



roger

## Accessories available to you

Additional accessories to help you more in everyday situations such as at home, at work, on the phone or in busy environments.

**PHONAK**  
life is on



The accessories of the Phonak Wireless Communication Portfolio offer great benefits in difficult listening situations and unparalleled speech understanding in noise or over distance.

With the latest products, making and receiving phone calls at home or on the go, watching TV, listening to music, streaming movies on a laptop or a tablet, all become more comfortable.

The Roger products work with virtually all hearing aids and cochlear implants, are easy to use and have proven benefits to you.



Customise your solution	I would like to improve...			
	Changing the volume or program on my hearing aid	Hearing mainly one person speaking	Hearing in small meetings or with a couple of people talking around a table	Hearing the television
PilotOne*/ PilotOne II 	✓			
EasyCall*/ EasyCall II 				
ComPilot*/ ComPilot II 	✓			
Roger Select 		✓	✓	✓
ComPilot Air II 	✓			✓
TVLink ComPilot bundle* 	✓			✓
Roger Clip on Mic (with receiver) 		✓		✓
Roger Pen (with receiver) 		✓	✓	✓
ComPilot Bundle** 	✓	✓	✓	✓
DECT II* 				
Roger TableMic II 			✓	✓

\* These devices require wireless Phonak hearing aids – please check your hearing aid compatibility with your Audiologist

\*\* Requires a Roger X receiver. The bundle contains a microphone suited to your needs. Please speak to your audiologist.

Listening to music clearly and without disturbing others	Hearing phone calls clearly from my mobile phone	Hearing a landline phone at home or at work	Hearing mainly one person in a busy pub or restaurant	Hearing in large meetings or several people talking around a table
	✓			
✓	✓			
✓	✓	✓ <sup>1</sup>	✓	
✓	✓			
✓	✓	✓ <sup>1</sup>		
✓			✓	
✓	✓	✓ <sup>1</sup>	✓	
✓	✓	✓ <sup>1</sup>	✓	
		✓ <sup>2</sup>		
✓				✓

<sup>1</sup> Additional equipment required.

<sup>2</sup> Only works in a single telephone line use, such as at home.



# Solutions for conversations

## Roger Select – Designed for noisy environments




The Roger Select is a state-of-the-art wireless microphone designed to help you clearly hear in noise. Placed in the middle of a table, it allows you to hear the conversation around you by automatically and seamlessly switching from one talker to another. When multiple conversations take place, you can take control by selecting which way the internal microphone 'points' to listen. Additionally, you can connect your TV, laptop and other multimedia devices via an audio input to hear directly. Using its in-built Bluetooth® functionality, you can also answer and stream your phone calls hands free, direct to your hearing aids.

### Feature overview

- Fully automatic settings
- 360° hearing capabilities
- Audio input for connection to multimedia devices
- Manual touch keys to select directionality
- Operating time: 8 hours
- Operating range up to: 22 meters / 66 feet

### How To Use

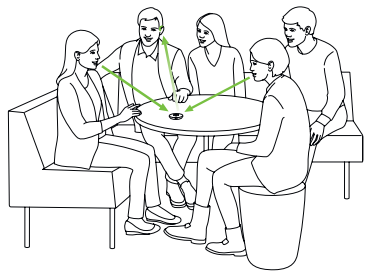

## Roger Pen – Hearing better than people with no hearing loss




The Roger Pen is designed with comfort and discretion in mind, providing you with greater speech understanding in a loud, noisy environment. It houses fully automatic settings allowing you to use it with ease, in most situations. With the Roger Pen, you can connect your TV, laptop and other multimedia devices via an audio input to hear directly. Using its in-built Bluetooth® functionality, you can also answer and stream your phone calls hands free, direct to your hearing aids.

### Feature overview

- Fully automatic settings
- Manual microphone mode option available
- Audio input for connection to multimedia devices
- Three different environmental modes
- Operating time: 7 hours (5 hours with a Bluetooth® call)
- Operating range up to: 22 meters / 66 feet

### How To Use

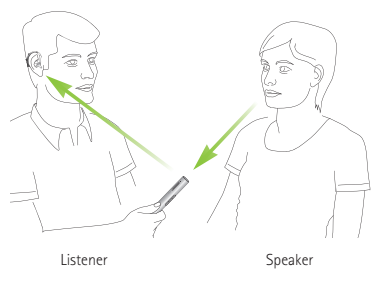

Please Note: The Roger Select, Roger Pen and Clip-On Mic require receivers to operate. Please confirm when placing your order.

## Roger Clip-On Mic – The most powerful wireless partner microphone




This lightweight mini microphone uses adaptive wireless transmission to help people with a hearing loss understand more speech in noise and over distance. It is ideal for single conversations, plus it includes an audio input for listening to multimedia and TV connectivity. It can also be used alongside other **Roger Clip-On Mics** and **Roger Pens** in a microphone network.

### Feature overview

- Proven Roger speech-in-noise performance
- One-click connection – no programming needed
- Audio input for connection to multimedia devices
- Used as standalone or with other Roger microphones
- Operating time: 6 hours
- Operating range: 10 meters / 33 feet (typical) 20 meters / 66 feet (ideal conditions)

### How To Use

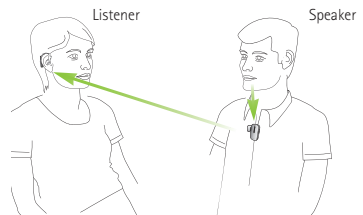

## Solutions for the workplace

### TableMic II – Especially designed for meetings




This microphone is dedicated for working adults who participate in various meetings. It selects the person who's talking and switches automatically between the meeting participants. Multiple Roger Table Mic II's can be paired up, making it ideal for large meetings. It can also transmit the sound of multimedia devices e.g. computers or TV's, to your hearing aids.

### Feature overview

- MultiBeam technology for clearer directionality
- Remote control for muting and adjusting the range
- Audio input for connection to multimedia devices
- Operating time: 16 hours
- Operating range: 10 meters / 33 feet (typical) 40 meters / 130 feet (ideal conditions)

### How To Use

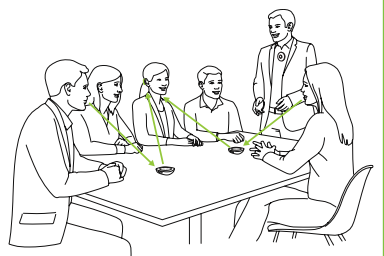

# Extended Streamer Solutions

## Phonak RemoteMic – The lightweight wireless microphone



**Phonak RemoteMic**, is a wireless microphone for one-on-one conversations over distance. The RemoteMic is clipped onto the speaker's clothing. Combined with Phonak ComPilot, it transmits the speaker's voice directly to both hearing aids over a distance of up to 20 meters (66 ft).


### Feature overview

- Lightweight and small
- Omnidirectional microphone
- Battery powered and rechargeable
- Up to 8 hours usage time

### Compatibility

- The Phonak RemoteMic can be used with all Phonak streamers
- Works in conjunction with the ComPilot and wireless hearing aids

### How To Use

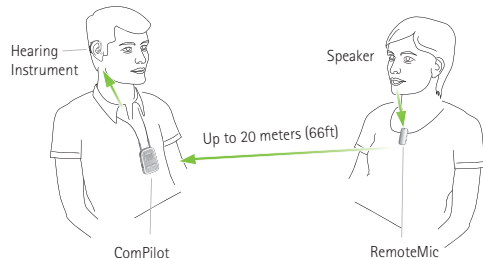

## ComPilot Air II – A Wireless Bluetooth device




This small and stylish device uses Bluetooth technology to connect Phonak hearing aids wirelessly to mobile phones or Bluetooth-enabled music sources. Phonak ComPilot Air II transmits a phone call directly into both hearing aids and offers excellent stereo sound quality when watching TV or listening to music.

### Feature overview

- No neckloop
- Clip for easy wearing
- Built-in remote control for hearing aids
- Spoken messages and callers name for easy interaction
- Operating time up to 4 hours Bluetooth streaming

### How To Use

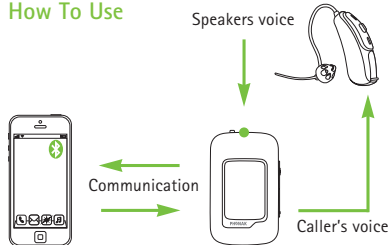



# Phone Solutions

---

## Phonak EasyCall – The mobile phone solution



**Phonak EasyCall** connects Phonak hearing aids wirelessly with any Bluetooth enabled mobile phone. It streams the phone call directly to both hearing aids for maximum speech understanding.

### Feature overview

- Unpack and use: no fitting required
- Works with any Bluetooth enabled mobile phone, even non-smartphones
- Attached onto the mobile phone, never lost or forgotten
- 10 hours talk time
- Standby time > 1 Week

### Compatibility

- Use Phonak EasyCall for Nathos S+ & Sky Q wireless hearing aids
- Use Phonak EasyCall II for Nathos™ Auto, Naida V and Sky V hearing aids

### How To Use



max. 25 cm / 10"

## Phonak DECT II – The home and small office phone solution



**Phonak DECT II** cordless phone is easy to use and designed for people with hearing loss. It is also suitable for people without hearing loss and can be used like a regular cordless phone. The main benefit is the direct binaural streaming to both hearing aids for maximum speech understanding.

### Feature overview

- Unpack and use: no fitting required
- Demo button for easy demonstration
- Simultaneous audio transmission to both hearing aids
- 1-click volume boost (up to +15 dB)
- Direct dial button for preferred number

### Compatibility

- Use Phonak DECT II for Nathos™ Auto, Sky V and Naida V Hearing aids

### How To Use



max. 25 cm / 10"

# TV, Music and Phone Solutions

## Phonak TV Link II basestation – Allows for improved audio listening



**Phonak TVLink II** basestation is a Bluetooth transmitter that turns Phonak hearing instruments into a light wireless TV headset. The TVLink II basestation wirelessly sends audio signals from the TV or any other audio source to the ComPilot, guaranteeing a perfect listening experience.


### Feature overview

- Easy and quick installation
- No line of sight to hearing instruments necessary
- Up to 30 meters (100 ft) transmission range
- Digital volume adjustment with audio-visual feedback
- Embedded charging slot for ComPilot
- Power saving option

### Compatibility:

- Use TV Link II basestation for Nathos™ Auto, Nathos S+, Sky Q, Sky V and Naida V wireless hearing aids
- Works in conjunction with the ComPilot

### How To Use

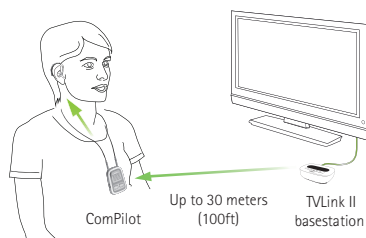

## Phonak ComPilot – The multi-purpose audio streamer



**Phonak ComPilot** connects wirelessly to all Phonak wireless hearing aids and supports audio inputs via 3.5mm Jack and the Europlug socket. Connecting the Roger X receiver to ComPilot allows streaming from Roger microphones. It provides high quality sound when watching TV or listening to music for a longer time period.

### Feature overview

- Built-in demonstration sound
- Bluetooth and wired audio inputs
- More than 20 hours continuous streaming time
- Built-in remote control for hearing aids
- Binaural streaming of audio signals in stereo
- Spoken messages and callers name for easy interaction
- Exchangeable neckloop available in two colors and lengths

### Compatibility:

- Use Phonak ComPilot for Nathos, Nathos S+ and Sky Q wireless hearing aids
- Use Phonak ComPilot II for Nathos™ Auto, Sky V and Naida V hearing aids

### How To Use



Together,  
we change lives

## Scheme 1 – Access to Work

### Funding support for adults in work or about to go into work

An **Access to Work (ATW)** grant offers financial support for assistive listening devices to enhance your hearing aids. The grant does not need to be repaid and will not affect your other benefits.

Below is a self-check list to give you an overview of the initial criteria:

- ✔ Do you wear hearing aids?
- ✔ Are you over 16 years old?
- ✔ Do you live in England, Scotland or Wales?
- ✔ Are you in employment, self-employment, work experience or about to start a job?

Once your application has been approved, please contact the department to purchase the necessary assistive technology.

## Scheme 2 – Disability Students' Allowance

### Funding support for students in higher education

As a higher education student living in the United Kingdom, you can apply for a **Disability Students' Allowance (DSA)** if you have a disability. If you wear a hearing aid(s), an assistive listening device to enhance your hearing can be applied for and may be fully funded. The support you can receive is in addition to any student grant or loan, is not affected by income and does not have to be repaid.

Below is a self-check list to give you an overview of the initial criteria:

- ✔ Do you wear hearing aids?
- ✔ Are you over 16 years old?
- ✔ Do you live in the United Kingdom?
- ✔ Are you an undergraduate or post-graduate?
- ✔ Do you qualify for Student Finance England, Student Finance Wales, Student Finance Northern Ireland or Student Awards Agency Scotland?

For more information on how to apply visit:  
[www.hearinggrants.co.uk](http://www.hearinggrants.co.uk)

## Scheme 3 – Veterans' Hearing Fund

### Medical funds programme for veterans who have served in the Armed Forces

The Royal British Legion's veterans medical fund is a programme for veterans that have been affected during their service in the Armed Forces. The **Veterans' Hearing Fund (VHF)** provides support to veterans who acquired hearing loss during Service. This funding can be used to purchase assistive listening devices to enhance your hearing aids.

Below is a self-check list to give you an overview of the initial criteria:

- ☒ Do you wear hearing aids?
- ☒ Have you served in the Armed Forces?
- ☒ Was your hearing loss acquired during service?

If you have answered 'yes' to the above, visit [www.hearinggrants.co.uk](http://www.hearinggrants.co.uk) for more information and to receive your application form.

Once you have received your stage 2 application form. Please contact the department for an appointment so that we can undertake an assessment of your needs and propose ways in which VHF support may be able to help you.

## Not eligible? – Self funding

### Purchasing products privately

There are a variety of additional products available to purchase outside of these schemes should you be ineligible, or a parent looking to acquire additional technology to work with your child's hearing aid(s).

The Technology can be purchased directly from the audiology department.

Email: [enquiries@hearingsupport.info](mailto:enquiries@hearingsupport.info)  
Call: 0800 035 6670

## Accessory Price List

(Prices correct as of 01/09/18)

ComPilot II .....	£252
TV Link II .....	£164
RemoteMic v2.0 .....	£213
Phonak DECT Phone II .....	£208
EasyCall II .....	£139
PilotOne II .....	£203

### Bundle Prices

ComPilot II & TVLink II .....	£346
ComPilot II & Remote Mic v.2.0 .....	£405
ComPilot II, TVLink II & Remote Mic v2.0 .....	£508

### Roger

Roger Pen & 1 Roger receiver (one hearing aid) .....	£799
Roger Pen & Roger receivers (two hearing aids) .....	£1079
Roger Clip-on Mic & 1 Roger receiver (one hearing aid) .....	£602
Roger Clip-on Mic & Roger receivers (two hearing aids) .....	£874
Roger Select & 1 Roger receiver (one hearing aid) .....	£888
Roger Select & 2 Roger receivers (Two hearing aids) .....	£1,160

### Accessories

Replacement neck loop for ComPilot .....	£20
Easy call Phone case .....	£20
(Available for Samsung Galaxy s4 Mini/s5 Mini/s6, iPhone 5/5s/6/6 Plus/6s Plus)	
Silicon protection cover for ComPilot .....	£10



# How do I access the products?

---

You can purchase or simply find out about the products highlighted in this brochure by speaking to your audiologist.

For demonstration and/or purchase, please contact the Audiology team on **01623 622 515** ext **6171**.

We do not offer a loan or trial of equipment, but we do offer advice and a demonstration service. To give you peace of mind, orders are placed with the assurance that if for any reason, you find the product unsuitable, we will refund the cost of the product, providing it is received by us in an 'as new resalable condition' in the original undamaged packaging. The product must be returned with proof of purchase and a covering letter explaining the reason for return, to arrive with us within 21 days of invoice date.

This offer of refund does not apply if you have used the equipment other than to assess whether it is satisfactory.

The definition of 'as new resalable condition' includes being returned as originally packaged with the original instruction booklets which have not been written in. A repackaging charge will be made if we have to re-coil leads neatly or replace missing bags or internal packaging etc.

An administration charge of £25 to cover the cost of advice and demonstration of the equipment will be deducted from the refund amount if the equipment is returned.

The accessories of the Phonak Wireless Communication Portfolio offer great benefits in difficult listening situations and unparalleled speech understanding in noise or over distance. With the latest products, making and receiving phone calls at home or on the go, watching TV, listening to music, streaming movies on a laptop or a tablet, all become more comfortable. Wireless accessories are easy to use and have proven benefits to you.

## Sherwood Forest Hospitals NHS Foundation Trust

Audiology Services  
Clinic 8, King's Treatment Centre  
King's Mill Hospital  
Mansfield Road  
Sutton-in-Ashfield  
Nottinghamshire NG17 4JL  
T: **01623 622 515** ext. **6171**  
E: **[sfh-tr.Audiology@nhs.net](mailto:sfh-tr.Audiology@nhs.net)**  
Textphone: **0792 0206720**

Audiology Services  
Newark Hospital  
Boundary Road  
Newark  
Nottinghamshire  
NG24 4DE  
T: **01623 622 515** ext. **6171**  
E: **[sfh-tr.Audiology@nhs.net](mailto:sfh-tr.Audiology@nhs.net)**  
Textphone: **0792 0206720**

If you would like this letter or information in an alternative format, for example large print or easy read, or if you need help with communicating with us, for example because you use British Sign Language, please let us know. You can call the Patient Experience Team on **01623 672222** or email **[sfh-tr.PET@nhs.net](mailto:sfh-tr.PET@nhs.net)**

Supported in conjunction with:

**PHONAK**  
life is on

To be reviewed September 2021

