**Artisan™
Acid-Fast Bacillus
(AFB) Stain Kit**

**ENGLISH
Code AR162****Intended use**

For In Vitro Diagnostic Use.

Acid-Fast Bacillus (AFB) Stain Kit is intended for laboratory use to identify, by light microscopy, acid-fast bacilli, such as Mycobacterium, in tissue samples.^{1,2} Routinely processed samples (paraffin-embedded) may be used. The preferred fixative is neutral buffered formalin. The clinical interpretation of any staining, or its absence, should be complemented by morphological studies and proper controls, and should be evaluated within the context of the patient's clinical history and other diagnostic tests by a qualified pathologist. All reagents in the AFB Stain Kit have been optimally prepared for use on Dako's Artisan Staining System and require no mixing or diluting.

Summary and explanation

The AFB Stain Kit is typically used to identify acid-fast bacilli, such as Mycobacterium. This procedure is based on the work of Ziehl-Neelsen¹ and utilizes a Carbol Fuchsin solution to which phenol has been added. The addition of phenol acts as a mordant between the bacilli and staining solution. This species of bacteria retains the Carbol Fuchsin after being treated with Acid Decolorizer, thus the name Acid-Fast.

Principle of procedure

Acid-fast organisms typically do not stain with Hematoxylin and Eosin and are difficult to demonstrate by the Gram technique, because they possess a hydrophobic capsule containing mycolic acid, a long-chain fatty acid.¹ The AFB Stain Kit is a modification of the original Ziehl-Neelsen method.²

The AFB Stain Kit is optimized for use on the Artisan Staining System. Wash steps follow all of the staining steps. Following staining, the air dried slides are removed from the instrument cleared with xylene or xylene substitute and mounted with appropriate mounting media. Results are interpreted using a standard light microscope.

Control Tissue: Lung infected with Acid Bacteria Bacillus

Reagents provided

The AFB Stain Kit is composed of three ready-to-use dispenser packs, each one containing:

Carbol Fuchsin**CARBOL FUCHSIN**

0.9% Basic Fuchsin, 4.3% Phenol and 9% Alcohol in deionized water

Acid Alcohol 1%**ACID ALCOHOL 1%**

70% Alcohol with 1% Hydrochloric Acid

Methylene Blue**METHYLENE BLUE**

1% Methylene Blue in deionized water containing antimicrobial agents

The Carbol Fuchsin reagent pack should be agitated before priming.

Dako provides these reagents at optimal formulation for use according to the instructions provided for staining on prepared tissue sections. Any deviation from the recommended test procedures may invalidate declared expected results.

Materials required, but not supplied

1. Microscope slides
2. Artisan Staining System
3. Wash Solution (code AR102)

Precautions

1. For professional users.
2. Minimize microbial contamination of reagents or increase in nonspecific staining may occur.
3. Reagent packs may contain residual solution after completing the prescribed number of tests.
4. Incubation times or temperatures other than those specified may give erroneous results. The user must validate any changes made to the factory-released procedure.
5. Store reagent packs in an upright position when not in use on the instrument.
6. Keep reagent packs out of direct sunlight and away from heat-generating sources.
7. As a general rule, persons under 18 years of age are not allowed to work with this product. Users must be carefully instructed in the proper work procedures, the dangerous properties of the product and the necessary safety instructions. Please refer to Safety Data Sheet (SDS) for additional information.
8. Wear appropriate Personal Protective Equipment to avoid contact with eyes and skin.
9. Unused solution should be disposed of according to local, State and Federal regulations.
10. Safety Data Sheet available for professional users on request.

Danger

Carbol Fuchsin: 1-5% Phenol, 0.1-1% 4,4'-(4-Iminocyclohexa-2,5-dienylidene)methyl)dianiline hydrochloride

H314	Causes severe skin burns and eye damage.
H360	May damage fertility or the unborn child.
H341	Suspected of causing genetic defects.
H351	Suspected of causing cancer.
H373	May cause damage to organs through prolonged or repeated exposure.
P201	Obtain special instructions before use.
P202	Do not handle until all safety precautions have been read and understood.
P281	Use personal protective equipment as required.
P280	Wear protective gloves. Wear eye or face protection. Wear protective clothing.
P260	Do not breathe vapor.
P264	Wash hands thoroughly after handling.
P314	Get medical attention if you feel unwell.
P308 + P313	IF exposed or concerned: Get medical attention.
P304 + P340 + P310	IF INHALED: Remove victim to fresh air and keep at rest in a position comfortable for breathing. Immediately call a POISON CENTER or physician.
P301 + P310 + P330 + P331	IF SWALLOWED: Immediately call a POISON CENTER or physician. Rinse mouth. Do NOT induce vomiting.
P303 + P361 + P353 + P363 + P310	IF ON SKIN (or hair): Take off immediately all contaminated clothing. Rinse skin with water or shower. Wash contaminated clothing before reuse. Immediately call a POISON CENTER or physician.
P305 + P351 + P338 + P310	IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. Immediately call a POISON CENTER or physician.
P405	Store locked up.
P501	Dispose of contents and container in accordance with all local, regional, national and international regulations.

Danger

Acid Alcohol, 1%: 30-60% Ethanol, 3-<10% Methanol

H225	Highly flammable liquid and vapor.
H314	Causes severe skin burns and eye damage.
H360	May damage fertility or the unborn child.
H371	May cause damage to organs.
H335	May cause respiratory irritation.
H336	May cause drowsiness and dizziness.
H373	May cause damage to organs through prolonged or repeated exposure.
P201	Obtain special instructions before use.
P202	Do not handle until all safety precautions have been read and understood.
P281	Use personal protective equipment as required.
P280	Wear protective gloves. Wear eye or face protection. Wear protective clothing.
P210	Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking.
P241	Use explosion-proof electrical, ventilating, lighting and all material-handling equipment.
P242	Use only non-sparking tools.
P243	Take precautionary measures against static discharge.
P233	Keep container tightly closed.
P271	Use only outdoors or in a well-ventilated area.
P260	Do not breathe vapor.
P270	Do not eat, drink or smoke when using this product.
P264	Wash hands thoroughly after handling.
P314	Get medical attention if you feel unwell.
P309 + P311	IF exposed or if you feel unwell: Call a POISON CENTER or physician.

P304 + P340 + P310 IF INHALED: Remove victim to fresh air and keep at rest in a position comfortable for breathing. Immediately call a POISON CENTER or physician.

P301 + P310 + P330 + P331 IF SWALLOWED: Immediately call a POISON CENTER or physician. Rinse mouth. Do NOT induce vomiting.

P303 + P361 + P353 + P363 + P310 IF ON SKIN (or hair): Take off immediately all contaminated clothing. Rinse skin with water or shower. Wash contaminated clothing before reuse. Immediately call a POISON CENTER or physician.

P305 + P351 + P338 + P310 IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. Immediately call a POISON CENTER or physician.

P405 Store locked up.

P403 Store in a well-ventilated place.

P235 Keep cool.

P501 Dispose of contents and container in accordance with all local, regional, national and international regulations.

Warning

Methylene Blue: 0.1-1%

H317 May cause an allergic skin reaction.

P280 Wear protective gloves.

P261 Avoid breathing vapor.

P272 Contaminated work clothing should not be allowed out of the workplace.

P302 + P352 + P363 IF ON SKIN: Wash with plenty of soap and water. Wash contaminated clothing before reuse.

P333 + P313 If skin irritation or rash occurs: Get medical attention.

P501 Dispose of contents and container in accordance with all local, regional, national and international regulations.

Storage

Each component should be stored at the temperature indicated on the label. Do not use after expiration date. If reagents are stored under any conditions other than those specified, the conditions must be verified by the user. There are no obvious signs to indicate instability of this product. Therefore, appropriate tissue must be run as controls for such verification. If unexpected staining is observed which cannot be explained by variations in laboratory procedures or improper storage, contact Dako Technical Support.

Instrument procedure / Staining procedure

Preliminary Comments

Please refer to the *Artisan Operator's Manual* for detailed instructions.

Priming Reagent Packs

Before beginning staining, reagent packs should be primed and checked for leakage. Check the prime of each reagent pack when a new kit is used and before the first staining run of the day.

1. Over a paper towel, if present, remove the red or black shipping cap from the dispense assembly of the reagent pack by gently pulling down on the cap.
2. To prime a reagent pack when it is first received, hold the reagent pack with the dispenser assembly pointed downward and lightly tap on the side of the reagent pack 2–3 times to release any air bubbles that may be trapped in the dispenser assembly.
3. Fully compress and release the plunger 3–4 times in rapid succession, taking care to absorb any expelled fluid into a paper towel, absorbent gauze, or similar material, until a steady stream is obtained.
4. Avoid dispensing excess amounts of reagent during priming. Excessive priming will result in reagent loss, which may cause the reagent pack to run out of fluid before the stated number of tests is performed.
5. Before each use, reagent packs may be primed 1–2 times, checking for a steady stream.
6. Holding the reagent pack in one hand, slide the mounting guide on the back of the reagent pack into the slots on the reagent carousel until the pack is completely seated.

Procedure

Refer to the *Procedure Report* section in the *Artisan Host Software* for the recommended detailed procedure.

Note: After staining is completed, slides will be dry. Do Not Dip in Alcohol as this will further differentiate the stain. Dip in xylene or xylene substitute and permanently mount.

The use of the Artisan Clearing Solution in combination with the AFB Kit may reduce the amount of target bacteria stained by the Acid Fast Bacillus Kit.

Staining interpretation

Acid-fast bacilli:**Varying shades of red**
 Background:**Light blue**

Performance characteristics

Reproducibility

Intra-run reproducibility of 100% was determined by staining 12 slides containing the same control tissue on one instrument.
Inter-run reproducibility of 100% was determined by staining 12 slides containing the same control tissue on three different instruments.

Troubleshooting

Refer to the Maintenance and Troubleshooting section of your *Artisan Operator's Manual* or contact our Technical Support.

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Dako North America, Inc.
Dako Denmark A/S