

**BIDMC Drug Dictionary - OCTOBER 2020**

<b>Aminocaproic Acid BOLUS</b>	<b>1</b>
Short name	Amcr Bol
Drug ID	
Quantity of active agent	5000.000 mg
Size of infusate	50 ml
Concentration	100.000 mg/ml
Standard profile	-----
Type of device	Perfusor
Dose mode	Yes -- in mg/hr
Rate	Softlimit min. = 50.00 ml/hr Default = 100.00 ml/hr Softlimit max. = 100.00 ml/hr Hardlimit = 100.00 ml/hr
Dose rate	Softlimit min. = 5000.000 mg/hr Default = 10000.000 mg/hr Softlimit max. = 10000.000 mg/hr Hardlimit = 10000.000 mg/hr
VTBI	
Bolus selectable	No
Categories	All Drugs Cardiac
Remarks	
Color	green
Alarm priority	high
Show warning symbol, if no softlimits are set	Yes

<b>Aminocaproic Acid INFUSE</b>	<b>2</b>
Short name	Amicar
Drug ID	
Quantity of active agent	
Size of infusate	
Concentration	100.000 mg/ml
Standard profile	-----
Type of device	Perfusor
Dose mode	Yes -- in mg/hr
Rate	Softlimit min. = 5.00 ml/hr Default = 20.00 ml/hr Softlimit max. = 20.00 ml/hr Hardlimit = 25.00 ml/hr

Dose rate	Softlimit min. = 500.000 mg/hr Default = 2000.000 mg/hr Softlimit max. = 2000.000 mg/hr Hardlimit = 2500.000 mg/hr
VTBI	
Bolus selectable	No
Categories	All Drugs Cardiac
Remarks	
Color	green
Alarm priority	high
Show warning symbol, if no softlimits are set	Yes

<b>Amiodarone</b>	<b>3</b>
Short name	AmioD
Drug ID	
Quantity of active agent	
Size of infusate	
Concentration	1.200 mg/ml
Standard profile	-----
Type of device	Perfusor
Dose mode	Yes -- in mg/min
Rate	Softlimit min. = 25.00 ml/hr Default = 50.00 ml/hr Softlimit max. = 75.00 ml/hr Hardlimit = 100.00 ml/hr
Dose rate	Softlimit min. = 0.500 mg/min Default = 1.000 mg/min Softlimit max. = 1.500 mg/min Hardlimit = 2.000 mg/min
VTBI	
Bolus selectable	No
Categories	All Drugs Cardiac
Remarks	USE IV FILTER
Color	green
Alarm priority	high

Show warning symbol, if no softlimits are set	Yes
---	-----

<b>Argatroban</b>	<b>4</b>
Short name	Argatrob
Drug ID	
Quantity of active agent	50.000 mg
Size of infusate	50 ml
Concentration	1.000 mg/ml
Standard profile	-----
Type of device	Perfusor
Dose mode	Yes -- in mcg/kg/min
Rate	Softlimit min. = Default = Softlimit max. = Hardlimit =
Dose rate	Softlimit min. = 0.250 mcg/kg/min Default = 1.000 mcg/kg/min Softlimit max. = 3.000 mcg/kg/min Hardlimit = 10.000 mcg/kg/min
VTBI	
Bolus selectable	No
Categories	All Drugs Cardiac
Remarks	
Color	green
Alarm priority	high
Show warning symbol, if no softlimits are set	Yes

<b>ATG</b>	<b>5</b>
Short name	ATG
Drug ID	
Quantity of active agent	
Size of infusate	
Concentration	1.000 mg/ml
Standard profile	-----
Type of device	Perfusor
Dose mode	Yes -- in mg/hr
Rate	Softlimit min. = 42.00 ml/hr Default = 42.00 ml/hr Softlimit max. = 42.00 ml/hr

	Hardlimit = 42.00 ml/hr
Dose rate	Softlimit min. = 42.000 mg/hr Default = 42.000 mg/hr Softlimit max. = 42.000 mg/hr Hardlimit = 42.000 mg/hr
VTBI	
Bolus selectable	No
Categories	All Drugs
Remarks	
Color	green
Alarm priority	high
Show warning symbol, if no softlimits are set	Yes

<b>Bivalirudin</b>	<b>6</b>
Short name	Bivalrud
Drug ID	
Quantity of active agent	25.000 mg
Size of infusate	50 ml
Concentration	0.500 mg/ml
Standard profile	-----
Type of device	Perfusor
Dose mode	Yes -- in mg/kg/hr
Rate	Softlimit min. = Default = Softlimit max. = Hardlimit =
Dose rate	Softlimit min. = 0.100 mg/kg/hr Default = 0.200 mg/kg/hr Softlimit max. = 2.500 mg/kg/hr Hardlimit = 2.500 mg/kg/hr
VTBI	
Bolus selectable	Yes
Quantity of bolus	Softlimit min. = Default = 0.500 mg/kg Softlimit max. = Hardlimit =
	Softlimit min. = Default =

Bolus rate	Softlimit max. = Hardlimit =
Categories	All Drugs Cardiac
Remarks	
Color	green
Alarm priority	high
Show warning symbol, if no softlimits are set	No

<b>Ciprofloxacin</b>	<b>7</b>
Short name	Cipro
Drug ID	
Quantity of active agent	
Size of infusate	
Concentration	10.000 mg/ml
Standard profile	-----
Type of device	Perfusor
Dose mode	Yes -- in mg/hr
Rate	Softlimit min. = 40.00 ml/hr Default = 40.00 ml/hr Softlimit max. = 80.00 ml/hr Hardlimit = 160.00 ml/hr
Dose rate	Softlimit min. = 400.000 mg/hr Default = 400.000 mg/hr Softlimit max. = 800.000 mg/hr Hardlimit = 1600.000 mg/hr
VTBI	
Bolus selectable	No
Categories	All Drugs Antibiotic Cardiac
Remarks	
Color	green
Alarm priority	high
Show warning symbol, if no softlimits are set	Yes

<b>Clevidipine</b>	<b>8</b>
Short name	Clevidip

Drug ID	
Quantity of active agent	
Size of infusate	
Concentration	0.500 mg/ml
Standard profile	-----
Type of device	Perfusor
Dose mode	Yes -- in mg/hr
Rate	Softlimit min. = Default = 2.00 ml/hr Softlimit max. = 32.00 ml/hr Hardlimit = 64.00 ml/hr
Dose rate	Softlimit min. = Default = 1.000 mg/hr Softlimit max. = 16.000 mg/hr Hardlimit = 32.000 mg/hr
VTBI	
Bolus selectable	No
Categories	All Drugs Cardiac
Remarks	
Color	green
Alarm priority	high
Show warning symbol, if no softlimits are set	Yes

<b>Clindamycin</b>	<b>9</b>
Short name	Clinda
Drug ID	
Quantity of active agent	600.000 mg
Size of infusate	10 ml
Concentration	60.000 mg/ml
Standard profile	-----
Type of device	Perfusor
Dose mode	Yes -- in mg/hr
Rate	Softlimit min. = 20.00 ml/hr Default = 30.00 ml/hr Softlimit max. = 60.00 ml/hr Hardlimit =
Dose rate	Softlimit min. = 1200.000 mg/hr Default = 1800.000 mg/hr Softlimit max. = 3600.000 mg/hr

	Hardlimit =
VTBI	
Bolus selectable	No
Categories	All Drugs Antibiotic MAC
Remarks	
Color	green
Alarm priority	high
Show warning symbol, if no softlimits are set	Yes

<b>Dexmedetomidine</b>	<b>10</b>
Short name	DexMet
Drug ID	
Quantity of active agent	200.000 mcg
Size of infusate	50 ml
Concentration	4.000 mcg/ml
Standard profile	-----
Type of device	Perfusor
Dose mode	Yes -- in mcg/kg/hr
Rate	Softlimit min. = Default = Softlimit max. = Hardlimit =
Dose rate	Softlimit min. = 0.100 mcg/kg/hr Default = 0.200 mcg/kg/hr Softlimit max. = 0.700 mcg/kg/hr Hardlimit = 1.400 mcg/kg/hr
VTBI	
Bolus selectable	Yes
Quantity of bolus	Softlimit min. = 0.025 mcg/kg Default = 0.250 mcg/kg Softlimit max. = 1.000 mcg/kg Hardlimit = 2.000 mcg/kg
Bolus rate	Softlimit min. = 5.00 ml/hr Default = 10.50 ml/hr Softlimit max. = 100.00 ml/hr Hardlimit = 200.00 ml/hr

Categories	All Drugs Cardiac MAC
Remarks	
Color	green
Alarm priority	high
Show warning symbol, if no softlimits are set	Yes

<b>Dextran 40</b>	<b>11</b>
Short name	Dextran
Drug ID	
Quantity of active agent	
Size of infusate	
Concentration	100.000 mg/ml
Standard profile	-----
Type of device	Perfusor
Dose mode	Yes -- in mg/hr
Rate	Softlimit min. = 0.20 ml/hr Default = 0.20 ml/hr Softlimit max. = 0.40 ml/hr Hardlimit = 0.40 ml/hr
Dose rate	Softlimit min. = 20.000 mg/hr Default = 20.000 mg/hr Softlimit max. = 40.000 mg/hr Hardlimit = 40.000 mg/hr
VTBI	
Bolus selectable	Yes
Quantity of bolus	Softlimit min. = Default = 20.00 ml Softlimit max. = Hardlimit =
Bolus rate	Softlimit min. = Default = Softlimit max. = Hardlimit =
Categories	All Drugs
Remarks	
Color	green



Alarm priority	high
Show warning symbol, if no softlimits are set	Yes

<b>DOBUTamine</b>	<b>12</b>
Short name	Dobuta
Drug ID	
Quantity of active agent	
Size of infusate	
Concentration	1.000 mg/ml
Standard profile	-----
Type of device	Perfusor
Dose mode	Yes -- in mcg/kg/min
Rate	Softlimit min. = Default = Softlimit max. = Hardlimit =
Dose rate	Softlimit min. = 0.500 mcg/kg/min Default = 5.000 mcg/kg/min Softlimit max. = 20.000 mcg/kg/min Hardlimit = 40.000 mcg/kg/min
VTBI	
Bolus selectable	No
Categories	All Drugs Cardiac
Remarks	
Color	green
Alarm priority	high
Show warning symbol, if no softlimits are set	Yes

<b>DOPamine</b>	<b>13</b>
Short name	DopaM
Drug ID	
Quantity of active agent	
Size of infusate	
Concentration	1.600 mg/ml
Standard profile	-----
Type of device	Perfusor
Dose mode	Yes -- in mcg/kg/min
	Softlimit min. = Default =

Rate	Softlimit max. = Hardlimit =
Dose rate	Softlimit min. = 1.000 mcg/kg/min Default = 2.000 mcg/kg/min Softlimit max. = 20.000 mcg/kg/min Hardlimit = 40.000 mcg/kg/min
VTBI	
Bolus selectable	No
Categories	All Drugs Cardiac
Remarks	
Color	green
Alarm priority	high
Show warning symbol, if no softlimits are set	Yes

<b>EpiduralAPS</b>	<b>14</b>
Short name	APS
Drug ID	
Quantity of active agent	
Size of infusate	
Concentration	1.000 mg/ml
Standard profile	-----
Type of device	Perfusor
Dose mode	Yes -- in mg/hr
Rate	Softlimit min. = 1.00 ml/hr Default = 6.00 ml/hr Softlimit max. = 12.00 ml/hr Hardlimit = 20.00 ml/hr
Dose rate	Softlimit min. = 1.000 mg/hr Default = 6.000 mg/hr Softlimit max. = 12.000 mg/hr Hardlimit = 20.000 mg/hr
VTBI	
Bolus selectable	Yes
Quantity of bolus	Softlimit min. = 2.00 ml Default = 5.00 ml Softlimit max. = 10.00 ml Hardlimit = 10.00 ml

Bolus rate	Softlimit min. = 30.00 ml/hr Default = 150.00 ml/hr Softlimit max. = 300.00 ml/hr Hardlimit = 300.00 ml/hr
Categories	All Drugs
Remarks	FOR EPIDURAL USE ONLY
Color	green
Alarm priority	high
Show warning symbol, if no softlimits are set	Yes

<b>EPINEPHrine</b>	<b>15</b>
Short name	Epi
Drug ID	
Quantity of active agent	
Size of infusate	
Concentration	0.008 mg/ml
Standard profile	-----
Type of device	Perfusor
Dose mode	Yes -- in mcg/kg/min
Rate	Softlimit min. = Default = Softlimit max. = Hardlimit =
Dose rate	Softlimit min. = 0.004 mcg/kg/min Default = 0.030 mcg/kg/min Softlimit max. = 0.500 mcg/kg/min Hardlimit = 0.500 mcg/kg/min
VTBI	
Bolus selectable	No
Categories	All Drugs Cardiac
Remarks	
Color	green
Alarm priority	high
Show warning symbol, if no softlimits are set	Yes

<b>Epoprostenol</b>	<b>16</b>
Short name	Epoprost

Drug ID	
Quantity of active agent	
Size of infusate	
Concentration	
Standard profile	-----
Type of device	Perfusor
Dose mode	Yes -- in mcg/kg/min
Rate	Softlimit min. = Default = Softlimit max. = Hardlimit =
Dose rate	Softlimit min. = 0.010 mcg/kg/min Default = 0.050 mcg/kg/min Softlimit max. = 0.050 mcg/kg/min Hardlimit = 0.050 mcg/kg/min
VTBI	
Bolus selectable	No
Categories	All Drugs Cardiac
Remarks	
Color	green
Alarm priority	high
Show warning symbol, if no softlimits are set	Yes

<b>Esmolol</b>	<b>17</b>
Short name	Esmolol
Drug ID	
Quantity of active agent	
Size of infusate	
Concentration	10.000 mg/ml
Standard profile	-----
Type of device	Perfusor
Dose mode	Yes -- in mcg/kg/min
Rate	Softlimit min. = Default = Softlimit max. = Hardlimit =
Dose rate	Softlimit min. = 5.000 mcg/kg/min Default = 5.000 mcg/kg/min Softlimit max. = 150.000 mcg/kg/min

	Hardlimit = 300.000 mcg/kg/min
VTBI	
Bolus selectable	Yes
Quantity of bolus	Softlimit min. = 100.000 mcg/kg Default = 500.000 mcg/kg Softlimit max. = 1000.000 mcg/kg Hardlimit = 1000.000 mcg/kg
Bolus rate	Softlimit min. = Default = Softlimit max. = Hardlimit =
Categories	All Drugs Cardiac
Remarks	Rate control higher (50)
Color	green
Alarm priority	high
Show warning symbol, if no softlimits are set	Yes

<b>Insulin</b>	<b>18</b>
Short name	Insulin
Drug ID	
Quantity of active agent	
Size of infusate	
Concentration	1.000 IU/ml
Standard profile	-----
Type of device	Perfusor
Dose mode	Yes -- in IU/hr
Rate	Softlimit min. = 0.25 ml/hr Default = 1.00 ml/hr Softlimit max. = 5.00 ml/hr Hardlimit = 30.00 ml/hr
Dose rate	Softlimit min. = 0.250 IU/hr Default = 1.000 IU/hr Softlimit max. = 5.000 IU/hr Hardlimit = 30.000 IU/hr
VTBI	
Bolus selectable	Yes
	Softlimit min. = 1.000 IU

Quantity of bolus	Default = 1.000 IU Softlimit max. = 5.000 IU Hardlimit = 10.000 IU
Bolus rate	Softlimit min. = Default = Softlimit max. = Hardlimit =
Categories	All Drugs Cardiac
Remarks	
Color	green
Alarm priority	high
Show warning symbol, if no softlimits are set	Yes

<b>Ketamine</b>	<b>19</b>
Short name	Ketamin
Drug ID	
Quantity of active agent	
Size of infusate	
Concentration	10.000 mg/ml
Standard profile	-----
Type of device	Perfusor
Dose mode	Yes -- in mg/kg/hr
Rate	Softlimit min. = Default = Softlimit max. = Hardlimit =
Dose rate	Softlimit min. = 0.100 mg/kg/hr Default = 0.100 mg/kg/hr Softlimit max. = 0.300 mg/kg/hr Hardlimit = 1.000 mg/kg/hr
VTBI	
Bolus selectable	Yes
Quantity of bolus	Softlimit min. = 0.100 mg/kg Default = 0.200 mg/kg Softlimit max. = 0.500 mg/kg Hardlimit = 0.750 mg/kg
	Softlimit min. =

Bolus rate	Default = Softlimit max. = Hardlimit =
Categories	All Drugs MAC
Remarks	
Color	green
Alarm priority	high
Show warning symbol, if no softlimits are set	Yes

<b>Lidocaine - Arrythmia</b>	<b>20</b>
Short name	LidoARR
Drug ID	
Quantity of active agent	
Size of infusate	
Concentration	10.000 mg/ml
Standard profile	-----
Type of device	Perfusor, Infusomat
Dose mode	Yes -- in mg/min
Rate	Softlimit min. = 6.00 ml/hr Default = 12.00 ml/hr Softlimit max. = 24.00 ml/hr Hardlimit = 24.00 ml/hr
Dose rate	Softlimit min. = 1.000 mg/min Default = 2.000 mg/min Softlimit max. = 4.000 mg/min Hardlimit = 4.000 mg/min
VTBI	
Bolus selectable	No
Categories	All Drugs Cardiac
Remarks	
Color	green
Alarm priority	high
Show warning symbol, if no softlimits are set	Yes

<b>Lidocaine - Pain</b>	<b>21</b>
Short name	LidoPAIN

Drug ID	
Quantity of active agent	
Size of infusate	
Concentration	10.000 mg/ml
Standard profile	-----
Type of device	Perfusor
Dose mode	Yes -- in mg/kg/hr
Rate	Softlimit min. = Default = Softlimit max. = Hardlimit =
Dose rate	Softlimit min. = 1.000 mg/kg/hr Default = 1.000 mg/kg/hr Softlimit max. = 2.000 mg/kg/hr Hardlimit = 2.000 mg/kg/hr
VTBI	
Bolus selectable	Yes
Quantity of bolus	Softlimit min. = Default = Softlimit max. = Hardlimit =
Bolus rate	Softlimit min. = Default = Softlimit max. = Hardlimit =
Categories	All Drugs
Remarks	
Color	green
Alarm priority	high
Show warning symbol, if no softlimits are set	Yes

<b>Milrinone</b>	<b>22</b>
Short name	Mlrinone
Drug ID	
Quantity of active agent	10.000 mg
Size of infusate	50 ml
Concentration	0.200 mg/ml
Standard profile	-----
Type of device	Perfusor



Dose mode	Yes -- in mcg/kg/min
Rate	Softlimit min. = Default = Softlimit max. = Hardlimit =
Dose rate	Softlimit min. = 0.250 mcg/kg/min Default = 0.500 mcg/kg/min Softlimit max. = 0.750 mcg/kg/min Hardlimit = 0.750 mcg/kg/min
VTBI	
Bolus selectable	Yes
Quantity of bolus	Softlimit min. = 25.000 mcg/kg Default = 50.000 mcg/kg Softlimit max. = 50.000 mcg/kg Hardlimit = 100.000 mcg/kg
Bolus rate	Softlimit min. = 105.00 ml/hr Default = 210.00 ml/hr Softlimit max. = 390.00 ml/hr Hardlimit = 800.00 ml/hr
Categories	All Drugs Cardiac
Remarks	
Color	green
Alarm priority	high
Show warning symbol, if no softlimits are set	Yes

<b>NiCARDipine</b>	<b>23</b>
Short name	Nicardip
Drug ID	
Quantity of active agent	
Size of infusate	
Concentration	0.200 mg/ml
Standard profile	-----
Type of device	Perfusor
Dose mode	Yes -- in mg/hr
Rate	Softlimit min. = 25.00 ml/hr Default = 25.00 ml/hr Softlimit max. = 75.00 ml/hr Hardlimit = 75.00 ml/hr

Dose rate	Softlimit min. = 5.000 mg/hr Default = 5.000 mg/hr Softlimit max. = 15.000 mg/hr Hardlimit = 15.000 mg/hr
VTBI	
Bolus selectable	No
Categories	All Drugs
Remarks	
Color	green
Alarm priority	high
Show warning symbol, if no softlimits are set	Yes

<b>NitroGLYcerin</b>	<b>24</b>
Short name	NitroGL
Drug ID	
Quantity of active agent	
Size of infusate	
Concentration	0.400 mg/ml
Standard profile	-----
Type of device	Perfusor
Dose mode	Yes -- in mcg/kg/min
Rate	Softlimit min. = Default = Softlimit max. = Hardlimit =
Dose rate	Softlimit min. = 0.050 mcg/kg/min Default = 0.500 mcg/kg/min Softlimit max. = 5.000 mcg/kg/min Hardlimit = 10.000 mcg/kg/min
VTBI	
Bolus selectable	Yes
Quantity of bolus	Softlimit min. = 0.100 mcg/kg Default = 0.200 mcg/kg Softlimit max. = 0.500 mcg/kg Hardlimit = 1.000 mcg/kg
Bolus rate	Softlimit min. = Default = Softlimit max. =

	Hardlimit =
Categories	All Drugs Cardiac
Remarks	
Color	green
Alarm priority	high
Show warning symbol, if no softlimits are set	Yes

<b>NitroPRUside</b>	<b>25</b>
Short name	NitroPRu
Drug ID	
Quantity of active agent	
Size of infusate	
Concentration	0.200 mg/ml
Standard profile	-----
Type of device	Perfusor
Dose mode	Yes -- in mcg/kg/min
Rate	Softlimit min. = Default = Softlimit max. = Hardlimit =
Dose rate	Softlimit min. = 0.050 mcg/kg/min Default = 0.500 mcg/kg/min Softlimit max. = 6.000 mcg/kg/min Hardlimit = 10.000 mcg/kg/min
VTBI	
Bolus selectable	No
Categories	All Drugs Cardiac
Remarks	
Color	green
Alarm priority	high
Show warning symbol, if no softlimits are set	Yes

<b>NORepinephrine</b>	<b>26</b>
Short name	NORepi
Drug ID	
Quantity of active agent	

Size of infusate	
Concentration	0.032 mg/ml
Standard profile	-----
Type of device	Perfusor
Dose mode	Yes -- in mcg/kg/min
Rate	Softlimit min. = Default = Softlimit max. = Hardlimit =
Dose rate	Softlimit min. = 0.005 mcg/kg/min Default = 0.025 mcg/kg/min Softlimit max. = 0.500 mcg/kg/min Hardlimit = 0.500 mcg/kg/min
VTBI	
Bolus selectable	Yes
Quantity of bolus	Softlimit min. = 0.010 mcg/kg Default = 0.030 mcg/kg Softlimit max. = 0.060 mcg/kg Hardlimit = 0.100 mcg/kg
Bolus rate	Softlimit min. = Default = Softlimit max. = Hardlimit =
Categories	All Drugs Cardiac
Remarks	
Color	green
Alarm priority	high
Show warning symbol, if no softlimits are set	Yes

<b>Octreotide</b>	<b>27</b>
Short name	Octetri
Drug ID	
Quantity of active agent	500.000 mcg
Size of infusate	100 ml
Concentration	5.000 mcg/ml
Standard profile	-----
Type of device	Perfusor
Dose mode	Yes -- in mcg/hr

Rate	Softlimit min. = 5.00 ml/hr Default = 10.00 ml/hr Softlimit max. = 10.00 ml/hr Hardlimit = 10.00 ml/hr
Dose rate	Softlimit min. = 25.000 mcg/hr Default = 50.000 mcg/hr Softlimit max. = 50.000 mcg/hr Hardlimit = 50.000 mcg/hr
VTBI	
Bolus selectable	Yes
Quantity of bolus	Softlimit min. = Default = 50.000 mcg Softlimit max. = Hardlimit =
Bolus rate	Softlimit min. = Default = Softlimit max. = Hardlimit =
Categories	All Drugs
Remarks	
Color	green
Alarm priority	high
Show warning symbol, if no softlimits are set	Yes

<b>Phenylephrine</b>	<b>28</b>
Short name	Neo
Drug ID	
Quantity of active agent	
Size of infusate	
Concentration	0.240 mg/ml
Standard profile	-----
Type of device	Perfusor
Dose mode	Yes -- in mcg/kg/min
Rate	Softlimit min. = Default = Softlimit max. = Hardlimit =
	Softlimit min. = 0.050 mcg/kg/min

Dose rate	Default = 0.300 mcg/kg/min Softlimit max. = 5.000 mcg/kg/min Hardlimit =
VTBI	
Bolus selectable	Yes
Quantity of bolus	Softlimit min. = 0.100 mcg/kg Default = 0.500 mcg/kg Softlimit max. = 2.000 mcg/kg Hardlimit = 2.000 mcg/kg
Bolus rate	Softlimit min. = Default = Softlimit max. = Hardlimit =
Categories	All Drugs Cardiac
Remarks	
Color	green
Alarm priority	high
Show warning symbol, if no softlimits are set	Yes

<b>Propofol</b>	<b>29</b>
Short name	PPF
Drug ID	
Quantity of active agent	
Size of infusate	
Concentration	10.000 mg/ml
Standard profile	-----
Type of device	Perfusor
Dose mode	Yes -- in mcg/kg/min
Rate	Softlimit min. = Default = Softlimit max. = Hardlimit =
Dose rate	Softlimit min. = 10.000 mcg/kg/min Default = 40.000 mcg/kg/min Softlimit max. = 300.000 mcg/kg/min Hardlimit = 300.000 mcg/kg/min
VTBI	

Bolus selectable	Yes
Quantity of bolus	Softlimit min. = 1.00 ml Default = 2.00 ml Softlimit max. = 5.00 ml Hardlimit = 10.00 ml
Bolus rate	Softlimit min. = Default = Softlimit max. = Hardlimit =
Categories	All Drugs Cardiac MAC
Remarks	
Color	green
Alarm priority	high
Show warning symbol, if no softlimits are set	Yes

<b>Protamine</b>	<b>30</b>
Short name	PROTami
Drug ID	
Quantity of active agent	
Size of infusate	
Concentration	10.000 mg/ml
Standard profile	-----
Type of device	Perfusor
Dose mode	Yes -- in mg/hr
Rate	Softlimit min. = 20.00 ml/hr Default = 60.00 ml/hr Softlimit max. = 150.00 ml/hr Hardlimit = 200.00 ml/hr
Dose rate	Softlimit min. = 200.000 mg/hr Default = 600.000 mg/hr Softlimit max. = 1500.000 mg/hr Hardlimit = 2000.000 mg/hr
VTBI	
Bolus selectable	No
Categories	All Drugs Cardiac

Remarks	
Color	green
Alarm priority	high
Show warning symbol, if no softlimits are set	Yes

<b>REMifentanil GA</b>	<b>31</b>
Short name	Remi GA
Drug ID	
Quantity of active agent	
Size of infusate	
Concentration	0.050 mg/ml
Standard profile	-----
Type of device	Perfusor
Dose mode	Yes -- in mcg/kg/min
Rate	Softlimit min. = Default = Softlimit max. = Hardlimit =
Dose rate	Softlimit min. = 0.025 mcg/kg/min Default = 0.500 mcg/kg/min Softlimit max. = 2.000 mcg/kg/min Hardlimit = 2.000 mcg/kg/min
VTBI	
Bolus selectable	Yes
Quantity of bolus	Softlimit min. = 0.150 mcg/kg Default = 0.500 mcg/kg Softlimit max. = 1.000 mcg/kg Hardlimit = 1.000 mcg/kg
Bolus rate	Softlimit min. = Default = Softlimit max. = Hardlimit =
Categories	All Drugs Cardiac
Remarks	
Color	green
Alarm priority	high
Show warning symbol, if no softlimits are set	Yes



<b>REMifentanil MAC</b>	<b>32</b>
Short name	RemMAC
Drug ID	
Quantity of active agent	
Size of infusate	
Concentration	0.050 mg/ml
Standard profile	-----
Type of device	Perfusor
Dose mode	Yes -- in mcg/kg/min
Rate	Softlimit min. = Default = Softlimit max. = Hardlimit =
Dose rate	Softlimit min. = 0.025 mcg/kg/min Default = 0.050 mcg/kg/min Softlimit max. = 0.200 mcg/kg/min Hardlimit = 1.000 mcg/kg/min
VTBI	
Bolus selectable	Yes
Quantity of bolus	Softlimit min. = 0.100 mcg/kg Default = 0.500 mcg/kg Softlimit max. = 1.000 mcg/kg Hardlimit = 1.000 mcg/kg
Bolus rate	Softlimit min. = Default = Softlimit max. = Hardlimit =
Categories	All Drugs Cardiac MAC
Remarks	
Color	green
Alarm priority	high
Show warning symbol, if no softlimits are set	Yes

<b>Tranexamic Acid Loading</b>	<b>33</b>
Short name	TXS BOL

Drug ID	
Quantity of active agent	20.000 mg
Size of infusate	50 ml
Concentration	0.400 mg/ml
Standard profile	-----
Type of device	Perfusor
Dose mode	No
Rate	Softlimit min. = Default = Softlimit max. = Hardlimit =
VTBI	
Bolus selectable	Yes
Quantity of bolus	Softlimit min. = Default = Softlimit max. = Hardlimit =
Bolus rate	Softlimit min. = 100.00 ml/hr Default = 100.00 ml/hr Softlimit max. = 100.00 ml/hr Hardlimit = 100.00 ml/hr
Categories	All Drugs Cardiac
Remarks	
Color	green
Alarm priority	high
Show warning symbol, if no softlimits are set	Yes

<b>Tranexamic Acid Infusion</b>	<b>34</b>
Short name	TXA NFS
Drug ID	
Quantity of active agent	
Size of infusate	
Concentration	20.000 mg/ml
Standard profile	-----
Type of device	Perfusor
Dose mode	Yes -- in mg/kg/hr
Rate	Softlimit min. = Default = Softlimit max. =

	Hardlimit =
Dose rate	Softlimit min. = 1.000 mg/kg/hr Default = 5.000 mg/kg/hr Softlimit max. = 10.000 mg/kg/hr Hardlimit = 10.000 mg/kg/hr
VTBI	
Bolus selectable	No
Categories	All Drugs Cardiac
Remarks	
Color	green
Alarm priority	high
Show warning symbol, if no softlimits are set	Yes

<b>Vancomycin</b>	<b>35</b>
Short name	Vanco
Drug ID	
Quantity of active agent	
Size of infusate	
Concentration	50.000 mg/ml
Standard profile	-----
Type of device	Perfusor
Dose mode	Yes -- in mg/hr
Rate	Softlimit min. = 10.00 ml/hr Default = 20.00 ml/hr Softlimit max. = 40.00 ml/hr Hardlimit = 60.00 ml/hr
Dose rate	Softlimit min. = 500.000 mg/hr Default = 1000.000 mg/hr Softlimit max. = 2000.000 mg/hr Hardlimit = 3000.000 mg/hr
VTBI	
Bolus selectable	No
Categories	All Drugs Antibiotic Cardiac MAC
Remarks	

Color	green
Alarm priority	high
Show warning symbol, if no softlimits are set	Yes

<b>Vasopressin</b>	<b>36</b>
Short name	VasoPR
Drug ID	
Quantity of active agent	
Size of infusate	
Concentration	0.400 IU/ml
Standard profile	-----
Type of device	Perfusor
Dose mode	Yes -- in IU/hr
Rate	Softlimit min. = 1.50 ml/hr Default = 3.00 ml/hr Softlimit max. = 9.00 ml/hr Hardlimit = 9.00 ml/hr
Dose rate	Softlimit min. = 0.600 IU/hr Default = 1.200 IU/hr Softlimit max. = 3.600 IU/hr Hardlimit = 3.600 IU/hr
VTBI	
Bolus selectable	No
Categories	All Drugs Cardiac
Remarks	
Color	green
Alarm priority	high
Show warning symbol, if no softlimits are set	Yes