

HKS880

Cavitation+RF

User&Training Manual

ALGIS



HKS880

Cavitation+RF
User&Training Manual


Cavitation Theory

What is CAV?

HKS776B cavitation that revolutionizes the treatment against cellulites and fat. It is a non invasive based on sound waves which generates empty micro bubbles in the liquid situated in the interior of the adipose tissue.

Its repeated action causes the accumulation of these micro bubbles, with each other to implode and collapse.

The result is the destruction of the adipose cells transforming the fat nodules in liquid substances that in a natural way the body removes through the lymph..

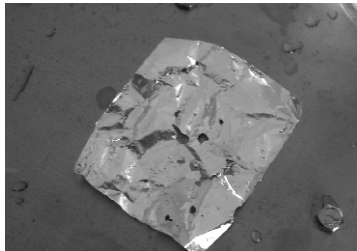

What areas can be treated with CAV systems?

Belly,waist,thigh,calf,buttocks.

What areas forbid treated with CAV systems?

The CAV is a deep model ultrasonic,fobid use around eyes and heart.

What do CAV treatments feel like?

When CAV 40K work,can hear zizizi voice like hexapod sound. body can feel warm and little vibrate.

How many treatments will I need?

3times in frist week,2times in second/third week.after that 1times per week until satisfying effect.

HKS880

Cavitation+RF

User&Training Manual

Radio Frequency Safety Guidelines

What is Radio Frequency?

RF is used for dermal skin heating with sparing of the epidermis. This will result in non-ablative skin tightening and lifting without injection and surgery.

What is the advantage?

The results are a firmer, refreshed and more youthful face, lifted brows and eyelids, less wrinkles and sharper jaw line again. Eyebags and double chin will improve with this treatment too.

How does it work?

First, heat generated by RF waves shrinks the bundle of collagen under the skin. The collagen becomes shorter and the skin tightens. The second phase begins as the body responds. The collagen will be gradually healed, renewed and even more collagen will be produced. The skin tightens and overall look is improved.

What do I expect during the treatment?

gentle warming sensation is felt throughout the procedures. As there is minimal redness after the treatment, normal activity can be resumed immediately.

How many sessions are required?

1times/2weeks

5-6 sessions is recommended for optimal longer lasting result in treating the wrinkles and lifting the face.



HKS880

Cavitation+RF

User&Training Manual

Device Information

APPLICATION

- *Localized fat
- *Cellulite
- *Body sculpture
- *Body Remodelling
- *Skin tightening and lifting
- *Wrinkle reduction
- *Systemic collagen regeneration

FEATURE

Cavitation

- *2Probe for CAV
- Deep Probe:25KHz with photon
- Middle Probe:40KHz
- *Power:60W
- *Frequency:25K+40KHz
- *Active Surface:18.6cm²
- *Operate timer:1-20min

Radio Frequency

- *1Probe for RF with 3model size,Bipolar+Tripolar
- *RF Energy:50J
- *Pulse type:continuous/pulse
- *Frequency:1/3/10MHz **Variable frequency**
- *Power:80W

Certifications:CE LVD,CE EMC

Pakcge size:81*27*35+61*51*24+53*6*52cm

N.W:22kg

G.W:31kg




HKS880

Cavitation+RF
User&Training Manual

Installation Alignment and operation procedures

1.1 Open the Package and checking,fix the probe holder in back of device.

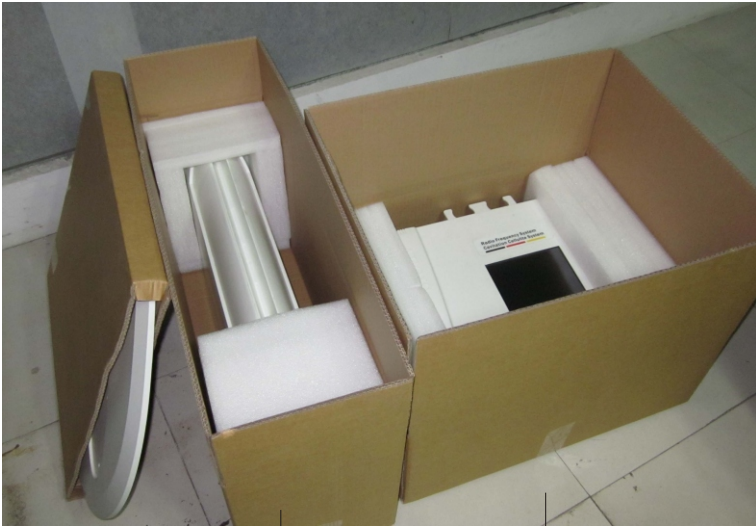

Base

Tubbiness

Top part

1.2 Fix all parts.

1.2.1 Fix the wheel under the Plastic base by screw.

1.2.2 connect the socket from top part to Tubbiness

1.2.3 Fix the top part on the Tubbiness by screw.

HKS880

Cavitation+RF
User&Training Manual

Installation Alignment and operation procedures



1.2.1



1.2.2



1.2.3

Fix This Device Only need
10minutes Max

But can **save**
shipment charge for you!

Normal stand Device:80-120kg
Our Device:32Kg by 3package


HKS880

Cavitation+RF

User&Training Manual

A.CAV Operate the Device

A.1 Operate interface

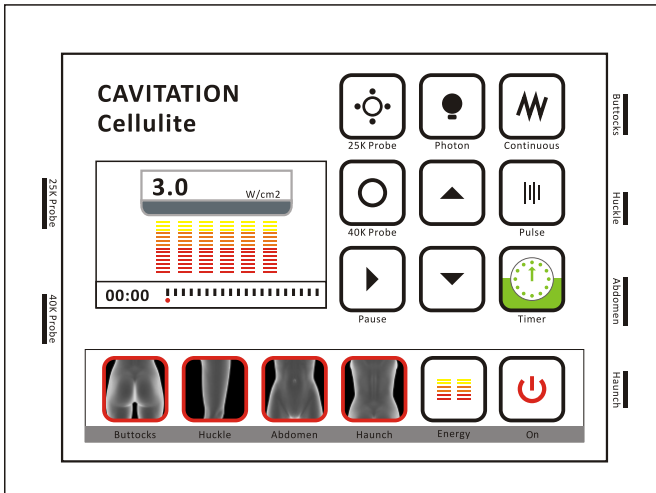

HKS880

Cavitation+RF

User&Training Manual

A.CAV Operate the Device

A.2 Auto Model Treatment



Buttocks



Huckle




Abdomen



Haunch

Choice one Treatment Part that you want, then Press OFF Button be ON, The Device start to work.

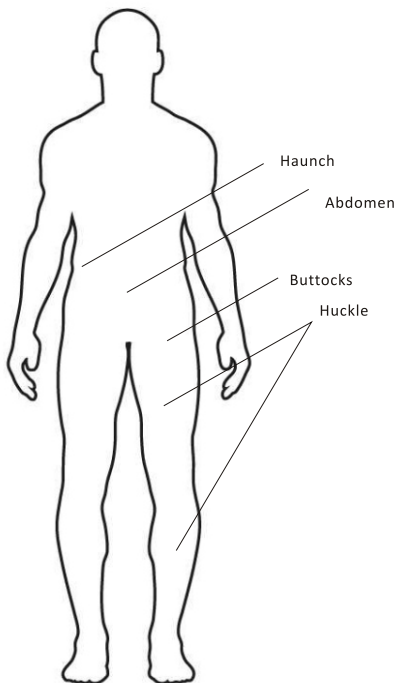

<p>Buttocks 25K probe 4.5W/cm² 10minutes Pulse</p>
<p>Huckle 40K probe 5.0W/cm² 12Minutes continuous</p>
<p>Abdomen 25K probe 5.5W/cm² 8minutes</p>
<p>Haunch 40K probe 4.0W/cm² 10minutes</p>

HKS880

Cavitation+RF

User&Training Manual

A.CAV Operate the Device

A.3 Manual Model Treatment

Choice right Probe that you want



25K Probe



40K Probe

Choice



Timer













Energy

Then press the UP and DOWN

For adjust.

Then Press OFF Button be ON, The Device start to work.

 25K Probe	▶	 Photon	This Photon only work with 25K probe
 Continuous	Continuous working	 Pulse	Discontinuous working
 ▲	Add Enrygy or Timer	 ▼	Reduce Enrygy or Timer
 On	Working	 OFF	Stop
 	Timer Pause	 ▶	Timer Working
<p>Pause</p>		<p>Play</p>	

HKS880



Cavitation+RF
User&Training Manual

B.CAV Treatment Procedure


B.1 Let the patient Drink 50-80ML Water before treatment.
Then make 5minutes massage,let the cellulite be relax.



B.2 Choice the fat area in body that you want make treatment.

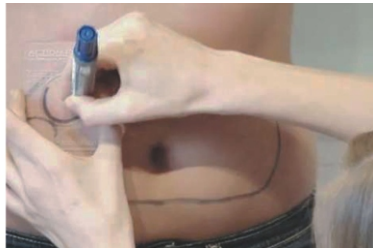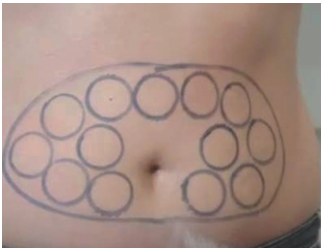

B.3 Put the GEL in this area.the GEL Thickness is around 3-5mm.

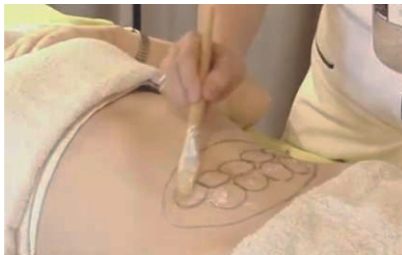


HKS880

Cavitation+RF
User&Training Manual

B.Treatment Procedure

B.4 Touch the Probe to Skin and move by slowly.
Let the treatment probe touch the target area and move(Or Slide)from the inner to the outer and round and round and from the lower place to the higher,do it in this way by reiterative, when you do the treatment on t he upper lid,Pls move a bit more quickly,when the gel on the skin get drier,please add some GEL and go on with the treatment.the ultrasonic GEL thickness is around 3-5mm.

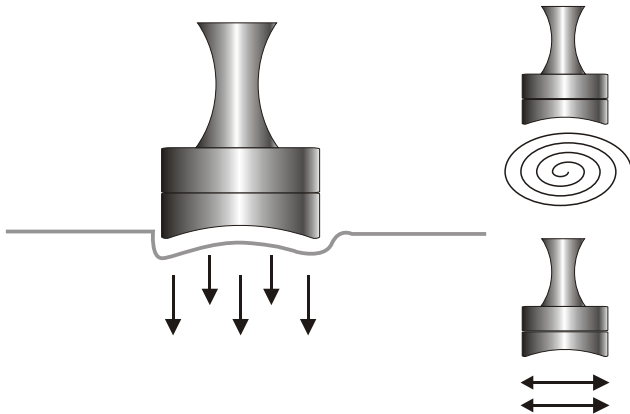

C.Cavitation Treatment General Knowledge

C.1.Curative effect

After frist one or two treatment,the Cellulite will lost 2-3kg.
The skin will become lift.

After one course of treatment;Skin becomes compact and
flabby skin is improved.The Cellulite will lost 5-10kg

C.2.treatment Caution And the principle

for treatment is that the patient should feel Warm but not very
hot.Keep the skin be wet,put more gel or spray more water.
Do not apply on areas such as around the face and heart.

C.3. Taboos

- a.To avoid using the machine on the abdomen of the pregnant
women or women in menses
- b.To avoid using the machine on around the parts under which
metal,plastic or silicon wee buried.
- c.To avoid using the machine on around the medical and
electric instrument,such as the hearing aid,heart starter,
mechanical heart,etc.
- d.To avoid using the machine on the epileptic and those in
bad health
- e.To avoid using the machine on the wound around the
treatment area and the areas with metals inside of the body.
- f.To avoid using the machine on people who has high blood
pressure or diabetes or serious skin diseases or heart diseases,
or who is sensitive to electric current,or who is reveiving
hormone treatment.

D.Our Cavitation Device superiority

D.1. Whole ABS Plastic Mould.

D.2. Variable frequencies depending on the chosen treatment.

D.3. Frequency of steady emission.

D.4. It does not present any contagious risk.

D.5. Maximum security for the client, default programs

D.6. Permits to adapt the used power used according to the zone and deepness to work.

We Are Looking for Sale agent over the world.





HKS880

Cavitation+RF
User&Training Manual

E.RF Operate the Device

E.1 Operate interface

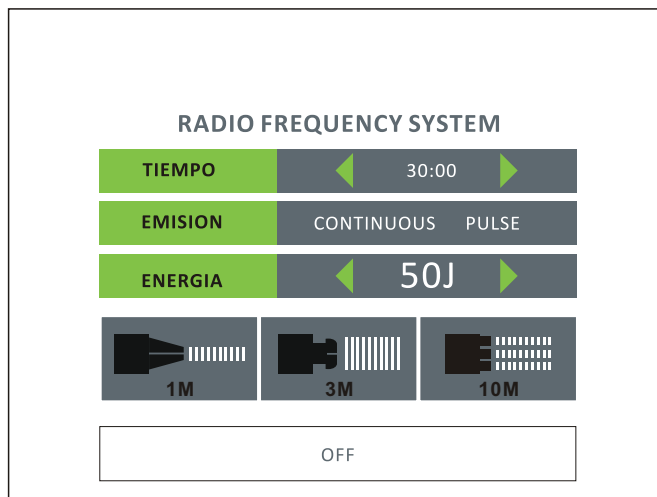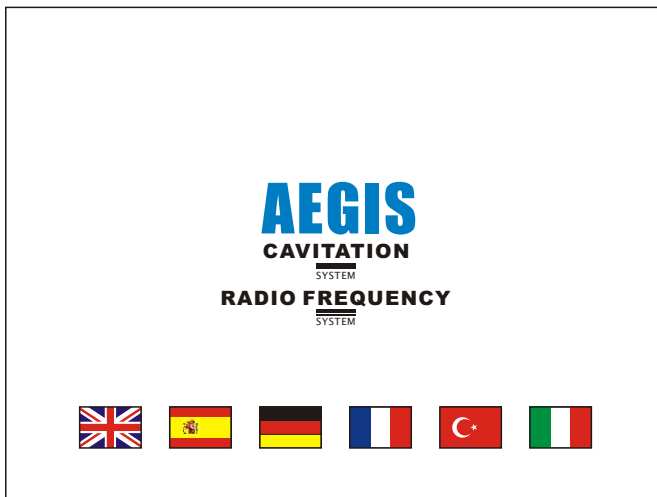

HKS880

Cavitation+RF
User&Training Manual


E.RF Operate the Device



Big Probe for body/neck

Tripolar For body/Jowl

Small Probe for Face

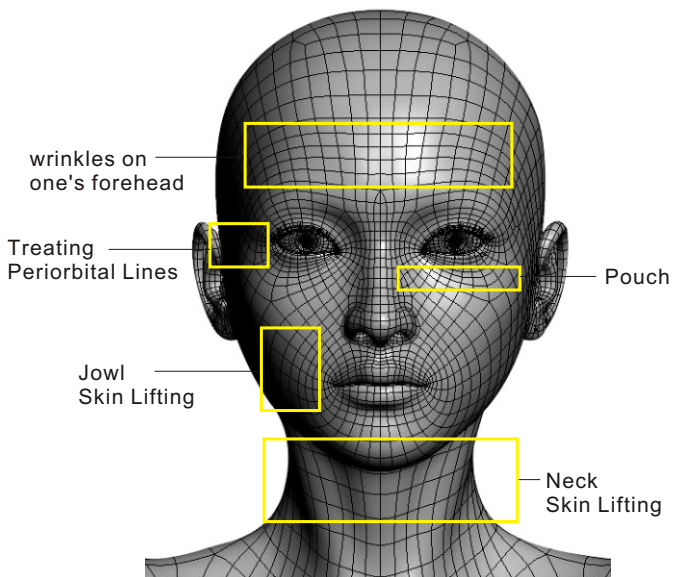

HKS880








Cavitation+RF

User&Training Manual

E.RF Operate the Device

E.3 Manual Model Treatment

Choice right Probe as you want and adjust the timer and energy,press OFF be ON,the Device start to work.
User need press the switch in handle.

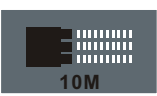	Adjust Button
	Timer Adjust,Max is 30Minutes
	Continuous or discontinuous
	Energy Adjust
	Choice small bipolar probe 1MHz
	Choice big bipolar probe 3MHz
	Choice tripolar probe 10MHz

HKS880

Cavitation+RF


User&Training Manual

F.RF Treatment Procedure

F.1 .Operateion technigue:Let the treatment probe touch the target area ad mover(Or Slide)from the inner to the outer and round and round and from the lower place to the higher, when you do the treatment ont he upper lid,Pls move a bit more quickly,when the gel on the face get drier,please add some GEL and go on with the treatment.

F.2.Face(Includeing cheek,eye,forhead,lip,neck)

Operation procedure(About 50minutes for whole face,15-20minutes for neck)

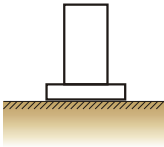

a. carry out a skin evaluation before treatment.

B. Clean skin

c. tone skin

d. massage with GEL

e. clean skin

f. smear gel on skin and apply treatment with trobe.

g. let the probe touch the skin and make treatment

Course of treatment:One course of treatments includes 5treatments at an interval of every 10-15days.

Curative effect

After one treatment,the heated subcutaneous fiber shrinks instantly,therefore fins wrinkles are improved,and skin becomes tender,sommth and compact.After onecourse of treatment:the collagen protein is regenerated for long term,skin becomes tender and smooth,wrikles are reduced, and skin is tightened and compact.

HKS880

Cavitation+RF

User&Training Manual

F.RF Treatment Procedure

F.3.Body(striae gravidarum)

Operation procedure(About 60-90minutes for topical area)

a. Carry out a skin evaluation before treatment.

B. clean skin

c. massage with body slimming distillate

d. Smear body slimming diatillate on skin and apply treatment with probe.


e. clean skin

f. let the probe touch the skin and make treatment.

Course of treatment:One course of treatments includes 5 treatments at a interval of every 10-15days.

Curative effectAfter one treatment,the heated subcutaneous fiber shrinks instantly,therefore skin become smooth,compact and elastic.

After one course of treatment;Skin becomes compact and flabby skin is improved.

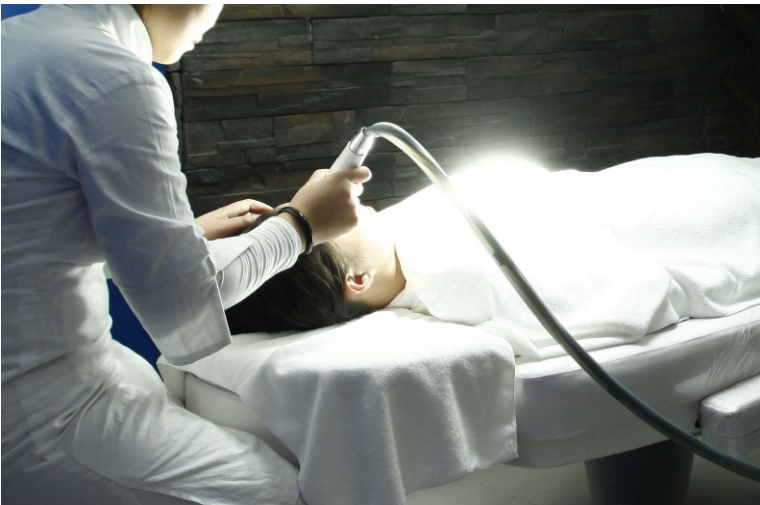

HKS880

Cavitation+RF

User&Training Manual

G. Treatment Caution

G.1. for treatment is that the patient should feel needling&hot.

G.2. The treatment head must be kept on the skin (sliding); otherwise, the skin will get burned. Before the treatment head leaves the skin, you must loose the button on the head or loose the foot switch.

G.3. When the temperature of the treatment head is over too high, you should stop treatment. The energy you adopt should not be too high; otherwise, the skin will get burned.

G.4. Please carry out skin moisturizing and sun proof cream work after treatments, do not wash faces with hot water (use water under the body temperature), and do not go for hot spring and massage.

H. Treatment Taboos

H.1. The abdomen of the pregnant women or women in menses

H.2. To avoid using the machine on around the parts under which metal, plastic or silicon wee buried.

H.3. To avoid using the machine on the epileptic and those in bad health

H.4. To avoid using the machine on people who has high blood pressure or diabetes or serious skin diseases or heart diseases, or who is sensitive to electric current, or who is reveiving hormone treatment.

HKS880

Cavitation+RF

User&Training Manual

I.RF Treatment Parameter

Therapy Position	Probe	Energy	Mode	Timer
eye side	small	15-20	■	10
	big	22-28	■	5
forehead	small	14-17	■	8
	big	22-28	■ ■ ■ ■	5
jowl	Tripolar	20-26	■	10
neck	big	22-28	■	15
Abdomen	big	22-28	■	10
buttocks	Tripolar	22-28	■ ■ ■ ■	10
Low-necked	Tripolar	18-24	■	10
arm	Tripolar	22-28	■ ■ ■ ■	10
leg	Tripolar	22-28	■ ■ ■ ■	10

J.HKS817CRF Device Superiority

- 7.1.Whole ABS Plastic Mould.
- 7.2.Variable frequencies depending on the chosen treatment.
- 7.3.Frequency of steady emission.
- 7.4.It does not present any contagious risk.
- 7.5.Maximum security for the client, default programs
- 7.6.Permits to adapt the used power used according to the zone and deepness to work.



We Are Looking for Sale agent over the world.