MINISAB 2T

- Instruction Manual
- Manuale di Istruzioni
- Gebrauchs Anweisungen
- Mode d'emploi
- Manual de manejo

CAUTION:

- **INSTALL THIS DEVICE IN A PROPER LOCATION**
- **TO BE USE BY AUTHORIZED AND INSTRUCTED PERSONNEL ONLY**

TISSIDental

Electromechanical Manufacturing Division

Dental, Jewellery and Laboratory Equipment Made in Italy

Tel. +39 02 5276 050 - Fax +39 02 5274 760 - www.tissidental.com

TISSIDental si riserva il diritto di modificare, senza preavviso, le caratteristiche indicate nel presente manuale
TISSIDental reserves the right to change the specifications of this equipment without notice
TISSIDental behält sich das Recht vor, jederzeit stillschweigend technische oder bauliche Änderungen vorzunehmen
TISSIDental se réserve le droit de modifier, sans préavis, les caractéristiques dans ce manuel
TISSIDental se reserva el derecho de modificar sin aviso previo las características incluidas en el presente manual de uso

Index

- Declaration of conformity
- Visual Instructions
- Exploded draw
- Spare parts list
- Electrical diagram

◆ Instruction Manual

1. Safety directives
2. Unpack
3. Installation
4. Maintenance
5. Guide to common questions
6. Specifications
7. Dimensions

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ESPAÑOL

TISSIDental

Electromechanical Manufacturing Division
20097 S.Donato – Milano – Italy

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IN BASE ALLE DIRETTIVE :
as for directives:

89/392/CEE
89/336/CEE
73/23/CEE
89/68/CEE

E SUCCESSIVE MODIFICHE
and subsequential modifies

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MODELLO: MINISAB 2T
Model

MATRICOLA: _____
Matriculation nr.

E' CONFORME ALLE NORME:
is made in conformity with normatives:

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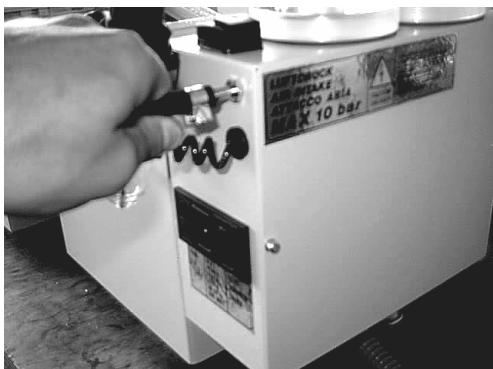
Visual Instructions

Please read instruction booklet carefully prior installation and use of this device

1

Connect air pressure hose (max 10 bars)

2

Insert dust aspirator hose on back connector

3

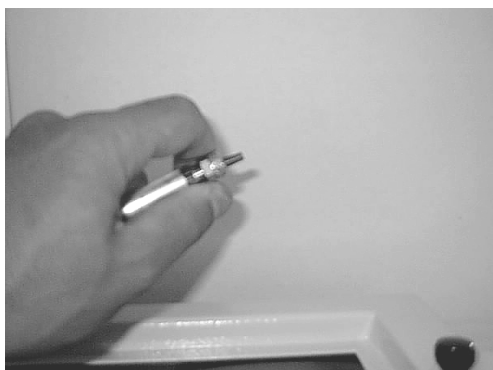
Connect aspirator plug in the sandblaster socket

4

Insert plug respecting voltage on back plate

5

Mount appropriate nozzles on ejector pen

6

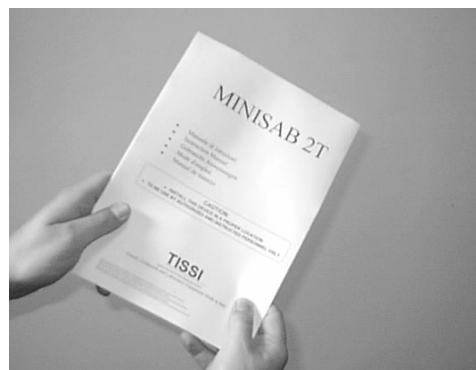
Fill containers with appropriate grain size of sand

7


Regulate air pressure (min 2 – max 6 bars)

8

Consult instruction booklet before operate

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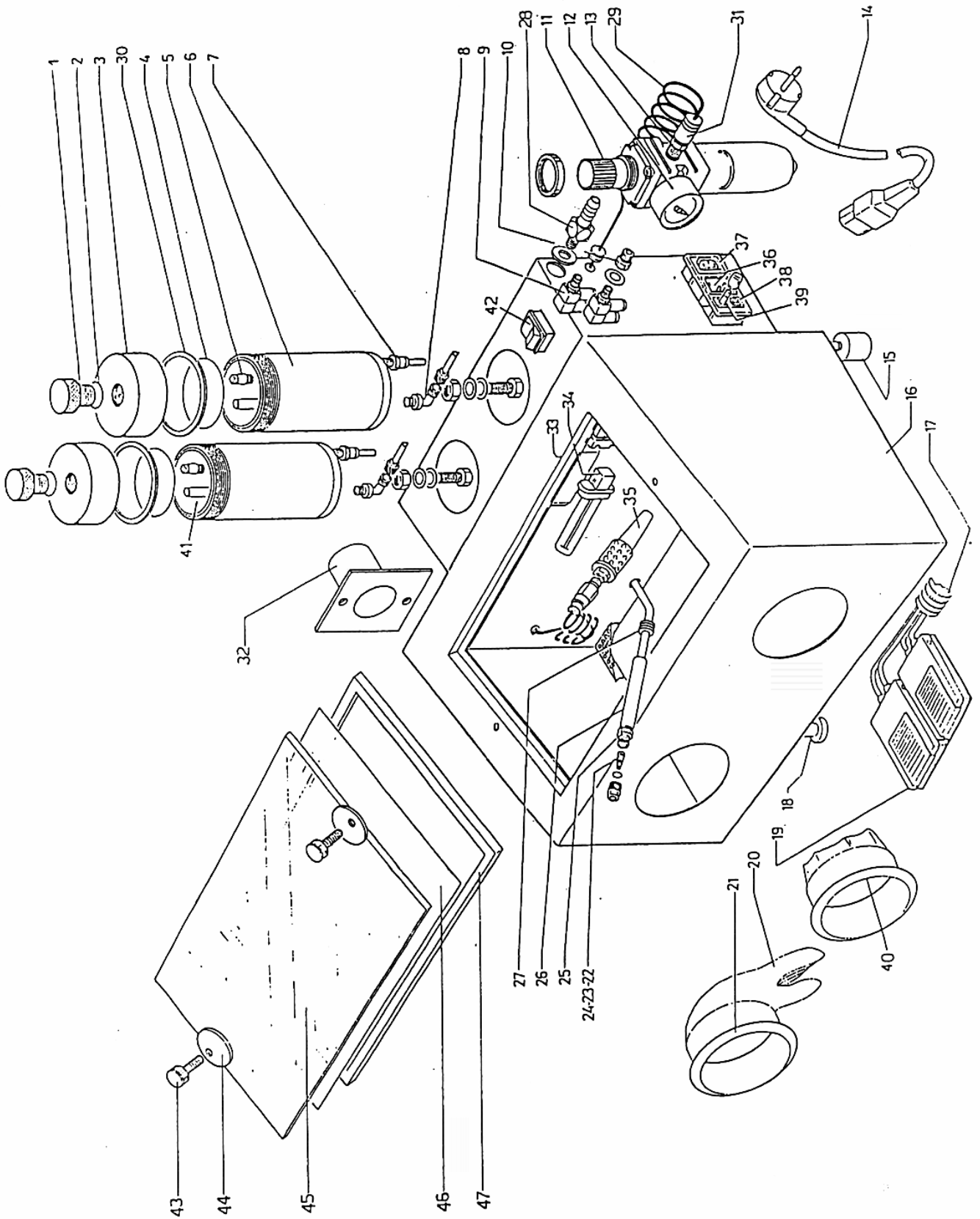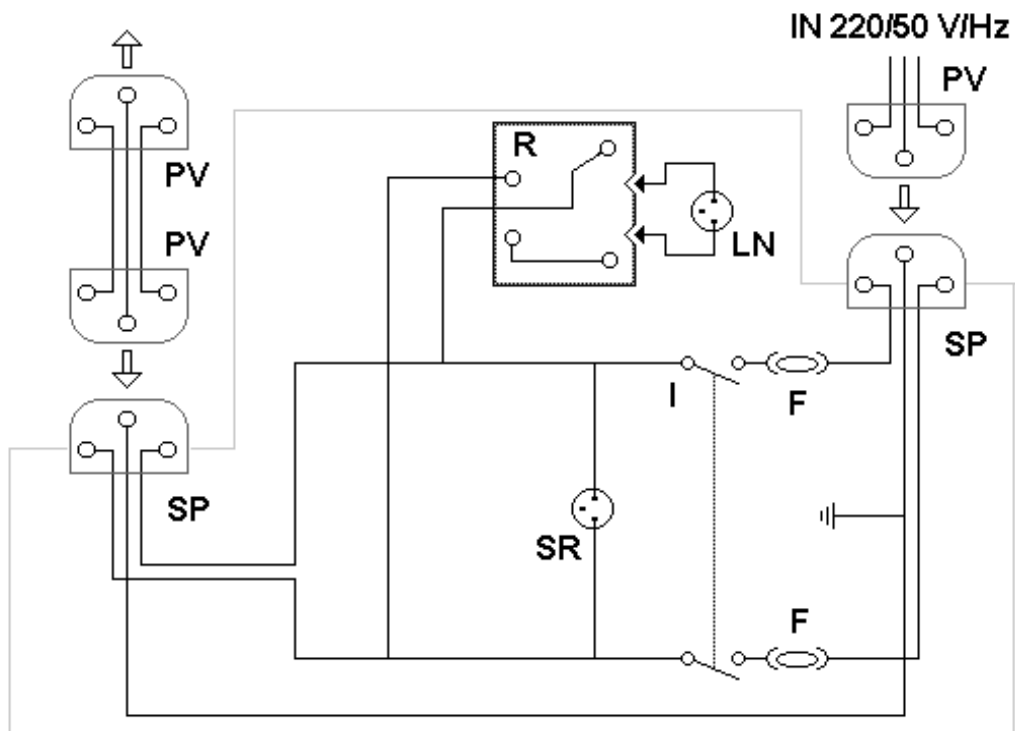
LN - NEON LAMP 9 W – LAMPADA NEON 9 W

CONGRATULATIONS FOR YOUR CHOICE!!

Choosing this apparatus you have purchased a particularly reliable unit that will facilitate your work.

This unit is supplied of the TUV quality brand, and complies with the directives 89/392/CE, therefore CE mark is admitted:

CE

IMPORTANT!

Before installing the apparatus read carefully this manual and keep it for future consultations.

WARNING

Do follow carefully all the indications brought back in this manual. During all the maintenance operations disconnect the power cord from the electrical outlet, and/or disconnect the compressed air and/or the water attack and/or other attacks of external parts to the apparatus. Do not introduce stranger objects inside the unit during the maintenance operations. Use only the accessories in equipment for the filling operations and/or maintenance. Do not execute different operations of maintenance from those brought back in the manual. Whichever operation not contemplated in this manual, can involve risks. For operations of this type, contact the technical service. Before turning on the unit and connecting the power cord to the electrical outlet, make sure that the voltage corresponds to the one brought back on the fabrication nameplate placed on the back part of the apparatus.

CAUTION:

- **Do not make any maintenance operation when the power cord is connected: FULGURATION DANGER!!!**
- **Use only genuine spare parts. Placing not genuine spare parts can void your warranty, moreover can make the apparatus not in compliance with the norms, and dangerous when in use.**
- **For whichever inherent information concerning installation, maintenance and use of the unit, contact the customers' service TISSIDental:
Tel +39 (02) 527 6050 – Fax +39 (02) 527 4760 – Internet: www.tissidental.com**

UNPACK

Verify integrity of the pack at the moment of the reception of the apparatus. Verify that in the packaging all the parts and the supplied accessories of series are contained together with the apparatus: power cords, neon lamp, set of nozzles (3 pcs.) funnel loading sands, technical documentation and warranty.

INSTALLATION AND USE

Take the apparatus out of the packaging, and place it on a flat surface strong enough to bear its weight. Do not place the apparatus close to humidity and/or vapor sources (steam generators, stoves, windows, etc.). Insert the neon lamp inside of cabin in the appropriate lamp holder (rif. 33 - 34) making a light pressure. Connect the tube of the compressed air to the connection placed on one side of the unit - MAX 8 BARS (rif. 28). When using our suction unit, connect the hose of AIRBOX GS in the appropriate connection placed on the back side of the unit. Then connect the power cord plug of AIRBOX GS to the outlet on the right side of the microsandblaster (rif. 37). Now connect the power cord of the microsandblaster to the electrical outlet in your laboratory.

ATTENTION: make sure that the voltage corresponds to the one brought back on the fabrication name plate placed on the apparatus. Turning on the main switch of the microsandblaster (rif. 42) will activate all the system. Regulate the aspiration power on the minimum acting on the power regulator placed on the suction unit (rif. 29)

Use of the nozzles:

Oxides from 25 up to 50 My = nozzle 0.8 millimeter - Small (rif. 22)

Oxides from 50 up to 120 My = nozzle 1.2 millimeter - Medium (rif. 23)

Oxides from 120 up to 250 My = nozzle 1.5 millimeter - Large (rif. 24)

Container with WHITE cap for aluminum oxides from 25 up to 120 My.

BLUE cap container for aluminum oxides from 120 up to 250 My.

LOADING THE CONTAINERS

Unscrew the aluminium stopper of the container for loading (rif. 1) and pour the prechosen abrasive using the funnel in equipment.

ATTENTION: do not fill over the level reference nut placed on the brass tube inside the container (rif. 41). Rescrew the aluminium stopper of the container, and make sure the threaded surface of the transparent stopper in plexiglass is perfectly clean from eventual residual of sand: it is advised to clean up after the loading the thread (rif. 3) with a normal small tooth brush. Set up the pressure rotating the black pressure regulator, following symbols +/- (rif. 11), depending on the type of the prechosen abrasive (2/3 bars). Select the flow of abrasives acting on the on the pneumatic pedal of corresponding colour (rif. 19).

MAINTENANCE

Substitution of the glass protection films: remove the screws (rif. 43) and make rotate the two disks (rif. the 44); push from the inside of the cabin the glass (rif. 45) upwards, and replace the sheet of protection (cod. 3109046 - confection of 5 pieces) with a new one, fixing it long the 4 sides with normal transparent adhesive tape, in a perfect contact position with the glass. Then place the glass in the center making all the operations to the contrary.

Substitution of gloves and sleeves: extract them pulling towards the outside the blue rubber holder (rif. 21); replace the glove and the sleeve (rif. 20) paying attention that the border of the glove is accurately inserted in the rabbet of the blue rubber holder. Place the holder again making a light pressure.

Substitution of the nozzles: unscrew the nut of the nozzle to replace it, controlling that it is of proper size in relation of the adapted sand in use, and fit it in the appropriate site (rif. 25).

Cleaning of the cabin: empty periodically the cabin of powders, the rubber holders and the black rubber stopper (rif. 18) placed under the base. Tilt the cabin in ahead, helping the outflow of the sand with the hand towards the hole.

ATTENTION: THE USED ABRASIVE CAN NOT BE RE-USED.

GUIDE TO COMMON QUESTIONS

The apparatus is not turning on:

1. Control that the power cord plug (rif. 14) is properly connected to the electrical outlet.
2. Check that fuses (rif. 39) are not damaged.

The sand does not exit:

1. Control that there is air pressure at the air connection of apparatus (rif. 28).
2. Control that the sand inside the container is of appropriate size, and that it is not humid; if humid remove the sand with a vacuum cleaner using the funnel in equipment as top of the vacuum cleaner.
3. Control that the used nozzle is of diameter correspondent to the type of sand. Little rubbles may interrupt the flow of the sand. In this case plug with a finger the hole of the nozzle and press the pedal creating pressure inside the container, therefore remove the finger. Repeat the operation several times until the release of the culverts.

Carried out these controls, in case the apparatus still presents disadvantages, contact the customers' technical service.

MAINTENANCE

Substitution of the used filter of powders:

Control the bag of used powders every 4/5 days and replace it before completely full (max to approximately 2/3 of its capacity). Do not re-use and do not empty out the used filters. When the filter is exhausted close it with a proper adhesive tape and waste appropriately in compliance with your country rules.

Substitution of the motor protection filter:

Check every two weeks the motor protection filter and clean it by blowing compressed air in the same way of passage of the sucked air. The filter must be replaced with a new one at least every 4 months. This preserves the motor and the perfect operating of the apparatus.

GUIDE TO COMMON QUESTIONS

The apparatus is not turning on:

1. Control that the power cord plug (rif. 13) is properly connected to the electrical outlet.

The suction power is low:

1. Replace the recovery dust filter (rif.10), and/or replace the motor protection filter (rif.7-8-9).
- Carried out these controls, in case the apparatus still presents disadvantages, contact the customers' technical service.

SPECIFICATIONS

Lamp: 9 neons Watt 41 Lumen

Fuses: 2 from 5 x 20 millimeter from 6.3 Ampere

Microsandblaster absorption 9 Watt; suction unit AIRBOX GS max 800 Watt.

Operating voltage V/Hz 230/50-60 or 110/50 (see nameplate on the apparatus)

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