

Invacare[®] TDX[®] SP



en Power Wheelchair Base User Manual

This manual **MUST** be given to the user of the product.
BEFORE using this product, read this manual and save for future reference.



Yes, you can.[®]

© 2016 Invacare® Corporation

All rights reserved. Republication, duplication or modification in whole or in part is prohibited without prior written permission from Invacare. Trademarks are identified by ™ and ®. All trademarks are owned by or licensed to Invacare Corporation or its subsidiaries unless otherwise noted.

Making Life's Experiences Possible is a registered trademark in the U.S.A.

Contents

I General	6
1.1 Symbols	6
1.2 Reference Documents	6
1.3 Restriction of Sale	6
1.4 Intended Use.	6
1.5 Indications For Use	6
1.6 General Guidelines	7
1.6.1 Set Up	10
1.6.2 Stairways and Escalators	11
1.6.3 Repair and Service Information — Dealers and/or Qualified Technicians.	12
2 Safety and Handling	14
2.1 Safety and Handling	14
2.2 A Note to Wheelchair Assistants	16
2.3 Stability and Balance.	16
2.3.1 Driving Surfaces	18
2.3.2 Coping with Everyday Obstacles	18
2.3.3 Negotiating Inclines.	19
2.3.4 Reaching, Leaning and Bending - Forward	20
2.3.5 Reaching, Leaning and Bending - Backward	21
2.3.6 Transferring To and From Other Seats	21
2.4 Pinch Points.	22
2.5 Footplates and Front Rigging.	22
2.6 Storage	23
2.7 Electrical	23
2.8 Recycling Batteries.	24
2.9 Weight Training.	24

3 Electromagnetic Compatibility (EMC) Information	25
3.1 Electromagnetic Interference (EMI) From Radio Wave Sources	25
3.2 Powered Wheelchair Electromagnetic Interference (EMI)	25
3.3 Powered Wheelchair Electromagnetic Emissions	26
4 Label Locations	27
4.1 All Wheelchairs	27
4.2 Wheelchairs with TRRO	29
5 Technical data	31
5.1 Specifications.	31
5.1.1 Models.	31
5.1.2 Overall Dimensions	31
5.1.3 Seat	31
5.1.4 Wheels	31
5.1.5 Axle.	32
5.1.6 Battery.	32
5.1.7 Footrests	32
5.1.8 Weight	33
5.1.9 Weight Capacity Warnings	33
5.1.10 Weight Capacity Values.	33
6 Wheelchair Operation	34
6.1 Preparing the Joystick for Use.	34
6.2 Turning the Power On/Off.	35
6.3 Using the Joystick to Drive the Wheelchair	35
6.4 SPJ+™, MK6i™ SPJ+ w/PSS and MK6i SPJ+ w/ACC Joystick Switches and Indicators	36
6.4.1 On/Off Button	37
6.4.2 Speedometer	37
6.4.3 Speed Control Buttons	37
6.4.4 Joystick	37
6.4.5 Charger/Programming Input	38

6.4.6	Information Gauge Display	38	7.8	Securing the Wheelchair	53
6.4.7	Service Indicator	38	7.9	Securing the Occupant	53
6.4.8	Mode Button	38	7.9.1	Wheelchair-Anchored Belts	53
6.5	CMPJ+ Joystick, Switches and Indicators	38	7.9.2	Vehicle-Anchored Belts	54
6.5.1	On/Off - Drive Select Toggle Switch (CMPJ+ Joystick)	38	7.9.3	Seating System	55
6.5.2	Speed Control	39	7.9.4	Positioning Belts	55
6.5.3	Joystick	39	8 Setup and Maintenance	57	
6.5.4	Charger/Programming Input	39	8.1	Setup/Delivery Inspection	57
6.5.5	LCD Display Screens	39	8.2	Service Life	57
6.5.6	User Settings	41	8.3	Wear and Tear Information	57
6.5.7	CMPJ+ Joystick - Programmable Mono Ports 1 and 2 with External Mode Switch	43	8.4	User/Attendant Inspection Checklists	58
6.5.8	Remote On/Off Switch	44	8.4.1	Inspect/Adjust Weekly	58
6.5.9	Mode Switch (CMPJ+ Joystick)	44	8.4.2	Inspect/Adjust Monthly	58
6.5.10	Memory Card Slot	44	8.4.3	Inspect/Adjust Periodically	58
6.6	When to Charge Batteries	44	8.5	Service Inspection	59
6.6.1	SPJ+, SPJ+ w/PSS and SPJ+ w/ACC Joysticks	44	8.5.1	Six Month Inspection	59
6.6.2	CMPJ+ Joystick	45	8.5.2	Inspect/Adjust Every 18 Months	60
6.7	Charging Batteries	45	8.5.3	Inspect/Adjust Every 2 Years	60
6.7.1	Description and Use of Battery Chargers	46	8.6	Cleaning	60
6.8	Disengaging/Engaging the Motor Lock Levers	47	8.7	Stability Lock	61
6.9	Disengaging/Engaging the Wheel locks	47	8.8	Removing/Installing the Shrouds	61
6.9.1	Engaging	48	8.8.1	Removing the Rear Shroud	61
6.9.2	Disengaging	48	8.8.2	Installing the Rear Shroud	61
7 Transport in Vehicles	49		8.8.3	Removing/Installing the Front Shroud/Battery Retention Bracket	62
7.1	About Transport Ready Packages	49	8.9	Batteries	62
7.2	Wheelchair Transport Brackets (TRBKTS)	50	8.10	Using the Proper Batteries	64
7.3	Transport Ready Option (TRRO)	50	8.11	Removing/Installing the Batteries From/Into the Wheelchair	65
7.4	Compliance Information	51	8.11.1	Removing the Batteries from the Wheelchair	65
7.5	Specifications	52	8.11.2	Installing Batteries into the Wheelchair	66
7.6	Positioning the Wheelchair in the Vehicle	52	8.12	Replacing Batteries and/or Battery Cables	67
7.7	Securement Points	52	8.12.1	Cleaning Battery Terminals	69

8.13	Adjusting Forks	69
8.14	Removing/Installing the Footboard or Adjusting the Footboard Height	70
8.15	Adjusting the Footboard Angle	71
8.16	Repositioning Joystick.	72
8.17	Disconnecting/Connecting the SPJ+ Joysticks	72
8.17.1	Disconnecting the SPJ+ Joysticks	72
8.17.2	Connecting the SPJ+ Joysticks	72
8.18	Disconnecting/Connecting the CMPJ+ Joysticks	73
8.18.1	Connecting the CMPJ+ Joysticks	73
8.18.2	Disconnecting the CMPJ+ Joysticks	73
9	Troubleshooting	74
9.1	Driving Performance	74
9.2	Electrical	74
9.2.1	SPJ+, SPJ+ w/PSS or SPJ+ w/ACC Joysticks	74
9.2.2	CMPJ+ Joysticks or Displays	76
9.3	Checking Battery Charge Level.	79
10	Warranty	80
10.1	Global Limited Warranty (Excluding Canada)	80
10.2	Canada Limited Warranty.	81
10.3	Usability Survey	83

I General

I.1 Symbols

Signal symbols and/or words are used in this manual and apply to hazards or unsafe practices which could result in personal injury or property damage. See the information below for definitions of the signal words.



DANGER!

– Danger indicates a imminently hazardous situation which, if not avoided, could result in death or serious injury.



WARNING!

– Warning indicates a potentially hazardous situation which, if not avoided, could result in death or serious injury.



CAUTION!

– Caution indicates a potentially hazardous situation which, if not avoided, may result in property damage or minor injury or both.



IMPORTANT

– Indicates a hazardous situation that could result in damage to property if it is not avoided.



Gives useful tips, recommendations and information for efficient, trouble-free use.

I.2 Reference Documents



Refer to the table below for part numbers of additional documents which are referenced in this manual.

MANUAL	PART NUMBER
MK6i™ Electronics Programming Guide	1141471
Adjustable ASBA Owner's Manual	1143192
Formula™ CG Seating System	1143155
Adjustable ASBA Service Manual	1143238

I.3 Restriction of Sale

Federal law restricts this device to sale by/on the order of a physician licensed by the law of the state in which he/she practices.

I.4 Intended Use

The intended use of the Invacare® TDX® SP Power Wheelchair is to provide mobility to persons limited to a sitting position, that have the capability of operating a powered wheelchair.

I.5 Indications For Use

The intended use of the Invacare® TDX® SP Power Wheelchair is to provide mobility to persons limited to a sitting position.

I.6 General Guidelines

The safety section contains important information for the safe operation and use of this product.



WARNING!

Risk of Death, Injury or Damage

Improper use of this product may cause injury or damage.

- If you are unable to understand the warnings, cautions or instructions, contact a health care professional or dealer before attempting to use this equipment.
- DO NOT use this product or any available optional equipment without first completely reading and understanding these instructions and any additional instructional material such as user manual, service manuals or instruction sheets supplied with this product or optional equipment.

Continued use of the wheelchair with damaged parts could lead to the wheelchair malfunctioning, causing injury to the user and/or caregiver.

- Check all wheelchair components and carton for damage and test components before use. In case of damage or if the wheelchair is not working properly, contact a qualified technician or Invacare for repair.



WARNING!

Risk of Serious Injury or Damage

Use of non-Invacare accessories may result in serious injury or damage.

- Invacare products are specifically designed and manufactured for use in conjunction with Invacare accessories. Accessories designed by other manufacturers have not been tested by Invacare and are not recommended for use with Invacare products.
- DO NOT use non-Invacare accessories.
- To obtain Invacare accessories, contact Invacare by phone or at www.invacare.com



DANGER!

Risk of Death, Serious Injury, or Damage

Use of incorrect or improper replacement (service) parts may cause death, serious injury, or damage.

- Replacement parts MUST match original Invacare parts.
- ALWAYS provide the wheelchair serial number to assist in ordering the correct replacement parts.



WARNING!

Risk of Serious Injury or Damage

Attaching hardware that is loosely secured could cause loss of stability resulting in serious injury or damage.

- After ANY adjustments, repair or service and before use, make sure that all attaching hardware is tightened securely.



WARNING!

Risk of Minor to Serious Injury

Pinch points can cause minor to serious injury.

- Be mindful of potential pinch points and use caution when using this product.



WARNING!

Risk of Serious Injury or Damage

Loss of power due to loose electrical connections could cause the wheelchair to suddenly stop resulting in serious injury or damage.

- ALWAYS ensure that all electrical connections are tightly connected so they don't vibrate loose.



DANGER!

Risk of Death, Serious Injury, or Damage

Lighted cigarettes dropped onto an upholstered seating system can cause a fire resulting in death, serious injury, or damage.

Wheelchair occupants are at particular risk of death or serious injury from these fires and resulting fumes because they may not have the ability to move away from the wheelchair.

- DO NOT smoke while using this wheelchair.



WARNING!

Risk of Injury, Damage or Death

Improper monitoring or maintenance may cause injury, damage or death due to ingestion or choking on parts or materials.

- Closely supervise children, pets, or people with physical/mental disabilities.



WARNING!

Risk of Injury or Damage

Improper mounting or maintenance of the Sip n' Puff control including the mouthpiece and breath tube may cause injury or damage.

Water inside the Sip n' Puff interface module may cause damage to the unit.

Excessive saliva residue in the mouthpiece/straw can reduce performance.

Blockages, a clogged saliva trap or air leaks in the system may cause Sip N' Puff not to function properly.

- Ensure moving parts of the wheelchair, including the operation of powered seating, DO NOT pinch or damage the Sip n' Puff tubing.
- Saliva trap MUST be installed to reduce risk of water or saliva entering the Sip n' Puff interface module.
- Occasionally flush the mouthpiece to remove saliva residue.
- The mouthpiece/straw MUST be completely dry before installation.
- If Sip n' Puff does not function properly, inspect system for blockages, clogged saliva trap or air leaks. As necessary, replace mouthpiece, breath tube and saliva trap.



Contact your Invacare dealer/provider for more information about maintaining and troubleshooting the Sip n' Puff system.

**WARNING!****Risk of Injury, Damage or Death**

- Exposure to liquids may cause injury, damage or death.
- DO NOT expose electrical connections to sources of liquid or dampness. This includes, but is not limited to, water, body fluids or cleaning agents.
 - DO NOT expose battery charger or other accessories to sources of liquid or dampness.
 - Wheelchairs that are used by incontinent users and/or are frequently exposed to water/liquids may require replacement and inspection of electrical components more frequently than normal schedule dictates.
 - Electrical components damaged by corrosion MUST be replaced immediately.

**CAUTION!****Risk of Damage**

- Operating the wheelchair in rain or dampness may cause the wheelchair to malfunction electrically and mechanically; may cause the wheelchair to prematurely rust or may damage the upholstery.
- DO NOT leave wheelchair in a rain storm of any kind.
 - DO NOT use wheelchair in a shower.
 - DO NOT leave wheelchair in a damp area for any length of time.
 - Check to ensure that the battery covers are secured in place, joystick boot is NOT torn or cracked where water can enter and that all electrical connections are secure at all times. DO NOT use if the joystick boot is torn or cracked. If the joystick boot becomes torn or cracked, replace IMMEDIATELY.



Leakage current has been tested in accordance to ISO 7176-14:2008.



Invacare has tested its power wheelchairs in accordance with RESNA Section 9 “Rain Test”.



THE INFORMATION CONTAINED IN THIS DOCUMENT IS SUBJECT TO CHANGE WITHOUT NOTICE.



As a manufacturer of wheelchairs, Invacare endeavors to supply a wide variety of wheelchairs to meet many needs of the end user. However, final selection of the type of wheelchair to be used by an individual rests solely with the user and his/her healthcare professional capable of making such a selection. Invacare recommends working with a qualified rehab technology provider, such as an ATP, (Assistive Technology Professional).

1.6.1 Set Up



DANGER!

Risk of Death, Serious Injury, or Damage

Continued use of the wheelchair that is not set to the correct specifications may cause erratic behavior of the wheelchair resulting in death, serious injury, or damage.

- Performance adjustments should only be made by professionals of the healthcare field or persons fully conversant with this process and the driver's capabilities.
- After the wheelchair has been set up/adjusted, check to make sure that the wheelchair performs to the specifications entered during the set up procedure. If the wheelchair does not perform to specifications, turn the wheelchair Off immediately and reenter set up specifications. Contact Invacare, if wheelchair still does not perform to correct specifications.



WARNING!

Risk of Serious Injury or Damage

Moving the seating system from the factory setting may reduce driver control, wheelchair stability, traction and increase caster wear resulting in serious injury or damage.

- Move the seating system ONLY when necessary to fit the wheelchair to the user.
- If the seating system must be moved, ALWAYS inspect the wheelchair to ensure the front rigging DOES NOT interfere with the front casters.
- If the seating system must be moved, ALWAYS inspect to ensure the wheelchair DOES NOT easily tip forward or backward.



WARNING!

Risk of Injury, Damage or Death

Improper routing of cord(s) may cause tripping, entanglement or strangulation hazard that may result in injury, damage or death.

- Ensure all cord(s) are routed and secured properly.
- Ensure there are no loops of excess cable extending away from the chair.
- Close supervision and attention is needed when operating the wheelchair near children, pets or people with physical/mental disabilities.



WARNING!

Risk of Injury, Damage or Death

Pinched or severed cord(s) may be a shock or fire hazard and may cause injury, damage or death.

- Ensure all cord(s) are routed and secured properly.
- Inspect cord(s) periodically for proper routing, pinching, chafing or other similar wear.
- Replace any damaged cords immediately.



WARNING!

Risk of Serious Injury

Sharp edges can cause serious injury.

- Be mindful that some parts may have sharp edges. Use caution when encountering these sharp edges.



WARNING!

Risk of Serious Injury

Hot surfaces can cause severe burns.

- Be mindful of potential hot surfaces and avoid touching.

**WARNING!**

- DO NOT connect any medical devices such as ventilators, life support machines, etc. directly to the batteries used to power the wheelchair. This could cause unexpected failure of the device and the wheelchair.

**WARNING!****Risk of Injury or Damage**

Grease may leak from the gearbox and cause injury or damage from falling or fire.

- Follow the maintenance instructions in this manual.
- Routinely inspect the gearbox for grease leakage.

If the gearbox is leaking grease:

- Stop use immediately.
- Keep the product away from sparks, flame, and open heat sources.
- Contact your dealer or Invacare for repair.

**WARNING!****Risk of Death, Serious Injury, or Damage**

Missing attaching hardware could cause instability resulting in death, serious injury or damage.

- Ensure all attaching hardware is present and tightened securely.

I.6.2 Stairways and Escalators**WARNING!****Risk of Injury, Damage or Death**

Improper use on stairways and escalators may cause injury, damage or death.

- DO NOT attempt to move an occupied wheelchair between floors using a stairway or escalator.
- DO NOT attempt to move an empty power wheelchair between floors using an escalator.
- Observe and follow all safety warnings.
- Use instructions provided for safe transportation between floors.

**WARNING!****Risk of Injury or Damage**

Improper lifting technique may cause injury or damage.

- Use proper lifting techniques, assistance and gear such as straps when available when lifting heavy loads.



When using a stairway to move the wheelchair and any accessories, move all wheelchair components away from the stairway prior to reassembly.

- Use an elevator or other accepted safe means of transportation between floors.
- Extreme caution is advised when it is necessary to move an unoccupied power wheelchair up or down the stairs or escalator.

- Follow this procedure for moving the wheelchair between floors when an elevator is NOT available:
 1. If stairway or escalator is only means of access, remove occupant of wheelchair and transport user and wheelchair separately.
 2. Invacare recommends using two assistants and making thorough preparations prior to transportation.
 3. Ensure that only secure, non-detachable parts are used for hand-hold supports.
 4. Use safe lifting techniques.
 5. Remove the batteries from wheelchair. Refer to 8.11.1 Removing the Batteries from the Wheelchair, page 65.
 6. Using non-removable (non-detachable) parts of the wheelchair, lift the wheelchair off of the ground and transfer the wheelchair up or down the stairs.
 7. The wheelchair should not be lowered until the last stair has been negotiated and the wheelchair has been carried away from the stairway.

I.6.3 Repair and Service Information — Dealers and/or Qualified Technicians



WARNING!

Risk of Injury, Damage or Death

Improper setup, service, adjustment or programming may cause injury, damage or death.

- Qualified technician **MUST** setup, service and program the wheelchair.
- **DO NOT** allow non-qualified individuals to perform any work or adjustments on the wheelchair.
- **DO NOT** setup or service the wheelchair while occupied except for programming or unless otherwise noted.
- Turn off power **BEFORE** adjusting or servicing the wheelchair. Note that some safety features will be disabled.
- Ensure all hardware is securely tightened after setup, service or adjustments.
- Warranty is void if non-qualified individuals perform any work on this product.

**DANGER!****Risk of Death, Serious Injury, or Damage**

Corroded electrical components due to water and/or liquid exposure, or incontinent users can result in death, serious injury, or damage.

- Minimize exposure of electrical components to water and/or liquids. Electrical components damaged by corrosion **MUST** be replaced immediately.
- Wheelchairs that are used by incontinent users and/or are frequently exposed to water/liquids may require replacement of electrical components more frequently.

**WARNING!**

- **DO NOT** overtighten hardware attaching to the frame. This could cause damage to the frame tubing.
- Replace gas-locking cylinders every two years or if performance issues are encountered. Performance issues include forward tipping and veering.

**WARNING!****Risk of Injury, Damage or Death**

Improper installation or service may result in injury, damage or death.

- Transport ready packages are not retrofittable to existing models and are not field serviceable.
- **DO NOT** overtighten hardware.

2 Safety and Handling

2.1 Safety and Handling

Safety and handling of the wheelchair requires the close attention of the wheelchair user as well as the assistant. This manual points out the most common procedures and techniques involved in the safe operation and maintenance of the wheelchair. It is important to practice and master these safe techniques until you are comfortable in maneuvering around the frequently encountered architectural barriers.

Use this information only as a “basic” guide. The techniques that are discussed on the following pages have been used successfully by many.

Users and assistants must be aware that the handling and maneuverability characteristics of the wheelchair are inherently different based on the drive wheel location. Handling and maneuverability differences will be most noticeable when traveling down declines (Example: ramps and slopes) or over obstacles and rough terrain as this may shift the users center of mass forward resulting in decreased stability. **ALWAYS** reduce speed and wear the seat positioning strap when driving under these conditions.

Individual wheelchair users often develop skills to deal with daily living activities that may differ from those described in this manual. Invacare recognizes and encourages each individual to try what works best for him/her in overcoming architectural obstacles that they may encounter. However all warnings and cautions given in this manual **MUST** be followed. Techniques in this manual are a starting point for the new wheelchair user and assistant with “safety” as the most important consideration for all.

Invacare strongly recommends that initial use of front wheel drive wheelchairs be supervised by an assistant.



DANGER!

Risk of Death, Serious Injury, or Damage

Misuse of the wheelchair may cause component failure and/or the wheelchair to start smoking, sparking, or burning. Death, serious injury, or damage may occur due to fire.

- **DO NOT** use the wheelchair other than its intended purpose. If the wheelchair starts smoking, sparking, or burning, discontinue using the wheelchair and seek service **IMMEDIATELY**.



DANGER!

Risk of Death or Serious Injury

Not wearing your seat positioning strap could result in death or serious injury.

- **ALWAYS** wear your seat positioning strap. Your seat positioning strap helps reduce the possibility of a fall from the wheelchair. The seat positioning strap is a positioning belt only. It is not designed for use as a safety device withstanding high stress loads such as auto or aircraft safety belts. If signs of wear appear, seat positioning strap **MUST** be replaced **IMMEDIATELY**.

**WARNING!****Risk of Injury, Damage or Death**

Use of the wheelchair while judgement or ability is impaired may result in injury, damage or death.

- DO NOT operate the wheelchair under the influence of alcohol, medications or other substances that impair judgement or function.
- Changing medications may affect your ability to operate the wheelchair. Discuss the impact on your ability to operate the wheelchair with a health care professional when changing medications.
- DO NOT operate the wheelchair under conditions where judgement or function may be impaired. This may include but is not limited to lack of sleep or poor sight.
- Always be aware of your surroundings.

**WARNING!****Risk of Injury, Damage or Death**

Misuse of wheelchair may result in injury, damage or death.

- DO NOT operate wheelchair on roads, streets or other roadways.
- DO NOT operate wheelchair when vision is impaired by poor lighting such as unlit rooms, during the night or similar situations.
- ALWAYS be aware of motor vehicles and your surroundings.
- DO NOT operate the wheelchair under the influence of alcohol, medications or other substances that impair judgement or function.

**WARNING!****Risk of Injury, Damage or Death**

Conditions such as restlessness, mental deterioration, dementia, seizure disorders (uncontrolled body movement) or sleeping problems may cause injury, damage or death.

- Monitor patients with these conditions frequently.
- Close supervision and attention is needed when operating the wheelchair near children, pets or people with physical/mental disabilities.

**WARNING!****Risk of Injury or Damage**

To avoid injury or damage from moving parts:

- ALWAYS keep hands and fingers clear of moving parts.
- Closely supervise children, pets, or people with physical/mental disabilities.

**WARNING!****Risk of Injury or Damage**

Improper operation may change the normal balance, center of gravity or weight distribution of the wheelchair causing injury or damage.

- Determine and establish your particular safety limits. Practice bending, reaching and transferring activities in several combinations in the presence of a qualified healthcare professional before attempting active use of the wheelchair.
- ALWAYS shift your weight in the direction you are turning. Shifting your weight in the opposite direction of the turn may cause the inside drive wheel to lose traction.

**WARNING!****Risk of Serious Injury**

Impacting objects in the surrounding environment can cause serious injury.

- When maneuvering the wheelchair around, ALWAYS have assured cleared distance with all objects in environment.

**WARNING!**

– Always check grips for looseness before using the wheelchair. If loose, contact a qualified technician for instructions.

**WARNING!****Risk of Injury or Damage**

Unintended movement or operation of wheelchair may cause injury or damage.

- Turn power OFF BEFORE entering or exiting the wheelchair.
- Close supervision and attention is needed when operating the wheelchair near children pets or people with physical/mental disabilities. Turn power off.
- Turn power off when near children, pets or people with physical/mental disabilities.

**WARNING!****Risk of Injury, Damage or Death**

Damaged parts due to collision or impact may result in injury, damage or death.

- Seek immediate attention and service if wheelchair is involved in a collision or impact event. This includes, but is not limited to, vehicle accidents, mishandling and impact events where the wheelchair strikes something or is struck by something that may cause damage.
- Ensure your wheelchair is working properly and is inspected by a qualified Invacare technician if the wheelchair is involved in a collision or impact event.

2.2 A Note to Wheelchair Assistants

When assistance to the wheelchair user is required, remember to use good body mechanics. Keep your back straight and bend your knees whenever tilting wheelchair or traversing curbs or other impediments.

Ensure that only secure, non-detachable parts are used for hand-hold supports.

When learning a new assistance technique, have an experienced assistant help you before attempting it alone.

2.3 Stability and Balance

To assure stability and proper operation of your wheelchair, you **MUST** at all times maintain proper balance. Your wheelchair has been designed to remain upright and stable during normal daily activities as long as you **DO NOT** move beyond the center of gravity. **DO NOT** lean forward out of the wheelchair any further than the length of the armrests.

The drive behavior initially experienced by the user may be different from other wheelchairs previously used. This power wheelchair has Invacare's SureStep® technology, a feature that provides the wheelchair with optimum traction and stability when driving forward over transitions and thresholds. Refer to 2.3.2 Coping with Everyday Obstacles, page 18 for maximum height of transitions and thresholds. The following warnings apply specifically to the SureStep feature:



WARNING!

Risk of Injury, Damage or Death

Improper use of wheelchair may cause instability and may result in injury, damage or death. The stability of the wheelchair is adversely affected by additional weight that shifts the center of gravity.

- This wheelchair has been designed to accommodate one individual. DO NOT operate with additional person(s).
- DO NOT carry heavy objects on your lap while operating the wheelchair.



WARNING!

Risk of Injury or Damage

Improper position and activity, such as reaching, bending or transferring may change the normal balance, center of gravity or weight distribution of the wheelchair causing injury or damage.

- Observe and follow all instructions and warnings regarding reach, weight, balance and positioning.
- Determine and establish your particular safety limits. Practice bending, reaching and transferring activities in several combinations in the presence of a qualified healthcare professional before attempting active use of the wheelchair.
- DO NOT move beyond the center of gravity.
- DO NOT lean forward out of the wheelchair any further than the length of the armrests.
- DO NOT attempt to reach objects if you have to move forward in the seat or pick them up from the floor by reaching down between your knees.
- DO NOT shift your weight or sitting position toward the direction you are reaching.
- DO NOT stand on the frame of the wheelchair.
- DO NOT lean over the top of the back of the wheelchair.

2.3.1 Driving Surfaces



WARNING!

Risk of Injury, Damage or Death

Loss of traction or stability on rough or unstable terrain may cause injury, damage or death.

- DO NOT operate the wheelchair on rough or unstable terrain. This would include but is not limited to areas of rock, mulch, mud, uneven pavement, roots and similar conditions.
- Be aware of your surroundings and conditions that might affect the ability to operate the wheelchair.

Your power wheelchair has been designed to operate on dry, level surfaces (i.e. concrete, blacktop, and asphalt). Other acceptable surfaces likely to be encountered include packed soil, grass, and gravel.



Do not operate the wheelchair in any type or depth of water (edges of streams, lakes, or oceans.)

If you approach an unfamiliar surface and feel uneasy about driving on that surface, avoid that surface.

2.3.2 Coping with Everyday Obstacles

Coping with the irritation of everyday obstacles can be somewhat alleviated by learning how to manage your wheelchair. Keep in mind your center of gravity to maintain stability and balance.

While the wheelchair is designed for use primarily in and around the home, the provider should determine whether this wheelchair is suitable for the actual environment in which the wheelchair will be used.




WARNING!

Risk of Injury, Damage or Death


Loss of traction or stability on curbs/obstacles may cause injury, damage or death.

- DO NOT cross curbs/obstacles that are not suitable for wheelchairs.
- DO NOT attempt to pass curbs/obstacles that are greater than 60 mm (2.36 in).
- Follow all instructions regarding negotiating curbs, obstacles and inclines.
- ALWAYS stop before negotiating an obstacle.
- Approach slowly until caster wheels are approximately 46 cm (18 in) away from the obstacle.
- Slowly apply power to move forward/reverse while negotiating the obstacle.

2.3.3 Negotiating Inclines

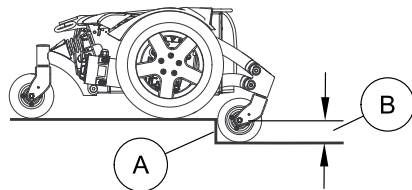

- i**
- Ⓐ = Curb/Obstacle
 - Ⓑ = Curb/Obstacle Height

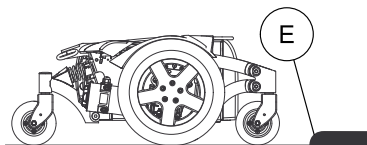

* Improved performance can be achieved when climbing over rolled curbs Ⓔ.

Acceptable Incline Angles 0° to 9°	Avoid Inclines of 10° and above

**WARNING!****Risk of Injury, Damage or Death**

Loss of traction or stability on inclines/grades or ramps may cause injury, damage or death. Lighter weight users may be at an increased risk. Surfaces that may be wet, icy, oily, slippery, painted, treated wood, rotten wood, rusted metal or other similar surfaces or materials may also increase risk.

- DO NOT use on inclines or ramps where surface is uncertain or compromised.
- DO NOT use on inclines greater than nine (9) degrees.
- DO NOT operate the seating system while the wheelchair is moving. Stop before operating seating system.
- DO NOT operate the seating system while on an incline. Operation on an incline may result in increased instability.
- To determine and establish your particular safety limits, practice use of this product on various sloping surfaces in the presence of a qualified healthcare provider before attempting active use of this wheelchair.
- DO NOT use on inclines where line of sight is impaired.
- Travel at a reduced, constant speed and DO NOT make sudden stops or direction changes. Release the joystick and allow the wheelchair to come to a full stop before changing directions. Traveling at high speeds reduces traction and increases stopping distance.
- DO NOT drive in an elevated position while on an incline.

- DO NOT leave elevating legrests in the fully extended position when proceeding down inclines/grades.
- DO NOT leave an unoccupied wheelchair unattended on inclines or ramps.

**DANGER!****Risk of Death or Serious Injury**

Traveling down inclines in reverse could cause the wheelchair to tip over resulting in death or serious injury

- DO NOT travel down inclines in reverse.

2.3.4 Reaching, Leaning and Bending - Forward

Many activities require the wheelchair user to reach, bend and transfer in and out of the wheelchair. These movements will cause a change to the normal balance, center of gravity, and weight distribution of the wheelchair. To determine and establish your particular safety limits, practice bending, reaching and transferring activities in several combinations in the presence of a qualified healthcare professional before attempting active use of the wheelchair



Engage motor locks and turn power off before reaching, leaning or bending only as far as your arm will extend without changing your sitting position. Position the casters so that they are extended away from the drive wheels and engage wheel locks/motor locks/clutches.

2.3.5 Reaching, Leaning and Bending - Backward



Position wheelchair as close as possible to the desired object. Position the casters so that they are extended away from the drive wheels to create the longest possible wheelbase, engage the motor locks and turn power off. Reach back only as far as your arm will extend without changing your sitting position.

2.3.6 Transferring To and From Other Seats



WARNING!

Risk of Serious Injury or Damage

Improper transfer techniques may cause serious injury or damage.

- Before attempting transfers, consult a health care professional to determine proper transfer techniques for the user and type of wheelchair.




WARNING!

Risk of Injury or Damage


Misuse of footplate may cause injury or damage.

- DO NOT stand on footplates.
- Ensure the footplates are in the upward position or swung outward when getting in or out of the wheelchair.

FRONT VIEW

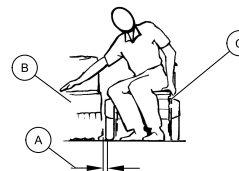


TOP VIEW

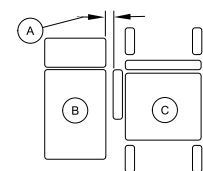

Adequate mobility and upper body strength is required to perform this activity independently.

1. Transfer to and from the wheelchair in the presence of a qualified healthcare professional to determine individual safety limits.
2. Turn power button OFF BEFORE entering or exiting the wheelchair.
3. Reduce gap between transfer surface and wheelchair seat to the minimum distance necessary to perform transfer.
4. ALWAYS engage both motor locks and wheel hubs (if equipped) to prevent the wheels from moving before transferring into or from the wheelchair.
5. Align casters parallel to the drive wheels to improve stability during transfer.
6. Invacare strongly recommends ordering wheel locks as an additional safeguard if not present.

7. Flip up footplates or swing footrests outward.
8. Shift body weight into seat with transfer

 During independent transfer, little or no seat platform will be beneath you. Use a transfer board if at all possible.

2.4 Pinch Points



WARNING!

Risk of Minor to Serious Injury

Pinch points can cause minor to serious injury.

- Be mindful of potential pinch points and use caution when using this product.




WARNING!

Risk of Injury

Pinch points can cause injury.

- Be aware that a pinch point **A** exists between the head tube cap and walking beam.
- Be aware that a pinch point **B** exists between the walking beam/head tube cap and telescoping tube when the wheelchair is at the lowest seat to floor height.
- Be aware that a pinch point **C** may occur when rotating the center mount front rigging assembly.

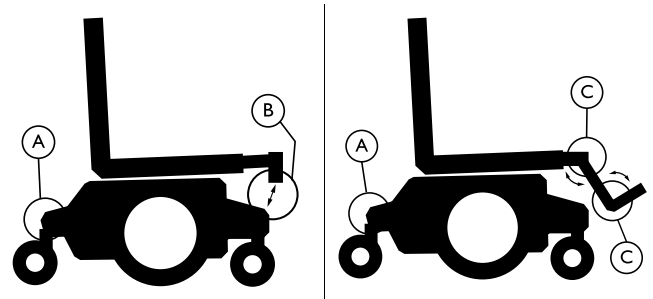

2.5 Footplates and Front Rigging



WARNING!

Risk of Serious Injury or Damage

Operating the wheelchair with a ground clearance of less than 75 mm (3 inches) between the footplates and the ground/floor may cause serious injury or property damage.

- ALWAYS maintain a minimum of 75 mm (3 inches) between the bottom of the footplates and ground/floor to ensure proper ground clearance while the wheelchair is in motion. If necessary, adjust the footplates height to achieve proper ground clearance. After footplates height adjustment, if the wheelchair dips forward and the footplates touch the ground while in motion, please contact your dealer for an inspection and avoid use of the wheelchair if possible.

**WARNING!****Risk of Injury or Damage**

Misuse of footplate may cause injury or damage.

- DO NOT stand on footplates.
- Ensure the footplates are in the upward position or swung outward when getting in or out of the wheelchair.

2.6 Storage

**WARNING!****Risk of Injury, Damage or Death**

Storage or use near heat sources and combustible products may result in injury or damage.

- DO NOT store or use wheelchair near open flames or other heat sources.
- DO NOT store or use wheelchair near combustible products.

2.7 Electrical

**DANGER!****Risk of Death or Serious Injury**

Electric shock can cause death or serious injury

- To avoid electric shock, inspect plug and cord for cuts and/or frayed wires. Replace cut cords or frayed wires immediately.

**WARNING!**


Power wheelchairs are equipped with three-prong (grounding) plugs for protection against possible shock hazards. Where a two-prong wall receptacle is encountered, it is the personal responsibility and obligation of the customer to contact a qualified electrician and have the two-prong replaced with a properly grounded three-prong wall receptacle in accordance with the National Electrical code. If you must use an extension cord, use only a three-wire extension cord having the same or higher electrical rating as the device being connected. DO NOT, under any circumstances, cut or remove the round grounding prong from any plug used with or for Invacare products. In addition, Invacare has placed RED warning tags on some equipment. DO NOT remove these tags.

**DANGER!****Risk of Death, Injury or Damage**

Shock hazards and risk of fire exist due to use of improper extension cord and/or use of three prong adapters.

- To avoid injury or product damage, when using an extension cord, use only a UL approved three wire extension cord having at least 16 AWG (American Wire Gauge) wire and the same or higher electrical rating as the device being connected.
- DO NOT use three prong to two prong adapters.

2.8 Recycling Batteries

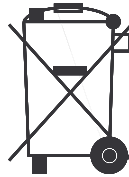

Lead acid batteries are almost entirely recyclable. Discarding these batteries in the trash is considered “improper disposal” and is illegal in most states. Old, used batteries are “hazardous material” and **MUST** be recycled through an approved agency. Contact your dealer or Invacare on proper disposal and recycling of your batteries.

2.9 Weight Training



WARNING!

Risk of Injury or Damage

Invacare **DOES NOT** recommend the use of its wheelchairs as a weight training apparatus. Invacare wheelchairs have **NOT** been designed or tested as a seat for any kind of weight training. Using said wheelchair for weight training could result in serious bodily injury to the user, damage to the wheelchair and surrounding property. Also, if occupant uses said wheelchair as a weight training apparatus, Invacare shall **NOT** be liable for bodily injury and the warranty is void.

– **DO NOT** use the wheelchair as a weight training apparatus.


3 Electromagnetic Compatibility (EMC) Information

3.1 Electromagnetic Interference (EMI) From Radio Wave Sources

Powered wheelchairs and motorized scooters (in this text, both will be referred to as powered wheelchairs) may be susceptible to electromagnetic interference (EMI), which is interfering electromagnetic energy (EM) emitted from sources such as radio stations, TV stations, amateur radio (HAM) transmitters, two way radios, and cellular phones. The interference (from radio wave sources) can cause the powered wheelchair to release its brakes, move by itself, or move in unintended directions. It can also permanently damage the powered wheelchair's control system. The intensity of the interfering EM energy can be measured in volts per meter (V/m). Each powered wheelchair can resist EMI up to a certain intensity. This is called its "immunity level." The higher the immunity level, the greater the protection. At this time, current technology is capable of achieving at least a 20 V/m immunity level, which would provide useful protection from the more common sources of radiated EMI.

There are a number of sources of relatively intense electromagnetic fields in the everyday environment. Some of these sources are obvious and easy to avoid. Others are not apparent and exposure is unavoidable. However, we believe that by following the warnings listed below, your risk to EMI will be minimized.

The sources of radiated EMI can be broadly classified into three types:

1. Hand-held Portable transceivers (transmitters/receivers with the antenna mounted directly on the transmitting unit. Examples include: citizens band (CB) radios, "walkie talkie", security, fire and police transceivers, cellular telephones, and other personal communication devices).
 -  Some cellular telephones and similar devices transmit signals while they are ON, even when not being used.
2. Medium-range mobile transceivers, such as those used in police cars, fire trucks, ambulances and taxis. These usually have the antenna mounted on the outside of the vehicle.
3. Long-range transmitters and transceivers, such as commercial broadcast transmitters (radio and TV broadcast antenna towers) and amateur (HAM) radios.



Other types of handheld devices, such as cordless phones, laptop computers, AM/FM radios, TV sets, CD players, cassette players, and small appliances, such as electric shavers and hair dryers, so far as we know, are not likely to cause EMI problems to your powered wheelchair.

3.2 Powered Wheelchair Electromagnetic Interference (EMI)

Because EM energy rapidly becomes more intense as one moves closer to the transmitting antenna (source), the EM fields from handheld radio wave sources (transceivers) are of special concern. It is possible to unintentionally bring high levels of EM energy very close to the powered wheelchair's control system while using these devices. This can affect powered wheelchair movement and braking. Therefore, the warnings listed below are recommended to prevent possible interference with the control system of the powered wheelchair.

Electromagnetic interference (EMI) from sources such as radio and TV stations, amateur radio (HAM) transmitters, two-way radios, and cellular phones can affect powered wheelchairs and motorized scooters.

FOLLOWING THE WARNINGS LISTED BELOW SHOULD REDUCE THE CHANCE OF UNINTENDED BRAKE RELEASE OR POWERED WHEELCHAIR MOVEMENT WHICH COULD RESULT IN SERIOUS INJURY.



WARNING!

- DO NOT operate handheld transceivers (transmitters receivers), such as citizens band (CB) radios, or turn ON personal communication devices, such as cellular phones, while the powered wheelchair is turned ON;
- Be aware of nearby transmitters, such as radio or TV stations, and try to avoid coming close to them;
- If unintended movement or brake release occurs, turn the powered wheelchair OFF as soon as it is safe;
- Be aware that adding accessories or components, or modifying the powered wheelchair, may make it more susceptible to EMI (Note: There is no easy way to evaluate their effect on the overall immunity of the powered wheelchair); and
- Report all incidents of unintended movement or brake release to Invacare and note whether there is a source of EMI nearby.



WARNING!

Important Information

- 20 volts per meter (V/m) is a generally achievable and useful immunity level against EMI (as of May 1994) (the higher the level, the greater the protection);
- This device has been tested to a radiated immunity level of 20 volts per meter.
- The immunity level of the product is unknown.
- Modification of any kind to the electronics of this power wheelchair as manufactured by Invacare may adversely affect the EMI immunity levels.

3.3 Powered Wheelchair Electromagnetic Emissions



CAUTION!

Risk of Injury or Damage

EMC interference affecting other products may result in injury or damage.

To avoid impacting the operation and function of other products:

- Products not specified by Invacare that may be used on or near the wheelchair may be impacted by emissions from this product if they have a sensitivity level that is lower than the recognized standard and provided by this wheelchair. Refer to the manufacturer specifications for any electronic device BEFORE use near this product to determine its level of immunity and potential risk.

4 Label Locations

4.1 All Wheelchairs



**DANGER!****Risk of Injury, Damage or Death**





Missing or damaged labels may contribute to injury, damage or death.




– Ensure labels are present and legible.

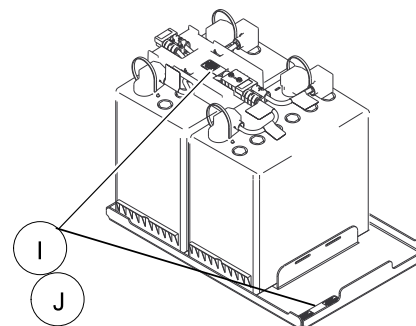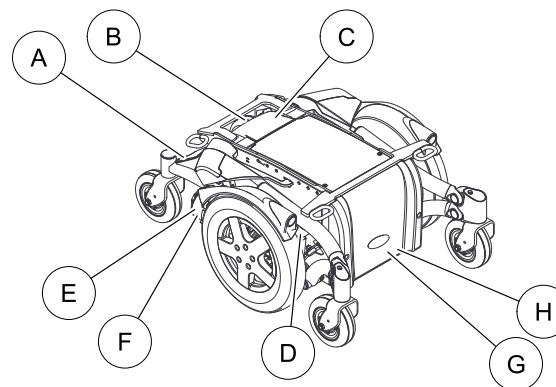


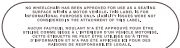

Labels are subject to change without notice.

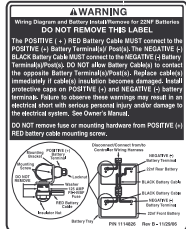


ITEM	PART NUMBER	DESCRIPTION
A	1183487	Serial Number Label 
B	1167422	Controller Label Label located under the controller 
C	1035900	Transportation Label 
D	See Notes	Weight Capacity Label (Base Only) Label depends on model 

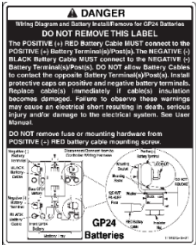E	1167424	Replacing Motor/Gearbox Label 
F	1104802	Push/Drive Label 
G	1114826 and 1114847	*22NF Battery Wiring Diagram Labels 

H	1114825 and 1114848	<p>*GP24 Battery Wiring Diagram Labels</p>  <p>The diagram shows a complex wiring system with various components labeled with letters and numbers. It includes a warning section at the top and a title 'GP24 Batteries' at the bottom.</p>
I	1118356	<p>** 22NF Battery Label</p>  <p>The label is rectangular with a black background and white text. It contains instructions for battery use and a warning symbol.</p>
J	1118355	<p>** GP24 Battery Label</p>  <p>The label is rectangular with a black background and white text. It contains instructions for battery use and a warning symbol.</p>


4.2 Wheelchairs with TRRO

Wheelchairs with TRRO

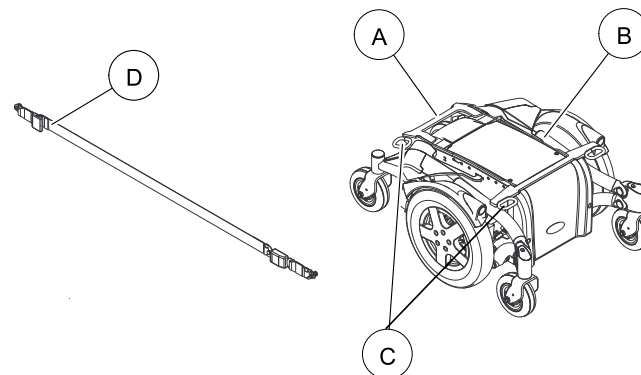





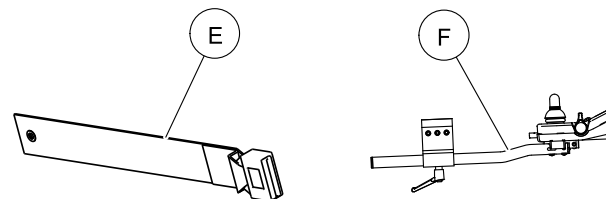

Wheelchairs without TRRO




Wheelchairs with Running Lights



* Labels located under front shroud. Only one set of labels will appear depending on chair model.

** Only one of the battery labels (22NF or GP24) will appear depending on chair model.

ITEM	PART NUMBER	DESCRIPTION
A	1082692	Compliance Label 
B	1134848	Adjustment TRRO Label  Also on opposite side of wheelchair. 
C	1083199	Tie Down Point Label  Also on opposite side of wheelchair. 
D	1134840	Buckle Label 

ITEM	PART NUMBER	DESCRIPTION
E	1134811	Non Compliance Label  Auto style seat positioning strap shown. This label is also on the airline style seat positioning strap. 
F	1171793	Running Lights Label 

5 Technical data

5.1 Specifications


5.1.1 Models

TDX SP	TDXSPBASE, TDXSP, TDXSP-CG, TDXSP-MCG, TDXSP-CG-GT, TDXSP-GT, TDXSP-MCG-GT
--------	---

5.1.2 Overall Dimensions

	SP
Base Width (Without Joystick):	
With two 22NF Batteries:	24 in (61 cm)
With two GP 24 Batteries	25.5 in (65 cm)
Base Length (Without Front Rigging):	35.25 in (89 cm)
Overall Height -	
With ASBA Seat:	35.5 into 39.5 in (90 cm to 100 cm)
With Formula CG Tilt Only:	36.5 in to 40.5 in (93 cm to 103 cm)

	SP
With Elevating ASBA Seat:	35.5 to 41.5 in (35.5 cm to 105 cm)
Overall Length -	
With Center Mount Front Rigging:	42.9 in (109 cm) @ 0°
Without Front Rigging:	35.25 in (89 cm)


 All dimensions are ± .50 inches unless otherwise indicated.

5.1.3 Seat

	SP
Tilt Angle Adjustment:	Adjustable (0° - 10°)

5.1.4 Wheels


Caster:	6 in x 2 in (15 cm x 5 cm) in Semi-pneumatic with Precision Sealed Bearings
Drive Wheel/Tires:	TDX SP – 14 in x 3 in (35 cm 8 cm) in Foam Filled or Pneumatic (Standard)
Caster Forks:	Two sided fork (Standard)

Drive Axle:	Non-adjustable
 The size of tire is marked on the side wall of each drive tire. The caster tires do not have the tire size marked as the caster tires are not serviceable. If new tire is needed, the entire caster wheel assembly MUST be replaced.	

5.1.5 Axle

	SP
Drive Axle:	Non-adjustable

5.1.6 Battery

	Battery Type	
	GP24	22NF
Number of Batteries Needed	2	2
Manufacturer	MK batteries	MK batteries
Model	p/n M24SLDG batteries only	p/n M22NFSLDG batteries only
Voltage	12	12
Capacity (Amp-hours) for 5 hour rating	63	43.2
 Only sealed non-spillable batteries that meet DOT CFR 173.159 (d), IATA Packing Instructions 806, and IATA Provision A67 shall be installed in this wheelchair.		

5.1.7 Footrests

	SP
Footrest:	Telescoping Front Rigging Supports, 2 in and 4 in. long Pivot Slide Tube

5.1.8 Weight

	SP
Weight (Base Only)	-
Without 22NF Batteries:	123 lbs (56 Kg)
With two 22NF Batteries:	221 lbs (100 Kg)
Without GP24 Batteries:	158 lbs (72 Kg)
With (2) GP24 Batteries:	260 lbs (118 Kg)



Weight limitation is total weight (user weight plus any additional items that the user may require [back pack, ventilator, etc.]). Example: If weight limitation of the wheelchair is 300 lbs and additional items equal 25 lbs, subtract 25 lbs from 300 lbs this means the maximum weight limitation of the user is 275 lbs.

5.1.9 Weight Capacity Warnings



WARNING!

Risk of Death or Serious Injury

Exceeding the weight capacity of the wheelchair/seating system could cause instability resulting in death or serious injury.

– DO NOT exceed the weight capacity.

5.1.10 Weight Capacity Values

	TDX SP	
	4 POLE	HD 4 POLE
With ASBA Seat:	Up to 300 lbs (136 Kg)	Up to 400 lbs (181 Kg)
With Formula CG Seating:	Up to 300 lbs (136 Kg)	Up to 400 lbs (181 Kg)
With Elevating ASBA Seat:	Up to 300 lbs (136 Kg)	N/A

6 Wheelchair Operation

6.1 Preparing the Joystick for Use



WARNING!

Risk of Serious Injury or Damage

Attaching hardware that is loosely secured could cause loss of stability resulting in serious injury or damage.

- After ANY adjustments, repair or service and before use, make sure that all attaching hardware is tightened securely.




WARNING!

Risk of Injury, Damage or Death

Malfunctioning or damaged joystick may cause unintended/erratic movement resulting in injury, damage or death.

- Ensure the joystick is securely connected to controller.
- DO NOT use if joystick does not spring back to the neutral position or becomes sticky or sluggish.
- DO NOT use if joystick boot is torn or damaged.
- DO NOT use with a broken or missing joystick knob.
- If unintended/erratic movement occurs, stop using the wheelchair immediately and contact a qualified technician.
- Ensure control knobs are secure before using the wheelchair. Stop using the wheelchair immediately and contact a qualified technician if control knobs are not secure.

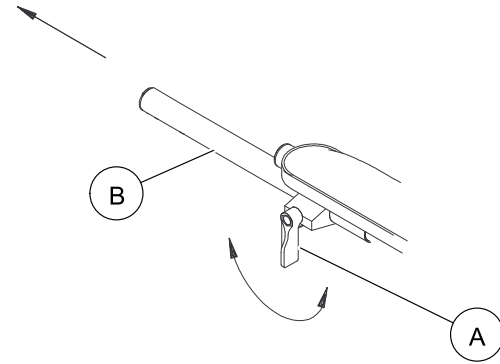


The joystick is factory installed on the right side of the wheelchair. To reposition the joystick onto the left side of the wheelchair, refer to 8.16 Repositioning Joystick, page 72.


1. Turn the adjustment lock lever (A) to release the adjustment lock from joystick mounting tube (B).
2. Slide joystick mounting tube to the desired position.
3. Turn the adjustment lock lever to secure the adjustment lock to the joystick mounting tube.

6.2 Turning the Power On/Off

SPJ+ Joysticks

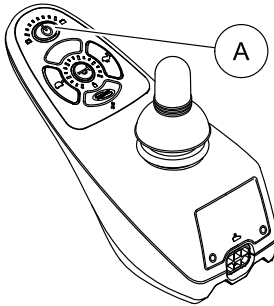

CMPJ+ Joysticks

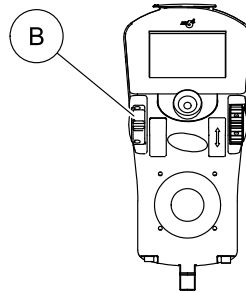

- To turn the power On, perform one of the following steps:

Joystick	Action
CMPJ+	Move the On/Off switch B Forward to the On position.
SPJ+	Press the On/Off button A .

- Turning the power Off can be achieved by performing one of the following steps:

Joystick	Action
CMPJ+	Move the On/Off switch B Back to the Off position.
SPJ+	Press the On/Off button A .

6.3 Using the Joystick to Drive the Wheelchair

The joystick is located on the joystick housing and provides smooth control of speed and direction. It is equipped with 360 degrees of mobility for ease of operation. The joystick is spring-loaded, and automatically returns to the upright (neutral) position when released. Pushing the joystick in a given direction causes the wheelchair to move in that direction.


The joystick has proportional drive control, meaning that the further it is pushed from the upright (neutral) position, the faster the wheelchair moves. The maximum speed, however, is limited by the setting of the speed-control knob.

To slow the wheelchair to a stop, simply release the joystick. The wheelchair has automatic speed and direction compensation to minimize corrections.

When first learning to drive, select a slow speed and try to drive the wheelchair as slowly as possible by pushing the joystick slightly forward. This exercise will help you learn to utilize the full potential of the proportional control and allow you to start and stop smoothly.

To drive the wheelchair, perform the following:

- Adjust speed control knob to the appropriate setting.
- Turn the power On. Refer to 6.2 Turning the Power On/Off, page 35.
- Maneuver the joystick in the following manner:

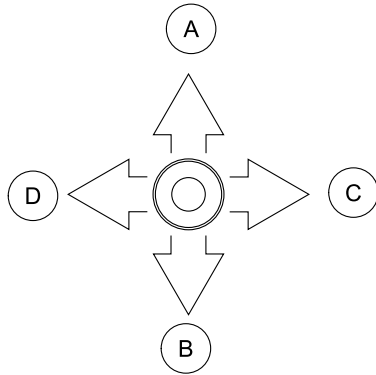


i For specific information about the joystick installed on the wheelchair, refer to 6.4 SPJ+™, MK6i™ SPJ+ w/PSS and MK6i SPJ+ w/ACC Joystick Switches and Indicators, page 36 or 6.5 CMPJ+ Joystick, Switches and Indicators, page 38.

Operating noise may change over time. Sudden or severe change may signal a problem with your wheelchair. Contact a qualified technician or Invacare if issue persists.


6.4 SPJ+™, MK6i™ SPJ+ w/PSS and MK6i SPJ+ w/ACC Joystick Switches and Indicators

MOVEMENT	ACTION
FORWARD A	Push joystick forward, towards the front of the wheelchair.
REVERSE B	Pull joystick back, towards the rear of the wheelchair.
Turn RIGHT C	Move joystick toward the right side of the wheelchair.
Turn LEFT D	Move joystick toward the left side of the wheelchair.
STOP	Release the joystick and the wheelchair will slow to a stop.

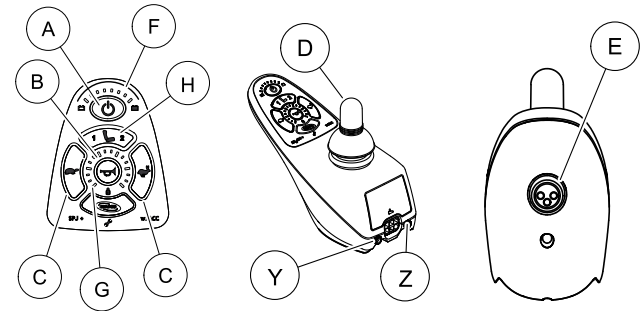



i **Y** Additional input for powered seating switch.
Z Active (if programmed)




6.4.1 On/Off Button









The On/Off button  is located at the front of the joystick housing. It is used to turn the wheelchair On and Off.

6.4.2 Speedometer


The speedometer  is used to show the maximum speed. The right-most LED indicates current maximum speed setting. The bottom left GREEN LED  flashes to indicate that the joystick is in speed limit mode. Speed limit mode limits the drive speed to a pre-programmed value, typically when the seat has been elevated and the wheelchair is required to drive at 20% speed.

6.4.3 Speed Control Buttons

The speed control buttons  [tortoise button ] and hare button ] are used to set and adjust the maximum speed.

- I. To adjust the speed, perform one of the following:
 - Adjust Speed in 20% Increments (5 Speed Mode) -
 - Press the tortoise button  or hare button  to decrease/increase the speed in 20% increments. The larger bars in the speedometer will light.
 - Adjust Speed in Smaller Increments (VSP Mode) - Perform the following steps:
 - a. Press and hold both the tortoise button  and hare button  until the joystick beeps.
 - b. Perform one of the following:
 - Press the tortoise button  or hare button  to decrease/increase the speed in 20% increments. The larger bars in the speedometer will light.
 - Press and hold the tortoise button  or hare button  to decrease/increase the speed in smaller increments. The smaller bars in the speedometer will light.

6.4.4 Joystick

The joystick  has proportional drive control, meaning that the further the joystick is pushed from the upright (neutral) position, the faster the wheelchair or seat moves. Your top speed, however, is limited by the programmed settings.

To slow the wheelchair to a stop, simply release the joystick. The wheelchair has automatic speed and direction compensation to minimize corrections.

6.4.5 Charger/Programming Input

The charger/programming input (E) is located at the front of the joystick housing. This provides easy access for charging the wheelchair batteries. This port also serves as the Remote Programmer Communication connection. Driving is prevented while the system is charging.

6.4.6 Information Gauge Display

The information gauge display (F) is located on the front of the joystick housing and provides the following information to the user on the status of the wheelchair:

1. Power is On.
2. True state-of-battery-charge, including notification of when the battery requires charging:
 - Green LEDs are lit, indicating well charged batteries.
 - Amber LEDs are lit, indicating batteries are moderately charged. Recharge batteries before taking a long trip.
 - Red LEDs are lit, indicating batteries are running out of charge. Recharge batteries as soon as possible.



The Information Gauge display also serves as a system diagnostic device when a fault is detected by the control module. A specific number of flashes of the LEDs indicate the type of fault detected.

Refer to [Information Gauge Display Diagnostics](#) in 9.2.1 SPJ+, SPJ+ w/PSS or SPJ+ w/ACC Joysticks, page 74 for the diagnostic indications of the wheelchair status.


6.4.7 Service Indicator

The AMBER service indicator (⚡) will light when an error or fault occurs.

6.4.8 Mode Button

The mode button (H) is only present on SPJ+ w/ACC joystick

6.5 CMPJ+ Joystick, Switches and Indicators

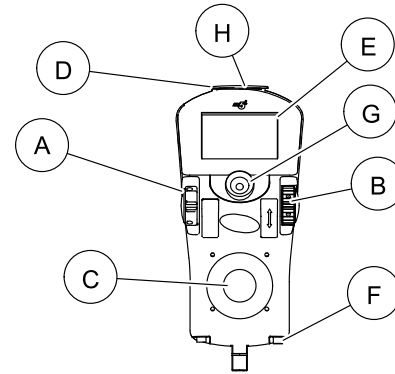

6.5.1 On/Off - Drive Select Toggle Switch (CMPJ+ Joystick)

The drive select toggle switch (A) is located on the left side, below the LCD. The drive select position is momentary, meaning that it will return to the neutral position after a selection is made.

This switch allows the operator to select the type of operation or performance which best suits a particular control need or situation. The DRIVE 1 program uses performance values which are independent of those used for the DRIVE 2 or 3 or 4 program. As an example, an operator may have a control need for spasticity in the morning and a very different need in the afternoon. DRIVE 1 can be programmed for higher speeds and quicker response while DRIVE

2 can be programmed for slower speeds and less responsiveness or vice versa. The other two drive programs could be indoor and outdoor versions of DRIVE 1 and DRIVE 2.

Selecting the Drive Mode

1. Move the toggle up and release. DRIVE 1 will appear on LCD.
2. Move the toggle up and release again. DRIVE 2 will appear on LCD.
3. Move the toggle up and release again. DRIVE 3 will appear on LCD.
4. Move the toggle up and release again. DRIVE 4 will appear on LCD.
5. Move the toggle up and release one more time to select DRIVE 1.

6.5.2 Speed Control

The speed control knob ⑧ is located on the side of the joystick housing.

1. Rotate the knob clockwise (forward) to increase the speed of the wheelchair to the programmed max speed.
2. Rotate the knob counterclockwise (backward) to decrease the speed of the wheelchair to the programmed max speed.

6.5.3 Joystick

The joystick ⑨ has proportional drive control, meaning that the further the joystick is pushed from the upright (neutral) position, the faster the wheelchair or seat moves. Your top speed, however, is limited by the programmed settings.

To slow the wheelchair to a stop, simply release the joystick. The wheelchair has automatic speed and direction compensation to minimize corrections.

6.5.4 Charger/Programming Input

The charger/programming input ⑩ is located at the front of the joystick housing. This provides easy access for charging

1143190-N-03

the wheelchair batteries. This port also serves as the Remote Programmer Communication connection. Driving is prevented while the system is charging.

6.5.5 LCD Display Screens

The LCD Display ⑪ is located in front of the joystick and provides information on the status of the wheelchair through a backlit display. The LCD display is readable in both bright sunlight and complete darkness.


Splash Screen




This screen is displayed at startup of the joystick for about 2 seconds. This screen displays the software version and date information.

After this screen, the joystick displays the Main Screen.

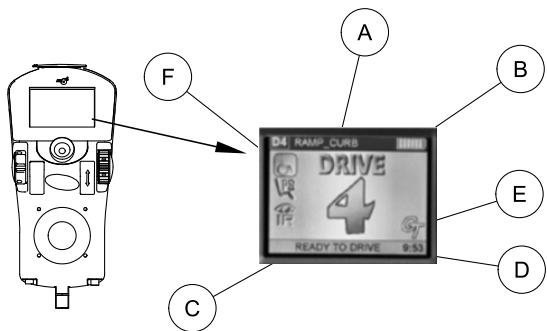


Main Screen








 The callouts in this illustration correspond to the LCD Display chart only.








During normal operation, the active drive is displayed in the upper half of the LCD display. Battery charge level is shown in the Battery Gauge Display (BGD) located on the right side of the LCD display. At full charge, solid blocks fill in all ten segments between E (Empty) and F (Full). As the battery becomes discharged, the top most segments will progressively disappear until no segments appear between E and F. At this level, the user should charge the batteries as soon as possible.

The lower half of the LCD display is the Information Center. The Information Center displays current data on the wheelchair.

LCD DISPLAY					
ITEM	DESCRIPTION				
DRIVE NAME — A	This field shows the currently selected Drive's Name. Available choices are as follows:				
	Color MPJ+				
	<table border="1" style="width: 100%; text-align: center;"> <tr> <td style="color: green;">DRIVE 1 Drive 1*</td> <td style="color: orange;">DRIVE 2 Drive 2*</td> <td style="color: red;">DRIVE 3 Drive 3*</td> <td style="color: green;">DRIVE 4 Drive 4*</td> <td> ** No Drive**</td> </tr> </table>	DRIVE 1 Drive 1*	DRIVE 2 Drive 2*	DRIVE 3 Drive 3*	DRIVE 4 Drive 4*
DRIVE 1 Drive 1*	DRIVE 2 Drive 2*	DRIVE 3 Drive 3*	DRIVE 4 Drive 4*	 ** No Drive**	
<p> *Drive names can be customized. Actual drive names may display differently. ** No Drive selected via the programmer</p>					
Battery Level Indicator — B	This symbol shows the Battery Level and will change depending on the available battery power. This indicator is shown on every screen.				
Status Message — C	This area displays status or instructions.				

Clock — D	Displays current time.
Status Indicator — E	<p>The status indicator will show a “Warning” (exclamation point inside a triangle) indicator when the chair has a condition that requires attention.</p> <p>The status indicator will show a “STOP” sign when a serious condition exists. The chair will not be allowed to operate.</p> <p>The status indicator shows an Attendant Icon if the attendant’s override switch is active.</p>

Modes — F	<p>The dotted-box shows the area that contains the available “modes” in the currently selected drive. The modes are programmed for each drive and are based upon the configuration of the chair.</p> <p>These modes are highlighted when the Mode is active. The operator changes modes by pressing the Mode Select Switch.</p> <p>The available modes are as follows:</p>			
 <p>Digital 3 Speed 1 - 3</p>	 <p>RIM Mode</p>	 <p>No Driving</p>	 <p>Automatic Positioning</p>	 <p>Powered Seating</p>

 <p>4-Switch Level 1 (L1, L1 Latched)</p>	 <p>4-Switch Level 2 (L2, L2 Latched)</p>	 <p>ECU Output Activated</p>	
 <p>ASM 2</p>	 <p>Infrared</p>	 <p>Mouse</p>	 <p>Mouse B</p>

Driving Screen








This screen is shown when the operator issues a drive command and the Drive Icon on the main screen was highlighted.



The Drive’s name, warning/info message, status icon and battery indicator are displayed on this screen.

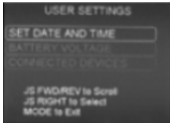6.5.6 User Settings

Depress the mode button of the CMPJ+ joystick for 10 seconds and the User Settings screen will appear with three choices shown. Move the joystick forward or reverse to scroll through the list. Move the joystick to the right to select a user setting.

User Settings	
	
Set Date and Time	Battery Voltage
	
Fault Codes	Connected Devices
	

User Settings	
Set Date and Time	<p>SET DATE AND TIME - Sets the clock on the CMPJ+ joystick. Adds date and time stamp to error codes.</p> <ul style="list-style-type: none"> • Move the joystick Up or Down to change the highlighted value (hour, minute, AM/PM, month, day, year) • Move the joystick Right or Left to select a value or the Set icon. • Highlight the Set icon and move the joystick forward to enter new date and time.
Battery Voltage	<p>BATTERY VOLTAGE - Displays current battery voltage. This is a diagnostic test a user can perform prior to a service call.</p>
Fault Code	<p>FAULT CODES - Displays time and date stamped fault codes. This information can be helpful to a provider prior to making a service call.</p>
Connected Devices	<p>CONNECTED DEVICES - Displays device connections.</p>

Connected Devices Screen

This screen is displayed if the Mode Select switch is held active for about 10 seconds. This screen shows an icon that represents any additional devices that are connected to the chair



ICON	DESCRIPTION	ICON	DESCRIPTION
	Intelligent Tilt Actuator		RIM Control
	Intelligent Recline Actuator		ECU 1/2 and ECU 3/4
	Intelligent Center Leg Actuator		Proportional Attendant Control
	Elevate Actuator		Compact Joystick
	Generic Tilt Actuator		Sip and Puff Control
	Generic Recline Actuator		Digital Attendant Control
	Generic Leg Actuator		Micro Extremity Control
	Generic Right Leg Actuator		Peachtree Control
	Generic Left Leg Actuator		ASL Digital Control

	Intelligent CG Tilt		Generic Analog Control
	Shark Power Module (SPM) Actuator		This is displayed if the controller supports G-Trac
	SANODE or Single Actuator Control Interface		Mouse Only
	4-way Switch Box		IR/Mouse
	Multiple Actuator Control Box		

6.5.7 CMPJ+ Joystick - Programmable Mono Ports 1 and 2 with External Mode Switch

The programmable mono port with external mode switch input is located at the rear of the joystick on the left side. The programmable mono port input offers the choice of three options:

- Remote drive select
- Remote stop/mode (reset) input
- Single actuator input

The single switch functions operate through mono port 1. An optional y-cable allows a second programmable function through mono port 2.

Remote Stop Switch

The remote reset switch may be used to stop the wheelchair if the wheelchair is in motion.


The remote reset switch also functions in the same way as the joystick mode switch when the wheelchair is not in motion. Refer to 6.5.9 Mode Switch (CMPJ+ Joystick, page 44).

6.5.8 Remote On/Off Switch

The remote On/Off switch input is located at the rear of the joystick on the right side and allows the power switch to be operated by an ability switch (normally open momentary switch with mono plug). To use the remote On/Off feature, the Drive Select/On/Off switch must be in the On position. Each activation of the ability switch will alternately turn the joystick On or Off.

6.5.9 Mode Switch (CMPJ+ Joystick)

The mode switch [CMPJ+ Joystick Ⓞ] is used to select the operating mode for the wheelchair. The mode switch is located on the joystick. A mode switch is needed whenever any of the following operating modes are programmed:

 *In these modes, Standby Select allows the reset switch to be bypassed for users unable to activate the switch.

- Environmental Controls (ECU 1, ECU 2, ECU 3, ECU 4)*
- 3 Speed Mode in Digital 3 Speed (Slow, Medium, Full)
- Sleep Mode
- RIM Mode*
- Remote Drive Selection Mode*
- Tilt/Recline Mode*
- Information Center Display Selection (does not require Reset activation at power up)

If any of the above modes are selected, the control will require activation of the switch immediately after the power switch is turned On in order to enter the drive mode. The second line of the LCD will display - PRESS RESET.

6.5.10 Memory Card Slot

The memory card slot is used with the basic or professional memory card for saving or reading wheelchair parameters.

6.6 When to Charge Batteries



Avoid draining battery to low or empty levels.

Fully charge battery before use if the battery becomes fully drained.

If battery charge becomes so low that no battery indicators are lit, allow the batteries to charge overnight.

New batteries MUST be fully charged prior to initial use to avoid reduced battery life.

Recharge daily and frequently to extend battery life and reduce charge time.


Charge battery when product will not be used for an extended period of time.

6.6.1 SPJ+, SPJ+ w/PSS and SPJ+ w/ACC Joysticks

The Information Gauge Display located on the front of the joystick housing, it provides the state-of-battery charge, including notification of when the battery requires charging. It also provides the following information to the user on the status of the wheelchair:

- Green LEDs are lit, indicating well charged batteries.
- Amber LEDs are lit, indicating batteries are moderately charged. Recharge batteries before taking a long trip.
- Red LEDs are lit, indicating batteries are running out of charge. Recharge batteries as soon as possible.

6.6.2 CMPJ+ Joystick

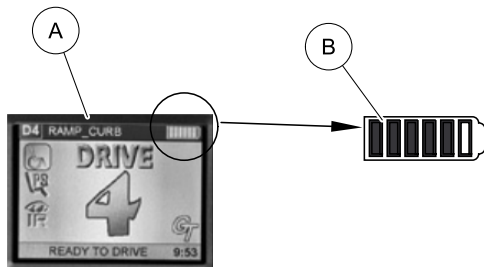

The far right side of the display screen **A** is the Battery Gauge Display **B**. It provides information on the remaining charge in the batteries.

All battery indicators will be lit at full charge. The battery discharges as power is used. Indicators will turn off as the battery discharges.

6.7 Charging Batteries



WARNING!

Risk of Injury, Damage or Death

Improper charging may cause injury or damage

- Charge battery in a cool, dry and well-ventilated area.
- DO NOT attempt to charge the battery by attaching cables directly to the battery terminals.
- DO NOT attempt to charge the battery and operate the wheelchair at the same time.
- DO NOT attempt to charge the battery when the wheelchair has been exposed to any type of moisture.
- DO NOT sit in the wheelchair while charging the battery.
- DO NOT leave the charger unattended when the breaker has tripped. Unplug and discontinue use immediately. Contact a qualified service technician or Invacare for service.



New batteries **MUST** be fully charged prior to initial use of the wheelchair.

As a general rule, batteries should be recharged daily to assure the longest possible life and minimize the required charging time. Plan to recharge the batteries when it is anticipated the wheelchair will not be used for a long period of time.


The range per battery charge using recommended batteries should be approximately 5 to 9 hours of typical operation. Extensive use on inclines may substantially reduce per charge mileage.

6.7.1 Description and Use of Battery Chargers

The charger automatically reduces the charge from an initially high rate to a zero reading at a fully charged condition. If left unattended, the charger should automatically shut-off when full charge is obtained or enter a trickle charge mode to maintain the batteries depending on charger model.

There are some basic concepts which will help you understand this automatic process. They are:

Once the charger has been connected to the wheelchair and wall outlet and, if necessary, the charger has been turned on, the battery charger indicator lights will flash and light to show the battery charger status and condition of batteries to be charged. Refer to owner's manual shipped with battery charger.

 If performing the charging procedures, READ and CAREFULLY follow the individual instructions for each charger (supplied or purchased).

If charging instructions are not supplied, consult a qualified service technician for proper procedures.

Required Items

TOOL	QTY	COMMENTS
Battery Charger	1	Supplied




1. Attach the battery charger connector to the charger port on the joystick.
2. Plug the charger's AC power cord, or extension, into the grounded 120 VAC wall outlet.
3. Wait until charging is complete.




Allow eight hours for normal charging. Larger batteries (greater than 55 ampere-hours) or severely discharged batteries may require up to sixteen hours to be properly charged and equalized.

It is advantageous to recharge frequently rather than only when necessary. In fact, a battery's life is extended if the charge level is maintained well above a low condition.

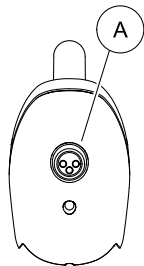If the batteries need to be charged more often or take longer to charge than normal, they may need to be replaced. Contact an Invacare dealer for service.

SPJ+, SPJ+ w/PSS and SPJ+ w/ACC Joysticks	CMPJ+ Joystick	Display
A — Charger Programming Port		
Located on Back of Joystick	Located on Front of Joystick	Located on Back of Display
		

6.8 Disengaging/Engaging the Motor Lock Levers

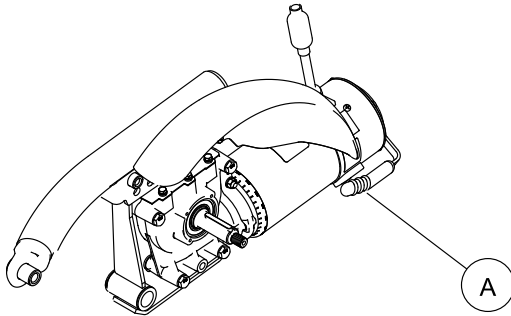

WARNING! Risk of Injury or Damage

Misuse of motor locks may result in injury or damage.
– DO NOT engage or disengage the motor locks until the power is in the off position.



Motor lock **A** disengagement/engagement allows free-wheeling or joystick controlled operation. Free-wheeling allows an assistant to maneuver the wheelchair without power.

Motor lock levers are located between the rear caster assembly and drive wheel on both sides of the wheelchair.

- I. Perform one of the following as if viewing the motors from behind the wheelchair (Detail “A”):
 - Disengage (PUSH) - Move the motor lock levers UP to disengage the motors and push the wheelchair.
 - Engage (DRIVE) - Move the motor lock levers DOWN to engage the motors and drive the wheelchair.

6.9 Disengaging/Engaging the Wheel locks




WARNING! Risk of Injury or Damage

Misuse of wheel locks may result in injury or damage.
– DO NOT attempt to stop a moving wheelchair with the wheel locks. Wheel locks are not brakes.
– DO NOT use the wheel locks (if equipped) when the wheelchair power is on. Invacare recommends the use of wheel locks.



Use the wheel locks whenever the clutches are disengaged and the wheelchair is being pushed.

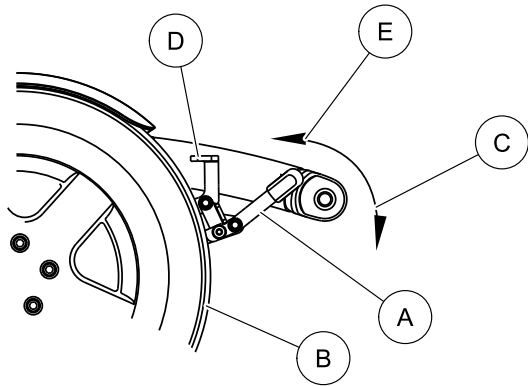

6.9.1 Engaging

1. Push handle forward away from tire to engage wheel lock.
2. Repeat STEP 1 for opposite wheel.

6.9.2 Disengaging

1. Pull handle back toward tire to disengage wheel lock.
2. Repeat STEP 1 for opposite wheel.

7 Transport in Vehicles

7.1 About Transport Ready Packages



WARNING!

Risk of Injury, Damage or Death

- Failure to observe and follow transport warnings and instructions may result in injury, damage or death.
- Wheelchair occupants should transfer into the vehicle seat and use the OEM (Original Equipment Manufacturer) vehicle-installed restraint system.
 - Ensure wheelchair is secured using proper restraint systems. Use **ONLY** Wheelchair Tie-down and Occupant Restraint Systems (WTORS) which meet the requirements of the SAE (Society of Automotive Engineers) J2249 Recommended Practice during travel in a motor vehicle.
 - Wheelchair-mounted accessories, including but not limited to IV poles, trays, respiratory equipment, backpacks, and other personal items should be removed and secured separately.
 - Spill proof batteries, such as “gel cells”, should be installed on wheelchair to be used during travel in a motor vehicle.
 - Contact Invacare Corporation with any questions about using this wheelchair for seating in a motor vehicle.



WARNING!

Risk of Injury, Damage or Death

- Improper installation or service may result in injury, damage or death.
- Transport ready packages are not retrofittable to existing models and are not field serviceable.
 - **DO NOT** overtighten hardware.



WARNING!

Risk of Injury, Damage or Death

- Alteration or substitution may result in injury, damage or death.
- **DO NOT** alter or substitute product parts, components or systems.



WARNING!

Risk of Injury, Damage or Death

- Damaged parts due to collision or impact may result in injury, damage or death.
- Seek immediate attention and service if wheelchair is involved in a collision or impact event. This includes, but is not limited to, vehicle accidents, mishandling and impact events where the wheelchair strikes something or is struck by something that may cause damage.
 - Ensure your wheelchair is working properly and is inspected by a qualified Invacare technician if the wheelchair is involved in a collision or impact event.



ANSI = American National Standards Institute, RESNA= Rehabilitation Engineering and Assistive Technology Society of North America.

7.2 Wheelchair Transport Brackets (TRBKTS)

As of this date, the Department of Transportation has not approved any tie-down systems for transportation of a user while in a wheelchair, in a moving vehicle of any type. It is Invacare's position that users of wheelchairs should be transferred into appropriate seating in vehicles for transportation and use be made of the restraints made available by the auto industry. Invacare cannot and does not recommend any wheelchair transportation system.

TRBKTS includes four factory-installed wheelchair transport brackets.



WARNING!

Risk of Injury, Damage or Death

- Improper use of wheelchair transport brackets (TRBKTS) may result in injury, damage or death.
- Use these transport brackets only to secure an unoccupied wheelchair during transport.
 - Ensure wheelchair is secured using proper restraint systems. Wheelchair transport brackets have not been crash-tested in accordance with ANSI/RESNA WC Vol I Section 19 Frontal Impact Test requirements for wheelchairs.
 - Only use the transport brackets for the purposes described in this manual.



ANSI = American National Standards Institute, RESNA= Rehabilitation Engineering and Assistive Technology Society of North America.

7.3 Transport Ready Option (TRRO)



As of January 1, 2017, Transport Ready Option (TRRO) has been discontinued on this product. Please contact your dealer or Invacare for legacy information or to answer questions regarding TRRO.

As of this date, the Department of Transportation has not approved any tie-down systems for transportation of a user while in a wheelchair, in a moving vehicle of any type. It is Invacare's position that users of wheelchairs should be transferred into appropriate seating in vehicles for transportation and use be made of the restraints made available by the auto industry. Invacare cannot and does not recommend any wheelchair transportation system.



WARNING!

Risk of Injury, Damage or Death

- Failure to observe and follow transport warnings and instructions may result in injury, damage or death.
- Use ONLY Wheelchair Tie-down and Occupant Restraint Systems (WTORS) which meet the requirements of the SAE (Society of Automotive Engineers) J2249 Recommended Practice during travel in a motor vehicle.
 - The wheelchair MUST be in a forward facing position during travel in a motor vehicle.
 - Only use the transport brackets included with TRRO for the purposes described in this manual.
 - This wheelchair is equipped, and has been dynamically tested to rely on WHEELCHAIR-ANCHORED pelvic belts. If desired, VEHICLE-ANCHORED pelvic belts may be used.
 - Use both pelvic and upper-torso belts.

**WARNING!****Risk of Injury, Damage or Death**

Lack or improper use of wheelchair transport systems may result in injury, damage or death.

- Use both pelvic and upper torso belts.
- The pelvic belt that is provided by Invacare has been tested for use in a motor vehicle on this wheelchair only. Do not replace the pelvic belt with a different style pelvic belt.



ANSI = American National Standards Institute, RESNA= Rehabilitation Engineering and Assistive Technology Society of North America.

TRRO includes four factory-installed transport brackets and a wheelchair anchored pelvic belt.

- The wheelchair has been tested for seating in a motor vehicle with the factory installed seating system only.
- This wheelchair has been dynamically tested in a forward-facing mode with the specified crash test dummy restrained by BOTH pelvic and upper-torso belt(s) (shoulder belts), and that BOTH pelvic and upper torso belt(s) should be used to reduce the possibility of head and chest impacts with vehicle components.
- TRRO has been crash-tested in accordance with ANSI/RESNA WC Vol I Section 19 Frontal Impact Test requirements for wheelchairs with a 130 lb (59 kg) crash test dummy, which corresponds to a person with a weight of 125 lb (57 kg) to 165 lb (75 kg) for Junior seat sizes or a 168 lb (76 kg) crash dummy, which corresponds to a person with a weight of 165 lb (75 kg) to 300 lbs (136 kg) for Adult seat sizes.

7.4 Compliance Information

This wheelchair conforms with the requirements of the ANSI/RESNA WC/Vol. I - Section 19 (Frontal Impact Test)



ANSI = American National Standards Institute, RESNA= Rehabilitation Engineering and Assistive Technology Society of North America.

This wheelchair has been dynamically tested in a forward-facing mode with the specified crash test dummy, which corresponds to a person with a weight of 114 lbs (52 kg) - 209 lbs (95 kg), restrained by Both pelvic and shoulder belts in accordance with ANSI/RESNA WC/Vol I – Section 19. Both pelvic and upper torso belts should be used to reduce the possibility of head and chest impacts with vehicle components.

7.5 Specifications

MODEL	MOTOR	WEIGHT LIMIT	
		Adult	Junior
TDX SP HD	4 Pole HD	Up to 400 lbs (181 kg)	N/A
TDX SP		Up to 300 lbs (136 kg)	Up to 150 lbs (68 kg)
TDX SR		Up to 300 lbs (136 kg)	Up to 150 lbs (68 kg)
TDX SR HD		Up to 400 lbs (181 kg)	N/A

7.6 Positioning the Wheelchair in the Vehicle

This wheelchair **MUST** be in a forward facing position during travel in a motor vehicle.

The recommended clear zones for wheelchair seated occupants restrained by both pelvic and upper torso belt(s) and only by a pelvic belt are shown in the diagrams and described below.


Frontal Clear Zones (FCZ) need to be **LARGER** when upper torso belt(s) are **NOT** used.

The rear clear zone of 16 inches (41 cm) is measured from the rearmost point on an occupant's head.

The frontal clear zone is measured from the frontmost point on an occupant's head and is 26 inches (66 cm) with pelvic and upper-torso belt(s) and 37 inches (94 cm) with **ONLY** a pelvic belt.


The frontal clear zone may not be achievable for wheelchair-seated drivers.

The estimated seated height (HHT) from the ground or floor to the top of the wheelchair-seated occupant's head ranges from approximately 47 inches (119 cm) for a small adult female to about 61 inches (155 cm) for a tall adult male

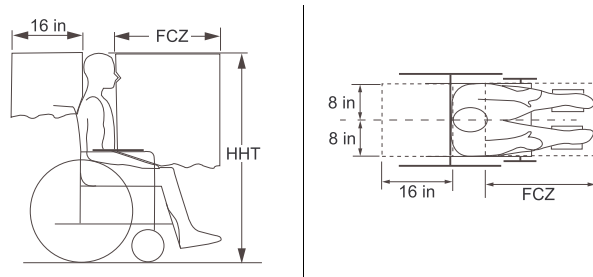

7.7 Securement Points

ITEM	DESCRIPTION
A	TIE DOWN BRACKETS

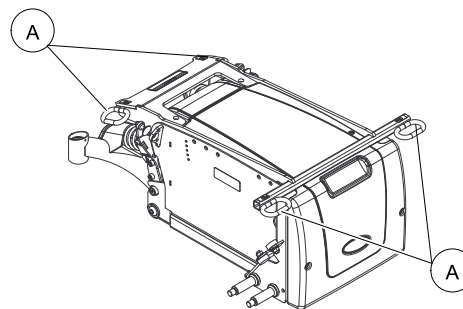

7.8 Securing the Wheelchair

This wheelchair is to be used only with Wheelchair Tie-down and Occupant Restraint Systems (WTORS) that have been installed in accordance with the manufacturer's instructions and SAE J2249.



A copy of SAE J2249 Wheelchair Tie-down and Occupant Restraint Systems (WTORS) for use in Motor Vehicles can be obtained from: SAE International, 400 Commonwealth Drive, Warrendale, PA 15096-0001, (877) 606-7232 or (724) 776-4970.

Attach WTORS to the tie-down brackets in accordance with the manufacturer's instructions and SAE J2249.

7.9 Securing the Occupant

7.9.1 Wheelchair-Anchored Belts





WARNING!

Risk of Injury, Damage or Death

Lack or improper use of wheelchair transport systems may result in injury, damage or death.


- Use both pelvic and upper torso belts.
- The pelvic belt that is provided by Invacare has been tested for use in a motor vehicle on this wheelchair only. Do not replace the pelvic belt with a different style pelvic belt.

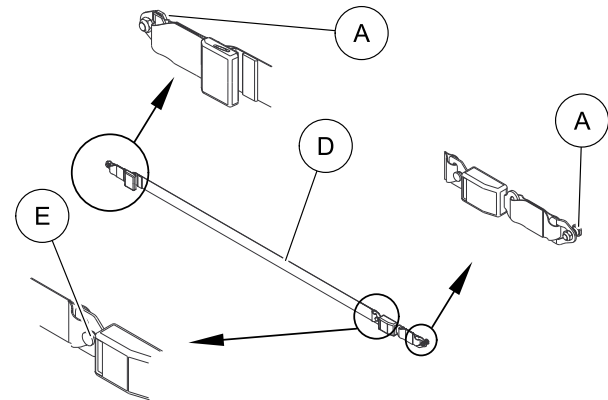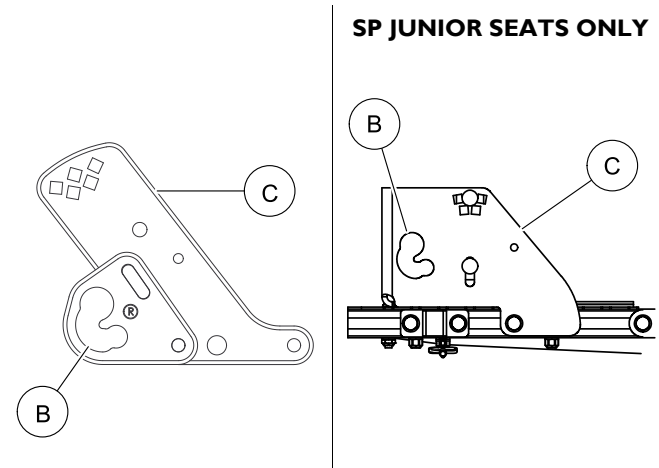

The wheelchair has been provided with a pelvic belt which meets the requirements of ANSI/RESNA WC/19.


The pelvic belt provided by Invacare has been designed to accommodate use on either side of the vehicle. If necessary, follow the instructions below to reverse the orientation of the pelvic belt to accommodate the vehicle-anchored upper torso belt.


1. Install the pelvic belt pin **A** into slot **B** in the pelvic belt bracket **C**. Pull upwards until it snaps into place.

 Both ends of the pelvic belt **D** have a pin which is used to secure the vehicle-anchored upper torso belt.

2. Repeat step 1 for the opposite pelvic belt bracket.
3. Install the vehicle-anchored upper torso belt onto the pin **E** located at either end of the pelvic belt.

7.9.2 Vehicle-Anchored Belts

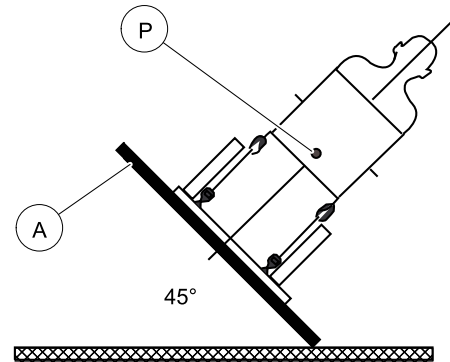

 Rear view of wheelchair and human surrogate secured on test platform **A** and tilted to 45°.

With regard to accommodating the use and fit of vehicle-anchored belts, this wheelchair has an overall rating of:

- TDX SP/TDX SR Adult/TDX SR Junior - “A”
- TDX SP Junior - “B”

This rating is scored as follows:

RATING	DESCRIPTION
A	Excellent
B	Good

RATING	DESCRIPTION
C	Fair
D	Poor

The test for Lateral Stability Displacement for Point P is shown in:


- TDX SP with Two 22NF Batteries -
 - TDX SP Adult - 0.53 in (13.4 mm)
 - TDX SP Junior - 0.57 in (14.4 mm)
- TDX SP/SR with Two GP24 Batteries -
 - TDX SP/SR Adult - 0.54 in (13.7 mm)
 - TDX SP Junior - 0.51 in (12.95 mm)
 - TDXR Junior - 0.29 in (7.46 mm)


7.9.3 Seating System

The wheelchair has been tested for seating in a motor vehicle with the factory installed seating system only.

Ensure that the factory installed seating system is secured to the wheelchair frame before operation. Refer to the seating system user manual.

7.9.4 Positioning Belts



	Preferred Zone
	Optional Zone

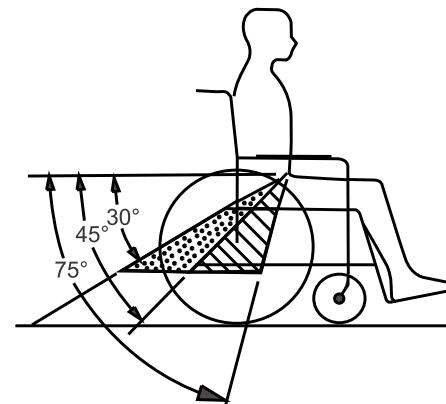

The angle of the pelvic belt should be within the preferred zone of 45 to 75 degrees to the horizontal OR within the optional zone of 30 to 45 degrees to the horizontal.

Steeper side-view pelvic belt angles are especially important if the pelvic belt is intended to be used for postural support in addition to occupant restraint in a frontal crash. Steeper angles will reduce the tendency for a vertical gap to develop between the user and the belt due to compliance of seat cushions and belt movement, thereby reducing the tendency for the user to slip under the belt and for the belt to ride up on the soft abdomen during normal use

Steeper belt angles also reduce the tendency for upper-torso belts to pull the pelvic belt onto the abdomen during frontal impact loading.
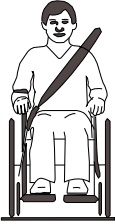
<p>DO position belts INSIDE of armrests, wheels, etc.</p>	<p>DO NOT position belts OUTSIDE of armrest, wheels, etc.</p>
	

1. The pelvic belt should be worn low across the front of the pelvis.
2. Position the upper torso belt(s) over the shoulders.
3. The belt(s) should not be held away from the body by wheelchair components or parts, including but not limited to wheelchair armrests or wheels.
4. Ensure the belt(s) are not be twisted.
5. Adjust belts as firmly as possible, being mindful of user comfort.

8 Setup and Maintenance

8.1 Setup/Delivery Inspection



Setup/delivery inspection should be performed by dealer at time of delivery/set up.

Initial adjustments should be made to suit your personal body structure needs and preference. Thereafter weekly, monthly and periodic inspections should be performed by user/attendant between the six month service inspections.

Every six months, and as necessary, take your wheelchair to a qualified technician for a thorough inspection and servicing.

- Check all parts for shipping damage. In case of damage, DO NOT use.
- Ensure wheelchair rolls straight (no excessive drag or pull to one side).
- Ensure clothing guards are secure.
- Ensure arms are secure but easy to release and adjustment levers engage properly.
- Ensure adjustable height arms operate and lock securely.
- Ensure axle nut or bolt and wheel mounting nuts are secure on drive wheels.
- Ensure caster/anti-tipper wheels are free of debris, and all mounting hardware is secure and not damaged/missing.
- Check that cables are routed and secured properly to ensure that cables do NOT become entangled and damaged during normal operation of seating system.
- Ensure proper operation of powered functions (Example: drive, seating and legrests).

8.2 Service Life

The expected service life is five years, presuming that the product is used daily and in accordance with safety instructions, maintenance instructions and intended use, stated in this manual.

8.3 Wear and Tear Information

General Information

Normal wear and tear items and components include but are not limited to: all upholstery items including seat and back upholstery, arm and calf pads, cushions, wheels, tires and casters, all types of batteries, joystick overlays and inductive rubberized protective boots.

Invacare reserves the right to ask for any item back that has an alleged defect in workmanship. Refer to the Warranty section in this manual for specific warranty information.

Refer to the Inspection Checklists for proper preventative maintenance schedule.

This is just a general guideline and does not include items damaged due to abuse and misuse.

Product Type	Product Wear and Tear
Wheelchairs	Wheels, Brake Assembly, Hand Grips
Mobility Hardware and Electronics	Rubber Urethane Tires and Casters, Handgrips, Joystick Inductive Tops, Joystick Overlays, Motors and Gearboxes (if exposed to prolonged moisture, urine, etc.), Stability Lock

Upholstery and Seating	Arm pads, Seat Cushion Foam, Seat Cushion Covers, Back Cushion Foam, Back Cushion Covers, Headrest Foam, Headrest Covers, Footplate Covers, Calf Pad (if applicable) Foam and Cover
Batteries	Lead acid/Lithium, Coin cell (watch type), Gel (6 months)

8.4 User/Attendant Inspection Checklists



Every six months, and as necessary, take your wheelchair to a qualified technician for a thorough inspection and servicing.

Weekly, monthly, and periodic inspections should be performed by user/attendant between the six month service inspections.

Regular cleaning will reveal loose or worn parts and enhance the smooth operation of your wheelchair. To operate properly and safely, your wheelchair **MUST** be cared for just like any other vehicle. Routine maintenance will extend the life and efficiency of your wheelchair.



DANGER!

Risk of Injury, Damage or Death

Overinflation of tires may cause tires to explode.

- Inflate tire to the proper tire pressure (P.S.I. / kilopascals) listed on the side wall of the tire.
- Only use wheelchair with tires at proper pressure.
- The wheels and tires should be checked periodically for cracks and wear and should be replaced if necessary.

8.4.1 Inspect/Adjust Weekly

- Ensure that the casters are free of debris.
- Inspect tires for flat spots and wear.
- Inspect all fasteners.
- Inspect TRBKTS fasteners and hardware.
- Ensure proper operation of powered functions (Example: drive, seating and legrests).

8.4.2 Inspect/Adjust Monthly

- Clean upholstery and armrests.
- Clean dirt and lint from axles.
- Clean dirt and lint from bearings.
- Ensure that the casters are free of debris.
- Inspect seat positioning strap for any signs of wear. Ensure buckle latches. Verify hardware that attaches strap to frame is secure and undamaged. Replace if necessary.

8.4.3 Inspect/Adjust Periodically

- Ensure wheelchair rolls straight (no excessive drag or pull to one side).
- Inspect all operator (user/attendant) adjustable fasteners including the back pan, back cane and angle adjustment fasteners, and the arm support, flip back and height adjustment fasteners. Ensure fasteners are securely tightened.
- Inspect TRBKTS fasteners and hardware.
- Ensure clothing guards are secure.
- Ensure arms are secure but easy to release and adjustment levers engage properly.
- Ensure adjustable height arms operate and lock securely.
- Ensure upholstery does not have any rips or tears.
- Armrest pad sits flush against arm tube.
- Ensure that the casters are free of debris.
- Inspect foam handgrips for damage. If damaged, have them replaced by a qualified technician.

- Check center mount front riggings for loose fasteners. Replace /tighten if necessary.
- Check that all labels are present and legible. Replace if necessary.

8.5 Service Inspection



Every six months take your wheelchair to a qualified technician for a thorough inspection and servicing.

Service inspections **MUST** be performed by a qualified technician.



DANGER!

Risk of Injury, Damage or Death

Overinflation of tires may cause tires to explode.

- Inflate tire to the proper tire pressure (P.S.I. / kilopascals) listed on the side wall of the tire.
- Only use wheelchair with tires at proper pressure.
- The wheels and tires should be checked periodically for cracks and wear and should be replaced if necessary.



WARNING!

Risk of Serious Injury or Damage

Attaching hardware that is loosely secured could cause loss of stability resulting in serious injury or damage.

- After ANY adjustments, repair or service and before use, make sure that all attaching hardware is tightened securely.

The following are recommended items to inspect during regular service inspections performed by a qualified technician. Actual items to be inspected during the service inspection may vary according to the specific wheelchair:

8.5.1 Six Month Inspection

- Inspect stability lock cylinders, front and rear swing arms to ensure proper operation.
- Ensure adjustable height arms operate and lock securely.
- Ensure arms are secure but easy to release and adjustment levers engage properly.
- Inspect seat positioning strap for any signs of wear. Ensure buckle latches. Verify hardware that attaches strap to frame is secure and undamaged. Replace if necessary.
- Ensure axle nut or bolt and wheel mounting nuts are secure on drive wheels.
- Inspect tires for flat spots and wear.
- Loosen/tighten caster locknut if wheel wobbles noticeably or binds to a stop.
- Ensure all caster/wheel/fork/headtube fasteners are secure and not damaged/missing.
- Check center mount front riggings for loose fasteners. Replace /tighten if necessary.
- Cables shall be inspected periodically to ensure that they are routed and secured properly. Periodic inspection is recommended as it may reveal loose and/or damaged cables. Re-secure all loose cables and replace by following the recommendations outlined in the Power Wheelchair Wiring Guide, p/n 1167603.
- Ensure proper operation of powered functions (drive, seating, legrests, etc.).
- Inspect motor brushes and gearbox coupling.
- Inspect electrical components for signs of corrosion. Replace if corroded or damaged.
- Inspect battery terminals for loose cable connection. Tighten if necessary.
- Ensure swingarm stops are in place and not deteriorated or damaged. Replace if necessary.

- Clean upholstery and armrests.
- Clean dirt and lint from axles.
- Clean dirt and lint from bearings.
- Check that all labels are present and legible. Replace if necessary.
- Ensure clothing guards are secure.
- Ensure upholstery does not have any rips or tears.
- Ensure armrest pad sits flush against arm tube.
- Ensure wheelchair rolls straight (no excessive drag or pull to one side).
- Ensure that the casters are free of debris.
- Inspect all fasteners including the back pan, back cane and angle adjustment fasteners, and the arm support, flip back and height adjustment fasteners. Ensure fasteners are securely tightened.
- Inspect TRBKTS fasteners and hardware.
- Inspect foam handgrips for damage. If damaged, have them replaced by a qualified technician.

8.5.2 Inspect/Adjust Every 18 Months

Replace motor brushes and gearbox coupling.

8.5.3 Inspect/Adjust Every 2 Years

The stability locking cylinders MUST be replaced every two years.

8.6 Cleaning



WARNING!

Risk of Injury, Damage or Death

- Electrical shock may cause injury, damage or death.
- Always unplug the product from the electrical outlet before cleaning.
 - Always unplug accessories from the electrical outlet before cleaning.



CAUTION!

Risk of Damage

- Cleaning or maintenance may cause damage to carpeting or flooring.
- Place the wheelchair in a well ventilated area where cleaning or maintenance can be performed without risk of damage to carpeting or flooring.



CAUTION!

Risk of Damage

- Exposure to liquids may damage components or accessories of wheelchair and electronics.
- DO NOT spray with any type of water or liquid.
 - Electrical components damaged by corrosion MUST be replaced immediately.



WARNING!

Risk of Injury, Damage or Death

- Excessive moisture or cleaning may reduce the flame retardancy of the upholstery and may result in injury, damage or death.
- Follow all cleaning instructions.
 - Avoid excessive moisture or cleaning.



Regular cleaning will reveal loose or worn parts and enhance the smooth operation of your wheelchair. To operate properly and safely, your wheelchair must be cared for just like any other vehicle.

For upholstery that is severely stained or surface finish that is badly damaged, contact Invacare for further information.

- Use the following instructions to clean this product unless otherwise specified.
 - Upholstery — Warm water and mild non-abrasive soap.
 - Metal — Hot water and mild non-abrasive soap. Car polish and soft wax may be used to remove abrasions and restore gloss.
 - Plastic — Hot water and mild non-abrasive soap.
- Dry the surface with dry cloth.
- DO NOT use solvents or kitchen cleaners.

8.7 Stability Lock




DANGER!

Risk of Death or Serious Injury

Not performing periodic maintenance on stability lock could result in death or serious injury.

– ALWAYS perform the periodic maintenance to the stability lock listed in the inspection checklist of this manual.

8.8 Removing/Installing the Shrouds

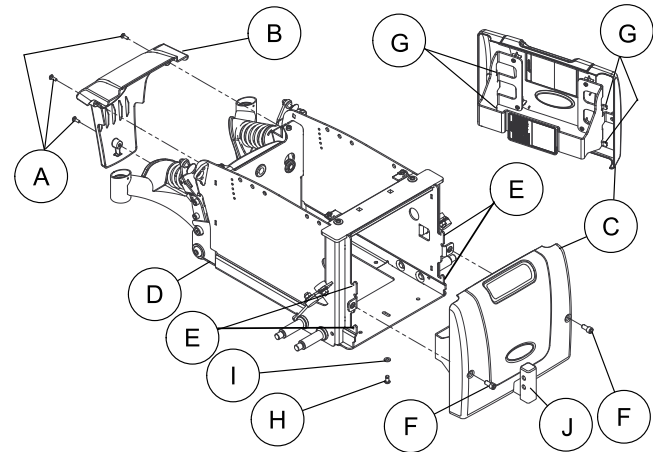

8.8.1 Removing the Rear Shroud

- Verify the joystick On/Off switch is in the Off position.
- Remove the three mounting screw (A) securing the rear shroud (B) to the wheelchair.
- Remove the rear shroud from the wheelchair (D).

8.8.2 Installing the Rear Shroud

- Install the rear shroud (B) and secure in place with mounting screw(s) (A).




New Batteries MUST be fully charged before using, otherwise the life of the battery(ies) will be reduced.

8.8.3 Removing/Installing the Front Shroud/Battery Retention Bracket

Removing

The front shroud/battery retention bracket ③ is removed from and installed onto the wheelchair frame ④ as a single unit.


Front shroud/battery retention bracket ③ mounts on hooks ⑤ located on the wheelchair frame ④.

 Have the following tools available: 1/4 inch Allen Wrench, 7/16 inch Box Wrench.

1. Remove the mounting screws ⑥ securing the front shroud/battery retention bracket to the wheelchair frame ④.
2. Lift up to unhook the front shroud/battery retention bracket from the base frame and remove.

Installing

 Wheelchairs with TRRO or TRBKTS Only - Battery retention brackets MUST be installed at all times.

 When performing STEP 1, ensure that the tabs ③ located on the sides of the front shroud/battery retention bracket ③ are positioned on the hooks ⑤ located on the front of the wheelchair frame ④.

1. Reinstall the front shroud/battery retention bracket onto front of wheelchair frame.
2. Secure the front shroud/battery retention bracket ③ to the wheelchair frame ④ with two mounting screws ⑥. Securely tighten.

3. Gently, pull on the front shroud to ensure the mounting screws are fully engaged into the battery box.
4. Wheelchairs with Footboards Only: Using the mounting screw ⑥ and washer ①, secure the footboard support ① to the wheelchair frame.

8.9 Batteries



WARNING!

Risk of Death or Serious Injury

Failure to observe these warnings can cause an electrical short resulting in death, serious injury, or damage to the electrical system.

- The POSITIVE (+) RED battery cable MUST connect to the POSITIVE (+) battery terminal(s)/post(s).
- The NEGATIVE (-) BLACK battery cable MUST connect to the NEGATIVE (-) battery terminal(s)/post(s).
- NEVER allow any of your tools and/or battery cable(s) to contact BOTH battery post(s) at the same time. An electrical short may occur and serious injury or damage may occur.
- Install protective caps on positive and negative battery terminals.
- Replace cable(s) immediately if cable(s) insulation becomes damaged.
- DO NOT remove fuse or mounting hardware from POSITIVE (+) red battery cable mounting screw.

**WARNING!****Risk of Injury or Damage**

Improper installation of the battery can result in injury or damage.

- Batteries can weigh up to 52 lbs (23.6 kg). ALWAYS use a battery lifting strap when lifting the battery. It is the most reliable method of carrying a battery and preventing serious injury.

**WARNING!****Risk of Injury or Damage**

Improper lifting technique may cause injury or damage.

- Use proper lifting techniques, assistance and gear such as straps when available when lifting heavy loads.

**WARNING!****Risk of Injury**

Exposure to battery acid may result in injury.

- The use of rubber gloves is recommended when working with batteries.
- DO NOT allow the liquid in the battery to come in contact with skin, clothes or other possessions. It is a form of acid and harmful or damaging burns may result. Should the liquid touch your skin, wash the area IMMEDIATELY and thoroughly with cool water. In serious cases or if eye contact is made, seek medical attention IMMEDIATELY.
- DO NOT install/reinstall a battery with a cracked or otherwise damaged case.



Invacare strongly recommends battery installation and replacement be performed by a qualified technician.

Read the installation instructions noted on the battery and in the manual.

Ensure there is no battery acid in the bottom or around the battery box or on the sides of the battery(ies). Neutralize battery acid with baking soda if found.

Clean battery tray and batteries prior to installation.

Use battery lifting strap when available.

Keep batteries in an upright position and avoid tipping.

Use only deep cycle gel cell battery.

Use box wrench when tightening clamps. Do Not wiggle the battery terminals/posts when tightening.

8.10 Using the Proper Batteries



WARNING!

Risk of Injury or Damage

Improper configuration of battery terminals may cause injury or damage.

- Batteries with terminal configuration as shown MUST be used.
- Terminals MUST have a cross hole in them as shown.
- The POSITIVE (+) RED battery cable MUST connect to the POSITIVE (+) battery terminals/posts.
- The NEGATIVE (-) BLACK battery cable MUST connect to the NEGATIVE (-) battery terminals/posts.
- DO NOT allow any of your tools and/or battery cables to contact both battery terminals at the same time.



CAUTION!

Risk of Damage

Use of wrong battery type or size may cause damage.

- DO NOT force a battery into place.
- Batteries with molded straps or flanges that interfere with proper battery placement should not be used.
- Use batteries listed in this manual for proper replacement.



Failure to use the correct battery size and/or voltage may cause damage to your wheelchair and give you unsatisfactory performance. The warranty and performance specifications contained in this manual are based on the use of deep cycle gel cell batteries. Invacare strongly recommends their use as the power source for this unit.

Carefully read battery/battery charger information prior to installing, servicing or operating your wheelchair.

1. Position battery on ground/flat surface as shown below.
2. Visually inspect the battery to ensure proper polarity:

FOR WHEELCHAIRS USING 22NF BATTERIES	
<p>Batteries with terminal configuration (POSITIVE A on the left and NEGATIVE B on the right) as shown below MUST be used. Batteries that have the reverse terminal configuration MUST not be used - otherwise injury and damage may occur.</p> <p>Terminals must have a cross hole C in them as shown below.</p>	
PROPER BATTERIES TO USE	DO NOT USE THIS TYPE OF BATTERY



FOR WHEELCHAIRS USING GP24 BATTERIES

Batteries with terminal configuration (POSITIVE Ⓐ on the right and NEGATIVE Ⓑ on the left) as shown below **MUST** be used. Batteries that have the reverse terminal configuration **MUST** not be used - otherwise injury and damage may occur.

Terminals must have a cross hole Ⓒ in them as shown below.

PROPER BATTERIES TO USE

DO NOT USE THIS TYPE OF BATTERY

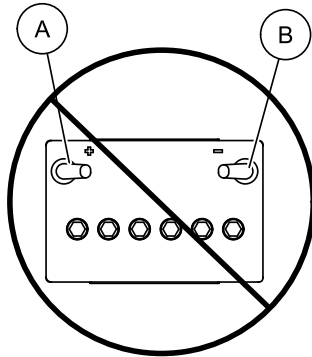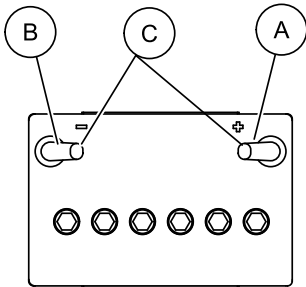

8.11 Removing/Installing the Batteries From/Into the Wheelchair

8.11.1 Removing the Batteries from the Wheelchair



CAUTION!


Risk of Damage

Cleaning or maintenance may cause damage to carpeting or flooring.


- Place the wheelchair in a well ventilated area where cleaning or maintenance can be performed without risk of damage to carpeting or flooring.



22NF Batteries shown

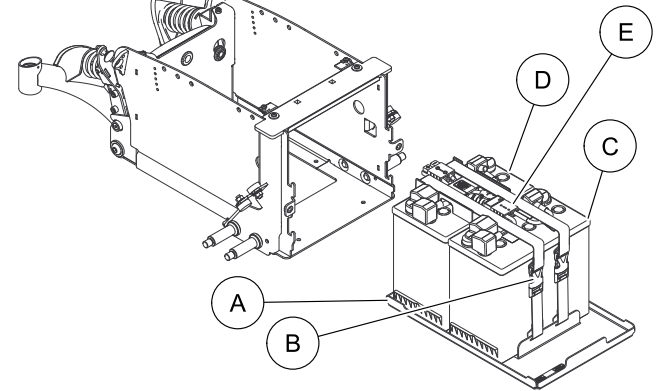


1. Place the wheelchair in a well ventilated area.
2. Verify the joystick On/Off switch is in the Off position and disconnect joystick. Refer to 8.17.1 Disconnecting the SPJ+ Joysticks, page 72 or 8.18.2 Disconnecting the CMPJ+ Joysticks, page 73.
3. Remove the rear shroud, front shroud and battery retention bracket. Refer to 8.8.3 Removing/Installing the Front Shroud/Battery Retention Bracket, page 62.
4. Disconnect the controller from the batteries at the rear of the wheelchair.
5. Slide battery tray (A) with batteries out.
6. Disconnect the battery straps (B).

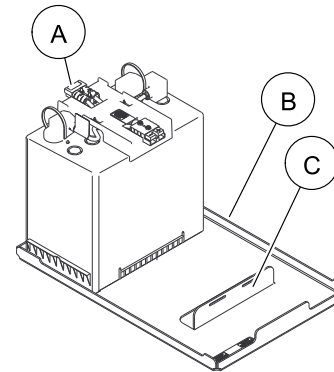

 Battery connection (E) located on top.

7. Disconnect batteries.
8. Remove the front (C) battery.
9. Slide the rear battery (D) forward and remove it from the tray.

8.11.2 Installing Batteries into the Wheelchair

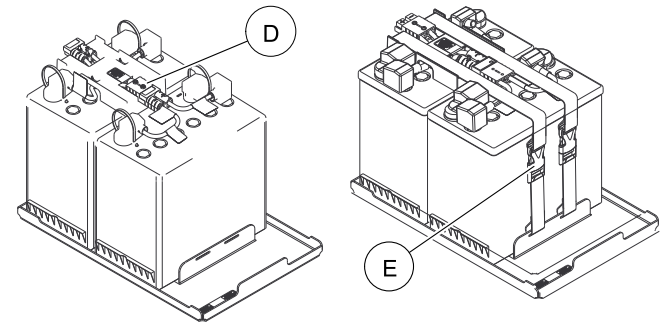

Positioning of the batteries into the battery tray is completed with battery tray positioned in wheelchair and partially pulled out. Illustrations are shown without the wheelchair for clarification purposes only.




WARNING!

Risk of Serious Injury

Improperly installed battery tray can cause instability resulting in serious injury.

- Ensure batteries and battery tray are installed properly to maintain stability.

- Position the battery with battery connector bracket **A** against the back edge of the battery tray **B** in the orientation as shown.


 Batteries **MUST** be installed in the orientation shown so that the wiring harnesses can be connected together.



- Position the remaining battery between the battery on the tray and battery stop **C**.
- Connect batteries together **D**.
- Connect battery straps **E**.
- Slide the battery tray into the wheelchair
- Install the rear shroud, front shroud and battery retention bracket. Refer to 8.8.3 Removing/Installing the Front Shroud/Battery Retention Bracket, page 62.

8.12 Replacing Batteries and/or Battery Cables



Have the following tools available: 7/16 inch (6pt) Box Wrench and Diagonal Cutters.

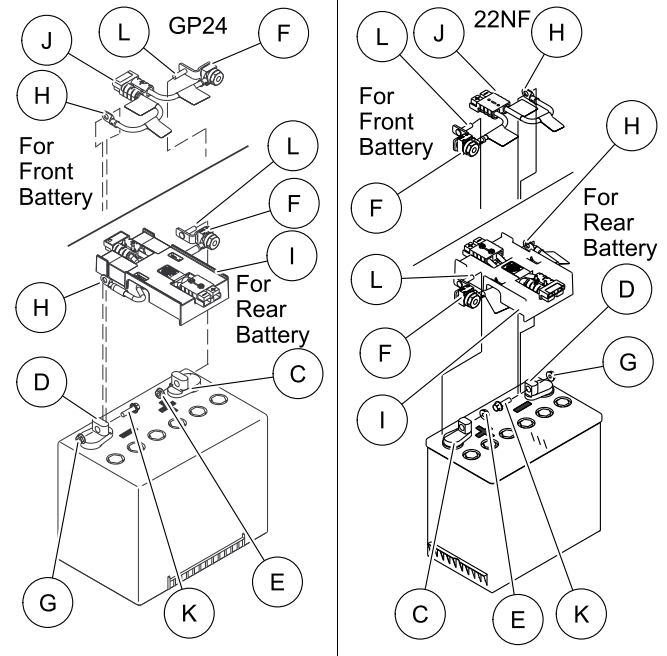

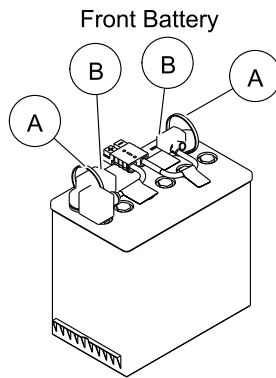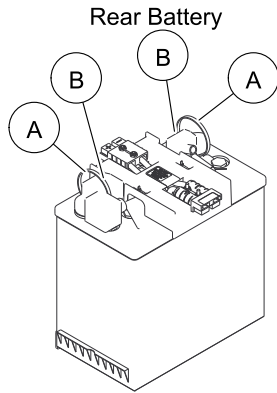

WARNING!

Risk of Death or Serious Injury

Failure to observe these warnings can cause an electrical short resulting in death, serious injury, or damage to the electrical system.

- The POSITIVE (+) RED battery cable MUST connect to the POSITIVE (+) battery terminal(s)/post(s).
- The NEGATIVE (-) BLACK battery cable MUST connect to the NEGATIVE (-) battery terminal(s)/post(s).
- NEVER allow any of your tools and/or battery cable(s) to contact BOTH battery post(s) at the same time. An electrical short may occur and serious injury or damage may occur.
- Install protective caps on positive and negative battery terminals.
- Replace cable(s) immediately if cable(s) insulation becomes damaged.
- DO NOT remove fuse or mounting hardware from POSITIVE (+) red battery cable mounting screw.

1. Remove the batteries from the wheelchair. Refer to 8.1.1 Removing/Installing the Batteries From/Into the Wheelchair, page 65.
2. Cut the tie-wraps (A) that secure the battery terminal covers (B) to the battery terminals.
3. Slide the RED battery terminal cover back on the red battery cable to expose the POSITIVE battery terminal (C).
4. Slide the BLACK battery terminal cover back on the Black battery cable to expose NEGATIVE battery terminal (D).
5. Remove the locknut (E) that secures the bracket (F) of the POSITIVE battery cable to the POSITIVE (+) battery post of the battery.

6. Remove the locknut ⑥ that secures the NEGATIVE battery cable ④ to the NEGATIVE (-) battery post of the battery.
7. Discard the existing battery.
8. Position battery connector bracket ① or wiring harness ① onto the new 22NF battery or GP24 battery as shown.
9. Secure the NEGATIVE battery cable ④ to the NEGATIVE (-) battery post with existing mounting screw ⑤ and locknut ⑥.
10. Secure the bracket ② of the POSITIVE battery cable to the POSITIVE (+) battery post with existing mounting screw ③ and locknut ⑦.
11. Position each battery terminal cover over top of each battery terminal.
12. Secure battery terminal covers in place with one tie-wrap.
13. Install batteries into wheelchair. Refer to 8.11 7.8 Removing/Installing the Batteries From/Into the Wheelchair, page 65.

8.12.1 Cleaning Battery Terminals



WARNING!

Risk of Injury

Exposure to battery acid may result in injury.

- The use of rubber gloves is recommended when working with batteries.
- DO NOT allow the liquid in the battery to come in contact with skin, clothes or other possessions. It is a form of acid and harmful or damaging burns may result. Should the liquid touch your skin, wash the area IMMEDIATELY and thoroughly with cool water. In serious cases or if eye contact is made, seek medical attention IMMEDIATELY.
- DO NOT install/reinstall a battery with a cracked or otherwise damaged case.


1. Examine battery terminals for corrosion.
2. Verify the plastic caps are in place over battery cell holes.
3. Clean terminals by using a battery cleaning tool, wire brush, or medium grade sand paper.



Upon completion, areas should be shiny, not dull.

4. Carefully dust off all metal particles.

8.13 Adjusting Forks

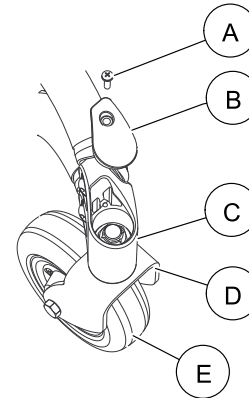


This procedure must be performed by a qualified technician.

This procedure applies to both Forks.

1. Remove the retaining screw ① securing the head tube cover ② in place.
2. Remove the head tube cover (not shown) from the caster head tube ③.

3. To properly tighten caster journal system and guard against flutter, perform the following check:
 - a. Tip the wheelchair forward, toward the floor.
 - b. Pivot both forks ④ and casters ⑤ to top of their arc simultaneously.
 - c. Let casters drop to bottom of arc (wheels should swing once to one-side, then immediately rest in a straight downward position).
 - d. Adjust locknuts ⑥ according to freedom of caster swing.
4. Test wheelchair for maneuverability.
5. Readjust locknuts if necessary, and repeat STEPS 1-3 until correct.
6. Snap headtube cover into the caster headtube.
7. Reinstall retaining screws.

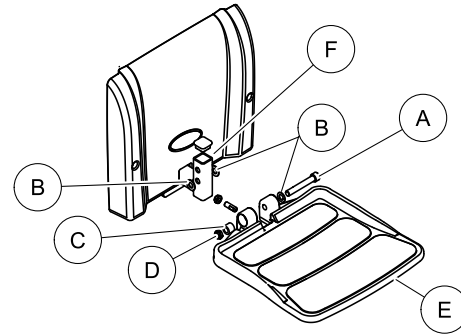

8.14 Removing/Installing the Footboard or Adjusting the Footboard Height



WARNING!

– **LIMITED CLEARANCE BETWEEN FOOTBOARD AND CASTER** - The user's feet must remain on the footboard while operating the wheelchair. If the user's feet are allowed to rest off the side of the footboard they may come in contact with the caster possibly resulting in injury.

Removing the Footboard

1. Remove the socket head screw ①, three washers ②, spacer ③ and locknut ④ that secures the footboard ⑤ to the footboard support ⑥.
2. Remove the footboard from the footboard support.

Installing the Footboard



WARNING!


– Pinch point may occur when rotating the footboard assembly.

1. Position the footboard onto the footboard support so that the mounting holes in the wheelchair frame align with the desired mounting holes in the footboard support.
2. Using the socket head screw, three washers, spacer and locknut secure the footboard to the footboard support.

Adjusting the Footboard Height


1. Remove the footboard. Refer to Removing the Footboard.
2. Adjust the footboard to one of two mounting holes in the footboard support to adjust the footboard height by 1 inch.
3. Install the footboard. Refer to Installing the footboard

8.15 Adjusting the Footboard Angle

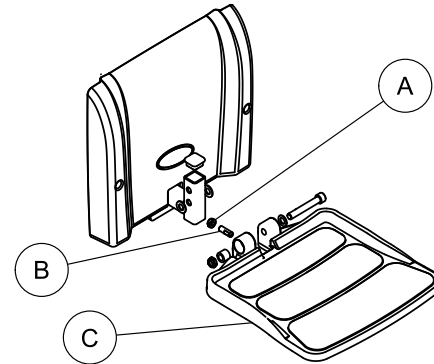


1. Loosen the jam nut **A** and set screw **B** located underneath on the rear of the footplate **C**.
2. Adjust the mounting screw in or out to obtain the desired footboard angle.
3. Thread the jam nut inward until it is flush with the footboard bracket.
4. Securely tighten the jam nut and washer to secure the mounting screw in place.


8.16 Repositioning Joystick

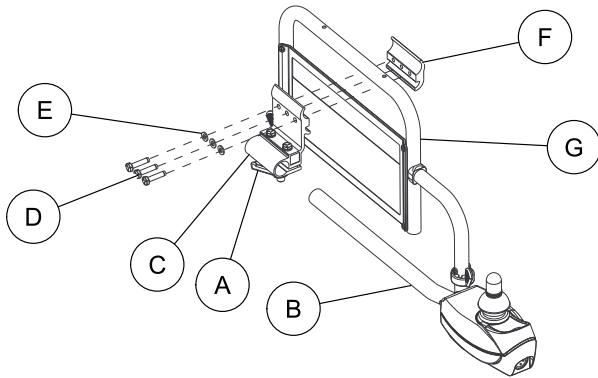

1. Turn the adjustment lock lever **A** to release the joystick mounting tube **B** from the joystick mounting bracket **C**.
2. Remove the joystick from the wheelchair.
3. Remove the three hex screws **D** and washers **E** that secures the joystick mounting bracket to the threaded mounting bracket **F**.
4. Reposition threaded mounting bracket on the inside of the opposite arm tube **G** as shown.
5. Using the three hex mounting screws and washers, secure joystick mounting bracket to the threaded mounting bracket.
6. Slide the joystick mounting tube through the joystick mounting bracket to the desired position.
7. Turn the adjustment lock lever to secure the joystick mounting tube into the mounting bracket.

8.17 Disconnecting/Connecting the SPJ+ Joysticks

 The joystick connector is located at the rear of the seat frame.

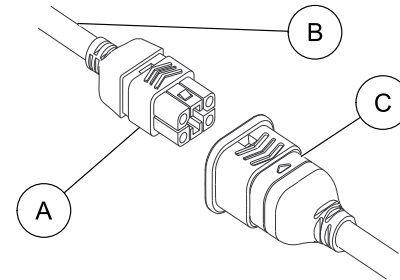

8.17.1 Disconnecting the SPJ+ Joysticks

1. Hold the light grey collar portion **A** of the joystick connector **B** with one hand and the controller connector **C** on the wheelchair in the other and disconnect them by pulling them apart.

8.17.2 Connecting the SPJ+ Joysticks




WARNING! **Risk of Injury or Damage**


Improper assembly or maintenance may cause injury or damage.

- The joystick connector and controller connector fit together in one way only. **DO NOT** force them together.


1. Hold the light grey collar portion **A** of the joystick connector **B** with one hand and the controller connector **C** on the wheelchair in the other and align them.
2. Lightly push to engage the joystick connector and the controller connector.

8.18 Disconnecting/Connecting the CMPJ+ Joysticks

 The joystick connector is located at the rear of the seat frame.

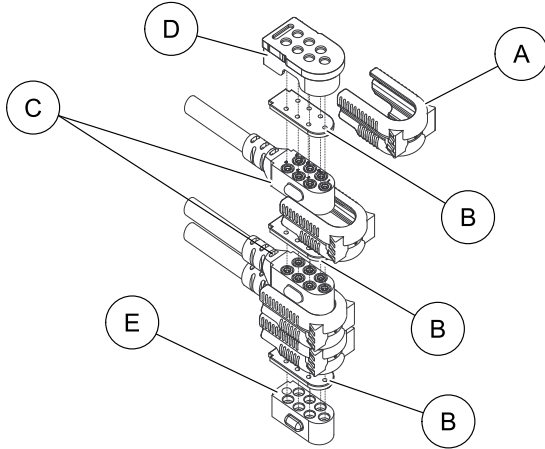

8.18.1 Connecting the CMPJ+ Joysticks

 Ensure the gaskets **B** are installed in the top connector cap and between network connectors **C**.

1. Ensure the latch is pulled away from the network connector.
2. Connect the network connector to the other connectors.
3. Top **D** and Bottom **E** Connectors - Install connector caps onto the network connector.
4. Push the latch in to secure the network connectors and caps.
5. If necessary secure excess cable using tie-wraps.

8.18.2 Disconnecting the CMPJ+ Joysticks

1. Pull the latch **A** away from the joystick connector.
2. Disconnect the joystick connector from the remaining connectors.

9 Troubleshooting

9.1 Driving Performance

	Solutions		
	Check for loose stem nuts/bolts, bearings or signs of wear.	Check for uneven tire wear, bent fork/frame or loose hardware.	If pneumatic, check tires for correct and equal pressure.
Wheelchair Veers Left or Right	X	X	X
Sluggish Turn or Performance	X		X
Casters Flutter	X	X	X
Squeaks and Rattles		X	
Looseness in Wheelchair		X	X
Wheelchair 3 Wheels		X	X

9.2 Electrical









For additional troubleshooting information and explanation of error codes, refer to the individual Electronics Manual supplied with each wheelchair.

9.2.1 SPJ+, SPJ+ w/PSS or SPJ+ w/ACC Joysticks

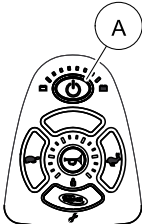
The joystick information gauge and the service indicator give indications of the type of fault or error detected by the control module. When a fault is detected, the wheelchair may stop and not drive. The LEDs on the information gauge may flash in a particular pattern or the service indicator light will flash. The number or type of flashes indicates the nature of the error. If multiple errors are found, only the first error encountered by the control module will be displayed.

Information Gauge Display Diagnostics

Display	Description	Definition	Comments
			
	All LEDs are off.	Power is off.	

Display	Description	Definition	Comments
	All LEDs are on.	Power is on.	Fewer than three LEDs on implies reduced battery charge.
	Left Red LED is flashing.	Battery charge is low.	The batteries should be charged as soon as possible.
	Left to Right "chase" alternating with steady display.	Joystick is in programming, inhibit and/or charging mode.	The steady LEDs indicate the current state of the battery charge.
	All LEDs are flashing slowly.	Joystick has detected Out-of-Neutral-at-Power-Up mode.	Release the joystick back to Neutral.

Service Indicator Light A Diagnostics

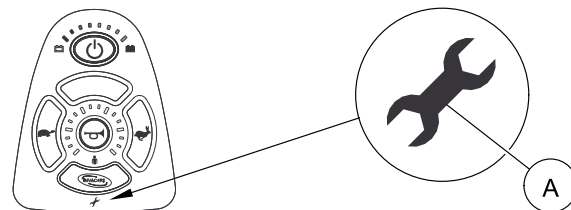

NUMBER OF FLASHES	ERROR CODE DESCRIPTION	POSSIBLE SOLUTION
1	User Fault	Release joystick to neutral and try again.
2	Battery Fault	Charge the batteries. Check that battery cables are connected properly. If necessary, replace batteries.
3	Left Motor Fault	Contact Invacare/Dealer for service.
4	Right Motor Fault	Contact Invacare/Dealer for service.

NUMBER OF FLASHES	ERROR CODE DESCRIPTION	POSSIBLE SOLUTION
5	Left Park Brake Fault	Ensure brake lever is in the drive position before turning on the wheelchair. Ensure motor cable is plugged into the controller. Contact Invacare/Dealer for service.
6	Right Park Brake Fault	Ensure brake lever is in the drive position before turning on the wheelchair. Ensure motor cable is plugged into the controller. Contact Invacare/Dealer for service.
7	Remote Fault	Check to make sure joystick is connected properly. Turn Joystick off then on. Contact Invacare/Dealer for service.
8	Controller Fault	Contact Invacare/Dealer for service.

NUMBER OF FLASHES	ERROR CODE DESCRIPTION	POSSIBLE SOLUTION
9	Communications Fault	Check joystick cable connections. Check joystick cable and connectors for damage. Contact Invacare/Dealer for service.
10	General Fault	Contact Invacare/Dealer for service.
11	Incompatible or incorrect Remote	Wrong type of remote connected. Contact Invacare/Dealer for service.

9.2.2 CMPJ+ Joysticks or Displays



Error codes are displayed in the Fault Log on the hand help programmer.

SYMPTOM	PROBABLE CAUSE	SOLUTIONS
LEFT BRAKE FAULT or RIGHT BRAKE FAULT displays and wheelchair does not drive.	Motor lock levers disengaged (Error code E9 or E10).	Engage motor lock levers. Refer to 6.8 Disengaging/Engaging the Motor Lock Levers, page 47.
CHARGER PLUGGED IN displays.	Battery charger connected (Error code E28).	Unplug battery charger from the wheelchair. Refer to 6.7 Charging Batteries, page 45.
BATTERY FAULT displays and the wheelchair does not drive.	Batteries need to be charged (Error code E14).	Charge batteries. Refer to 6.7 Charging Batteries, page 45. If batteries fail to charge properly, check battery charger or replace batteries. Refer to 8.12 Replacing Batteries and/or Battery Cables, page 67.

SYMPTOM	PROBABLE CAUSE	SOLUTIONS
JOYSTICK TIMEOUT displays and the wheelchair does not drive.	Joystick or input device is disconnected (Error code 32).	Turn off power, reconnect the joystick of input device and turn power on.
JOYSTICK FAULT displays and the wheelchair does not drive.	The joystick or input device is sending a value outside of the reverse, forward, left or right limits (Error codes E01, E02, E03 or E04).	Replace joystick or input device.
NEUTRAL TESTING displays.	The joystick neutral test has failed (Error code E18).	Release the joystick and try to get the joystick back into the center-most position.
BAD JOYSTICK CAL VALUES displays and the wheelchair does not drive.	The joystick calibration values are outside of the expected range (Error code E19).	Recalibrate the joystick (joystick throw procedure).

SYMPTOM	PROBABLE CAUSE	SOLUTIONS
CTRL NOT CONNECTED	The CMPJ or Display module is not communicating with the control module (Error code E200).	Check the connections between the joystick or display and the controller. Turn the power off and then back on. Replace the controller if necessary.
CTRL COM FAULT displays and the wheelchair drives slowly.	The controller has determined a fault during a previous turn-off process (Error code E41).	Turn the wheelchair off and back on.
MISSING CONFIGURATION displays at power up.	One or more devices have been removed or disconnected from the wheelchair.	Reconnect the device.
ATTENDANT ACTIVE and displays.	The Proportional or Digital Attendant control is active and can be used to drive the chair.	This is normal behavior.

SYMPTOM	PROBABLE CAUSE	SOLUTIONS
Batteries draw excessive current when charging.	Battery failure. Electrical malfunction.	Have batteries checked for shorted cell. Replace if necessary. Contact Dealer/Invacare for service.
Battery indicator flashes the charge level is low - immediately after recharge.	Battery failure. Malfunctioning battery charger. Electrical malfunction.	Check batteries for shorted cell. Replace if necessary. Contact Dealer/Invacare for Service. Contact Dealer/Invacare for Service.
Battery indicator flashes the charge level is low - too soon after being recharged.	Batteries not charged. Weak batteries.	Have charger checked. Replace batteries if necessary. Refer to 8.12 Replacing Batteries and/or Battery Cables, page 67.

SYMPTOM	PROBABLE CAUSE	SOLUTIONS
Motor “chatters” or runs irregular.	Motor/gearbox malfunction.	Stop use of Wheelchair. Contact Dealer/Invacare for Service.
Joystick erratic or does not respond as desired.	Damaged motor coupling. Electrical malfunction. Controller programmed improperly.	Contact Dealer/Invacare for Service. Contact Dealer/Invacare for Service. Contact Dealer/Invacare to have controller reprogrammed.
Wheelchair does not respond to commands.	Electrical malfunction.	Contact Dealer/Invacare for Service.
Power indicator off - even after recharging.	Poor battery terminal connection.	Have terminals cleaned.

9.3 Checking Battery Charge Level

The following “Do’s” and “Don’ts” are provided for your convenience and safety.

DON'T	DO
Don't perform any installation or maintenance without first reading this manual.	Read and understand this manual and any service information that accompanies a battery and charger before operating the wheelchair.
Don't perform installation or maintenance of batteries in an area that could be damaged by battery spills.	Move the wheelchair to a work area before cleaning terminals, or opening battery box.
Don't make it a habit to discharge batteries to the lowest level.	Recharge as frequently as possible to maintain a high charge level and extend battery life.
Don't use randomly chosen batteries or chargers.	Follow recommendations in this manual when selecting a battery or charger.
Don't put new batteries into service before charging.	Fully charge a new battery before using.
Don't tip or tilt batteries.	Use a carrying strap to remove, move or install a battery.
Don't tap on clamps and terminals with tools.	Push battery clamps on the terminals. Spread clamps wider if necessary.
Don't mismatch your battery and chargers.	Use only a gel charger for a gel battery.

10 Warranty

10.1 Global Limited Warranty (Excluding Canada)



PLEASE NOTE: THE WARRANTY BELOW HAS BEEN DRAFTED TO COMPLY WITH FEDERAL LAW APPLICABLE TO PRODUCTS MANUFACTURED AFTER JULY 4, 1975.

This warranty is extended only to the original purchaser who purchases this product within any country excluding CANADA when new and unused from Invacare or a dealer. This warranty is not extended to any other person or entity and is not transferable or assignable to any subsequent purchaser or owner. Coverage under this warranty will end upon any such subsequent sale or other transfer of title to any other person. For product purchased in Canada, please refer to the Canada Limited Warranty.

This warranty gives you specific legal rights and you may also have other legal rights which vary from state to state.

Invacare warrants the base frame to be free from defects in materials and workmanship for a period of five (5) years from the date of purchase from Invacare or a dealer, with a copy of the seller's invoice required for coverage under this warranty. Invacare warrants all electronics and electrical components (excluding batteries), motors, powered seating actuators and gearboxes to be free from defects in materials and workmanship for a period of one (1) year from the date of purchase from Invacare or a dealer, with a copy of the seller's invoice required for coverage under this warranty. Invacare warrants all batteries to be free from defects in materials and workmanship for a period of six (6) months from the date of purchase from Invacare or a dealer, with a copy of the seller's invoice required for coverage

under this warranty. Invacare warrants all remaining components (excluding all upholstered materials, padded materials, tires and wheels) to be free from defects in materials and workmanship for a period of one (1) year from the date of purchase from Invacare or a dealer, with a copy of the seller's invoice required for coverage under this warranty. If within such warranty periods any such product component shall be proven to be defective, the product component shall be repaired or replaced, at Invacare's option. This warranty does not include any labor or shipping charges incurred in replacement part installation or repair of any such product. Invacare's sole obligation and your exclusive remedy under this warranty shall be limited to such repair and/or replacement.

For warranty service, please contact the dealer from whom you purchased your Invacare product. In the event you do not receive satisfactory warranty service, please write directly to Invacare at the address on the bottom of the back cover. Provide dealer's name address, date of purchase, indicate nature of the defect and, if the product is serialized, indicate the serial number. Do not return products to our factory without our prior consent.

Limitations and Exclusions: The foregoing warranty shall not apply to serial numbered products if the serial number has been removed or defaced, products subject to negligence, accident, improper operation, maintenance or storage, commercial or institutional use, products modified without Invacare's express written consent (including, but not limited to, modification through the use of unauthorized parts or attachments); products damaged by reason of repairs made to any component without the specific consent of Invacare, or to a product damaged by circumstances beyond Invacare's control, and such evaluation will be solely determined by Invacare. The warranty shall not apply to problems arising from normal wear and tear or failure to adhere to the product instructions. A change in operating noise, particularly relative to

motors and gearboxes does not constitute a failure or defect and will not be repaired; all devices will exhibit changes in operating noise due to aging.

The foregoing express warranty is exclusive and in lieu of any other warranties whatsoever, whether express or implied, including the implied warranties of merchantability and fitness for a particular purpose, and the sole remedy for violations of any warranty whatsoever, shall be limited to repair or replacement of the defective product pursuant to the terms contained herein. The application of any implied warranty whatsoever shall not extend beyond the duration of the express warranty provided herein and Invacare shall not be liable for any consequential or incidental damages whatsoever; **SOME STATES DO NOT ALLOW THE EXCLUSION OR LIMITATION OF INCIDENTAL OR CONSEQUENTIAL DAMAGE, OR LIMITATION OF HOW LONG AN IMPLIED WARRANTY LASTS, SO THE ABOVE EXCLUSION AND LIMITATION MAY NOT BE APPLICABLE.**

THIS WARRANTY SHALL BE EXTENDED TO COMPLY WITH STATE/PROVINCIAL LAWS AND REQUIREMENTS.

10.2 Canada Limited Warranty

This warranty is extended only to the original purchaser who purchases this product within Canada when new and unused from Invacare or a dealer. This warranty is not extended to any other person or entity and is not transferable or assignable to any subsequent purchaser or owner. Coverage under this warranty will end upon any such subsequent sale or other transfer of title to any other person.

This warranty gives you specific legal rights and you may also have other legal rights which vary from state to state.

Invacare warrants the base frame to be free from defects in materials and workmanship for a period of five (5) years from the date of purchase from Invacare or a dealer, with a copy of the seller's invoice required for coverage under this warranty. Invacare warrants all electronics and electrical components (excluding batteries), powered seating actuators, 2-pole motors and gearboxes to be free from defects in materials and workmanship for a period of two (2) years from the date of purchase from Invacare or a dealer, with a copy of the seller's invoice required for coverage under this warranty. Invacare warrants all batteries to be free from defects in materials and workmanship for a period of six (6) months from the date of purchase from Invacare or a dealer, with a copy of the seller's invoice required for coverage under this warranty. Invacare warrants all remaining components (excluding all upholstered materials, padded materials, tires and wheels) to be free from defects in materials and workmanship for a period of one (1) year from the date of purchase from Invacare or a dealer, with a copy of the seller's invoice required for coverage under this warranty. If within such warranty periods any such product component shall be proven to be defective, the product component shall be repaired or replaced, at Invacare's option. This warranty does not include any labor or shipping charges incurred in replacement part installation or repair of any such product. Invacare's sole obligation and your exclusive remedy under this warranty shall be limited to such repair and/or replacement.

For warranty service, please contact the dealer from whom you purchased your Invacare product. In the event you do not receive satisfactory warranty service, please write directly to Invacare at the address on the bottom of the back cover. Provide dealer's name address, date of purchase, indicate nature of the defect and, if the product is serialized, indicate the serial number. Do not return products to our factory without our prior consent.

Limitations and Exclusions: The foregoing warranty shall not apply to serial numbered products if the serial number has been removed or defaced, products subject to negligence, accident, improper operation, maintenance or storage, commercial or institutional use, products modified without Invacare's express written consent (including, but not limited to, modification through the use of unauthorized parts or attachments); products damaged by reason of repairs made to any component without the specific consent of Invacare, or to a product damaged by circumstances beyond Invacare's control, and such evaluation will be solely determined by Invacare. The warranty shall not apply to problems arising from normal wear and tear or failure to adhere to the product instructions. A change in operating noise, particularly relative to motors and gearboxes does not constitute a failure or defect and will not be repaired; all devices will exhibit changes in operating noise due to aging.

The foregoing express warranty is exclusive and in lieu of any other warranties whatsoever, whether express or implied, including the implied warranties of merchantability and fitness for a particular purpose, and the sole remedy for violations of any warranty whatsoever, shall be limited to repair or replacement of the defective product pursuant to the terms contained herein. The application of any implied warranty whatsoever shall not extend beyond the duration of the express warranty provided herein and Invacare shall not be liable for any consequential or incidental damages whatsoever; SOME STATES DO NOT ALLOW THE EXCLUSION OR LIMITATION OF INCIDENTAL OR CONSEQUENTIAL DAMAGE, OR LIMITATION OF HOW LONG AN IMPLIED WARRANTY LASTS, SO THE ABOVE EXCLUSION AND LIMITATION MAY NOT BE APPLICABLE.

THIS WARRANTY SHALL BE EXTENDED TO COMPLY WITH STATE/PROVINCIAL LAWS AND REQUIREMENTS.

10.3 Usability Survey

Please complete the survey below to evaluate this manual.
 Your participation in the evaluation assists in the development of effective and usable manuals for our customers.
 The survey is also available online:
<http://www.invacare.com/TechnicalDocumentSurvey>



Yes, you can.

1. Please indicate your primary involvement with the product (choose one):

- | | |
|---|--|
| <input type="checkbox"/> Product User/Owner | <input type="checkbox"/> Product Service Technician |
| <input type="checkbox"/> User Assistant | <input type="checkbox"/> Health Care Provider |
| <input type="checkbox"/> Product Dealer | <input type="checkbox"/> Other (please specify): _____ |

2. Please indicate which product manual you are evaluating: _____

3. Evaluate the content:

	YES	NO
After reading this document, do you have a better understanding of how to use the product?	<input type="checkbox"/>	<input type="checkbox"/>
Do you have a better understanding of any limitations on the use of this product?	<input type="checkbox"/>	<input type="checkbox"/>
Is there any irrelevant information?	<input type="checkbox"/>	<input type="checkbox"/>
Is the Table of Contents useful?	<input type="checkbox"/>	<input type="checkbox"/>
Does any information seem inaccurate/misleading?	<input type="checkbox"/>	<input type="checkbox"/>
Do you understand that misuse of the product can cause injury or damage?	<input type="checkbox"/>	<input type="checkbox"/>
Explain:	_____	

4. Evaluate the Warnings/Cautions:

	YES	NO
Are there any warnings/cautions that you do not understand?	<input type="checkbox"/>	<input type="checkbox"/>
Are there too many warnings/cautions?	<input type="checkbox"/>	<input type="checkbox"/>
Are there warnings/cautions that you feel do not apply to this product?	<input type="checkbox"/>	<input type="checkbox"/>
Explain:	_____	

5. Evaluate the style:

	YES	NO
Is anything hard to locate/follow?	<input type="checkbox"/>	<input type="checkbox"/>
Are any headings missing/confusing?	<input type="checkbox"/>	<input type="checkbox"/>
Are there too many headings?	<input type="checkbox"/>	<input type="checkbox"/>
Should any material be a bulleted list or checklist instead of numbered steps or a paragraph?	<input type="checkbox"/>	<input type="checkbox"/>
Is there material that might be clarified by a visual?	<input type="checkbox"/>	<input type="checkbox"/>
Explain:	_____	

6. Evaluate the illustrations:

	YES	NO
Are the illustrations useful?	<input type="checkbox"/>	<input type="checkbox"/>
Do the illustrations need more or less detail?	<input type="checkbox"/>	<input type="checkbox"/>
Is the number/size of illustrations adequate?	<input type="checkbox"/>	<input type="checkbox"/>

Explain:

7. Do you have suggestions for other ways of making this document easier to use?

Explain:	YES	NO
<hr/>	<input type="checkbox"/>	<input type="checkbox"/>

Thank You! Thank you for completing this survey. If you have any questions or we may be of assistance to you, please feel free to contact us.

Send your survey to Invacare Technical Writing Department: TechnicalWriting@invacare.com or Invacare Corporation: One Invacare Way, Elyria, Ohio 44035 FAX: 440-329-6975

Notes

Notes

Invacare Corporation

USA

One Invacare Way

Elyria, Ohio USA

44035

440-329-6000

800-333-6900

www.invacare.com



1143190-N-03 2016-11-07



Canada

570 Matheson Blvd E Unit 8

Mississauga Ontario, L4Z 4G4

Canada

800-668-5324

www.invacare.ca

Making Life's Experiences Possible®



Yes, you can.®