




the ROHO group
shape fitting technology®

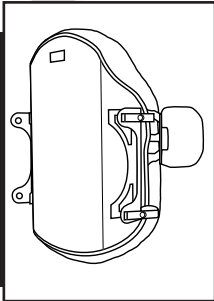

JetStream Pro™ Lateral Pads Operations Manual

Manuel d'utilisation des coussins latéraux JetStream Pro™

PROVIDER: THIS MANUAL MUST BE GIVEN TO THE USER OF THIS PRODUCT.

USER: BEFORE USING THIS PRODUCT, READ THIS ENTIRE MANUAL AND SAVE FOR FUTURE REFERENCE.

FOURNISSEUR : CE MANUEL DOIT ÊTRE REMIS À L'UTILISATEUR DE CE PRODUIT.

UTILISATEUR : AVANT D'UTILISER CE PRODUIT, LIRE CE MANUEL EN ENTIER ET LE CONSERVER POUR CONSULTATION ULTÉRIEURE.

shape fitting technology®

TABLE OF CONTENTS

Intended Use	2
Warnings & Precautions	3
Parts Detail	4
Attachment & Adjustment Instructions	5-6
Cleaning and Disinfecting	7
Maintenance	7
Disposal	7
Warranty / Contact Information	8

©The ROHO Group, Inc. 2007

The following are trademarks and registered marks of The ROHO Group, Inc.: ROHO®, shape fitting technology®, and JetStream Pro®.

The ROHO Group Inc. has a policy of continual product improvement and reserves the right to amend specifications presented in this manual.

©The ROHO Group, Inc. 2007

Vous trouverez ci-après les marques déposées et enregistrées de ROHO Group, Inc. : ROHO®, shape fitting technology® et JetStream Pro®.

ROHO Group, Inc. a une politique d'amélioration continue de ses produits et se réserve le droit de modifier les spécifications présentées dans ce manuel.

INTENDED USE

The JetStream Pro Lateral Pads are designed to assist in maintaining functional posture, as well as promote stability. JetStream Pro Lateral Pads are designed to be used only with the JetStream Pro Adjustable Back Replacement System.

The Pads come in Large (Swing Away or Stationary) and Active (Stationary only) sizes. The Pads consist of aluminum hardware covered by foam and a removable nylon cover, and attach to the Lateral Bracket. Each Pad size is compatible with any size Lateral Bracket.

Large Swing Away Pads are designed for users who need both trunk support for stability, and a removal of the supports for transfers. Large Stationary Pads are designed for users who need moderate external support for trunk stability. Active Pads are designed for users who need some lateral trunk support for stability, with potential to progress beyond the need of lateral support.

The Lateral Brackets come in a Standard size (separate left and right brackets) to fit Standard JetStream Pro Adjustable Back Replacement Systems and a Low size to fit Low JetStream Pro Adjustable Back Replacement Systems. The Lateral Brackets mount directly to the inside of the Carbon Fiber Shell using existing back post mounting hardware holes and bolt pattern.



WARNING

DO NOT use the JetStream Pro Lateral Pads without first reading and fully understanding this manual. If you need assistance contact your clinician, ROHO provider, or The ROHO Group.




PRECAUTIONS

• THIS LATERAL PAD OPERATIONS MANUAL IS SUPPLEMENTAL TO AND DOES NOT REPLACE THE JETSTREAM PRO BACK ATTACHMENT OR ADJUSTMENT PROCEDURES.

- As with any addition to your wheelchair, the Lateral Pads may affect the stability of the wheelchair. Consult with your clinician.
- Check screws, bolts, brackets, and hinges for wear, breakage, and tightness daily. Retighten loose screws following the Attachment Instructions in this manual. **DO NOT** continue to use the JetStream Pro Lateral Pads if any component cannot be tightened or is broken.
- **DO NOT** attach the JetStream Pro Lateral Pads with the user in the chair. The JetStream Pro Back must be removed from the wheelchair in order to install the Lateral Pads.
- Consult your clinician regarding proper positioning of the Lateral Pads.
- **DO NOT USE** the Lateral Pads with any wheelchair back replacement system other than a JetStream Pro Adjustable Back Replacement System.
- **DO NOT** use if user is over 250 lbs. (113.4 kg).
- **DO NOT** use for heavy duty applications or if user is high-tone.
- **DO NOT** use the Lateral Pads to transfer in or out of the wheelchair. The Lateral Brackets are not designed to support the full body weight of the user.
- **DO NOT** use the Lateral Pads to lift your wheelchair.
- Protect the Lateral Pads from blunt force shocks that may cause damage and/or breakage.
- Altering the Lateral Pads may affect performance and will void the warranty.
- Only use hardware supplied with the Lateral Pads and JetStream Pro Back. If replacement parts are needed, please contact your ROHO provider.
- **DO NOT USE** the JetStream Pro Lateral Pads near an ozone generator, as the ozone generator may deteriorate the foam pad and other materials, and void your warranty.

PARTS DETAIL

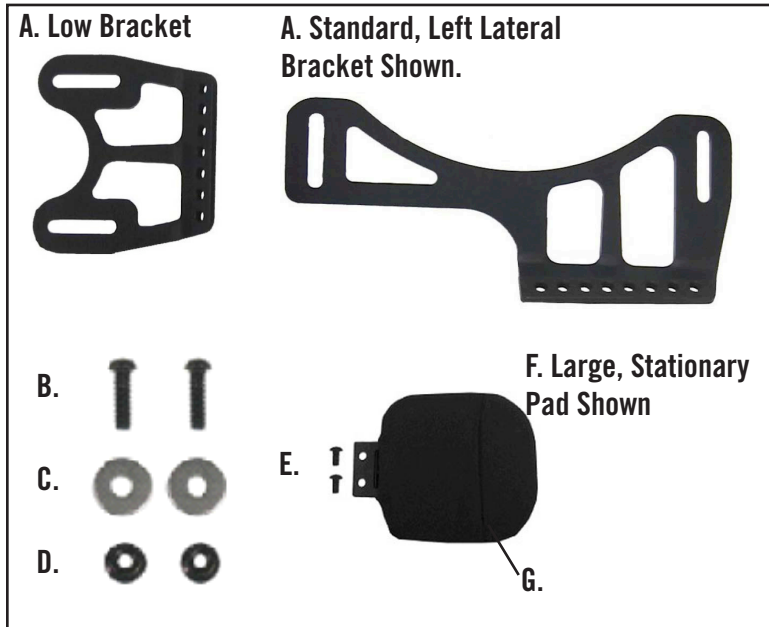

- A. Lateral Bracket - Comes in Standard (left or right) and Low sizes. Attaches Pad to inside of Carbon Fiber Shell.
- B. M6 x 20 mm screws - Used to attach Lateral Bracket to carbon fiber shell. Qty. 2 per Lateral Bracket.
- C. Washers - Used to attach Lateral Bracket to carbon fiber shell. Qty. 2 per Lateral Bracket.
- D. Nuts - Used to attach Lateral Bracket to carbon fiber shell. Qty. 2 per Lateral Bracket.
- E. M5 x 10mm screws - Used to attach Pad to Lateral Bracket. Qty. 2 per Lateral Pad.
- F. Pad - Comes in Large (Stationary or Swing Away) and Active (Stationary only) sizes. Provides positioning and support for user.
- G. Cover - Fits over the Pad.

Tools Needed for Assembly (Not Included):



3 mm, 4 mm, & 5 mm Allen wrenches. 10 mm wrench.

ATTACHMENT & ADJUSTMENT INSTRUCTIONS

Notes:

- Before installation, remove existing JetStream Pro Back, leaving the Bracket Clamps in place on the wheelchair back posts. Note existing height, depth and angle orientation before removing the Carbon Fiber Back.
- Install Lateral Pads before the JetStream Pro Back is installed on wheelchair.
- After Lateral Pads are installed but before Lateral Pad height, width and angle adjustments are made, install and adjust the JetStream Pro Back following the instructions in the JetStream Pro Manual.
- Left and right side designations are from the end user's point of view when sitting in chair.
- Be sure that Standard Lateral Brackets are attached to the correct side of the Carbon Fiber Shell.

INSTALL - LATERAL BRACKET

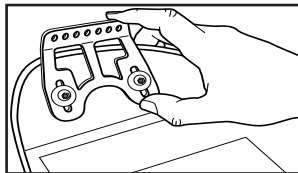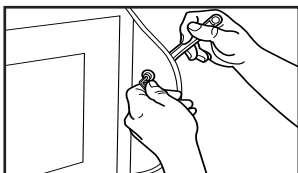

STEP 1. Remove both screws (M6 x 16mm) that secure the Adjustable Bracket to the Carbon Fiber Shell using a 5mm Allen wrench and a 10mm wrench.


NOTE: If spacer falls out during the removal of the original screws, be sure to replace the spacer before continuing with installation.

STEP 2. Line up slots on Lateral Bracket (part A) and Adjustable Bracket (Lateral Bracket on inside of Carbon Fiber Shell, and Adjustable Bracket on outside). Insert both screws (M6 x 20mm) (part B) with flat washers (part C) and nuts (part D) and hand tighten each.

NOTE: Reuse flat washer when attaching Lateral Bracket.

STEP 3. Repeat steps 1 & 2 for opposite side bracket.

INSTALL - PAD

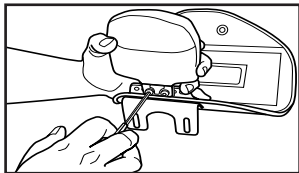

STEP 1. With the cover (part G) opening facing outward, insert both screws (M5 x 10mm) (part E) into the Lateral Bracket (part A) and tighten each with 3mm Allen wrench.

NOTE: All Pads (part F) can be used on the left or right side. Pad (part F) attaches to the outside of Lateral Bracket (part A).


ATTACHMENT & ADJUSTMENT INSTRUCTIONS

PAD HEIGHT ADJUSTMENT

STEP 1. Remove each screw (M5 x 10mm) (part E) and move Pad (part F) up or down to achieve desired height/location.

STEP 2. Reinstall and tighten each screw (M5 x 10mm (part E).


NOTE: Use 3mm Allen wrench. Total overall height adjustment- 1 1/2" (3.8cm)

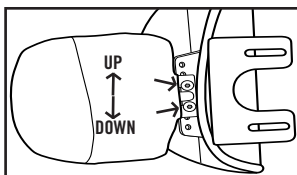

PAD WIDTH ADJUSTMENT

STEP 1. To adjust overall width, loosen each screw (M6 x 20mm) (part B), and slide Lateral Bracket (part A) in or out to achieve desired width.


NOTE: Use 4mm Allen wrench. Total overall width adjustment- 1" (2.5 cm)

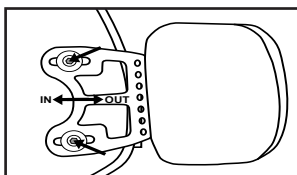

PAD ANGLE ADJUSTMENT

STEP 1. To adjust the Lateral Pad angle to better fit the user's torso, loosen each screw (M6 x 20mm) (part B) and slide the Lateral Bracket (part A) top and bottom (in opposite directions) to achieve desired angle.


NOTE: Use 4mm Allen wrench. Low Lateral Pads have approx. 40° of overall adjustment. Standard Lateral Pads have approx. 16° of overall adjustment.

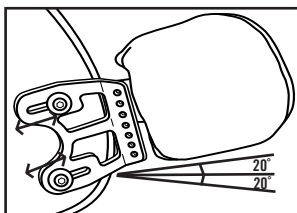

SWING AWAY PAD FUNCTION

STEP 1. Press the swing away hinge in and push the Pad (part F) back .

STEP 2. Pull the Pad (part F) inward until the hinge clicks.

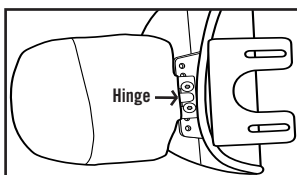

PAD / COVER REMOVAL


STEP 1. Loosen each screw (M5 x 10mm) (part E) from Lateral Bracket (part A) using 3mm Allen wrench.

STEP 2. Remove each screw (M5 x 10mm) (part E) and Pad (part F).

STEP 3. Repeat steps 1-2 for opposite side.

STEP 4. On the side opposite of the bracket, pull the cover (part G) over the edge of the foam & remove cover.

NOTE: Use 3mm Allen wrench.

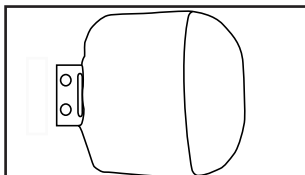

CLEANING & DISINFECTING

Cleaning and Disinfecting are two separate processes. To be effective, cleaning must precede disinfecting.

Cleaning the Cover: Remove the cover by following instructions on the Attachment & Adjustment Instructions Page. Machine wash in hot water. No bleach. Tumble dry low.



Cleaning the Brackets and Hardware: Using a soft cloth, hand wash the brackets and hardware with mild soap and warm water. Rinse thoroughly. Towel dry with a soft cloth.

Cleaning the Foam: Using a soft cloth, hand wash the foam with mild soap and warm water. Allow foam to air dry thoroughly.

Note: **DO NOT** attempt to remove the foam from the metal bracket.

Disinfecting all parts: Wipe gently with a household disinfectant dampened cloth. Air dry thoroughly.

MAINTENANCE

Check screws, bolts, brackets, and hinges for wear, breakage, and tightness daily. Re-tighten loose screws following the Attachment Instructions in this manual.

DO NOT continue to use the JetStream Pro Lateral Pads if any component cannot be tightened or is broken. ROHO recommends that you have your JetStream Pro Lateral Pads thoroughly inspected by a qualified technician every six months, or as necessary.

DISPOSAL

When properly used and disposed, none of the components of your JetStream Pro Lateral Pads will present an environmental hazard. Dispose of the product and/or components in accordance with the applicable regulations in your country.