

Pala Mondial

Proven Function that Caters to All Demands.

Pala Mondial is the time tested denture tooth that meets all the requirements of modern dental technology. It is the first line of denture teeth to feature NanoPearls and guarantees the highest abrasion resistance. The large selection of mould designs provides much flexibility when choosing teeth for the denture.




[Jump to Mould Chart](#)

Mondial - the facts


- Fast and unambiguous set-up
- Precisely defined centric means simple set-up
- Precise function without time consuming spot grinding
- NanoPearls guarantees high abrasion resistance and durability
- Lifelike and vivid surface structures match the basic tooth form and appear harmonious
- Widened tooth base and increased volume aid papillae contouring and eliminate dark recesses and vertical edges

CHARACTERISTICS

Mondial 6

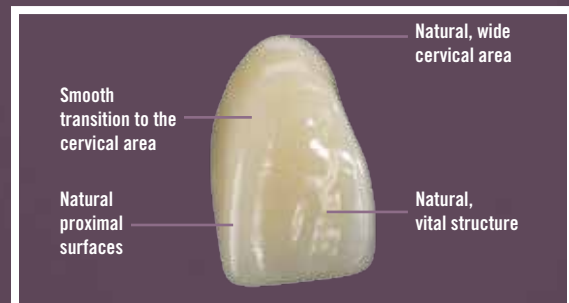

Mondial 8

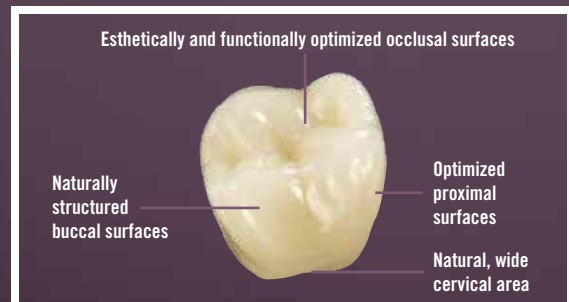










The Pala Mix & Match combination principle: The freedom to combine esthetic and functionality at your will.

Combining CAD/CAM technology and the INCOMP procedure maintains our exemplary form and functional accuracy. You always receive reliable, dependable shade values for all Pala denture teeth and thus an even better adaptation to facings and any residual dentition.

Mondial and Mondial i provide maximum flexibility with reliable, easily set up dentures for all indications. Identical manufacturing processes and materials as well as excellent color stability permits the lines to be combined freely.

Anterior Indications Even better through new tooth moulds R8, S8, L24, and L19			
Mondial 6	 Full Denture	 Implant-Borne Denture	 Partial Denture
Mondial 6i	 Implant-Borne Denture	 Partial Denture	 Full Denture

Posterior Indications			
Mondial 8	 Full Denture	 Implant-Borne Denture	 Partial Denture
Mondial 8i	 Partial Denture	 Implant-Borne Denture	 Full Denture
NEW Mondial 8i 5°	 Partial Denture	 Implant-Borne Denture	 Full Denture

The Pala Mix & Match Advantages

By blending our premium tooth lines together, we have set a new standard in denture teeth, by providing you capabilities of functionality, esthetics and color stability. The ability to combine our **Mondial and Mondial i** denture teeth provides you with advantages you will only find with the Pala system.

Advantages

- Perfectly matched tooth lines cover all indications, esthetic and functional desires individually
- Identical manufacturing processes and materials as well as excellent color stability permit the lines to be combined freely
- Maximum flexibility with reliable and easy set up
- Age appropriate restorations
- Better matching with the remaining dentition and veneered restorations











Pala Mondial and Mondial i Mould Combinations









Discover new possibilities.

The combination chart below shows examples of Pala Mondial and Mondial i teeth, and how flexible and customized your solutions can be.

By integrating these two products, your customer is getting an alternative for posterior setup without losing esthetics in the anterior region. This offers a wider range of bigger, smaller and additional moulds, and allows an additional posterior degree.

		Mondial 6i UPPER 	Mondial 6i LOWER 	Mondial 8i 5°* & 33° 	Mondial 8 10°-20° 
Ovoid Ovoide 	O2 O4 O6	L12, L14 L16, L18, L19 L19, L20, L22	XS/XSL + S/SL, 31, 33 S/SL + M/ML, 35 L/LL, 37	28, 30, 31, 33 30, 32, 35 32, 37	
Rectangular Rectangulare 	R2 R4 R6 R8	L12 L14, L16 L18, L19 L24, L22	XS/XSL + S/SL, 31, 33 S/SL + M/ML, 33, 35 M/ML + L/LL, 35, 37 L/LL, 37	28, 30, 31, 33 30, 32, 33, 35 32, 35, 37 37	
Square Carré 	S2 S4 S6 S8	L14, L16 L18, L19 L20, L22 L24, L22	XS/XSL + S/SL, 31, 33 M/ML, 33, 35 L/LL, 37 L/LL, 37	28, 30, 31, 33 32, 33, 35 32, 37 37	
Triangular Triangulare 	T2 T4 T6	L14 L16, L19 L18, L19	XS/XSL + S/SL, 31, 33 S/SL + M/ML, 33, 35 L/LL, 35	28, 30, 31, 33 30, 32, 33, 35 32, 35	

*Please note: Mondial i - 5° teeth have the same moulds as Mondial. There is a point of differentiation in the layering of the tooth and the volume base.

		Mondial 6 UPPER 	Mondial 6 LOWER 	Mondial 8i 5°* & 33° 	Mondial 8 10°-20° 
Ovoid Ovoide 	O444 O457 O510 O525	L341, L373, L369 L369, L362 L384, L398 L398	S/SL, 33 S/SL + M/ML, 35 L/LL, 37 L/LL, 37	30, 33 32, 35 32, 37 32, 37	
Rectangular Rectangulare 	R410 R420 R449 R455 R483	L331, L341 L331, L341 L362, L369, L373 L362, L369 L378, L382	XS/XSL, 31 XS/XSL + S/SL, 31, 33 S/SL + M/ML, 33, 35 S/SL + M/ML, 33, 35 M/ML + L/LL, 35, 37	28, 29, 31 28, 30, 31, 33 30, 32, 33, 35 30, 32, 33, 35 32, 35, 37	
Square Carré 	S443 S445 S453	L373, L369 L341, L369 L373, L362	S/SL, 33 S/SL, 33 S/SL, 33	30, 33 30, 33 30, 33	
Triangular Triangulare 	T405 T442 T450 T465 T477 T515	L315 L331, L341, L362 L341, L369 L362, L378 L382, L378 L384	- XS/XSL + S/SL, 31, 33 S/SL, 33 M/ML, 35 M/ML, 35 L/LL, 35, 37	28, 29 28, 30, 31, 33 30, 33 32, 35 32, 35 32, 35, 37	

*Please note: Mondial i - 5° teeth have the same moulds as Mondial. There is a point of differentiation in the layering of the tooth and the volume base.

Mould Comparison Charts

Mondial

Heraeus	Dentsply Portrait® IPN®	Ivoclar BlueLine®
R 410	21C, 43D	A41, P15
R 420	21D, 42D, 55D	A44, P1, P2, P14
R 449	12E, 22E, 54F	P17
R 455	12F, 25G, 42F	A66, P5
R 483	12G, 32G, 45H, 62G, 74H, 75G	A37, P11, P8
S 443	13E, 24F, 32E	A13
S 445	–	P4
S 453	22G, 21X, 75E	–
T 405	22C, 32B	A11
T 442	–	–
T 450	–	A12, A42, P7
T 465	43F	A36, A54, P13, P6
T 477	11G, 42G	A14, P16
T 515	21E	P12
O 444	31F	–
O 457	55F	A32
O 510	21G	A16
O 525	–	A17
L 315	C, D	–
L 331	E, F, H	A2, A3, A4
L 341	L, N, G	–
L 362	J, M	A5, A6
L 369	O, P	–
L 373	–	–
L 378	I	AL
L 382	S	–
L 384	K1, U	A8
L 398	W	–

Mondial i

Heraeus	Dentsply Portrait IPN	Ivoclar BlueLine	Ivoclar Phonares®
R 2	42D/22C	A44	S61
R 4	22E	P3	S81
R 6	21E	N/M	S72
R 8	21J	P18	B63
S 2	12E	A11	B81
S 4	43F	A24B	B82
S6	12G	P9	B83
S8	41J	–	B73
T2	43F	P13	B61
T 4	32G	P8	B82
T 6	42G	A25	B72
O 2	55D	A12	B61
O 4	55F	A56	S72
O 6	21G	A15	B73
L 12	P	A2L	L50
L 14	M	A5	L51
L 16	K1	A7	L51
L 18	W	A8	L53
L 19	V	A8	–
L 20	K1	A8	L25
L 22	W	N/M	L52
L 24	K1	–	L52

Teeth Shade Conversion Chart

Heraeus Mondial and Mondial i*	VITA® 3D-Master	VITA Classic	VITA MFT	Dentsply Portrait IPN	Dentsply Bioform® IPN	Dentsply Bioblend® IPN	Dentsply Trublend® SLM	Ivoclar Chromoscop	Ivoclar Vivodent-PE	Ivoclar BlueLine	Ivoclar Phonares
A1	2M1	A1	A1	P1	B51, B59	100	T9	120	1A	A1	A1
A2	2M2	A2	A2	P2	B53, B59, B62	104	T11	130	2A	A2	A2
A3	3M2	A3	A3	P3	B54, B65	108	T13	140	1C	A3	A3
A3.5	3R2.5	A3.5	A3.5	P3.5	B83, B96	116	T20	230	1E	A3.5	A3.5
A4	4M2	A4	A4	P4	B78, B81, B84, B96	118	T20	340	3E	A4	A4
B1	1M1	B1	B1	P11	B59	100	T22	120	1A	B1	B1
B2	2L1.5	B2	B2	P12	B52, B63, B66, B68, B92	104	T24	120	1A	B2	B2
B3	3L2.5	B3	B3	P13	B55, B56, B64, B65, B67, B77, B95	109	T23	210	2B	B3	B3
B4	3M3	B4	B4	P14	B56, B70, B83, B84	110	T10	320	5B	B4	B4
C1	3M1	C1	C1	P21	B59, B69, B91	102	T6	120	1A	C1	C1
C2	3L1.5	C2	C2	P22	B94	106	T7	430,440	4B, 6C	C2	C2
C3	4L1.5	C3	C3	P23	B95	–	T8	430, 510	4B, 6D	C3	C3
C4	5M1	C4	C4	P24	B96	–	T3	520	4C	C4	C4
D2	3M1	D2	D2	P32	B92	–	T14	130	2A	D2	D2
D3	3M2	D3	D3	P33	B65, B93	–	T5	410	4A	D3	D3
D4	3L1.5	D4	D4	P34	B69	113	T2	440	6C	D4	D4
BL1	–	–	OM1	PW7	–	–	–	–	–	–	–
BL2	–	–	–	PW4	–	–	–	–	–	010	–
BL3	–	–	OM3	PW2	–	–	–	–	–	030	BL3
–	–	–	–	–	–	–	–	–	–	–	BL4

*Heraeus Shades are equivalent to Vita Shades. Mondial and Mondial i have the exact same shading.

Mondial and Mondial i are available in 18 shades (A1–D4 and BL2 and BL3).

Heraeus Kulzer teeth meet and exceed all the requirements of EN ISO 22112:2006 and ANSI/ADA Specification No. 15-1999.

Mondial is a registered trademark of Heraeus Kulzer, GmbH.

VITA and 3D-Master are registered trademarks of VITA Zahnfabrik, GmbH.

Portrait, Bioform, Bioblend, Trublend and IPN are registered trademarks of Dentsply International, Inc.

BlueLine and Phonares are registered trademarks of Ivoclar Vivadent.




Pala Mondial i & Pala Mondial Mould Chart

Mondial 6i Uppers




Jump to Product
Item Numbers

Mondial 6i Uppers









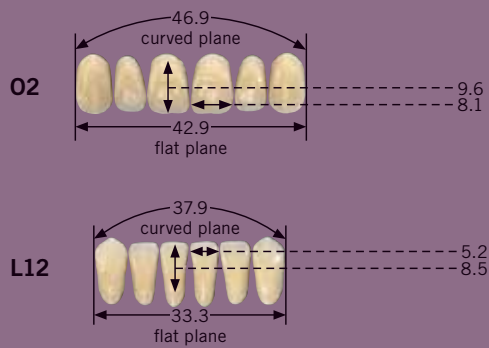
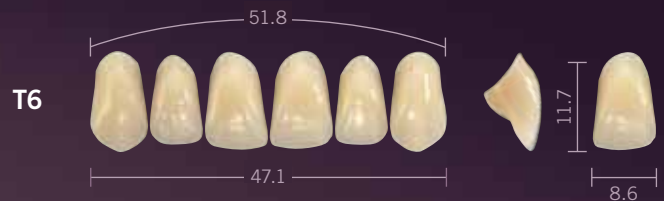
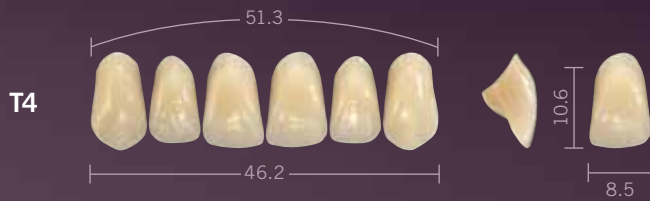


Mondial 6i Lower


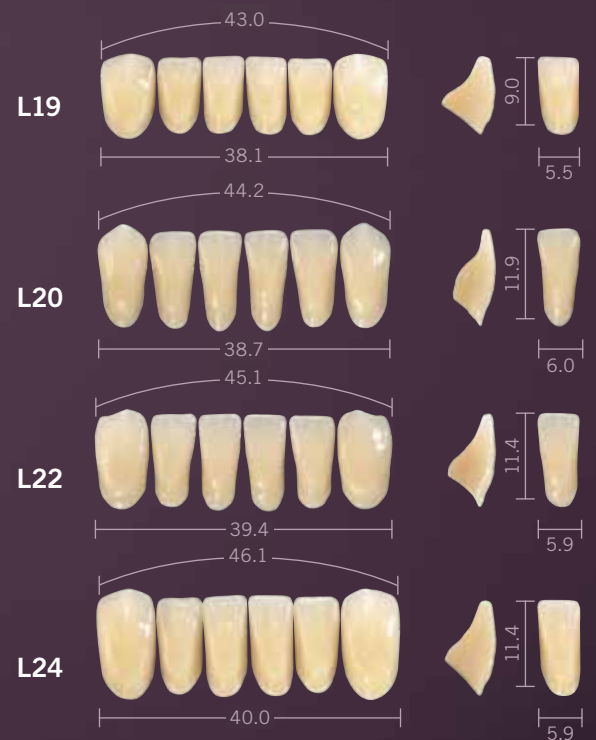





Mondial 6 Uppers

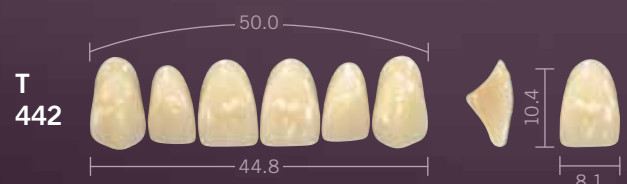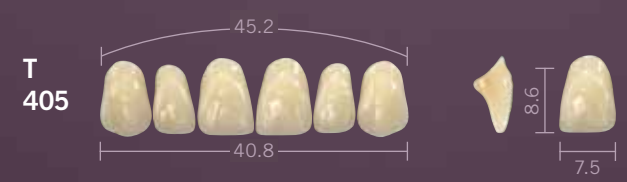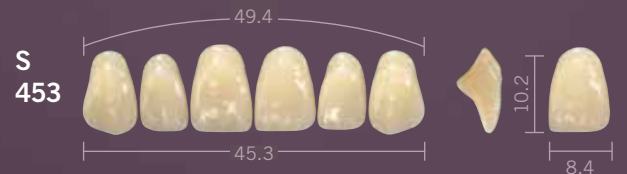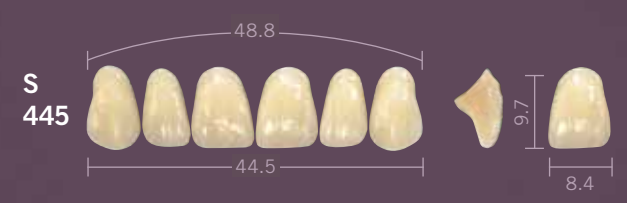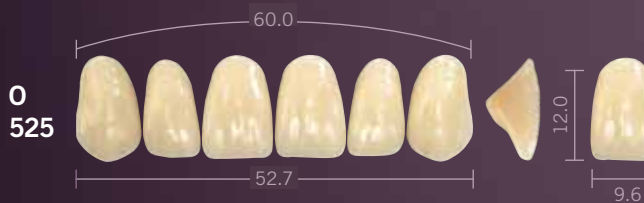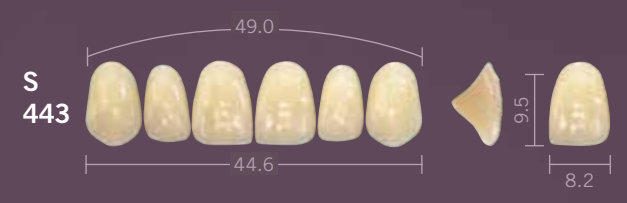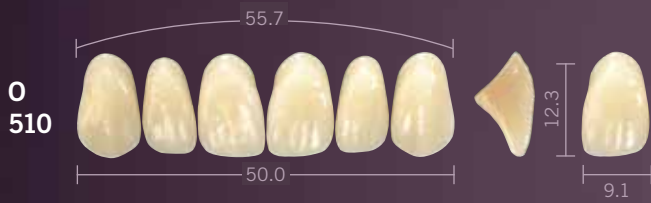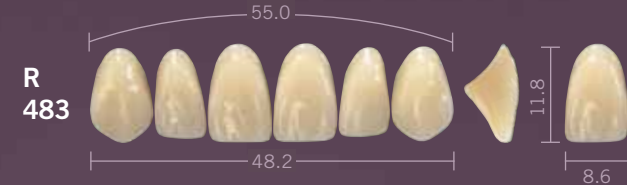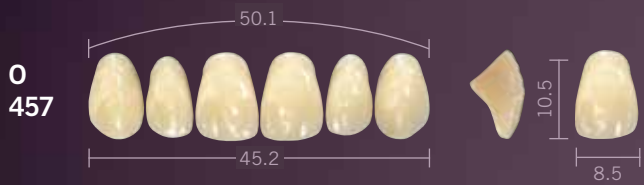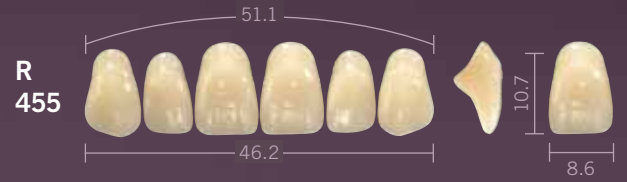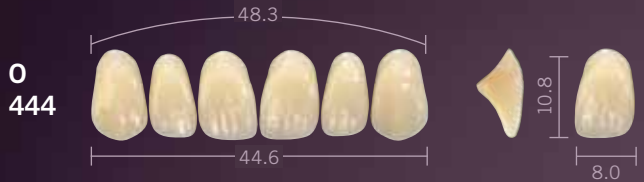


Mondial 6 Uppers

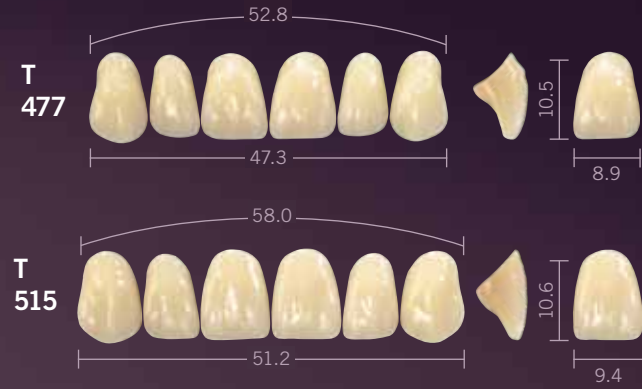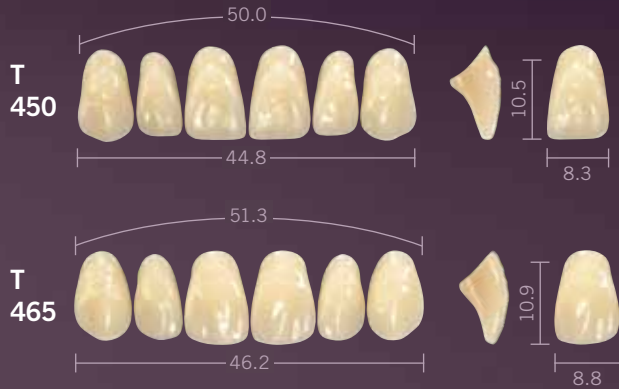

Mondial 6 Lower

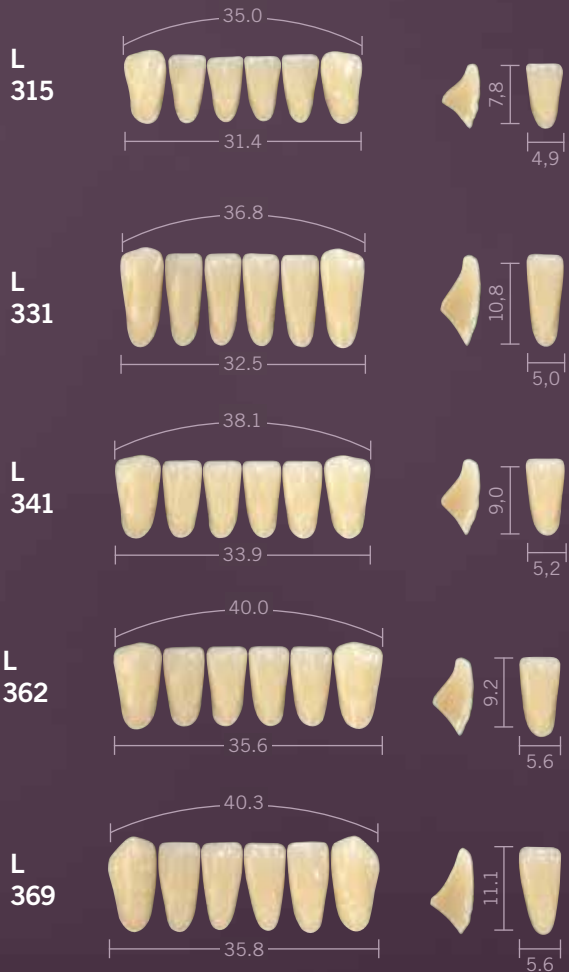

Mondial 8i Posteriors

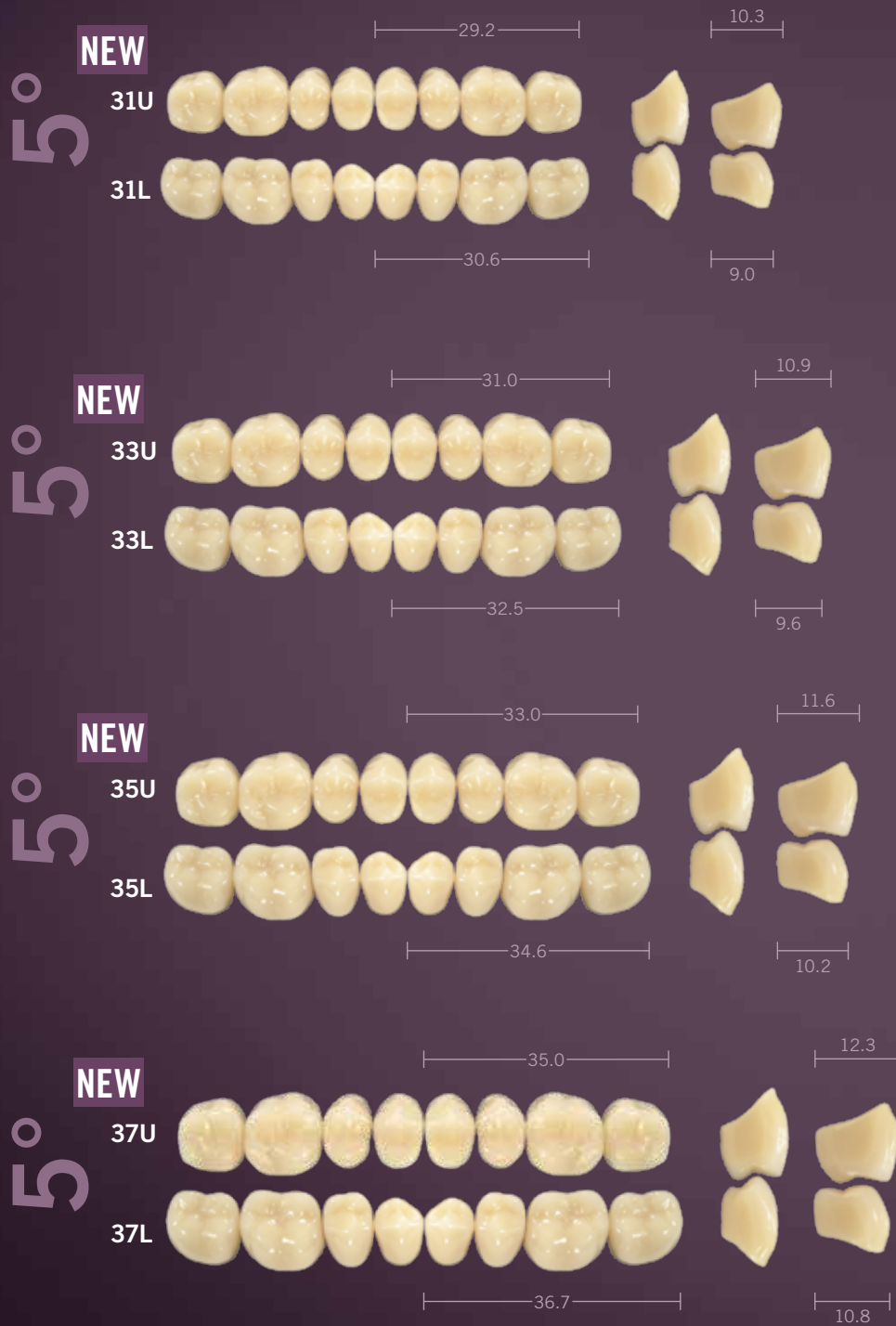

Mondial 8i Posteriors



Mondial 8 Posteriors

