**medi GmbH & Co. KG**  
**Medicusstraße 1**  
**D-95448 Bayreuth**  
**Germany**  
**T +49 921 912-0**  
**F +49 921 912-780**  
**ortho@medi.de**  
**www.medi.de**

medi Austria GmbH  
 Adamgasse 16/7  
 6020 Innsbruck  
 Austria  
 T +43 512 57 95 15  
 F +43 512 57 95 15 45  
 Vertrieb@medi-austria.at  
 www.medi-austria.at

medi Belgium bvba  
 Posthoornstraat 13/1  
 3582 Koersel  
 Belgium  
 T +32 011 24 25 60  
 F +32 011 24 25 64  
 info@medibelgium.be  
 www.medibelgium.be

medi Brasil  
 Rua Neuza 216  
 Diadema-Sao Paulo  
 Cep 09941-640  
 Brazil  
 T +55 11 3201 1188  
 F +55 11 3201 1185  
 sac@medibrasil.com  
 www.medibrasil.com

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## Important notes

This product is made for single patient use only. If it is used for treating more than one single patient, the manufacturer's product liability according to the Medical Devices Act will become invalid. If undue pain or an unpleasant sensation develops while you are wearing the support, please consult your doctor or orthotist immediately. Do not wear the brace over open wounds, and use it only as instructed by your doctor or orthotist.

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Elastan, Polyester, Baumwolle, Polyamid, Aluminium

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---

## Intended purpose

Spinomed active men is a back brace. The product is exclusively to be used for the orthotic fitting of the spine and only on unbroken skin.

---

## Indications

- Osteoporosis with kyphosis and/or back pain (long-term care)
- Scheuermann's disease
- Kyphosis with chronic back pain

## Contraindications

None known to date

## Putting on the orthosis

- Open the zip and unfasten the hooks.
- Put the corset on and fasten the hooks (1); then do up the zip (2).
- If required you can adjust the hooks at the groin in three stages (3).
- Slide the rod that your orthotist preshaped for you into the pocket provided at the back of the brace (4). Make sure that the ribs are facing outwards, and the narrower end is at the top.

## Mode of action

The integrated straps (pelvic strap/shoulder strap) use the biofeedback principle to straighten up the spine; this activates the muscles of the trunk and helps to straighten the rounded back that is associated with osteoporosis. As your muscles grow stronger you become more mobile and pain is reduced.

## Instructions for use

During the familiarisation phase and to gradually restore the back muscles the

time spent wearing Spinomed active should be increased gradually. Depending on the physical condition, 30 minutes uninterrupted wearing may be sufficient at the beginning. The wearing time can be increased step by step during the course of treatment. Your acceptance and co-operation play an important part in treatment with Spinomed active. If any problems arise please talk to your doctor or orthotist. He may be able to adjust the brace. Please follow your doctor's instructions exactly to ensure the success of this course of therapy.

---

## Care instructions

The back support has to be removed before washing. Please close any fasteners before washing (Velcro, zipper, hooks and eyes). Soap residues, lotions and ointments can cause skin irritation and material wear.

- Wash the product by hand, preferably using a medi clean detergent, or in delicate cycle at 30°C using a mild detergent without fabric conditioners.
- Do not bleach.
- Leave to dry naturally.
- Do not iron.
- Do not dry clean.

To avoid discoloration, always wash the body with similar colours.

---

## Storage instructions

Keep the product in a cool, dry place and do not expose to direct sunlight.

**Material composition**

Elastane, Polyester, Cotton, Polyamide,  
Aluminium

**Warranty / contractual guarantee**

Consistent with legal guidelines.

**Disposal**

The product can be disposed of in the  
domestic waste.

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