trophon[®] EPR

User Manual

This manual must be read prior to operating the **trophon® EPR** to ensure correct procedures are followed and the specified disinfection results are achieved.

All technical specifications and system approvals are found in Appendix I of this manual.

©2014 Nanosonics Limited. All rights reserved.

The written and graphic product descriptions in this manual are correct at the time of printing.

trophon® and NanoNebulant® are registered trademarks of Nanosonics Limited.

Nanosonics' technologies are protected by a combination of patents, trademarks, and exclusive proprietary rights.

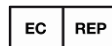
Your sales representative is:

Attach Business Card or information sticker/stamp here
--

Visit the Nanosonics website to obtain this user manual in other languages.

Nanosonics Limited
Unit 24, 566 Gardeners Rd
Alexandria NSW 2015
Australia
Tel: +61 2 8063 1600
Fax: +61 2 9317 5010

Nanosonics Europe GmbH
Falkenried 88,
House A, D-20251
Hamburg – Germany
Tel: +49 40 46856885
Fax: +49 40 46856179

Russian Distributor: AVA Medica
Kondratyevskiy prospect
15/3 Office 344,
St Petersburg Russia
Tel: +7 911 101 6710

Email: info@nanosonics.com.au
www.nanosonics.com.au

www.nanosonics.eu

 0197 Conforming to MDD 93/42/EEC

FSZ 2009/05867 25/12/2009

Table of Contents

PART A – INTRODUCTION AND INITIAL SETUP	4
SECTION A1: INTRODUCTION TO THE TROPHON EPR	4
SECTION A2: IMPORTANT WARNINGS, LABELS and SYMBOLS	5
SECTION A3: OVERVIEW OF DEVICE FEATURES	8
SECTION A4: INSTALLATION GUIDE.....	10
PART B – ROUTINE USE AND MAINTENANCE	14
SECTION B1: LOADING THE DISINFECTANT	14
SECTION B2: ROUTINE HIGH LEVEL DISINFECTION CYCLE.....	15
SECTION B3: PURGE CYCLE.....	19
SECTION B4: REMOVING and DISPOSING OF USED DISINFECTANT CARTRIDGES ..	20
SECTION B5: INCOMPLETE OR FAILED CYCLES	21
SECTION B6: ROUTINE CARE AND MAINTENANCE	23
SECTION B7: DISPOSAL OF DEVICE	23
PART C – TROUBLESHOOTING.....	24
PART D – TROPHON PRINTER (Accessory)	26
SECTION D1: INTRODUCTION AND SETUP	26
SECTION D2: ROUTINE CARE AND MAINTENANCE.....	29
SECTION D3: TROPHON PRINTER TROUBLESHOOTING	30
SECTION D4: TROPHON PRINTER DISPOSAL.....	32
SECTION D5: REORDER INFORMATION	32
SECTION D6: TROPHON PRINTER TECHNICAL SPECIFICATIONS	32
PART E – SERVICE AND WARRANTY.....	33
APPENDIX 1: TROPHON EPR TECHNICAL SPECIFICATIONS.....	33
APPENDIX 2: PRODUCT WARRANTY AND REGISTRATION FORM.....	34

PART A – INTRODUCTION AND INITIAL SETUP

SECTION A1: INTRODUCTION TO THE TROPHON EPR

For any operating, fault or maintenance queries, please contact your customer service representative.

A1.1 Validated Probes

NOTE: Do NOT use the trophon EPR to disinfect non-approved devices or instruments.

Refer to www.nanosonics.eu/trophon-EPR/Brochures for a list of Validated Probes for use with trophon EPR.

Refer to the original equipment manufacturers of the ultrasound equipment for their recommendations and the most up-to-date list of validated disinfectants for use with their probes.

A1.2 Training

It is the owner's responsibility to ensure that all users:

- Are trained as per the instructions contained in this manual to ensure safe operation.
- Are aware of the potential hazards in dealing with the disinfectant and safety procedures associated with the device.
- Visit the Nanosonics website to complete the online training module and receive your trophon EPR certification, before setting up or using your trophon EPR, www.nanosonics.eu

A1.3 User and Environment Profile

NOTE: The following description is intended for general information only. For specific operating instructions, please refer to the relevant sections of this manual.

The trophon EPR will be used in typical health care environments, under the control of health professionals such as:

- Sonographers
- General Practitioners
- Nurses
- Radiographers
- Specialist Doctors (e.g. cardiologists, obstetricians, gynaecologists, sonologists)

In locations such as:

- Hospitals with centralised cleaning rooms
- Hospitals without specific cleaning rooms
- Radiology/Ultrasound sites with centralised cleaning rooms
- General practices and specialised doctors rooms without cleaning rooms

Intended Use:

- The sole purpose of the trophon EPR is to high level disinfect validated ultrasound probes (see section B2.1) according to the specified processes outlined in this manual. It is not intended for any other use. Do **NOT** use this device for any application other than its expressed purpose.
- The trophon EPR together with the NanoNebulant is a high-level instrument grade disinfectant system.
- The trophon EPR is **NOT** intended to reprocess single use devices
- The trophon EPR is **NOT** intended to pre-clean ultrasound probes
- The cable management system is an accessory designed for use with the trophon EPR (see section A4.2)
- Chemical Indicator use is required with every disinfection cycle. More information can be found in the Chemical Indicator Instructions for use, provided with the Chemical Indicator.
- The trophon Printer, trophon Connect, trophon Wall Mount and the trophon Cart are accessories designed for use with the trophon EPR; contact your customer service representative or visit the Nanosonics website for additional information. Refer to Part D for information on the trophon Printer.

Disinfection Process:

At the beginning of the cycle, the trophon EPR creates an aerosol of concentrated hydrogen peroxide. This is quickly and evenly distributed over the surface of the probe, including very small crevices. This process provides thorough, high level disinfection of the shaft and the handle of the probe. The device breaks down the hydrogen peroxide into small amounts of water and oxygen and safely vents them into the external environment.

Notes:

- 1) NanoNebulant is the product name of the trophon Disinfectant.
- 2) The contact conditions are fixed cycle parameters that are not able to be modified by the end user.

SECTION A2: IMPORTANT WARNINGS, LABELS and SYMBOLS

Failure to read the following sections may result in damage to the trophon EPR, disinfectant cartridges or other equipment, or cause serious injury to the operator or other persons. If the device is used in a manner not specified by the manufacturer, the protection provided by the device may be impaired.

A2.1 Warnings

Hot Temperatures

- Risk of burns from the hot surfaces in the internal chamber. Do NOT touch these surfaces. A hot surface symbol is located inside the trophon EPR chamber as a reminder.
- Failure to correctly load the probe into the chamber may result in damage to the probe. Probe must NOT come into contact with the chamber wall. Refer to section B2.3 for correct positioning of probe.

Malfunctions

- Do NOT attempt to open the chamber door during a cycle or in the event of a power failure or system malfunction (see section B5.1).
- All repairs must be carried out by trained personnel ONLY. Do NOT attempt to repair or modify any part of the device. The trophon EPR contains no end user serviceable parts.

Transporting the Device

- The device weighs approximately 17kg (38lb). Use safe lifting techniques as per your Occupational Health and Safety Lifting Guidelines for your institution.
- Do not move, relocate or transport the device if hydrogen peroxide is present; purge the device before moving or relocation (See section B3).

Electrical Device

- Equipment must be connected to an earthed power outlet. Ensure power cable supplied with the device is used.
- Spilled fluid can result in electrical shock. Do not allow any fluid to spill onto or around the device or immerse any parts of the trophon EPR in liquid.
- Connect the device only to an electrical source with the proper voltage and frequency as specified in Appendix 1. If incorrect voltage is used, the device may be damaged when it is switched on.
- Attempting to open any component of the device to gain access to the internal mechanics may result in electrical shock.

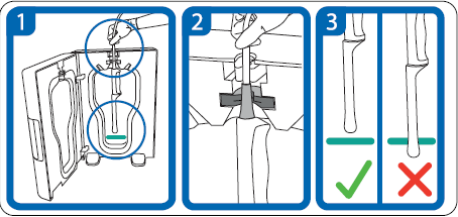
trophon Chemical Indicator

- ONLY to be used with the trophon EPR. Do NOT use with non-approved devices.
- Use one Chemical Indicator with every trophon EPR cycle.
- Do NOT use damaged and/or out-of-date indicators.
- Ensure indicator is red and not exposed before use.
- More information can be found in the Chemical Indicator Instructions for Use, provided with the Chemical Indicator.

trophon NanoNebulant

- Read the Instructions for Use (IFU) leaflet enclosed with the trophon NanoNebulant and the Material Safety Data Sheet (MSDS) enclosed with the device.
- Always wear disposable gloves when handling disinfectant cartridges and running the trophon EPR device.
- Ensure appropriate personal protective equipment (PPE) is worn during spills (according to the Occupational Health and Safety Guidelines for your institution and NanoNebulant MSDS).

A2.2 Labels and Symbols

	Caution		Corrosive
	Consult instructions for use		Warning
	Start (of action)		Single Use Only
	Fragile / Handle With Care		Warning: Hot Surface
	Do not disassemble		Dangerous Voltage
	Separate collection for electrical and electronic equipment		Keep Dry
	Keep Out of Direct Sunlight		Expires (year and month)
	Batch Number		Product Number
	Cannot be transported by air freight		This Way Up
	UN 2014		Wear Gloves when handling cartridges
 <p>Probe Guide</p>			

SECTION A3: OVERVIEW OF DEVICE FEATURES

A3.1 Front Panel and Back Panel

Figure 1

Front Closed

(Figure 1)

1. User screen: displays messages and menu
2. Start button
3. Soft key buttons
4. Chamber door handle


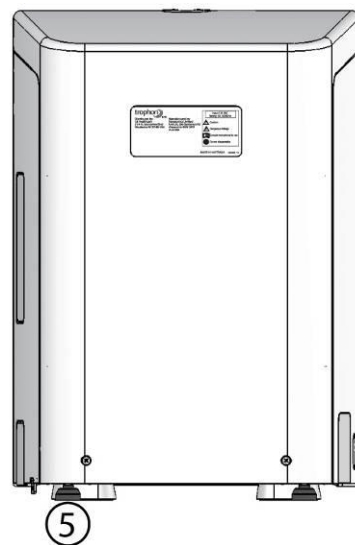
Figure 2

Back


(Figure 2)


5. Adjustable Feet (for levelling the device)
- Note:** Serial Number (SN) can be located on the back label

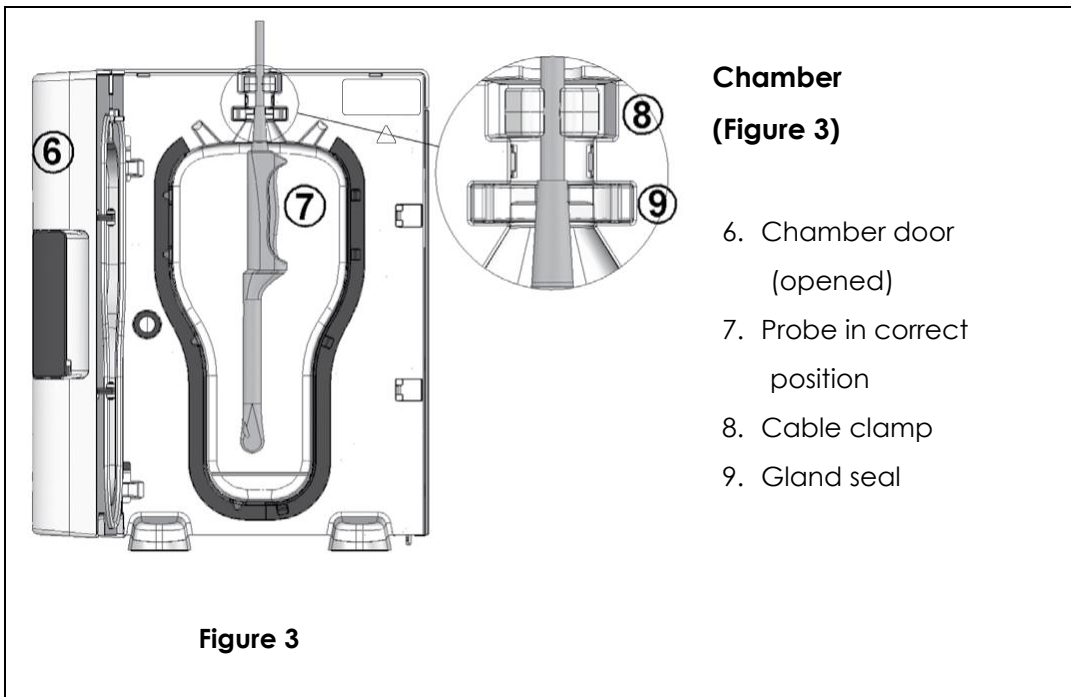A3.2 Chamber (Figure 3)

A3.3 Side Panels (Figure 4 & 5)

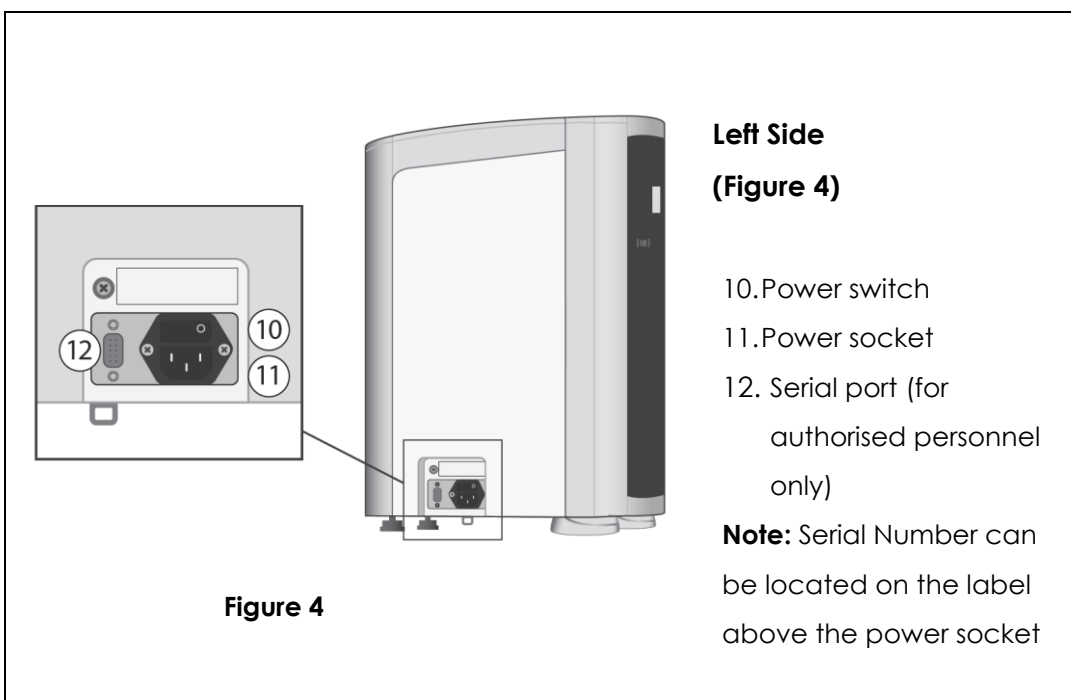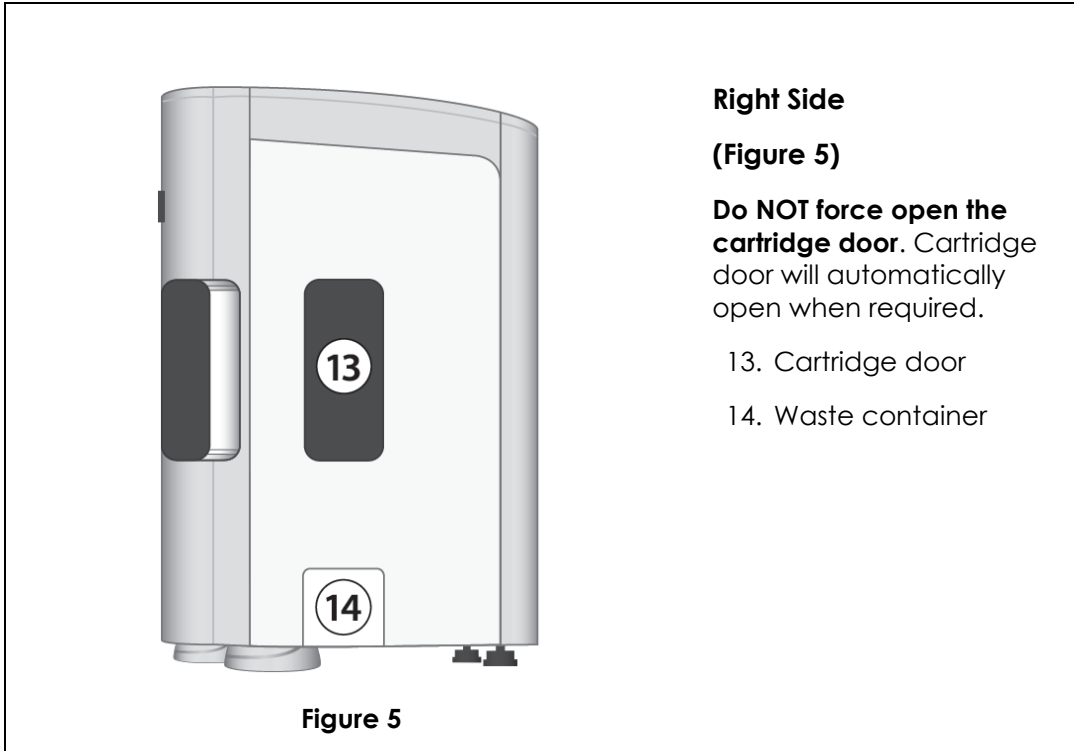
SECTION A4: INSTALLATION GUIDE

A4.1 Positioning the device

NOTE: Ensure the device is placed on a surface that can support the weight of the device (see Appendix 1).

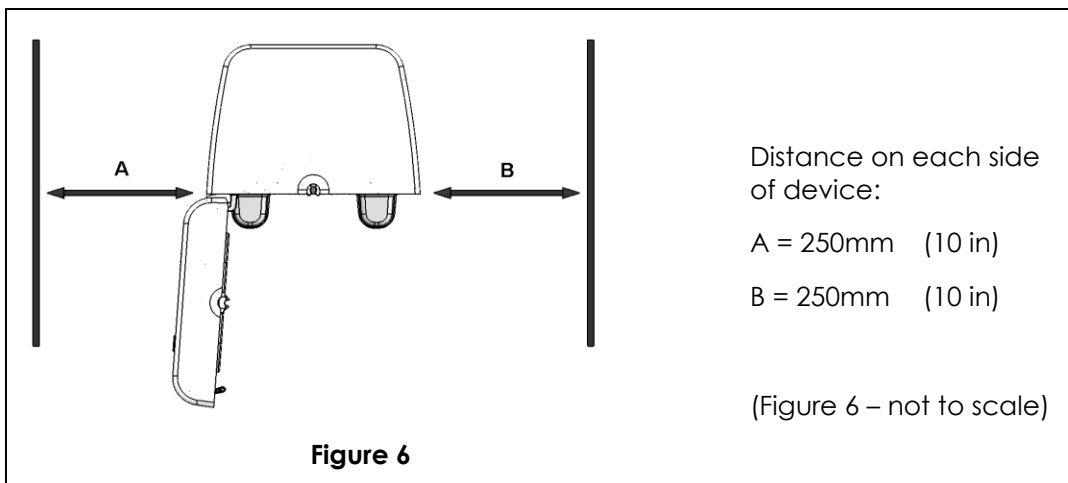1. Ensure the area around the device is free from other equipment and clutter. Positioning the trophon EPR as shown in Figure 6 will ensure access to all features including the cartridge system and for disconnecting the device.

2. Level the device by adjusting the back feet – turning feet clockwise or anti-clockwise (see Figure 2). Ensure feet are present at all times, and are not loose.

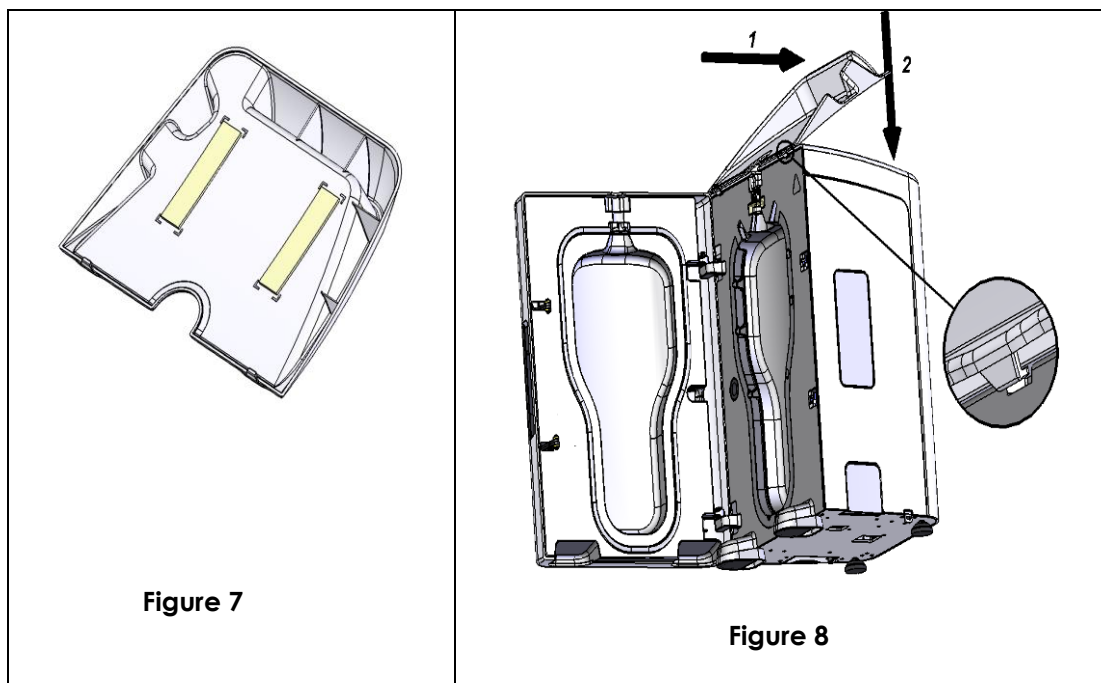
NOTE: If the device needs to be relocated, see section B3.3 – Transportation

A4.2 Cable Management System Installation

The Cable Management System is designed to facilitate ease of use of the trophon EPR by holding the ultrasound cable away from the chamber door, and providing a convenient location to store the cable during High Level Disinfection. Refer to section B2.3 for using the Cable Management System.

To install the Cable Management System:

1. Open the chamber door before mounting the Cable Management System to the device.
2. Peel back the protective strip from the double sided tape on the bottom of the Cable Management System (see Figure 7)
3. Align and slide the two hooks on the front of the Cable Management System into the slots in the device chamber (see Figure 8-1) until it cannot slide any further.
4. Push Cable Management System down on top of the device to secure it in place (see Figure 8-2).

A4.3 Powering On

1. Attach the power cable supplied with the device to the device power socket. Equipment must be connected to an earthed outlet.
2. Switch on at the mains power
3. Turn on the power switch, located on the side of the device to 'ON'

A4.4 Basic Settings

Date and Time

1. Press the soft key button underneath MENU on the screen
2. Using the soft key buttons under the LCD:
 - Scroll to SETUP using the buttons under the arrows and press OK
 - Scroll to SET DATE AND TIME using the buttons under the arrows
 - Set the date and time by using the button under NEXT to move through each set point and the arrows to scroll to the desired date and time
 - Once each set point is set press OK
 - The screen will now revert back to the menu screen
 - To change date and time format select SETUP using the buttons under the arrows and press OK
 - Scroll to CHANGE DATE FORMAT or CHANGE TIME FORMAT using the buttons under the arrows
 - Press OK once complete
 - The screen will now revert back to the menu screen.

Language

1. Press the soft key button underneath MENU on the screen
2. Using the soft key buttons under the LCD:
 - Scroll to SETUP using the buttons under the arrows and press OK
 - Scroll to LANGUAGE using the buttons under the arrows and press OK
 - Scroll to the desired language using the buttons under the arrows and press OK
 - The screen will now revert back to the menu screen

Alarm Settings

1. Press the soft key button underneath MENU on the screen
2. Using the soft key buttons under the LCD:
 - Scroll to SETUP using the buttons under the arrows and press OK
 - Scroll to ALARM SETTINGS using the buttons under the arrows and press OK
 - Set the alarm volume by pressing the up & down buttons to set the volume on high or low. Press Ok to confirm selection.
 - Scroll to ALARM REPEAT using the buttons under the arrows and press OK
 - Set the alarm to repeat by pressing the up & down buttons to turn on or off the alarm repeat. Press OK to confirm selection
 - If alarm repeat is enabled, the alarm will repeat every minute for the following situations:
 - a. The probe is left in the device
 - b. Error message is displayed and requires a response

A4.5 Warm-up Cycle

1. This cycle prepares the device for operation
2. The warm up will begin automatically

Screen Message	Approximate Warm Up Time (minutes)
Quick Warm Up	< 2 minutes
Warming Up	2 – 30 minutes
Extended Warm Up	> 30 minutes

3. During the warm up cycle some instructions may appear on the screen. Please follow these instructions, which may include:
 - Close Chamber Door
 - Cartridge Empty. Replace Cartridge Now (Refer to section B1)
 - Remove Probe From Chamber
4. When completed, the screen message will read: LOAD PROBE AND INDICATOR
5. The machine is now ready for use

NOTE: It is recommended that the device remains switched on at all times to maximise the life of the disinfectant cartridge, unless the machine needs to be moved.

PART B – ROUTINE USE AND MAINTENANCE

SECTION B1: LOADING THE DISINFECTANT

B1.1 Disinfectant Specifications and Handling

Read the Instructions for Use (IFU) leaflet enclosed with the trophon NanoNebulant and the Material Safety Data Sheet (MSDS) enclosed with the device.

Always wear disposable gloves when handling disinfectant cartridges.

WARNING: Failure to follow specifications and handling instructions may compromise the effectiveness of the disinfection process and/or cause injury to the operator.

B1.2 Inserting the Disinfectant Cartridge

1. Cartridge door will automatically open when cartridge needs replacing. Do NOT force the Cartridge Door open as this may damage and render the device non functional.
2. Screen message: CARTRIDGE EMPTY. REPLACE CARTRIDGE NOW is displayed.
3. Press the button under OK to open the cartridge door.
4. Do NOT insert empty cartridges into the device.

NOTE: Always check the expiration date on the cartridge before use. If the cartridge has expired, dispose of as per local environment and government regulatory requirements. Do not attempt to open or load a damaged or distorted cartridge. Do not manually pierce the cartridge.

Refer to IFU enclosed with the trophon NanoNebulant for detailed instructions on how to install and/or replace trophon NanoNebulant cartridges

NOTE: Cartridges will last for approximately one month from date of installing, depending on usage and whether the device has been switched off.

B1.3 Spillage of Disinfectant

- Wear PPE appropriate for the spill (according to the Occupational Health and Safety Guidelines for your institution and NanoNebulant MSDS).
- Never return spills to original cartridges for re-use.
- Contain and clean-up the spill by placing spill control materials over the entire spill area.

B1.4 Customised Disinfectant Cartridges

- Use ONLY the trophon NanoNebulant cartridges which are validated for use with the trophon EPR
- Each cartridge is to be used ONCE. Do NOT refill or reuse cartridges

SECTION B2: ROUTINE HIGH LEVEL DISINFECTION CYCLE

The effectiveness of the device cannot be guaranteed if non-approved accessories are used. Do NOT use the trophon EPR to disinfect non-approved devices or instruments.

B2.1 Validated Probes for use with trophon EPR

For details of probes that are able to be used in the trophon EPR refer to the following:

- Nanosonics website www.nanosonics.eu
- The manufacturer of ultrasound equipment for their recommendations and the most up-to-date list of validated disinfectants for use with their probes

NOTE: Only validated probes should be placed in the trophon EPR. All probes referred to on the Validated Probe List have been tested and validated according to the manufacturer's specifications.

B2.2 Preparing the Probe

The probe must be pre-cleaned and dried BEFORE the High-Level Disinfection process can commence in the trophon EPR.

NOTE: Failure to clean and dry the probe may result in:

- high-level disinfection will not be achieved during the trophon EPR disinfection cycle
- residual disinfectant remaining on the probe
- potential exposure to residual disinfectant, which may cause injury, when removing probe; this may lead to temporary bleaching and/or irritation of the skin if no gloves are worn

Refer to probe manufacturer's instructions for correct cleaning process.

B2.3 Positioning the Probe

NOTE: Probe must be correctly inserted in the device for a high-level disinfection cycle to occur.

1. When the device is ready, screen message: LOAD PROBE AND INDICATOR
2. Open chamber door
3. The probe is held securely in the chamber by the use of two clamps; refer to Figure 9(a).

The probe has a short sleeve at the back of the handle, covering the electrical cable. This is referred to as the PROBE GLAND - Figure 9(a)

Wearing gloves, insert the Probe correctly as follows:

- Press the probe's electrical cable into the cable clamp (at the top of the chamber)
- Hold the probe by its handle, press the top of the PROBE GLAND into the gland seal - Figure 9(a), ensuring the top of the probe cable gland is above the gland seal - Figure 9(b)
- Ensure the probe is straight and not touching the walls or the bottom of the chamber

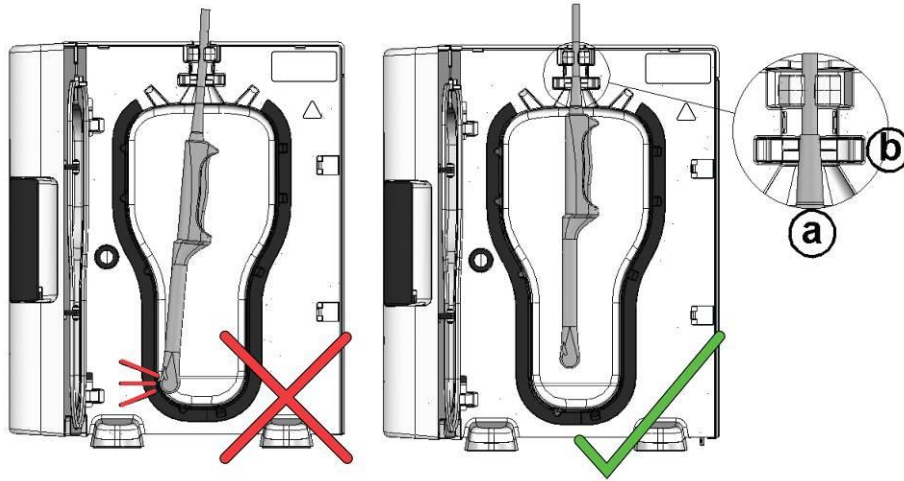
Figure 9

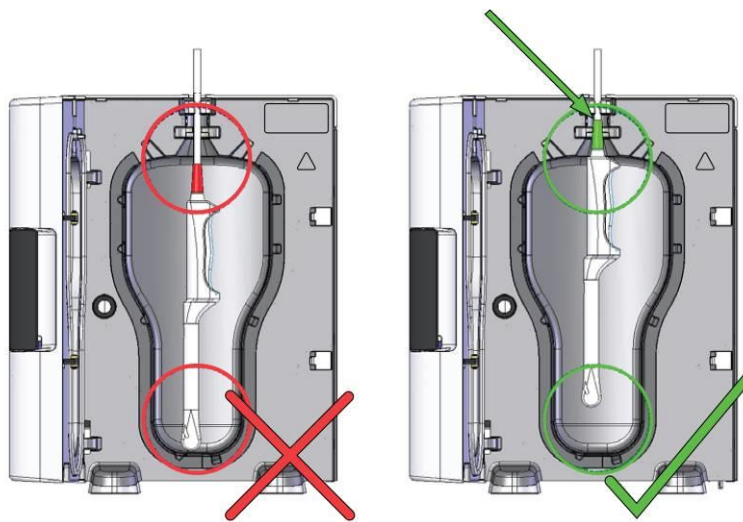
Figure 10


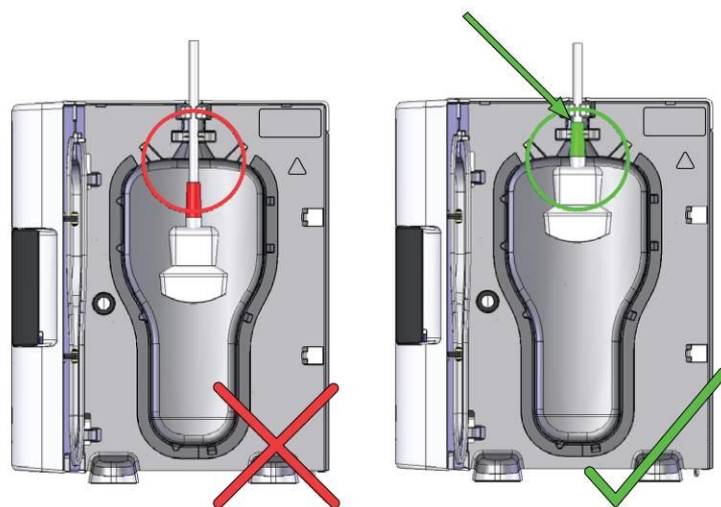
Figure 11

4. Additionally the probe shall be suspended with the tip located no lower than the line across the chamber (see Figures 10 & 11)
5. If a cable management system has been installed on your device, (see section A4.2), secure the external portion of the cable and connector by:
 - Placing the connector carefully inside the tray
 - Coiling the cable neatly and safely around the cable holder on the side of the tray assembly.

NOTE: Incorrect positioning of the probe may result in:

- High-level disinfection will not be achieved during the trophon EPR disinfection cycle
- Residual disinfectant remaining on the probe surface. This may lead to temporary bleaching and/ or irritation of the skin if no gloves are worn.
- Damage to the probe may occur if in contact with the chamber wall

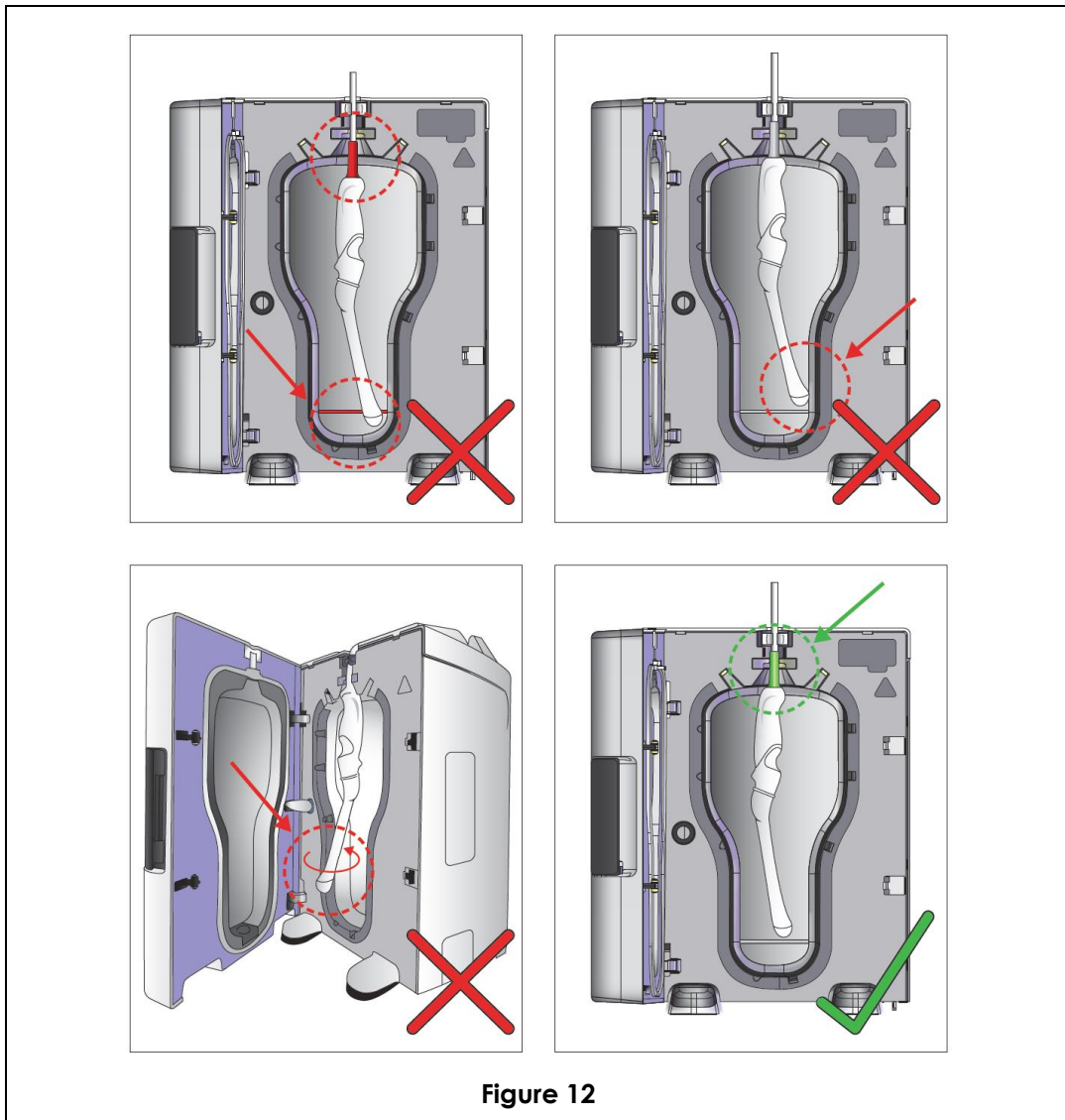NOTE: Curved Probes must be correctly inserted in the device. Refer to Figure 12 for incorrect and correct positioning of probe.

B2.4 Inserting the Chemical Indicator

A Chemical Indicator must be used for each disinfection cycle and can only be used once. After correctly loading the probe into the chamber, a chemical indicator shall be placed into the holder. Refer to the Chemical Indicator Instructions For Use (IFU).

B2.5 Closing the Chamber Door

- The door will automatically lock at the start of a high-level disinfection cycle.
- If the door is not properly closed, screen message: CLOSE CHAMBER DOOR

B2.6 Disinfecting the Probe

Screen message: IS THE PROBE CLEAN AND DRY?

YES

1. If the probe has been pre-cleaned and dried according to section B2.2 above, press YES using the soft key button. The device will then check device readiness to perform a disinfection cycle. Once ready screen message: PRESS START TO BEGIN
2. Press the START button to initiate the cycle or CANCEL using the soft key buttons to unlock the machine and remove the probe
3. The progress of the disinfection cycle is indicated on the screen
4. Screen message: DISINFECTING
5. High Level Disinfection cycle will take 7 minutes to complete.

NO

1. If the probe has NOT been pre-cleaned and dried according to section B2.2, select NO using the soft key button
2. Screen message: REMOVE AND CLEAN THE PROBE
3. Remove the probe and complete pre-clean as directed in section B2.2. Then follow instructions from section B2.3.

NOTE: If mist is visibly escaping from the chamber, remain at a distance from the device until completion of the operating cycle, and until the mist stops. Do not come into direct contact with the mist. Contact your customer service representative (see Part C - Troubleshooting).

B2.7 Removing the Probe

It is important to wear clean gloves when handling probes before and after a disinfection cycle.

1. When the cycle has been successfully completed, the device will sound an audible alarm. Screen message: CYCLE COMPLETE REMOVE AND WIPE PROBE

NOTE: If the probe is not removed within 1 minute of the cycle being complete, the warm-up time will increase for the subsequent cycle.

2. Wear new gloves. Remove the used Chemical Indicator from the device and compare colour change against the chart on the Chemical Indicator carton (refer to the Chemical Indicator IFU for further instructions).
3. Discard used Chemical Indicator
4. Remove the probe carefully using minimal contact after the cycle is complete. Avoid touching the probe against the chamber's hot surface

5. Wipe the probe with an absorbent, single-use, dry, low lint cloth. Visually inspect the probe and ensure any disinfectant residue is removed
6. Discard gloves
7. Close the chamber door
8. The probe is now ready for use or storage.

NOTE: After correct use, at the completion of a successful disinfection cycle, the Ultrasound probe may have surface temperatures up to 45°C. The probe will feel warm to touch and is safe for handling and use.

B2.8 Sleep Mode and Shut Down Procedures

- If the device is not used for 120 minutes or a probe has been left inside the device for an extended amount of time, it will automatically enter sleep mode. To restart the device from sleep mode press RESTART. The device will begin to warm up.
- To obtain maximum use of disinfectant from each NanoNebulant cartridge, it is advised the trophon EPR is left connected to power and switched ON at all times. The device will automatically switch into sleep mode if it is not used for extended periods in order to save power. Switching the system off for more than 24 hours will result in reduced usage from each NanoNebulant cartridge.

SECTION B3: PURGE CYCLE

The purge cycle removes any remaining disinfectant from the cartridge and the inside of the device and converts the NanoNebulant into oxygen and water. Oxygen is vented into the atmosphere. Water is collected in the waste container inside the device (maximum capacity 150ml).

B3.1 Reasons for Running a Purge Cycle

1. A purge cycle is required when the device detects that the disinfectant has expired or if an error has been detected by the device that cannot be rectified without a service call.
2. An onscreen message will be displayed when your device requires purging.
 - To purge the liquid, press PURGE to initiate cycle using the soft key button and proceed to section B3.2.
3. A purge cycle must be manually initiated before lifting or moving the device. To do this:
 - Select MENU using the soft key button
 - Select PURGE.
 - Screen message: REMOVE ALL DISINFECTANT FROM SYSTEM
 - Select YES using the soft key button
 - Then proceed to section B3.2.

NOTE: Once the purge cycle has been commenced it may be paused for a period of time but it cannot be cancelled. Do not switch the device off during purging as this will restart the purge cycle.

B3.2 Running the Purge Cycle

NOTE: Always wear disposable gloves when handling the waste container.

1. Ensure the empty waste container is fully inserted into the device
2. Purge cycle will commence automatically

NOTE: The device will not purge if the waste container is not present

3. The purge cycle will typically take 35 minutes (maximum of 100 minutes). Screen message: PURGING

NOTE: Do NOT attempt to open the cartridge door during the purge cycle.

4. Purge cycle can be paused by pressing the soft key button under PAUSE. Screen message: PURGE PAUSED
5. To continue the cycle, press the soft key button underneath RESUME
6. When purging is complete screen message: REMOVE AND EMPTY WASTE CONTAINER
7. Remove the waste container from the device and ensure it is empty
8. Screen message: LOAD WASTE CONTAINER
9. Screen message: CARTRIDGE EMPTY. REPLACE CARTRIDGE NOW
10. The empty cartridge can now be removed – proceed to section B4.

B3.3 Transporting the Device

- Before transporting the trophon EPR, you must purge the disinfectant and switch off the device at the power switch.
- Do not move excessively or drag the device across the mounting surface.
- Keep device upright at ALL times even during transportation. The device should only be moved in an upright position.

SECTION B4: REMOVING and DISPOSING OF USED DISINFECTANT CARTRIDGES

IMPORTANT: Cartridges are punctured at the top and on the side near the bottom when the cartridge door is closed and locked. A small amount of disinfectant may remain in the cartridge, even when it has been fully used. Follow the instructions carefully to avoid injury.

B4.1 Removing the Cartridge

1. Wear gloves
2. Screen message: REPLACE THE CARTRIDGE AND CLOSE CARTRIDGE DOOR

NOTE: Cartridge door opens automatically. Do NOT force the cartridge door open.

3. Lift the cartridge out by touching the areas exposed while the bottle is in the holder and avoid touching pierced area
4. Do NOT shake or change the orientation of the cartridge
5. Refer to IFU enclosed with the trophon NanoNebulant for detailed instructions on how to install a new cartridge.

B4.2 Disposing of Empty Cartridge

Empty used cartridges should be disposed of in the nearest waste receptacle or according to the disposal guidelines of your Institution.

NOTE: Do NOT insert empty cartridges into the device.

B4.3 Expired Cartridge containing Disinfectant

Follow procedures of your institution for the disposal of CORROSIVE or OXIDIZING materials.

B4.4 Deformed Cartridge

1. Turn disinfectant cartridge the right way up, to allow the cartridge to degas.
2. Contact your customer service representative.

SECTION B5: INCOMPLETE OR FAILED CYCLES

This section describes the most common situations in which a cycle has not been completed satisfactorily and the required actions to take (see also PART C – TROUBLESHOOTING).

B5.1 Mains Power Failure

If the mains power supply to the device is lost while in operation, the current cycle will not complete.

- If the probe is not urgently required, wait for power to come back on. As soon as power is restored the machine will safely recover. Follow the on screen prompts to remove the probe.
- If the probe is urgently required, and power cannot be restored, follow section B5.3.
- Discard the used Chemical Indicator and replace with a new one. Repeat the disinfection cycle.

B5.2 Cycle Fault (during or at the end of the cycle)

If a problem occurs during or at the end of the cycle, a cycle fault will be detected. Follow the on screen instructions:

- Screen message: DISINFECTION CYCLE FAILED, UNLOCK. A beep will be heard.
- Press the soft key button underneath UNLOCK to unlock the chamber door.
- Screen message: DISINFECTION CYCLE FAILED. REMOVE AND WIPE PROBE.
- Wear new gloves. Remove probe immediately and discard the used Chemical Indicator and replace with a new one. Close chamber door and repeat the disinfection cycle.
- In case of a repeated fault or serious malfunction, contact your customer service representative citing the error message shown on the screen.

NOTE: Do NOT attempt to use the device.

B5.3 Manual Door Lock Override

Use ONLY in EXCEPTIONAL circumstances when the probe is locked in the chamber and must be urgently retrieved for use.

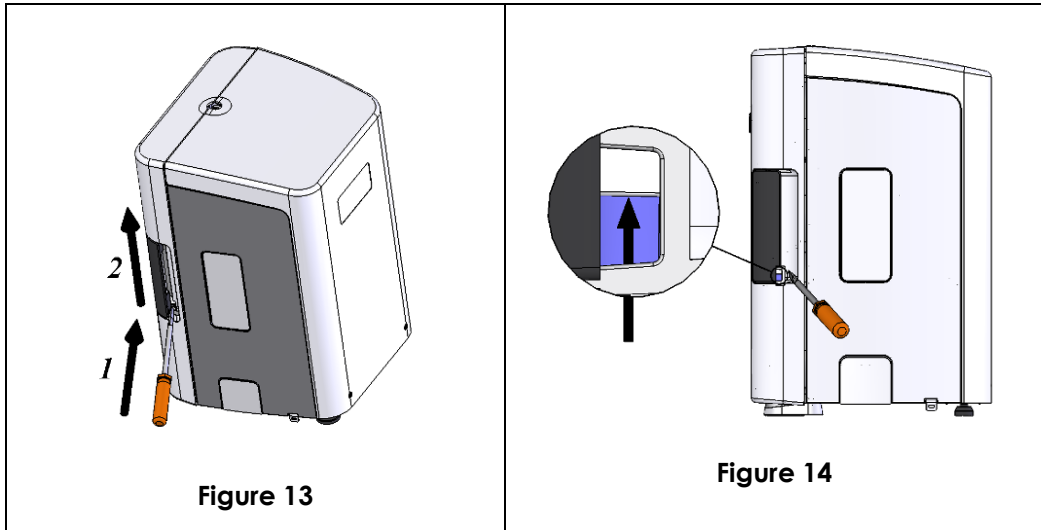
WARNING: THERE MAY STILL BE DISINFECTANT IN THE CHAMBER and CHAMBER SURFACES MAY STILL BE HOT. Personal protective equipment such as gloves should be worn to avoid contact with disinfectant.

By turning off the power and turning it back on, the device will attempt to recover. At this time if the probe still cannot be removed then the following may be undertaken:

1. Turn off device
2. Insert the tip of a screwdriver into the slot behind the chamber door handle gently until it stops. (see Figures 13 and 14)

3. Lift the screw driver in an upwards motion
4. The door will unlock and the probe can be removed.

NOTE: The probe is **NOT DISINFECTED** and **CANNOT** be used until it has completed a successful disinfection cycle or been disinfected by an alternative method.

SECTION B6: ROUTINE CARE AND MAINTENANCE

B6.1 Periodic Clean-Up

1. Wipe the chamber with a damp cloth when cool
2. Close the chamber door
3. To clean the outside of the device, wipe with a soft damp cloth. Do NOT submerge the device, or pour liquids over the device
4. Take care not to have liquid come in contact with power socket (see Figure 4).
5. Inspect vent holes in base of the device; remove any obstruction preventing air flow into the device. Maintain level state during the inspection & removal of any obstruction.

B6.2 Service

Once the service interval of 12 months or 5000 cycles is displayed on the screen, please contact your customer service representative to arrange a service for the device.

SECTION B7: DISPOSAL OF DEVICE

B7.1 Device Disposal

When it is time to decommission the device, please contact your distributor or Nanosonics Europe GmbH. The device will then be disposed to the applicable collection point for the recycling of electrical and electronic equipment.

PART C – TROUBLESHOOTING

The following table will enable a user to diagnose basic problems and implement a solution that may enable the device to resume operation. If a probe is present and you need to retrieve it urgently, see section B5.3.

If the problem persists, contact your Nanosonics representative.

Symptom	Check for the following
1. There is no power to the device 2. The screen is blank	<ul style="list-style-type: none"> The device is plugged in The correct power cable for the region is used The power is switched ON at the wall outlet The power switch on the device is set to ON
3. The chamber door will not open	<ul style="list-style-type: none"> There is power to the device Follow screen messages for instructions Any cycle (purge or disinfect) is complete Chamber door handle can be completely opened
4. The chamber door will not close	<ul style="list-style-type: none"> Follow screen messages for instructions The probe is validated for use in the device The probe is loaded correctly Door lock position: if in the 'locked' position, the door will not close. To check, try opening the handle as if opening the chamber. If the handle cannot be turned, the door is in the locked position – see section B5.3
5. The cartridge door does not open	<ul style="list-style-type: none"> There is power to the device Follow screen messages for instructions The cycle (purge or disinfecting) is complete The waste container is empty and fully inserted
6. The cartridge door will not close	<ul style="list-style-type: none"> Follow screen messages for instructions Correct cartridge type: trophon NanoNebulant Cartridge is correctly positioned Cartridge lid has been removed
7. The probe will not sit correctly in the chamber	<ul style="list-style-type: none"> The probe is validated for use in the trophon EPR The probe is loaded correctly
8. The cycle will not start	<ul style="list-style-type: none"> There is power to the device Follow screen messages for instructions The probe is loaded correctly The chamber door is closed If at the 'Clean and dry probe screen' and the YES button will not register, contact your customer service representative If the screen reads READY and the start button will not respond, contact your customer service representative
9. The device is beeping	<ul style="list-style-type: none"> Follow screen message for instructions Probe inside the device which needs to be removed
10. Liquid is leaking from the device	<p>Warning: Any fluid leaking from the device may contain hydrogen peroxide. If liquid or mist is seen coming from the device at any time:</p> <ul style="list-style-type: none"> Do not come into contact with the mist or liquid Wear appropriate PPE Ensure area is well ventilated Allow the device to complete the cycle Turn off the device and remove the power cord Contact your customer service representative
11. The chamber door is	<ul style="list-style-type: none"> If the device is in sleep mode, select RESTART on the LCD

open and locked.	<p>screen</p> <ul style="list-style-type: none"> • At all other times, power the device OFF using the power switch and back ON (Figure 4, item 10) <p>The device should then automatically unlock the chamber door.</p> <ul style="list-style-type: none"> • If the above are not successful, power OFF the device, and follow section B5.3 for the manual door lock override. Close the Chamber door before powering the trophon EPR ON again.
12. Ultrasound probe – not secure in cable clamp or comes into contact with chamber wall	<ul style="list-style-type: none"> • Refer to trophon Validated Probes list to confirm probe compatibility with trophon EPR • Contact your customer service representative.
13. The device is failing cycles.	<ul style="list-style-type: none"> • Record any error codes and colour of the indicator and contact your customer service representative.
14. The device takes too long to warm up between cycles	<ul style="list-style-type: none"> • Ensure that the probe is removed immediately after the end of each cycle.
15. I cannot locate the serial number for trophon EPR	<ul style="list-style-type: none"> • The trophon EPR serial number can be located on the back label of your device and also on the side label above the power socket.

PART D – TROPHON PRINTER (Accessory)

SECTION D1: INTRODUCTION AND SETUP

The trophon Printer prints a label in real time recording the most recent disinfection cycle directly from the trophon EPR. The label is generated at the completion of each high-level disinfection cycle and can be placed directly on a patient medical record or other documentation. Printed label records remain legible for up to 7 years.

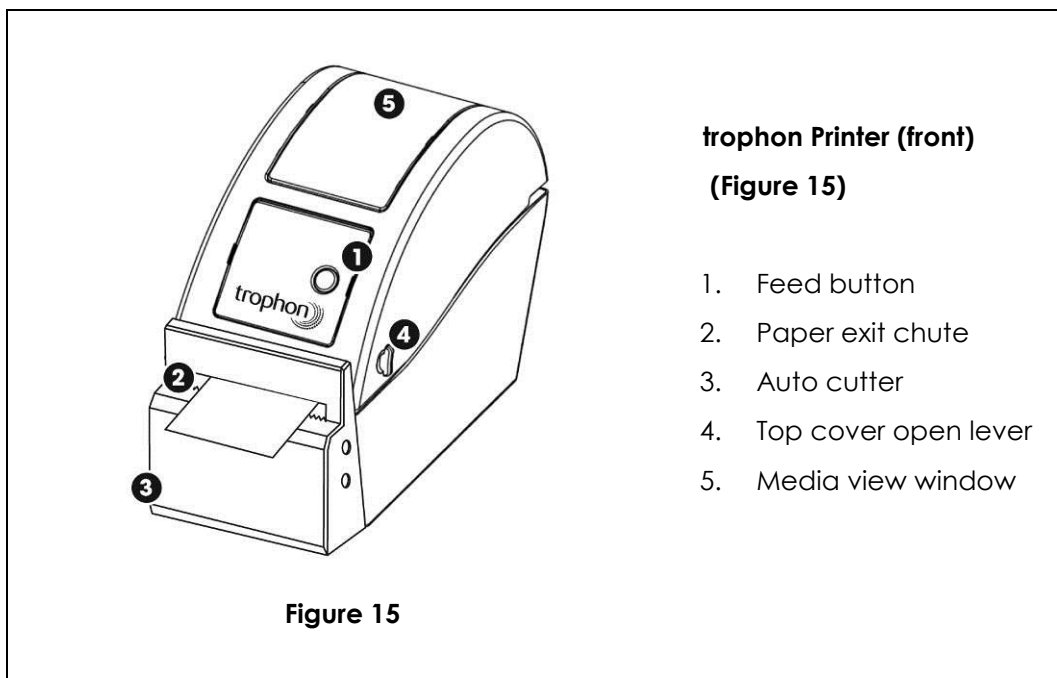
The trophon Printer is designed for use specifically with the trophon EPR. Accuracy and life time of records is not guaranteed if an unapproved accessory (printer or label roll) is used.

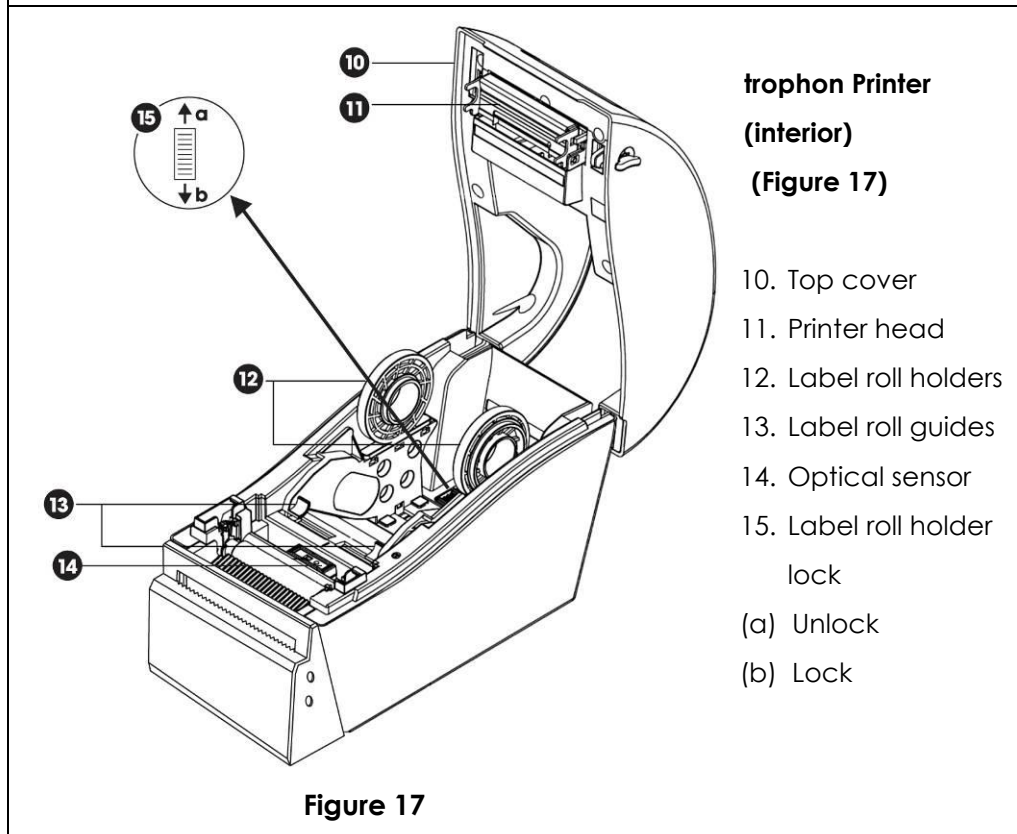
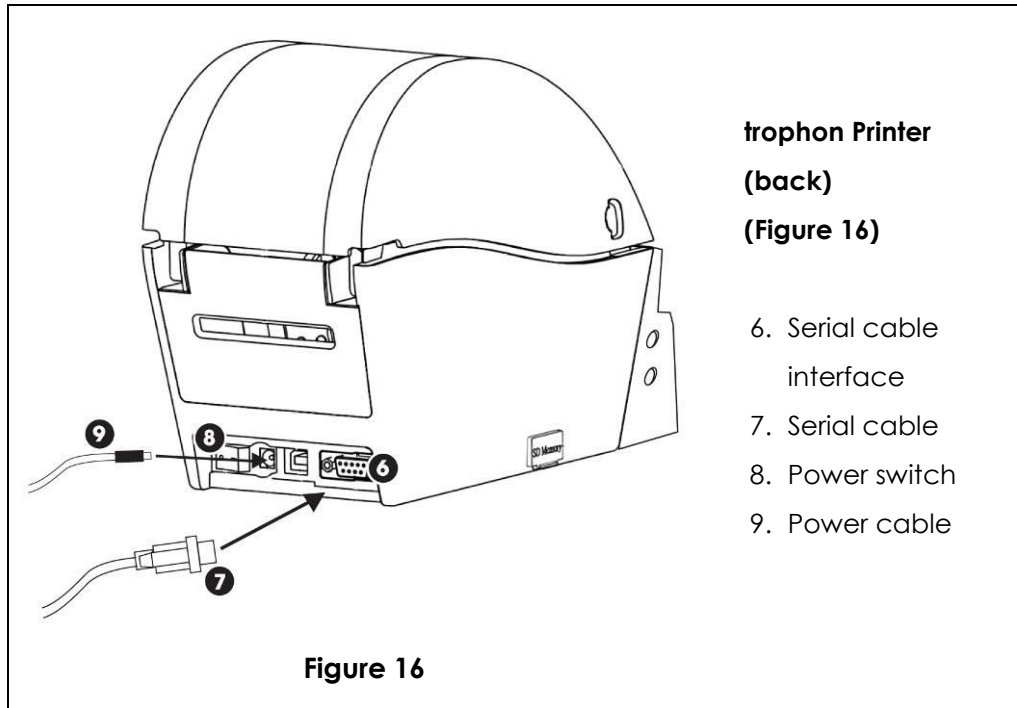
For any operation, fault or maintenance query, please contact your customer service representative.

D1.1 Important Warnings, Labels and Symbols



Please refer to Section A2 of this User Manual for description of all warnings, labels and symbols.

D1.2 Overview of trophon Printer Features

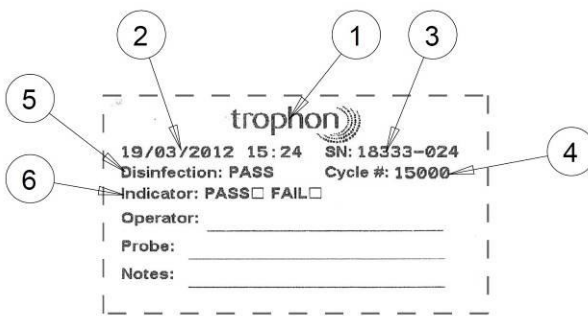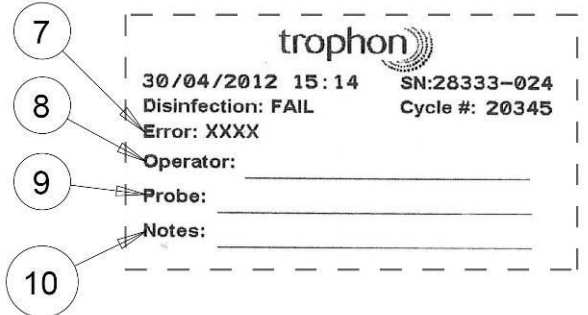D1.3 Overview of trophon Printed Label Record Features

 <p>Figure 18</p>	<p>Printed Label Record (Pass disinfection)</p> <ol style="list-style-type: none">1. trophon logo2. Date and Time3. trophon EPR serial number4. trophon cycle number5. Disinfection result (PASS)6. Chemical indicator result (manual entry field)
 <p>Figure 19</p>	<p>Printed Label Record (Failed disinfection)</p> <ol style="list-style-type: none">7. Error code8. Operator ID (manual entry field)9. Probe ID (manual entry field)10. Notes (manual entry field)

D1.4 trophon Printer – Initial Set-Up

1. Unpack the trophon Printer from its packaging, including the Quick Installation Guide.
2. Position the trophon Printer securely either on a bench beside the trophon EPR, in the container of the trophon Cart or mounted on a wall.
3. Follow the setup instructions included in the box, for powering on the printer and connecting it to your trophon EPR.

NOTE: Ensure the printer power cable is connected to the trophon Printer before connecting to mains power or switching power switch to on position.

4. Follow the setup instructions included in the box for loading the label roll into the trophon Printer.

NOTE: Keep the label roll holders in the "unlock" position. Please note the orientation of the label roll holders in Figure 17, as they can be rotated. Incorrect orientation will prevent correct loading of the label roll into the trophon Printer

D1.5 trophon Printer - Instructions for Use

1. Once the trophon Printer is connected to the trophon EPR via the cable provided and powered on, the printer will be automatically detected.
2. It is recommended that a Test Print is completed as the initial printed label before trophon EPR High-Level Disinfection labels are printed (Refer to D1.6).
3. Printed label records will be available at the completion of each high-level disinfection cycle completed by the trophon EPR.

NOTE: Records of previous high-level disinfection cycles can be re-printed from the trophon printer by selecting MENU → PRINTER → REPRINT LAST RECORD.

D1.6 trophon Printer - Test Print

1. Check the printer connection on the trophon EPR LCD screen:
 - Select MENU using the soft key button
 - Select PRINTER
 - Select TEST PRINT
2. The printer will print and cut a sample label, confirming correct connection.

D1.7 trophon Printer – Number of Label Copies

Select the number of printed label record copies (1- 4) printed for each disinfection cycle.

1. Use the MENU soft key and select PRINTER
2. Select NUMBER OF LABELS
3. Select desired number of printed copies (1 – 4) per disinfection cycle

SECTION D2: ROUTINE CARE AND MAINTENANCE

D2.1 Replacing trophon Printer Label Roll

The trophon EPR will display notification when the label roll requires replacement via an on screen LCD message: PRINTER OUT OF PAPER.

Follow the setup instructions included in the printer box to load label roll.

D2.2 Cleaning the trophon Printer

1. Turn off the printer and unplug from the power socket before cleaning.
2. To clean the outside of the device, wipe with a soft damp cloth. Do NOT submerge the device, or pour liquids over, or spray liquids onto the printer.
3. Use warm mild soapy solution to clean the outside covers of the printer, taking care not to have liquid come in contact with the power socket, the interior of the trophon Printer or label roll.
4. Clean the printer head when replacing the label roll using a soft cloth with ethanol. Refer to Figure 17 for printer head location.

SECTION D3: TROPHON PRINTER TROUBLESHOOTING

D3.1 Connecting and Disconnecting the trophon Printer

The trophon Printer is automatically detected and enabled when connected to the trophon EPR.

To disable the trophon Printer, use the trophon EPR LCD screen:

- Select MENU using the soft key button
- Select PRINTER and then select ENABLE/DISABLE
- Select DISABLED and select OK

D3.2 trophon Printer – Error Messages

trophon Printer error messages are displayed on the LCD screen of the trophon EPR. If an error message is displayed, two options are available:

1. RETRY - select this option via the trophon EPR soft keys
 - After the error has been addressed, select RETRY
 - The trophon will attempt to re-print the last disinfection record. Check the printed record before proceeding to remove the probe from the trophon EPR Chamber.
 - If the error was not addressed, the two options, RETRY and IGNORE will again display
2. IGNORE – select this option via the trophon EPR soft keys
 - If the printed record is not required and you do not want to RETRY the print, select IGNORE.
 - After IGNORE has been selected, remove the probe from the trophon EPR Chamber.

NOTE: If IGNORE is selected, the disinfection label record will not be printed.

TROPHON PRINTER ERROR MESSAGES

Error	Check for the following
No Power to the printer	<ul style="list-style-type: none"> • The printer is plugged in to power • Power is switched ON at the wall outlet and the printer • The printer will display a small green light when powered on.
Printer LED Indicator – solid Red light	<ul style="list-style-type: none"> • The paper cover is closed securely • Printing error - printer head open or cutter error. • Check the trophon EPR LCD Screen for further information
Printer LED Indicator – flashing Red light	<ul style="list-style-type: none"> • Printing error - head open, paper empty, or paper jam • Check the trophon EPR LCD Screen for further information
Printer LED Indicator – Amber light	<ul style="list-style-type: none"> • The system is clearing data from printer. • Check the trophon EPR LCD Screen for further information
Printer LED Indicator – flashing Green light	<ul style="list-style-type: none"> • Printer is paused • Check the trophon EPR LCD Screen for further information • Press the button on the Printer to re-start the Printer
trophon Printer Not Detected	<ul style="list-style-type: none"> • The printer is plugged in correctly to the trophon EPR • The power is switched ON at the wall outlet and the printer • The printer is a Nanosonics approved printer - a trophon logo is printed on the front screen
trophon Printer Not Compatible	<ul style="list-style-type: none"> • The printer is a Nanosonics approved printer - a trophon logo is printed on the front screen
trophon Printer Paused	<ul style="list-style-type: none"> • Select Retry on the trophon EPR LCD screen
trophon Printer Out of Paper	<ul style="list-style-type: none"> • Follow the setup instructions included in the box for loading new label roll
trophon Printer Paper Jam	<ul style="list-style-type: none"> • Labels may be stuck inside the printer mechanism. Check and remove any jammed labels • Once cleared, select Retry on the trophon EPR LCD screen
trophon Printer Cover Open	<ul style="list-style-type: none"> • Close the printer cover • Select Retry on the trophon EPR LCD screen
trophon Printer Cutter Error	<ul style="list-style-type: none"> • Labels may be stuck inside the printer mechanism. Check and remove any jammed labels • Once cleared, select Retry on the trophon EPR LCD screen
trophon Printer Head Overheat	<ul style="list-style-type: none"> • Select Retry on the trophon EPR LCD screen • If the error recurs, switch the printer off and allow to cool, then switch the printer on again • Switch the printer on again and select Retry on the trophon EPR LCD screen • Ensure the printer is positioned in a ventilated area and not confined space.
trophon Printer Error	<ul style="list-style-type: none"> • Switch the printer off and then on again • Select Retry on the trophon EPR LCD screen
No Print on Label	<ul style="list-style-type: none"> • Check the label roll is loaded correctly and the printer is switched on • Hold the feed button down to ensure the label roll is fed through properly • Recommend a test print be done when loading a new label roll into the trophon Printer to ensure correct loading

SECTION D4: TROPHON PRINTER DISPOSAL

Following information is for EU-member states:

The crossed out wheeled bin symbol is used to indicate that the product must not be treated as general waste. By ensuring that this product is disposed of correctly you will be helping to prevent potentially negative consequences for the environment and human health, which could otherwise be caused by incorrect waste handling of this product. For more detailed information about the take-back and recycling of this equipment, please contact the supplier that provided this product.

SECTION D5: REORDER INFORMATION

Please contact your local distributor for re-order information:

trophon Printer:

- 1x trophon Printer
- 1x trophon Printer Power Cable and Power Adapter
- 1x trophon Printer Cable

trophon Printer Label Roll

Replacement trophon Printer Cable

SECTION D6: TROPHON PRINTER TECHNICAL SPECIFICATIONS

Electrical Specification – trophon Printer	External universal switching power supply <ul style="list-style-type: none">• Input: AC 100- 240V• Output: DC 24V 2A
Electrical Specification – Transformer	<ul style="list-style-type: none">• Input Voltage:<ul style="list-style-type: none">-Maximum voltage: 264Vac-Normal voltage: 100 – 240Vac-Minimum voltage: 90Vac• Input Frequency:<ul style="list-style-type: none">-Maximum frequency: 63Hz-Normal frequency: 50 – 60Hz-Minimum frequency: 47Hz• Input Current:<ul style="list-style-type: none">-1.8A (Max) at 115 Vac with full load-0.9A
Environmental Specification	<ul style="list-style-type: none">• Operation: 41-104°F (5-40°C), 25 – 85% non-condensing• Storage: -40-140°F (-40-60°C), 10 – 90% non-condensing
Physical Dimension	<ul style="list-style-type: none">• 250mm (L) x 110mm (W) x 175mm (H)
Interface	RS-232c serial cable (max 115,200 bps)
Safety Regulation	FCC Class B, CE Class B, C-Tick Class B, UL, TÜV/Safety

PART E – SERVICE AND WARRANTY

If you have any questions or concerns regarding the trophon EPR, trophon Printer or disinfectant supplies, contact your Nanosonics representative.

The trophon EPR and trophon Printer has a comprehensive warranty against defects in material and workmanship for 12 months from the date of delivery.

Please ask your Nanosonics customer service representative for details about your trophon Printer warranty.

A warranty registration form is enclosed within this manual. Complete the warranty registration form including the date at the time of initial set-up and return to your customer service representative. If you have not received a warranty registration form or would like to obtain further information about post warranty contracts please contact your Nanosonics representative immediately.

Service Schedule: To ensure correct operation, the trophon EPR must be serviced by authorised service personnel only. Service Schedule is every 12 months or at every 5000 cycles; this will be indicated by the device by displaying the screen message: DEVICE SERVICE DUE prior to commencing a disinfection cycle. This message will be displayed weekly on the screen until a service is performed. Service intervals can also be accessed via the system information, located in the trophon EPR LCD menu. When required, a disinfection history log can be downloaded from your trophon EPR device and printed via trophon Connect software or provided by your service centre. Contact your Nanosonics representative to obtain a print out for your records.

NOTE: Unauthorised modification to the trophon EPR or service by someone other than authorised service personnel will void your warranty.

APPENDIX 1: TROPHON EPR TECHNICAL SPECIFICATIONS

Electrical Specification	<ul style="list-style-type: none">• Rated input voltage: 230V AC• Rated input current: 3Amp, 50/60Hz• Input IEC• Output DB9• Equipment must be connected to an earthed outlet using power cable supplied with the device
Environmental Specification	<ul style="list-style-type: none">• Operating temperature range: 17-27°C• IP20
Physical Characteristics	<ul style="list-style-type: none">• Weight of device: 17kg• Dimensions of device: 490mm high x 345mm wide x 345mm depth
EMC Compliance	This device has been tested and found to comply with the limits for emission requirements (Electro-Magnetic Interference) pursuant to EN61000-4-2: 2005 & EN 61000-4-3: 2006. CISPR 11 Group 1 Class A equipment to CISPR 16-1 & CISPR 16-2, EN61000-4-2: 2005 & EN 61000-4-3 : 2006.

APPENDIX 2: PRODUCT WARRANTY AND REGISTRATION FORM

This REGISTRATION FORM must be completed in full and returned to your Nanosonics customer service representative at the address below upon purchase of the trophon EPR.

Terms

Nanosonics Limited ABN 11 095 076 896 warrants to the customer that the trophon EPR is free from defects in material and workmanship that materially affect its function under normal use and service for a period of **12 months commencing upon the date of purchase**.

The following statement only applies if you are a 'consumer' for the purpose of the Australian Consumer Law.

Our goods come with guarantees that cannot be excluded under the *Australian Consumer Law*. You are entitled to a replacement or refund for a major failure and for compensation for any other reasonably foreseeable loss or damage. You are also entitled to have the goods repaired or replaced if the goods fail to be of acceptable quality and the failure does not amount to a major failure. The benefits under this product warranty are in addition to other rights and remedies you may have under law in relation to our goods.

The following statement only applies if you are a 'consumer' for the purpose of the Consumer Guarantees Act 1993.

Our goods come with guarantees that cannot be excluded under the *Consumer Guarantees Act 1993*. This guarantee applies in addition to the conditions and guarantees implied by that legislation.

Conditions

For this warranty to be valid, you must have completed all details on this form and returned it to your Nanosonics customer service representative within 30 days of purchase.

We are also entitled to refuse to honor this warranty in the following circumstances (regardless of how those circumstances arise):

- (a) where the trophon EPR has not been used, handled, installed, stored, cleaned and serviced in accordance with the relevant user manual or other written instructions issued by Nanosonics (including where used in temperature or other external conditions exceeding those set out in the product specification, or serviced by persons other than Nanosonics' approved service personnel);
- (b) where modifications have been made to the trophon EPR, other than by Nanosonics or its authorised service providers;
- (c) where unauthorised consumables, accessories or other chemicals or items have been used with the trophon EPR;
- (d) where the trophon EPR is used in conjunction with other equipment or products (other than multiple use ultrasound probes as described in the user manual), without Nanosonics' prior written consent;
- (e) where the trophon EPR has been damaged due to external or environmental causes of any kind (including factors such as voltage fluctuations, excess voltage or power failure);
- (f) where the trophon EPR has been damaged as a direct or indirect result of any malicious or negligent act or omission by any person (other than Nanosonics or its authorised service providers);
- (g) where the defect does not materially affect the function of the trophon EPR (for example scratches or marks on the external surface of the trophon EPR); or

- (h) where the serial number or product label has been removed, changed, deleted or made unrecognizable, or if the number or label is no longer clearly distinguishable for other reasons beyond Nanosonics' control and therefore it is not possible to conclusively identify the product.

This warranty applies to the trophon EPR only; the warranty does not cover the replacement of used NanoNebulant or of parts which need periodic replacement during the life of the product as a result of the ordinary use made of them, unless the item itself is defective.

Further information and making a claim

Please contact your Nanosonics customer service representative (details below) with any queries regarding this warranty or post warranty repairs, or if you wish to make a warranty claim:

Nanosonics Limited
Unit 24/566 Gardeners Road
Alexandria NSW 2015
Australia
+61 2 8063 1660
support@nanosonics.com.au

Your Nanosonics customer service representative will make arrangements for the collection and repair or replacement (as relevant) of your trophon EPR. You will be responsible for uninstalling, reinstalling and recommissioning the trophon EPR, regardless of whether or not it is found to be defective. We may require you to bear shipping costs if the trophon EPR is not defective, or require you to bear repair and shipping costs if you are not entitled to the benefit of this warranty.

We will repair or replace any defective trophon EPR at our discretion. If we are unable to do so for any reason, we will discuss with you for an appropriate solution including upgrading you to a newer model or refund the purchase price.

Please note that goods presented for repair may be replaced by refurbished goods of the same type rather than being repaired. Refurbished parts may be used to repair the goods.

Please retain the top half of this form

Name: _____ **Email:** _____

Phone: (____) _____ **Mobile:** _____ **Fax:** _____

Contact Address: _____

City: _____ **State:** _____

Post Code: _____ **Country:** _____

Device model: _____ **Serial No:** _____

Date of purchase: _____ **Invoice No:** _____

I understand and accept that the trophon EPR is warranted against defects in material and workmanship for **12 months** from the date of purchase, on the terms and conditions set out above.

I also understand that the warranty will be void in certain circumstances, such as where unauthorised modifications are made to the above product, or it is serviced by someone other than Nanosonics' approved customer service personnel.

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