

TRUMPF | OR TABLES

Components and accessories



INNOVATIONS FOR PATIENT CARE

TRUMPF



Components and accessories for TRUMPF operating tables

Operating tables by TRUMPF are distinguished in particular by their functionality, flexibility and individuality. The basis for this is formed by the universal mechanical coupling system of the TRUMPF operating tables as well as a wide range of components and accessories. These were developed in close collaboration with doctors and OR personnel and are tailor-made to suit the needs of our customers. This means that you can put together the ideal operating table for every discipline and for every individual requirements. In addition to this, our accessories guarantee the optimum positioning of the patients.

Needless to say, our components and accessories are subject to the same high quality requirements as all other TRUMPF products.

We use rust-free stainless steel as the material for our tables and accessories. In order to offer you optimum conditions for intraoperative imaging, our product portfolio also includes a wide range of components and accessories in carbon fibre.

Our standard pad Comfort Plus is made from soft, dimensionally stable polyurethane foam with a thickness of 50 mm. Fashioned from a single piece, it has no seams, is washable and antistatic.

In order to prevent decubitus, two further pad versions are available to the user:

- Medifoam: dynamic foam
- FoamLine: visco-elastic foam, antistatic

These pads adapt to the body contours of the patient particularly well and reduce the pressure at the various pressure points.



CONTENTS


Components and accessories for table tops	5
Standard head sections	6
Special head rest systems	8
Skull clamps for neurosurgery	16
Back section extensions	21
Pelvis extensions	23
Leg sections	27
Leg sections for gynaecology and urology	29
Standard accessories	31
Clamps	32
Anaesthesia screens	33
Arm supports	35
Lateral support	37
Patient fixation	40
Arm support tables	44
Side rails/Side rail extensions	45
Urology	47
Spine positioning	49
Radiology	51
Storage systems	52
Instrument tables and kick buckets	53
Mounting steps and swivel chairs	54
Accessories for orthopaedics	55
Knee positioning	56
Arm positioning	57
Shoulder operation	59
Extension	61
Hip surgery	65
Thorax	66
Pelvis	67
Storage systems	69
Positioning accessories	71

TRUMPF

Components and accessories for table tops



Standard head sections

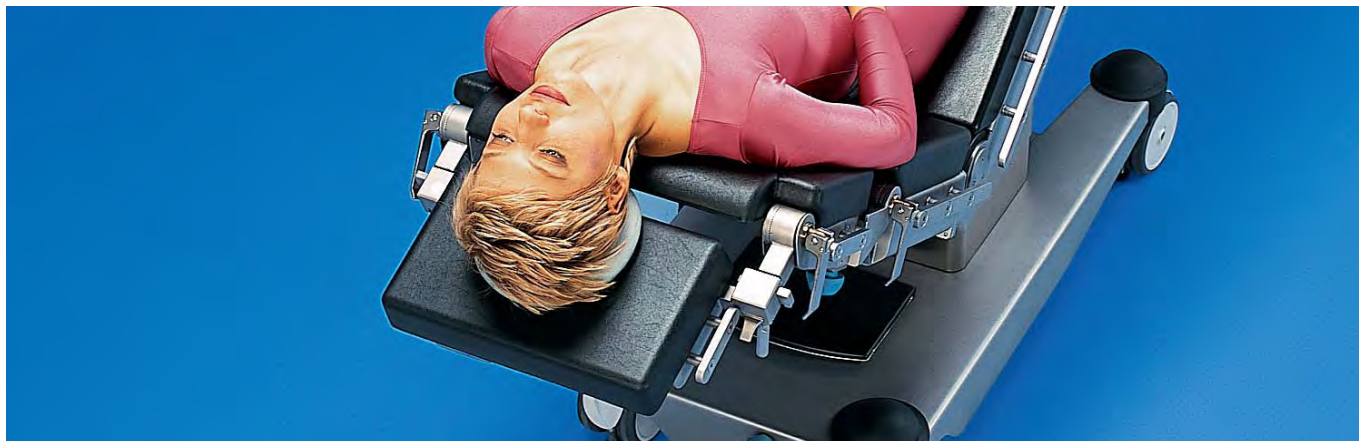

HEAD SECTION SINGLE JOINT COMFORT

Part No. 4549151 / 4549600

To attach to TRUMPF coupling point system; with single joint and one hand adjustment; Comfort Plus pad integrally foamed, electrically conductive and soft; stainless steel frame

Length: 245 mm

Width over side rails: 424 mm

Dimensions pad: 225 mm x 375 mm

Adjustment range: $\pm 55^\circ$

HEAD SECTION PAD DETACHABLE 4549600

HEAD SECTION with Medifoam pad: 1375602

HEAD SECTION with FoamLine pad: 1397909



HEAD SECTION DOUBLE JOINT

Part No. 4549105

Only to attach to TRUMPF coupling point system with joint; head section only with single joint; realisation double joint function through joint at the operating table; Comfort Plus pad integrally foamed, electrically conductive and soft; stainless steel frame

Length: 230 mm

Width over side rails: 424 mm

Dimensions pad: 225 mm x 375 mm

Adjustment range: $+ 55^\circ$



HEAD SECTION DOUBLE JOINT COMFORT**Part No. 4549152**

To attach to TRUMPF coupling point system; with double joint and one hand adjustment; Comfort Plus pad integrally foamed, electrically conductive and soft; stainless steel frame

Length: 350 mm

Width over side rails: 424 mm

Pad: 225 mm x 375 mm

Adjustment range front joint: $\pm 55^\circ$

Adjustment range back joint: $+ 55^\circ$

**HEAD SECTION 530 DOUBLE JOINT****Part No. 4549601**

To attach to TRUMPF coupling point system; with double joint and one hand adjustment; Comfort Plus pad integrally foamed, electrically conductive, soft and removable; pad fastening via Velcro; with lateral and frontal side rails; stainless steel frame

Length: 320 mm

Width over side rails: 580 mm

Pad: 260 mm x 520 mm

Adjustment range front joint: $\pm 60^\circ$

Adjustment range back joint: $+ 60^\circ$

HEAD SECTION 530 M with Medifoam pad: 1314342


HEAD SECTION 530 DOUBLE JOINT T with FoamLine pad: 1397893



Special head rest systems


Head rest, horseshoe-shaped, 3D-adjustment

- 1 ADAPTER FOR HEAD POSITIONING · Part No. 4544785
- 2 CENTRAL HOLDER FOR HEAD REST SYSTEM · Part No. 4544786
- 3 CONNECTION HOLDER WITH JOINT SHORT · Part No. 4544733
Optional: CONNECTION HOLDER WITH JOINT · Part No. 4544729
- 4 CROSS JOINT · Part No. 4544787
- 5 HEAD REST HORSESHOE-SHAPED TWO PARTS · Part No. 4549732

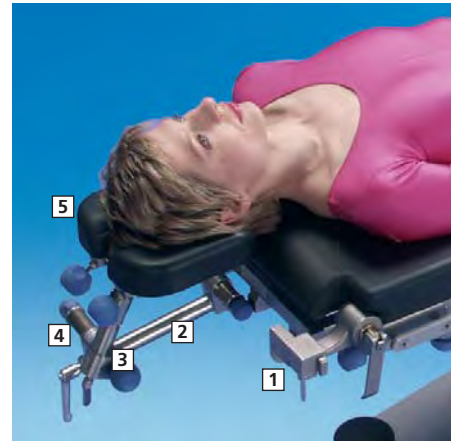


Head rest, horseshoe-shaped, basic

- 1 CENTRAL HOLDER HEAD REST SYSTEM BASIC · Part No. 4544788
- 2 CONNECTION HOLDER · Part No. 4544728
- 3 HEAD REST HORSESHOE-SHAPED TWO PARTS · Part No. 4549732

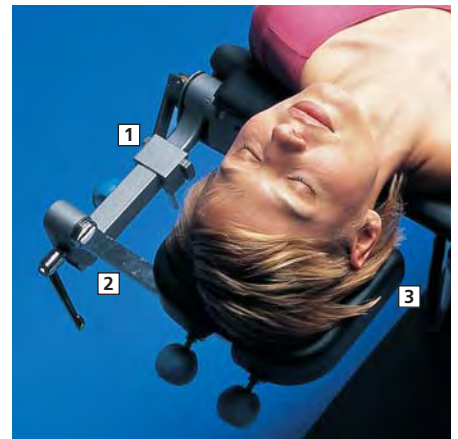

Head rest with RETAINER FOR HEAD RESTS VARIOUS

- 1 ADAPTER FOR HEAD POSITIONING · Part No. 4544785
- 2 RETAINER FOR HEAD RESTS VARIOUS · Part No. 1300079
- 3 CONNECTION HOLDER WITH JOINT · Part No. 4544729
- 4 HEAD REST HORSESHOE-SHAPED TWO PARTS · Part No. 4549732

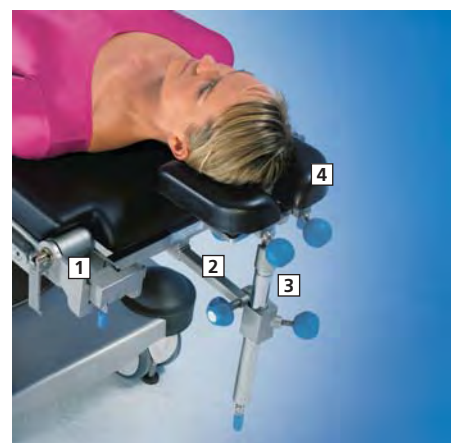

ADAPTER FOR HEAD POSITIONING**Part No. 4544785**

To attach to TRUMPF coupling point system; for attaching head positioning accessories; stainless steel
Distance retainer Mayfield: 220 mm

**ADAPTER FOR HEAD POSITIONING MERKUR****Part No. 4544734**

To attach to side rails of MERKUR operating tables; for retaining head positioning accessories; stainless steel

**RETAINER FOR HEAD RESTS VARIOUS****Part No. 1300079**

To attach to adapter for head positioning, shoulder segment, shoulder adapter and ophthalmology adapter; stainless steel
Pivoting range: $\pm 90^\circ$
Length adjustment up to 39 cm


**CONNECTION HOLDER BASIC DISPLACED****Part No. 1305449**

To attach to retainer for head rest; stainless steel
CONNECTION HOLDER BASIC · Part No. 1254739 (not shown)

**CONNECTION HOLDER WITH JOINT****Part No. 4544729 / 4544733**

To attach to central holder for head rest system 4544786, retainer for head rests various 1300079 and cross joint 4544787; stainless steel
Length of side rail: 185 mm
Adjustment ball joint:
Rotable: 360° / tilt: $\pm 30^\circ$

- 1 CONNECTION HOLDER WITH JOINT · Part No. 4544729
- 2 CONNECTION HOLDER WITH JOINT SHORT · Part No. 4544733

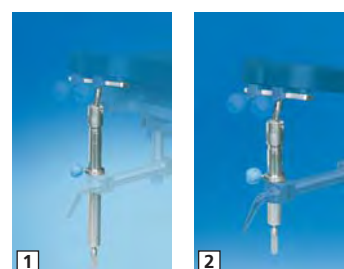

CENTRAL HOLDER HEAD REST SYSTEM BASIC

Part No. 4544788

To attach to TRUMPF coupling point system; stainless steel



CONNECTION HOLDER

Part No. 4544728

To attach to central holder head rest system basic; stainless steel

Dimension: 10 mm x 25 mm

Length: 300 mm



CENTRAL HOLDER FOR HEAD REST SYSTEM

Part No. 4544786

To attach to adapter for head positioning and ophthalmology adapter; central operating unit for setting height and angle of head rest; stainless steel



CROSS JOINT

Part No. 4544787

For the lateral positioning of connection holder with joint; to increase the free movement of head rests; stainless steel



HEAD REST BASIC**Part No. 4549730**

With clamp; to attach to connection holder; Comfort Plus pad integrally foamed, electrically conductive and soft; stainless steel frame
 Dimensions: 250 mm x 200 mm

**HEAD REST HORSESHOE-SHAPED ONE PART****Part No. 4549731**

With clamp; to attach to connection holder; Comfort Plus pad integrally foamed, electrically conductive and soft; stainless steel frame
 Dimensions: 180 mm x 180 mm

**HEAD REST HORSESHOE-SHAPED TWO PARTS****Part No. 4549732**

With clamp; to attach to connection holder; adjustable to head size; Comfort Plus pad integrally foamed, electrically conductive and soft; stainless steel frame
 Dimensions: 180 mm x 180 mm

**HASLINGER HEAD REST****Part No. 4545680**

To attach to adapter for head positioning and ophthalmology adapter; precise height adjustment through hand crank; stainless steel frame



OPHTHALMOLOGY ADAPTER MOTORISED

Part No. 4549525 + 4549732

To attach to TRUMPF coupling point system; with connection holder for TRUMPF head rest system; sensor monitored, electric motor driven adjustment of the connection holder:

upwards: + 90 mm / downwards: -70 mm

Manual head rest adjustment on the connection holder:

longitudinal adjustment: ± 30 mm / height adjustment: ± 28 mm /

tilting backwards: ca. 45°

Comfort Plus pad integrally foamed, electrically conductive and soft; stainless steel frame

Dimensions: 600 mm x 530 mm

HEAD REST HORSESHOE-SHAPED TWO PARTS · Part No. 4549732

Other models of head sections and head rests can be combined, see previous side.



Head rest "one hand" and ophthalmology adapter

1 OPTHALMOLOGY ADAPTER · Part No. 4549127

To attach to TRUMPF coupling point system; with square shank; Comfort Plus pad integrally foamed, electrically conductive, soft and detachable; stainless steel frame

Dimensions: 300 mm x 500 mm / 230 mm

2 OPTHALMOLOGY ADAPTER SWORD RETAINER · Part No. 4549176

To attach to TRUMPF coupling point system; with a square shank and sword retainer; Comfort Plus pad integrally foamed, electrically conductive, soft and detachable; stainless steel frame

OPHTHALMOLOGY ADAPTER SWORD RETAINER S · Part No.1344301

Model for SATURN

3 HEAD REST FLAT 20 · Part No. 1256701


With one hand adjustment; to attach to ophthalmology adapter

4 HEAD REST FLAT · Part No. 1256031

With one hand adjustment; to attach to ophthalmology adapter with sword retainer

5 GUIDING ROLLER FOR CERVIX TRACTION · Part No. 1446211

For head-side traction; to attach to ophthalmology adapter with sword retainer; stainless steel

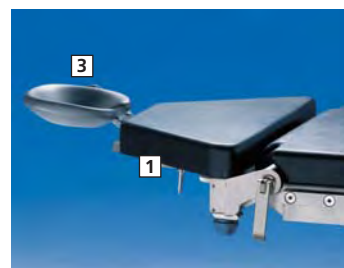

WRIST SUPPORT**Part No. 1240299**

To attach to the side rail of the connection holder; for the surgeon's hand during micro-surgical interventions on eyes; bow hinged, height and transversal adjustable; stainless steel

Height adjustment: 220 mm

Transversal shift: ± 40 mm

**WRIST SUPPORT 20****Part No. 1295542**

To attach to ophthalmology adapter with sword retainer; for the surgeon's hand during micro-surgical interventions on eyes; bow hinged, height and transversal adjustable; stainless steel

Height adjustment: 220 mm

Transversal shift: ± 40 mm

**HEAD RING****Part No. 1218975**

Foam material; electrically conductive

Diameter: 215 mm

Cross section: 40 mm



Head rests with carbon fibre elements



ADAPTER X-RAY NARROW

Part No. 1297206

For attaching head rest accessories X-Ray; made of carbon fibre; single joint
Length x width: 250 mm x 400 mm

ADAPTER X-RAY WIDE · Part No. 1312733 (not shown)

For attaching head rest accessories X-Ray; made of carbon fibre; single joint
Length x width: 250 mm x 550 mm



ADAPTER X-RAY DOUBLE JOINT NARROW

Part No. 1312734

For attaching head rest accessories X-Ray; made of carbon fibre; double joint
Rotable 360°/swivel-mounted $\pm 45^\circ$
Length 400 mm x width 400 mm

ADAPTER X-RAY DOUBLE JOINT WIDE · Part No. 1312735 (not shown)

For attaching head rest accessories X-Ray; made of carbon fibre; double joint
Rotable 360°/swivel-mounted $\pm 45^\circ$
Length 400 mm x width 550 mm



ADAPTER X-RAY 3-JOINT NARROW

Part No. 1430023

For attaching head rest accessories X-Ray; made of carbon fibre
Length 550 mm x width 400 mm

ADAPTER X-RAY 3-JOINT WIDE · Part No. 1430024

For attaching head rest accessories X-Ray; made of carbon fibre
Length 550 mm x width 550 mm



HEAD SECTION X-RAY NARROW**Part No. 1300664**

For head positioning; to attach to all adapters X-Ray; Comfort Plus pad integrally foamed, electrically conductive, soft and removable

Length x width: 180 mm x 380 mm

HEAD SECTION X-RAY NARROW M with Medifoam pad: 1424566

HEAD SECTION X-RAY NARROW T mit Polster FoamLine: 1424588

**HEAD SECTION X-RAY WIDE****Part No. 1297207**

For head positioning; to attach to all adapter X-ray wide; fully translucent; Comfort Plus pad integrally foamed, electrically conductive, soft and removable

Length x width: 180 mm x 530 mm

HEAD SECTION X-RAY WIDE M with Medifoam pad: 1424565

HEAD SECTION X-RAY WIDE T with FoamLine pad: 1424577

**HEAD REST X-RAY****Part No. 1300662**

For supporting the head; to attach to all adapter X-Ray; adjustable to head sizes; Comfort Plus pad integrally foamed, electrically conductive and soft

Dimensions: 200 mm x 250 mm x 100 mm

Pivoting range: $\pm 45^\circ$


**HELMET X-RAY****Part No. 1300663**

For supporting the head; to attach to all adapter X-Ray; fully radiolucent; u-shaped helmet with soft pads and 2 fixation straps with velcro fasteners for safe positioning and lateral fixation of the head while the patient is anaesthetised; adapted to varying the head sizes from adults to children; pads removable



Skull clamps for neurosurgery

Neurosurgery

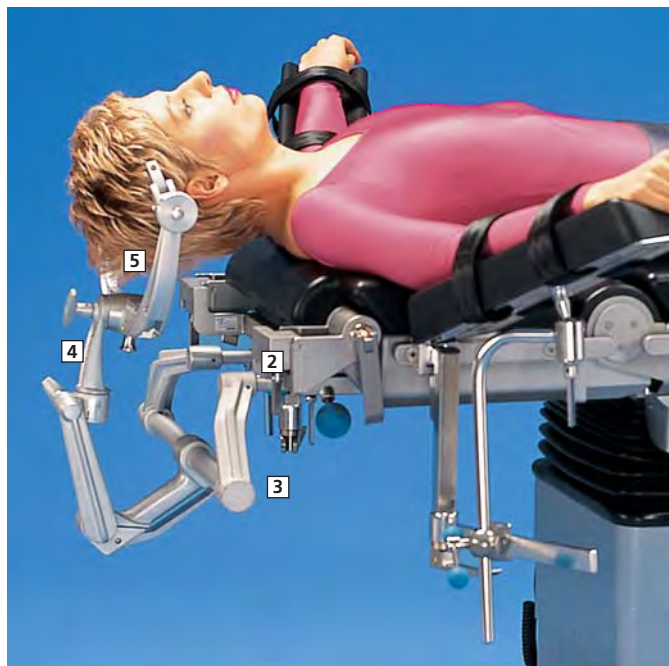

- 1 CROSSBAR ATTACHMENT · Part No. 4544436
2 radial setting clamps 1218804 required
- 2 ADAPTER FOR HEAD POSITIONING · Part No. 4544785
- 3 BASIC UNIT NEURO · Part No. 1218970
- 4 SKULL CLAMP ADAPTER · Part No. 1218969
- 5 SKULL CLAMP · Part No. 1218968
(Supply without skull pin holders and without skull pins)

CROSSBAR ATTACHMENT

Part No. 4544436

Without clamp; for attaching neurosurgery accessories; stainless steel;
height adjustment: 230 mm



ADAPTER FOR HEAD POSITIONING**Part No. 4544785**

To attach to TRUMPF coupling point system; for attaching head positioning accessories; stainless steel
Distance retainer Mayfield: 220 mm

**ADAPTER FOR HEAD POSITIONING MERKUR****Part No. 4544734**

To attach to side rails of MERKUR operating tables; for retaining head positioning accessories; stainless steel

**BASIC UNIT NEURO****Part No. 1218970**

Simultaneous fixation of the longitudinal and vertical position; cast aluminium

**SKULL CLAMP ADAPTER****Part No. 1218969**

connection between basic unit NEURO 1218970, skull clamp 1218968 or multi-purpose head piece 4145424; cast aluminium

**SKULL CLAMP****Part No. 1218968**

Stable and precise 3-point fixing of skull; cast aluminium



MULTIPURPOSE HEAD PIECE

Part No. 4145424

Suitable for children and adults of all ages; modular fixation system for skull pins and elastic pads



HEAD SUPPORT

Part No. 4145417

For attachment of skull pin holder and gel pads; for additional support of the skull



GEL PADS

Part No. 4145416

For mounting to skull clamp 1218968, multi purpose head piece 4145424 and head support 4145417



DUAL PADS FOR HEAD REST

Part No. 4145409

For mounting to skull clamp 1218968, multi purpose head piece 4145424 and head support 4145417



HEAD REST FOR ADULTS

Part No. 1218974

Adjustable to the head size; for connection to the skull clamp adapter 1218969; with elastic pad; cast aluminium



HEAD REST FOR CHILDREN**Part No. 1218973**

Adjustable to the head size; for connection to the skull clamp adapter 1218969;
with elastic pad; aluminium cast

**Skull pin holders**

SINGLE PIN HOLDER · Part No. 4145406 (not shown)
Stainless steel, autoclavable

SKULL PIN HOLDER FOR ADULTS · Part No. 4145407
Holder for two skull pins for adults 4145420
Stainless steel, autoclavable

SKULL PIN HOLDER FOR CHILDREN · Part No. 4145415
Holder for two skull pins for children 4145421
Stainless steel, autoclavable

SKULL PIN HOLDER FLEXIBLE · Part No. 1251207 (not shown)
Holder for two skull pins for children 4145421
Autoclavable

**Skull pins**

SKULL PINS FOR ADULTS · Part No. 4145420
Stainless steel, autoclavable – package with three pieces

DISPOSABLE SKULL PINS · Part No. 4145419 (not shown)
sterile; package with 36 pieces

DISPOSABLE SKULL PIN X-RAY · Part No. 4545688 (not shown)
sterile, radiolucent with titanium tip; package with 36 pieces

SKULL PINS FOR CHILDREN · Part No. 4145421
Stainless steel, autoclavable; package with three pieces



SKULL CLAMP X-RAY**Part No. 1317569**

Radiolucent skull pin made of carbon fibre, for a stable and precise 3-point fixation of skull; modular fixation system for skull pins and elastic pads; suitable for children and adults of all ages; material CT and MR compatible
Dimensions: 302 mm – 410 mm x 140 mm x 360 mm

Attachment to table tops Carbon X-Tra and table top segments Carbon 600 and narrow possible with the following accessories:

- 1 SWIVEL ADAPTER X-RAY · Part No. 1317786
Connection between skull clamp X-Ray 1317569 and retainer X-Ray 1345749; material CT and MR compatible
- 2 RETAINER X-RAY · Part No. 1345749
To attach to all adapter X-Ray; for retaining the swivel adapter X-Ray 1317570; CT and MR compatible
- 3 COUPLING PIECE X-RAY LOW · Part No. 1368880
For lower positioning of the patient's head when using the skull clamp X-Ray 1317569; made of carbon fibre
- 4 ADAPTER X-RAY 3-JOINT NARROW · Part No. 1430023
To attach to storage areas carbon fibre and table top segments carbon fibre with coupling point carbon fibre; to attach head positioning accessories; made of carbon fibre

ADAPTER X-RAY 3-JOINT WIDE · Part No. 1430024

**HEAD REST SYSTEM RADIOLUCENT****Part No. 1251251**

To attach to the side rails; for invasive and non-invasive applications in head and neck surgery



Back section extensions



TABLE TOP SEGMENT 130

Part No. 4549339

To position the upper part and head of the patient in the supine, lateral or prone position; without mechanical joint; Comfort Plus pad integrally foamed, electrically conductive, soft and detachable; frame made of rust-free stainless steel; suitable for SATURN; JUPITER; MARS and TITAN

Length 130 mm

TABLE TOP SEGMENT 130 with Medifoam pad: 1424567

TABLE TOP SEGMENT 130 with FoamLine pad: 1424954



UPPER BACK SECTION


Part No. 4549101

To position the upper part and head of the patient in the supine, lateral or prone position; Comfort Plus pad integrally foamed, soft and detachable; stainless steel frame; suitable for JUPITER, MARS and TITAN

Length 225 mm

UPPER BACK SECTION with Medifoam pad: 1314330

UPPER BACK SECTION with FoamLine pad: 1397891

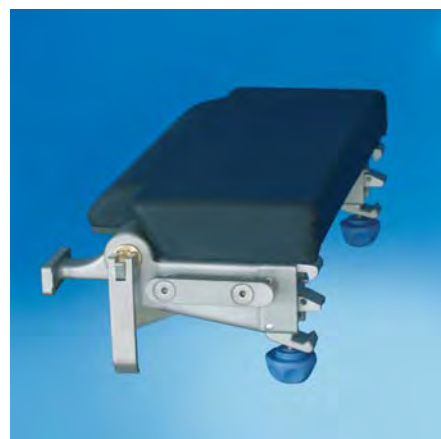

Back section extensions carbon fibre

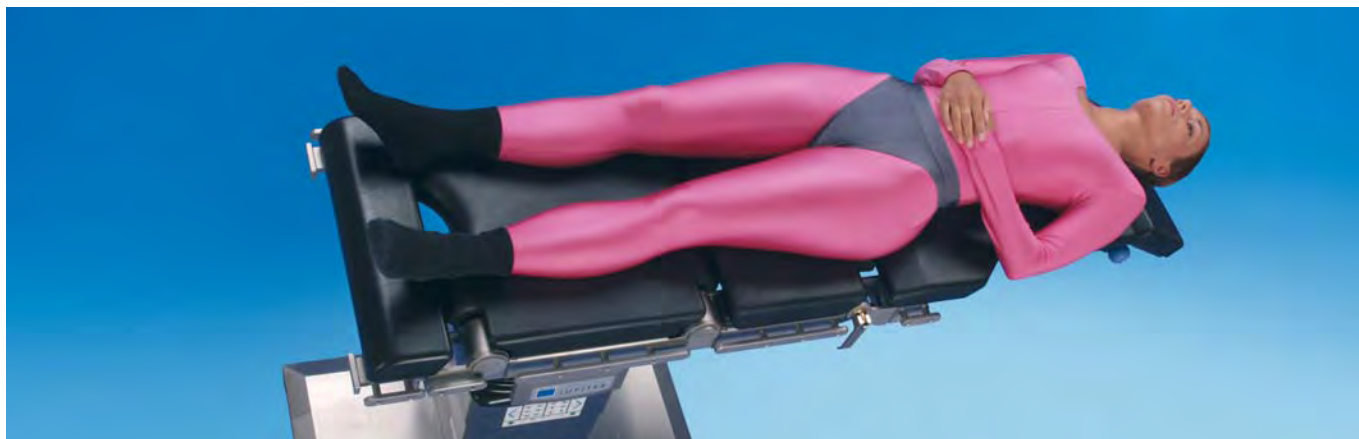


TABLE TOP SEGMENT CARBON NARROW

Part No. 1277923

Coupling to table top on foot and head side; to attach to TRUMPF coupling point system; CT compatible; Comfort Plus pad integrally foamed, electrically conductive, soft and removable; carbon fibre; coupling points made of stainless steel; (Fig. without pad)

Length x width: 600 mm x 530 mm / 370 mm

360° translucent: from coupling point up to the end of table top: 480 mm

ap-translucent: outside the coupling point 100 %

With carbon fibre coupling point to attach the adapters X-Ray



1 SIDE RAIL ADAPTER CARBON VARIO · Part No. 1266933

TABLE TOP SEGMENT CARBON NARROW with Medifoam pad: 1424570

TABLE TOP SEGMENT CARBON NARROW with FoamLine pad: 1424957

TABLE TOP SEGMENT CARBON 600

Part No. 1277922


Coupling to table top on foot and head side; to attach to TRUMPF coupling point system; CT compatible; Comfort Plus pad integrally foamed, electrically conductive, soft and removable; carbon fibre; coupling points made of stainless steel; (Fig. without pad)

Length x width: 600 mm x 530 mm

360° translucent: from coupling point up to the end of table top 480 mm

ap-translucent: outside the coupling point 100 %

With carbon fibre coupling point to attach the adapters X-Ray

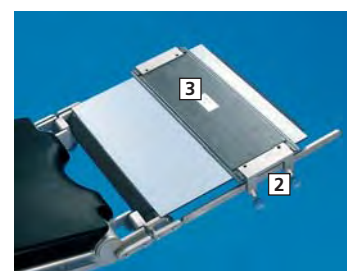

2 CLAMP PAIR FOR JOINT PLATE CARBON · Part No. 1272324

3 JOINT PLATE CARBON 520 · Part No. 1266934

TABLE TOP SEGMENT CARBON 600 with Medifoam pad: 1424569

TABLE TOP SEGMENT CARBON 600 with FoamLine pad: 1424956

Pelvis extensions



The pelvis extensions are used to position the patient's pelvis. By spreading the patient's feet outwards on the operating table the working area for the surgeon and the radiolucent area are enlarged.

PELVIS EXTENSION MOTORIZED

Part No. 1397285

To attach to operating tables and table tops with electrical bus system; with integrated rails for holding X-ray cassettes; with motor driven leg section joints; operated via remote control of the operating table; electrical cable connection to the operating table top; pad integrally foamed, electrically conductive, soft and detachable; stainless steel frame

Elongation: 360 mm

PELVIS EXTENSION MOTORIZED with Medifoam pad: 1421833

PELVIS EXTENSION MOTORIZED with FoamLine pad: 1421834



PELVIS EXTENSION JUPITER / MARS

Part No. 4549106

To attach to TRUMPF coupling point system; with manual adjustment of leg section joints; with integrated rails for holding X-ray cassettes; Comfort Plus pad integrally foamed, electrically conductive, soft and detachable; stainless steel frame

Elongation: 275 mm

PELVIS EXTENSION JUPITER / MARS with Medifoam pad: 1397906

PELVIS EXTENSION JUPITER / MARS with FoamLine pad: 1397907

PELVIS EXTENSION MARS ENDOURO: Part No. 4549382 (not shown)



PELVIS EXTENSION TITAN

Part No. 1268088

To attach to TRUMPF coupling point system; with integrated rails for holding X-ray cassettes; Comfort Plus pad integrally foamed, electrically conductive, soft and detachable; stainless steel frame

Elongation: 250 mm

PELVIS EXTENSION TITAN with Medifoam pad: 1424589

PELVIS EXTENSION TITAN with FoamLine pad: 1424590



PELVIS EXTENSION SATURN SELECT

Part No. 4549340

To attach to TRUMPF coupling point system; with integrated rails for holding X-ray cassettes; Comfort Plus pad integrally foamed, electrically conductive, soft and detachable; stainless steel frame

Elongation: 250 mm



PELVIS EXTENSION LONG


Part No. 1344246

To attach to TRUMPF coupling point system; with manual adjustment of leg section joints; with integrated rails for holding X-ray cassettes; Comfort Plus pad integrally foamed, electrically conductive, soft and detachable; stainless steel frame;

Elongation: 360 mm

PELVIS EXTENSION LONG with Medifoam pad: 1421069

PELVIS EXTENSION LONG with FoamLine pad: 1421075

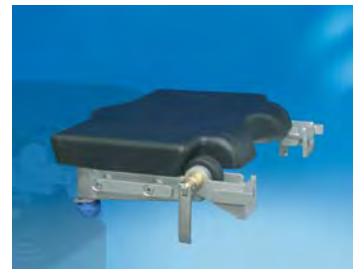

PELVIS EXTENSION LONG SATURN

Part No. 1421053

To attach to TRUMPF coupling point system; with manual adjustment of leg section joints; with integrated rails for holding X-ray cassettes; Comfort Plus pad integrally foamed, electrically conductive, soft and detachable; stainless steel frame

Elongation: 360 mm



TABLE TOP SEGMENT CARBON 450

Part No. 1277921



To attach to motorised TRUMPF coupling point system; for pelvis operation and low-artefact fluoroscopy; Comfort Plus pad integrally foamed, electrically conductive, soft and removable; carbon fibre; stainless steel coupling points
Length x width: 485 mm x 530 mm

TABLE TOP SEGMENT CARBON 450 with Medifoam pad: 1424571

TABLE TOP SEGMENT CARBON 450 with FoamLine pad: 1424958



COMPONENTS AND ACCESSORIES FOR TABLE TOPS

TABLE TOP SEGMENT CARBON THREE PARTS

Part No. 1227688



To attach the motorised TRUMPF standard coupling points of OR tables to the foot side; 360° radiolucent; width adjustment for general and visceral surgery as well as paediatrics; Comfort Plus pad integrally foamed, electrically conductive, soft and removable; coupling points stainless steel, bearing plate carbon fibre
Length: 1,220 mm

Width: 530 mm / 370 mm / 220 mm

360° translucent: 1050 mm

TABLE TOP SEGMENT CARBON THREE PARTS with Medifoam pad: 1424568

TABLE TOP SEGMENT CARBON THREE PARTS with FoamLine pad: 1424955

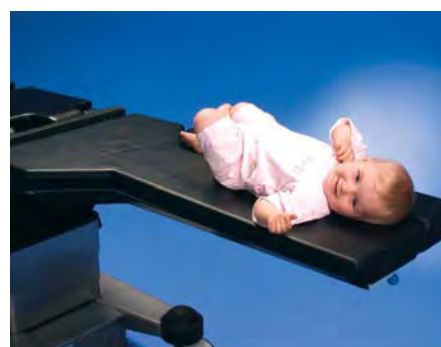


TABLE TOP SEGMENT CARBON ONE PART

Part No. 1446213


To attach to TRUMPF coupling point system; for patient positioning; Comfort Plus pad integrally foamed, electrically conductive, removable and soft; coupling points stainless steel, bearing plate carbon fibre

Length x width: 1.220 mm x 530 mm

360° translucent: 1.050 mm



Leg sections

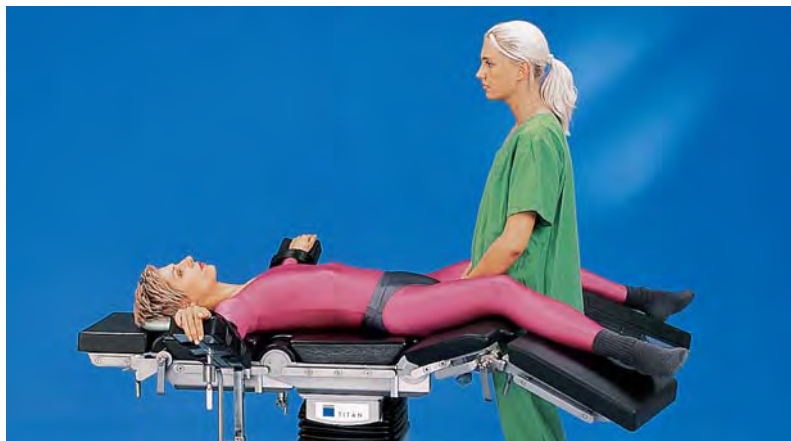

LEG SECTION FOUR PARTS

Part No. 4549188

With spread joint; to attach to TRUMPF coupling point system; Comfort Plus pad integrally foamed, electrically conductive and soft; stainless steel frame
Length: 750 mm

LEG SECTION FOUR PARTS with VS velcro fastener: 1227274

LEG SECTION FOUR PARTS with Medifoam pad: 1386804

LEG SECTION FOUR PARTS with FoamLine pad: 1397837



LEG SECTION TWO PARTS SPREAD JOINT

Part No. 4549189

To attach to TRUMPF coupling point system; Comfort Plus pad integrally foamed, electrically conductive and soft; stainless steel frame
Length: 740 mm

LEG SECTION TWO PARTS SPREAD JOINT with VS velcro fastener: 1227273

LEG SECTION TWO PARTS SPREAD JOINT with Medifoam pad: 1314344

LEG SECTION TWO PARTS SPREAD JOINT with FoamLine pad: 1397896



LEG SECTION TWO PARTS

Part No. 4549184

To attach to TRUMPF coupling point system; Comfort Plus pad integrally foamed, electrically conductive and soft; stainless steel frame
Length: 730 mm

LEG SECTION TWO PARTS with VS velcro fastener: 1227272

LEG SECTION TWO PARTS with Medifoam pad: 1397897

LEG SECTION TWO PARTS with FoamLine pad: 1397898



LEG SECTION TWO PARTS LONG

Part No. 4544153

To attach to TRUMPF coupling point system; for preparation of the patient before gynaecological and urological surgery; pad integrally foamed, electrically conductive, soft and detachable; stainless steel frame

Length: 1.040 mm



LEG SECTION TWO PARTS SHORT

Part No. 4549185

To attach to TRUMPF coupling point system; Comfort Plus pad integrally foamed, electrically conductive and soft; stainless steel frame

Length: 430 mm

LEG SECTION SHORT TWO PARTS with VS velcro fastener: 1227275

LEG SECTION TWO PARTS SHORT with Medifoam pad: 1388507

LEG SECTION TWO PARTS SHORT with FoamLine pad: 1397889



LEG SECTION SHORT ONE PART

Part No. 4549186

To attach to TRUMPF coupling point system; Comfort Plus pad integrally foamed, electrically conductive and soft; stainless steel frame

Length: 430 mm

LEG SECTION SHORT ONE PART with VS velcro fastener: 1227271



Leg sections for gynaecology and urology



LEGMASTER

Part No. 1412708 + 4544473 + 4544547

Four-parts leg section with combined lift- and spread mechanism for procedures in lithotomic position; lower leg pads are longitudinally adjustable and detachable; upper leg pads are detachable; pads integrally foamed, electrically conductive and soft; stainless steel frame

Length: 750 mm, extendable by 120 mm

Spread ankle: 45°

Leg up/down: +95°/-130°

PAD FOR LEGMASTER 4544547

LEG HOLDER GOEPEL FOR LEGMASTER 4544473



LEGHOLDER SET

Part No. 1232693

For easy positioning for gynaecological surgery in lithotomy position

Consists of:

- Legholder left 1228196
- Legholder right 1228197
- Clamp for legholder 1228198



LEG HOLDER GOEPEL

Part No. 4544736

Without clamp; pad integrally foamed, electrically conductive and soft; stainless steel frame

Recommended radial setting clamps: 2 x part no. 1218804



LEG HOLDER GOEPEL FOR CHILDREN

Part No. 4544789

Without clamp; pad integrally foamed, electrically conductive and soft; stainless steel frame

Recommended radial setting clamps: 2 x part no. 1218804



LEG HOLDER WITH STRAP PADDED, PAIR

Part No. 4544796

Without clamp; height adjustable and rotatable; strap polyester; stainless steel frame

Recommended radial setting clamps: 2 x part no. 1218804



LEG SECTION LIGHT LONG

Part No. 1419731

To attach to TRUMPF coupling point system; pad electrically conductive, soft and detachable; strap black polyester fabric with velcro fastener; stainless steel frame

Dimensions pad: 930 mm x 490 mm x 100 mm

Dimensions frame: 960 mm x 490 mm



LEG SECTION LIGHT WEIGHT

Part No. 4544522


Attach to side rails; tensioned cloth washable and electrically conductive; stainless steel frame

Dimensions: 880 mm x 600 mm

PAD FOR LEG SECTION LIGHT WEIGHT · Part no. 4544533 (not shown)

For height compensation of leg section light weight 4544522

Dimensions: 750 mm x 500 mm 50 mm

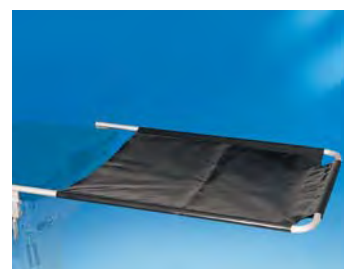

TRUMPF

Standard accessories




Clamps

CLAMP BASIC

Part No. 1305173

To attach to side rails; suitable for rods of 15 mm–20 mm diameter; stainless steel

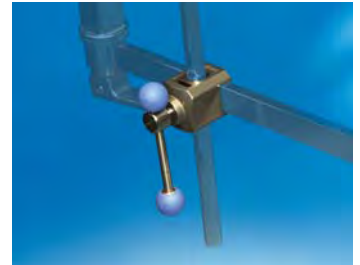

Y-CLAMP

Part No. 4544701


To attach to side rail by frontal placement; suitable for rods of 15 mm–18 mm diameter; stainless steel



Y-CLAMP RADIAL SETTING

Part No. 4544700

To attach to side rails by frontal placement; 360° adjustment; suitable for rods of 15 mm–18 mm diameter; stainless steel

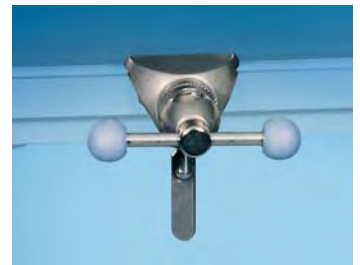

CLAMP RADIAL SETTING

Part No. 1218804

Version for heavy loads; to attach to side rails by frontal placement; 360° adjustment; suitable for rods of 15 mm–18 mm diameter; stainless steel



Anaesthesia screens

ANAESTHESIA SCREEN FLEXIBLE

Part No. 1218858

With clamps; Attachment by pushing on to side rails
 Maximum effective height under the screen: 420 mm
 Length of screen: 1,420 mm



ANAESTHESIA SCREEN EXTENDABLE

Part No. 4544710

Without clamp; L-shaped, with two lateral extendable rods; with electric insulation and safety-ring; height adjustable by using a clamp; stainless steel
 Diameter of vertical rod: 18 mm
 Height adjustment: 500 mm



ANAESTHESIA SCREEN WITH ELONGATION

Part No. 1294003

Without clamp; L-shaped; with electric insulation and safety-ring; height adjustable by using a clamp; with rotatable elongations; stainless steel
 Diameter of vertical rod: 18 mm
 Length of elongations: 300 mm
 Height adjustment: 500 mm



ANAESTHESIA SCREEN

Part No. 1218820

Without clamp; L-shaped; with electric insulation and safety-ring; height adjustable by using a clamp; stainless steel
 Diameter of vertical rod: 18 mm
 Height adjustment: 500 mm



HOLDER FOR ANAESTHESIA ACCESSORIES

Part No. 4544712

Attachable to anaesthesia screen; for fixing of supply hoses and measuring cables; stainless steel



SUSPENDED ARM STRAP

Part No. 4544703

Fix to anaesthesia screen with cross-bar diameter up to 22 mm; with velcro strap; washable and disinfectant resistant; strap black polyester fabric; holding plate stainless steel
Strap width: 100 mm




INFUSION POLE WITH 4 HOOKS

Part No. 4544708

With clamp; to attach to side rails by frontal placement; with ball joint; height adjustable; stainless steel
Height adjustment: 620 mm



Arm supports

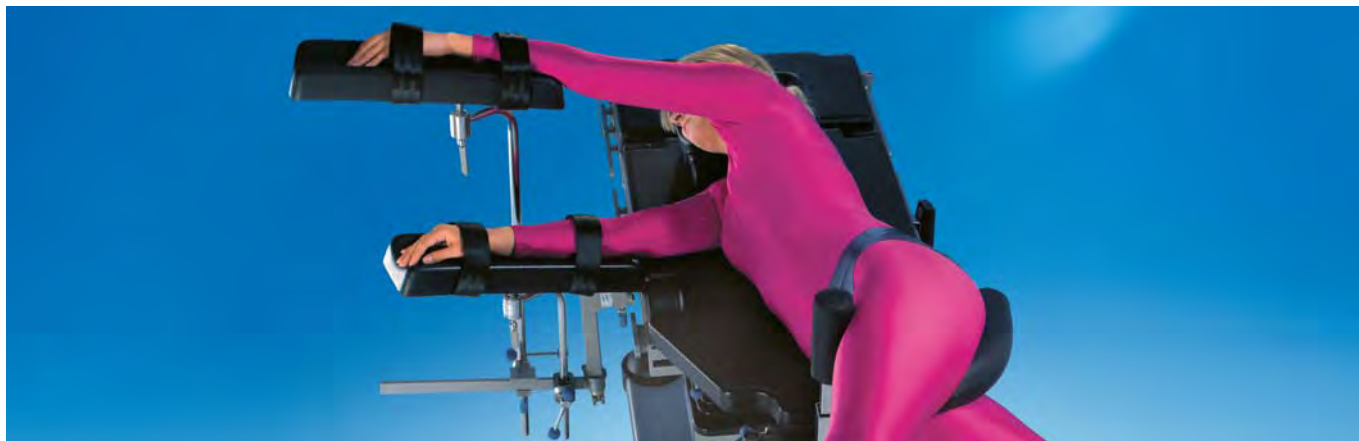

ARM HOLDER FOR LATERAL POSITION

ARM HOLDER 450 FOR LATERAL POSITION · Part No. 4549755

Without clamp; Comfort Plus pad integrally foamed, electrically conductive, soft and detachable; with velcro strap of black polyester fabric; stainless steel frame
Pad: 450 mm x 130 mm

Height adjustment: 650 mm

ARM HOLDER 300 FOR LATERAL POSITION · Part No. 4549756 (not shown)

Without clamp; Comfort Plus pad integrally foamed, electrically conductive, soft and detachable; with velcro strap of black polyester fabric; stainless steel frame;
Pad: 300 mm x 130 mm

Height adjustment: 650 mm

Recommended radial setting clamp: 1218804



ARM SUPPORT ONE HAND

Part No. 4549529

With clamp; to attach to existing arm supporting devices and holding device for arm and leg 1317577; swivel mounted with ball and socket joint and height adjustable; Comfort Plus pads integrally foamed, electrically conductive, soft and detachable; with velcro strap of black polyester fabric; stainless steel frame

Pad: 620 mm x 150 mm

Height adjustment: 250 mm

Pivoting range: $\pm 40^\circ$

1 HOLDING DEVICE FOR ARM AND LEG · Part No. 1317577



Arm supporting devices

With clamp; to attach to side rails by frontal placement; pad with a ball joint; swivel mounted, height and transversal adjustable; Comfort Plus pads integrally foamed, electrically conductive, soft and detachable, with velcro strap of black polyester fabric; stainless steel frame

ARM SUPPORTING DEVICE 600 · Part No. 1316895

Pad: 600 mm x 130 mm

ARM SUPPORTING DEVICE 450 · Part No. 1316894

Pad: 450 mm x 130 mm

ARM SUPPORTING DEVICE 300 · Part No. 1316893

Pad: 300 mm x 130 mm



ARM SUPPORT DEVICE ONE HAND OPERATION

Part No. 4549528

With clamp and guide rail; to attach to side rails by frontal placement; swivel mounted with ball and socket joint and height adjustable; Comfort Plus pads integrally foamed, electrically conductive, soft and detachable; with velcro strap of black polyester fabric; stainless steel frame

Pad: 620 mm x 150 mm

Height adjustment: 250 mm

Pivoting range: $\pm 40^\circ$



Arm supports, single joint

With clamp; to attach to side rails by frontal placement; swivel mounted and height adjustable; Comfort Plus pads integrally foamed, electrically conductive, soft and detachable; with velcro strap of black polyester fabric; stainless steel frame

ARM SUPPORT 600 · Part No. 4549538

Pad: 600 mm x 130 mm

ARM SUPPORT 450 · Part No. 4549778

Pad: 450 mm x 130 mm

ARM SUPPORT 300 · Part No. 4549779

Pad: 300 mm x 130 mm



Lateral support

Example of application for lateral support with pad



UNIVERSAL LATERAL SUPPORT

Part No. 4544720

With clamp; to attach to side rails by frontal placement; pad position individually adjustable; stainless steel
Support and pad freely rotatable



UNIVERSAL LATERAL SUPPORT HEIGHT ADJUSTABLE

Part No. 4544719

With clamp; to attach to side rails by frontal placement; pad position individually adjustable; stainless steel
Height adjustment: 150 mm
Longitudinal shift: 250 mm–390 mm
Support and pad freely rotatable



LATERAL SUPPORT HEIGHT ADJUSTABLE

Part No. 1302598

With clamp; to attach to side rails by frontal placement; pad position individually adjustable; stainless steel
Height adjustment: 60 mm
Pad freely rotatable



PELVIS SUPPORT

Part No. 4544773

With clamp; for lateral positioning; pads adjustable to width of pelvis;
pads integrally foamed, electrically conductive, soft and detachable;
stainless steel frame
Pad: 85 mm x 85 mm x 50 mm



Pad for lateral support

LATERAL SUPPORT 85

Part No. 4544723

For pubis, sakrum and sternum support; to attach to universal lateral support; pad integrally foamed, electrically conductive, soft and detachable; stainless steel frame
 Pad: 85 mm x 85 mm x 50 mm



LATERAL SUPPORT 170

Part No. 4544722

For back and buttocks support; to attach to universal lateral support; pad integrally foamed, electrically conductive, soft and detachable; stainless steel frame
 Pad: 170 mm x 120 mm x 50 mm



LATERAL SUPPORT 215

Part No. 4544721

To attach to universal lateral support; pad integrally foamed, electrically conductive, soft and detachable; stainless steel frame
 Pad: 215 mm x 100 mm x 50 mm



LATERAL SUPPORT 215 CONCAVE

Part No. 4544764

To attach to universal lateral support; pad intergrally foamed, electrically conductive, soft and detachable; stainless steel frame
 Pad: 215 mm x 100 mm x 50 mm



PAD SWIVEL MOUNTED

Part No. 4544192

To attach to universal lateral support height adjustable; pad integrally foamed, electrically conductive, soft and detachable; stainless steel frame
 Pad: 215 mm x 100 mm x 50 mm



SUPPORT ROLL FOR LATERAL SUPPORT**Part No. 4544762**

To attach to universal lateral support height adjustable; pad integrally foamed, electrically conductive and soft; stainless steel frame
 Pad: 80 mm x 240 mm

**Fixed lateral supports****LATERAL SUPPORT****Part No. 4544724**

With clamp; pad integrally foamed, electrically conductive, soft and detachable; stainless steel frame
 Pad: 215 mm x 100 mm x 50 mm

**LATERAL SUPPORT KONKAVE****Part No. 4544763**

With clamp; pad integrally foamed, electrically conductive, soft and detachable; stainless steel frame
 Pad: 215 mm x 100 mm x 50 mm

**LATERAL SUPPORT SHORT****Part No. 1436371**

With clamp; pad integrally foamed, electrically conductive, soft and detachable; stainless steel frame
 Pad: 150 mm x 30 mm x 130 mm



Patient fixation

ARM SAFETY STRAP

Part No. 4544704

To attach to side rails by frontal placement; continuously adjustable to width of arm via velcro fastener; disinfectant resistant and washable; strap black polyester fabric; stainless steel frame; strap width: 100 mm



ARM SAFETY STRAP WITH LOCKING CLAMP

Part No. 4544705

To attach to side rail; continuously adjustable to width of arm via velcro fastener; disinfectant resistant and washable; strap black polyester fabric; stainless steel clamp
Strap width: 100 mm



Fixation by strap with arm support pad

Fixation of the patient arm in combination with strap 1306719; pad electrically conductive and soft; to position the patient arm on the side of the body

ARM SUPPORT PAD 50 · Part No. 1306717

Length: 350 mm

Diameter: 100 mm / 50 mm

ARM SUPPORT PAD 100 · Part No. 1306718

Length: 450 mm

Diameter: 150 mm / 100 mm

STRAP FOR ARM SUPPORT PAD · Part No. 1306719

With clamp; body strap made of black polyester fabric; stainless steel clamp

Length: 650 mm

Width: 100 mm



Fixation with universal restraint

UNIVERSAL RESTRAINT CUFF · Part No. 1346755

With clamp; to attach to side rails by frontal placement; continuously adjustable to width of arm via velcro fastener; disinfectant resistant and washable; strap black polyester fabric; stainless steel frame
Strap width: 100 mm

PAD CUFF · Part No. 1358558

To position the patient arm close to the body in combination with universal restraint cuff 1346755; pad electrically conductive and soft
Length: 350 mm



BODY STRAP WITH LOCKING CLAMPS

Part No. 4544702

To attach to side rails; continuously adjustable to abdominal girth via velcro fastener; disinfectant resistant and washable; strap black polyester fabric; stainless steel locking clamp
Strap width: 100 mm
Max. length: 1.800 mm



BODY STRAP ROTATABLE

Part No. 4544706

To attach to side rail; continuously adjustable to abdominal girth via velcro fastener; disinfectant resistant and washable; strap black polyester fabric; stainless steel locking clamp
Strap width: 100 mm
Max. length: 1.800 mm



BODY STRAP WITH TIGHTENER

Part No. 4544164

To attach to side rails; continuously adjustable to abdominal girth via tightener; disinfectant resistant and washable; strap black polyester fabric; stainless steel locking clamp
Strap width: 100 mm
Max. length: 1.800 mm



LEG RESTRAINT SINGLE

Part No. 1218828

Fixation strap padded leather



LEG RESTRAINT TWO PARTS

Part No. 1218829

Fixation strap padded leather



TUNNEL PILLOW

Part No. 1238766

For side location of the patient; protection of legs against violations;
dimension: 650 mm x 400 mm x 220 mm

Pad thickness: 70 mm



Side guard, foldable

1 SIDE GUARD, PAIR · Part No. 4544434

With clamp; foldable and removable; stainless steel

Length x height: 700 mm x 220 mm

2 PAD FOR SIDE GUARD · Part No. 1376418

To pull over the side guard 4544434; for a better protection of the patient's arm



SHOULDER SUPPORT, PAIR

Part No. 4544726

With clamp; to attach to side rails by frontal placement; adjustable along and across the operating table top; pad integrally foamed, electrically conductive, soft and detachable; stainless steel frame

Longitudinal shift: 130 mm

Transversal shift: 90 mm



FOOT SUPPORT

Part No. 4544421

With clamp; continuously adjustable to width of table top; pad integrally foamed, electrically conductive and soft; stainless steel frame
 Pad: 470 mm x 220 mm



FOOT SUPPORT TWO PARTS

Part No. 1265545

To attach to side rails; with swivel joint allowing correct patient positioning; Comfort Plus pad integrally foamed, electrically conductive, soft and removable; stainless steel frame



MENISCUS POSITIONING DEVICE

Part No. 4544791

Without clamp; height adjustable through fixation of vertical rod at clamp; radiolucent; pad integrally foamed, electrically conductive, soft and removable; stainless steel frame

Recommended radial setting clamps: 1218804



Arm support tables



Hand operating table Carbon

- 1 HAND OPERATING TABLE CARBON · Part No. 1227687
To support the arm; Comfort Plus pad electrically conductive, integrally foamed, soft and removable; made of carbon fibre
360° radiolucent; can also be used without support rod
Dimensions: 840 mm x 370 mm / height adjustment: 450 mm
- 2 SIDE RAIL EXTENSION H · Part No. 1232675
- 3 SUPPORT ROD H · Part No. 1232679



ARM SUPPORT TABLE

Part No. 4544772

With clamp; pad integrally foamed, electrically conductive and soft; stainless steel frame
Dimensions: 650 mm x 260 mm



HAND OPERATING TABLE

Part No. 4549735

With clamp; table top and supports on castors height adjustable; Comfort Plus pad electrically conductive and soft; stainless steel frame; pad: 840 mm x 370 mm
Height adjustment: table top: 70 mm / support on castors: 400 mm / pivoting range: $\pm 45^\circ$

EXTENSION FOR HAND OPERATING TABLE · Part No. 4544769 (not shown)
With clamp; to attach to the side rails of hand operating table 4549735; to be used as instrument tray
Length: 300 mm



HOLDER FOR HAND OPERATING TABLE · Part No. 1303645 (not shown)

HAND SUPPORT FOR SURGEON, PAIR

Part No. 4544757

With clamp; height adjustable; pads integrally foamed, electrically conductive and soft; stainless steel frame
Dimensions: 170 mm x 120 mm
Height adjustment: 60 mm



Side rails / Side rail extensions

SIDE RAIL WITH CLAMP

Part No. 4544792

To attach to TRUMPF coupling point system; stainless steel



SIDE RAIL ADAPTER

Part No. 4544165

Stainless steel



Side rails

SIDE RAIL · Part No. 4544166

Stainless steel

Length: 520 mm

SIDE RAIL 90° GEWINKELT · Part No. 4544167

Stainless steel

SIDE RAIL 35° GEWINKELT · Part No. 4544168

Stainless steel



Side rail extensions

SIDE RAIL EXTENSION · Part No. 1218646

With clamp; stainless steel

Length of side rail: 520 mm

SIDE RAIL EXTENSION 160 · Part No. 1309436 (not shown)

With clamp; stainless steel

Length of side rail: 160 mm

SIDE RAIL EXTENSION ANGLED 90° · Part No. 1218872

With clamp; stainless steel



Side rails for carbon fibre segments

- 1 SIDE RAIL ADAPTER CARBON VARIO · Part No. 1266933
Width adjustment from 220 mm to 520 mm for all carbon fibre segments; attachment at the end; without side rails 4544166 or 4544167 (p. 45)

- 2 CLAMP PAIR FOR JOINT PLATE CARBON · Part No. 1272324
To retain joint plates Carbon and side rails; this combination can be attached about the whole length of the segments and table tops without any impairment of the X-raying's ability; clamp made of stainless steel

- 3 JOINT PLATE CARBON 520 · Part No. 1266934
Made of carbon fibre/width: 520 mm



- JOINT PLATE CARBON 370 · Part No. 1279867
Width: 370 mm

- JOINT PLATE CARBON 220 · Part No. 1229593
Width: 220 mm

- BAG FOR JOINT PLATES CARBON · Part No. 1324258

- SIDE RAIL ADAPTER CARBON · Part No. 1249777
With two short lateral side rails; for operating table tops Carbon X-tra; to attach side rail 4544536

- SIDE RAIL LONG · Part No. 4544536
Fixed between side rail of operating table tops carbon fibre and side rail adapter Carbon 1249777; stainless steel

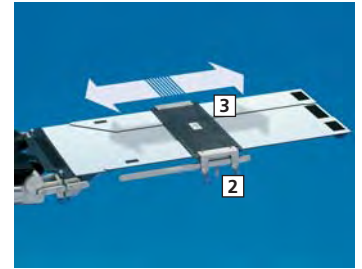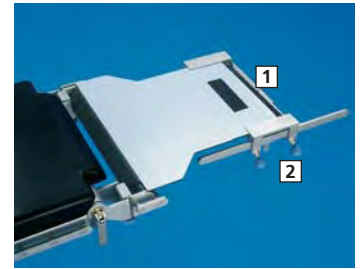


HANDLES FOR OPERATING TABLES

Part No. 1321641

- 1 To attach to side rails; for an easier moving of operating tables or an operating table top at the shuttle; stainless steel

- 2 CABLE REMOTE CONTROL TERMINAL · Part No. 1446212
To attach to lateral side rails or handles for operating tables 1321641; to retain remote controls; stainless steel

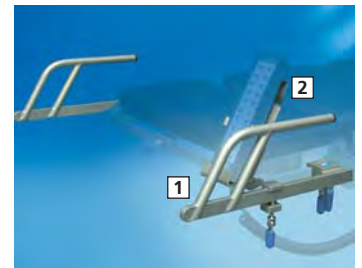

HOLDER REMOTE CONTROL

Part No. 1351241

To attach to side rails; for mounting of remote controls; stainless steel



Urology



DRAINAGE BOWL ADAPTER


Part No. 4544619

With clamp; to attach to side rail; precisely adjustable in x- and y-direction through guide rail; stainless steel

DRAINAGE BOWL ADAPTER ELONGAT. SEAT SECT · Part No. 1296302

To attach to pelvis extensions


To attach to pelvis extension TITAN 1256030: part no. 1300601

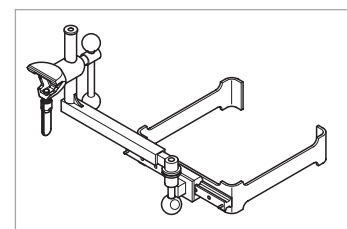

DRAINAGE BOWL ADAPTER SWIVEL-MOUNTED

Part No. 4544128

With clamp; to attach to side rails; stainless steel

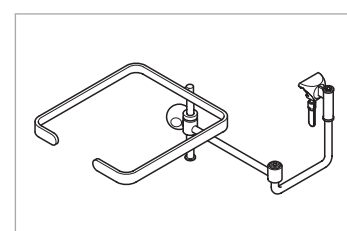

DRAINAGE BOWL STAND

Part No. 4544492

For mounting of drainage bowls; mobile; stainless steel



DRAINAGE BOWL WITH WATER INLET

Part No. 4544495

With sieve; with hose connectors for drainage and supply; protection of overflow; stainless steel



DRAINAGE BOWL

Part No. 4544618

With sieve; with hose connectors for drainage; protection of overflow; stainless steel



COLLECTION BOWL

Part No. 1218883

Suitable for all drainage bowl adapters; stainless steel



INSTRUMENT TRAY

Part No. 1286562

Suitable for all drainage bowl adapters; stainless steel
Dimensions: 353 mm x 323 mm x 32 mm



Spine positioning

Rectal and genupectoral positioning



Spinal cord positioning device

1 SPINAL CORD POSITIONING DEVICE · Part No. 1361398

To attach to TRUMPF coupling point system at foot side; it is used for positioning the patient's knees during operations on the spinal column; with crank for height adjustment; up and down movements by the motoric driven leg section joints; radiolucent via C-arm; pads electrically conductive, soft and detachable; pad fastening via velcro; bearing structure stainless steel; height adjustment: 300 mm

SPINAL CORD POSITIONING DEVICE SATURN · Part No. 1440415 (not shown)

To attach to TRUMPF coupling point at foot side of OR tables SATURN Select and OR table tops of the SATURN System

2 TROLLEY SPINAL CORD POSITIONING DEVICE BASIC · Part No. 1432336

To position and transport the spinal cord positioning device



ILLIAC ROLL WITH LATERAL SUPPORT

Part No. 4544761

Without clamp; height adjustable; adjustable to the width of table top; lateral support adjustable to the width of the body; pads integrally foamed, electrically conductive and soft; stainless steel frame; height adjustment: 400 mm

Dimensions of the pad cylinder: width 420 mm x Ø 80 mm

Radial setting clamps (2) · Part no. 1218804



UNIVERSAL SUPPORT ROLL

Part No. 1429987

Without clamp; height adjustable; adjustable to the width of table top; pad integrally foamed, electrically conductive and soft; stainless steel frame; height adjustment: 80 mm–310 mm (over pad)



KNEE SUPPORT RIGHT / LEFT

Part No. 1395188 (right) / 1395189 (left)

To attach to TRUMPF leg sections; swivel mounted; for rectal positioning; pad integrally foamed, electrically conductive and soft; stainless steel frame



SPINE BRIDGE WILSON

Part No. 1337640



Spine bridge Wilson is used to position the patient for spinal surgery, individually adjustable to the width of the patient; flexion range manually adjustable with hand crank; completely radiolucent
Dimensions: 730 mm x 500 mm x 240 mm–310 mm

Aluminium transport and storage box: 1337641



Pad for operation at intervertebral disc

The radiolucent and electrically conductive pads with abdominal cutoff are used for anatomical positioning of the patient for operations at the intervertebral disc

PAD FOR OPERATION AT INTERVERTEBRAL DISC SMALL · Part No. 1409493

Dimensions: 600 mm x 400 mm x 100 mm

PAD FOR OPERATION AT INTERVERTEBRAL DISC · Part No. 1409494

Dimensions: 840 mm x 460 mm x 100 mm



Radiology

X-RAY CASSETTE HOLDER COMFORT

Part No. 1305492

With clamp, for horizontal radiograms; rotatable with rotation joint;
with lateral adjustment; stainless steel
Lateral adjustment: 330 mm–570 mm
Height adjustment: 70 mm
Rotation angle: 360°
Maximum weight of X-ray cassettes: 3 kg



Positioner for X-ray cassettes

POSITIONER FOR X-RAY CASSETTES · Part No. 1256558

For X-ray cassettes with maximum dimension of 340 mm x 385 mm;
stainless steel

POSITIONER FOR X-RAY CASSETTES NARROW · Part No. 4544140

For X-ray cassettes with maximum dimension of 340 mm x 385 mm;
stainless steel

All tables can be equipped with positioners for X-ray cassettes



Storage systems

Stainless steel trolley

- 1 UNILATERAL HOOK TROLLEY · Part No. 1218503
 One side with 3 pairs of hooks for accessory:
 Accessory rail for operating accessory or stainless steel baskets
 Dimensions: 555 mm x 625 mm x 1.450 mm
- BILATERAL HOOK TROLLEY · Part No. 1218493 (not shown)
 3 pairs of hooks on both sides for accessory:
 Accessory rail for operating accessory or stainless steel baskets
 Dimensions: 555 mm x 625 mm x 1.450 mm
- 2 STERILIZATION WIRE MESH BASKET 1/2 · Part No. 1215712
 Dimensions: 575 mm x 280 mm x 130 mm; please order by the piece
- STERILIZATION WIRE MESH BASKET 1/1 · Part No. 1215760 (not shown)
 Dimensions: 575 mm x 280 mm x 260 mm; please order by the piece
- 3 ACCESSORY RAIL FOR OR TABLE ACCESSORIES · Part No. 4544725



TROLLEY FOR ACCESSORIES

Part No. 4545632

For storing accessories; with 6 shelves, 2 rails and 2 sterilisation wire mesh baskets; stainless steel
 Maximum carrying capacity per shelf 25 kg
 Dimensions: 1.350 mm x 670 mm x 1.620 mm
 Without accessory



Instrument tables and kick buckets

INSTRUMENT TABLE

Part No. 1218867

Without clamp; with ball joint; stainless steel
Dimension of tray: 530 mm x 325 mm



Oil pump instrument tables

Maximum weight to carry: 30 kg

OIL PUMP INSTRUMENT TABLE 640 · Part No. 4544827

Dimensions: 640 mm x 440 mm x 950 mm

height adjustment: 400 mm

OIL PUMP INSTRUMENT TABLE 740 · Part No. 4544828

Dimensions: 740 mm x 540 mm x 950 mm

Height adjustment: 400 mm



OIL PUMP INSTRUMENT TABLE 1200

Part No. 4544829

Maximum weight to carry: 30 kg

Dimensions: 1.200 mm x 600 mm x 950 mm

Height adjustment: 400 mm



Kick buckets

KICK BUCKET WITHOUT PUSHING HANDLE · Part No. 1218658

With bucket in stainless steel

Dimensions: 470 mm x 470 mm x 380 mm

KICK BUCKET WITH PUSHING HANDLE · Part No. 1218662

With bucket in stainless steel

Dimensions: 470 mm x 470 mm x 800 mm



Mounting steps and swivel chairs

Surgeon's mounting steps

SURGEON'S DOUBLE MOUNTING STEP 220/400 · Part No. 1218784

Footboard removeable

Dimensions: 600 mm x 650 mm x 220 mm / 400 mm

SURGEON'S MOUNTING STEP 300 · Part No. 1218783 (not shown)

Footboard removeable

Dimensions: 600 mm x 350 mm x 300 mm



MOBILE SWIVEL CHAIR WITH ROUND SEAT

Part No. 1218912

Stainless steel

Height adjustment: 600 mm–700 mm

Dimensions: 480 mm x 600 mm



SWIVEL STOOL

Part No. 1218628

With 5-star pedestal and castors; stainless steel

Height adjustment: 500 mm–670 mm

Dimensions: 460 mm x 500 mm



TRUMPF

Accessories for orthopaedics



Knee positioning



ARTHROSCOPY POSITIONING DEVICE

Part No. 4544664

With clamps, for easy and exact positioning and tight fixation of the leg;
with foam-padded tightening strap



ARTHROSCOPY POSITIONING DEVICE BASIC

Part No. 4544534

Without clamp; height adjustable through fixation of vertical rod at clamp;
adjustable to the width of the leg; pad integrally foamed, electrically conductive
and soft; stainless steel frame

Height adjustment: 460 mm

Width adjustment: 110 mm–180 mm

Pad roll: width 150 mm x Ø 80 mm

Recommended radial setting clamp: Part No. 1218804



CONDYLEN FIXATION DEVICE

Part No. 1434382

For mounting to counter traction post for tibia and fibula 4544770; for exact
positioning of the knee; pad integrally foamed, electrically conductive, soft
and removable; stainless steel frame

Height adjustment: 125 mm



MENISCUS POSITIONING DEVICE

Part No. 4544791

Without clamp; height adjustable through fixation of vertical rod at clamp;
radiolucent; pad integrally foamed, electrically conductive, soft and removable;
stainless steel frame

Recommended radial setting clamp: 1218804



Arm positioning



Upper arm support

- 1 HOLDING DEVICE FOR UPPER ARM SUPPORT · Part No. 1302539
To attach to side rails; in combination with upper arm pads or pad rolls for the positioning and support during surgical interventions at the upper arm, elbow and forearm; lateral shift at side rail; stainless steel frame
Height adjustment: 210 mm
- 2 UPPER ARM PAD · Part No. 1297244
Length x width: 230 mm x 210 mm
- 3 UPPER ARM PAD SHORT · Part No. 1322666
Length x width: 215 mm x 100 mm
- 4 PAD ROLL 80 FOR UPPER ARM SUPPORT · Part No. 1297245
Length 250 mm x Ø 80 mm

PAD ROLL 60 FOR UPPER ARM SUPPORT · Part No. 1297247 (not shown)
Length 250 mm x Ø 60 mm

All pads are radiolucent



Upper arm extension


- 1 TRACTION DEVICE FOR HUMERUS · Part No. 4544766
Without clamp; continuously adjustable to length of arm; stainless steel
Recommended radial setting clamp: 2x Part No. 1218804
- 2 COUNTER TRACTION FOR HUMERUS · Part No. 4544765
Without clamp; height adjustable to length of humerus through clamp;
padded and radiolucent; stainless steel frame
- 3 HAND TRACTION DEVICE WEINBERGER · Part No. 1218836
With clamp; to attach to traction device for humerus 4544766,
arm extension unit 4544542 and spindle traction mechanism 1218832



Forearm extension

- 4 AXILLA SUPPORT · Part No. 1218848
With clamp; pad integrally foamed, electrically conductive and soft; stainless
steel frame
- 5 ELBOW SUPPORT · Part No. 1218847
With clamp; swivel-mounted and adjustable to the length of the arm;
pad integrally foamed and electrically conductive; stainless steel frame
- 3 WEINBERGER HAND TRACTION DEVICE · Part No. 1218836
With clamp; to attach to traction device for humerus 4544766, arm extension
unit 4544542 and spindle traction mechanism 1218832

Additionally, extension parts (OR adapter, dual joint bar, spindle traction mechanism) are required.

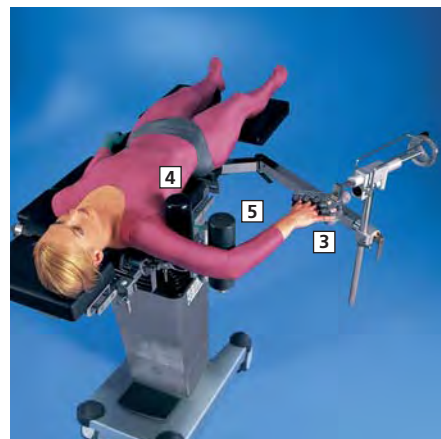

Shoulder operation

Shoulder adapter



The shoulder adapter enables free access to the right or left shoulder by adjustable and detachable pads; coupling to back section of MARS, JUPITER and TITAN.

- 1 SHOULDER POSITIONER · Part No. 4544103
The side pads (right and left) 215 mm x 235 mm are detachable and height adjustable 200 mm
Shoulder positioner with pad Medifoam: 1397910
- 2 RETAINER FOR HEAD RESTS · Part No. 1317948
Length adjustment 390 mm
Height adjustment 80 mm, length 275 mm
- 3 CONNECTION HOLDER BASIC · Part No. 1254739
CONNECTION HOLDER BASIC DISPLACED · Part No. 1305449
- 4 HEAD REST HORSESHOE-SHAPED TWO PARTS · Part No. 4549732



SHOULDER SEGMENT

Part No. 1266242

To attach to TRUMPF coupling point system; with 5 padded segments and a separate cervical vertebra support; lateral pads height adjustable and removable; Comfort Plus pad integrally foamed, electrically conductive and soft; stainless steel frame

SHOULDER SEGMENT with Medifoam pad: 1389817

SHOULDER SEGMENT SATURN · Part No. 1266243

To attach to coupling points for OR adapter; with 5 padded segments and a separate cervical vertebra support; lateral pads height adjustable and removable; Comfort Plus pad integrally foamed, electrically conductive and soft; stainless steel frame



ADDITIONAL PAD, PAIR


Part No. 4544799

- 1 For shoulder adapter, pads integrally foamed, electrically conductive, soft and detachable, stainless steel frame
Dimensions: 250 mm x 200 mm x 50 mm



Shoulder support

- 1 SHOULDER PADS, PAIR · Part No. 4544126
With clamp; pad integrally foamed, electrically conductive and soft; stainless steel frame
Dimensions: 250 mm x 200 mm x 50 mm
- 2 SHOULDER PAD NARROW · Part No. 4544191
Pad integrally foamed, electrically conductive, soft and removable; stainless steel frame
Dimensions: 215 mm x 100 mm x 50 mm

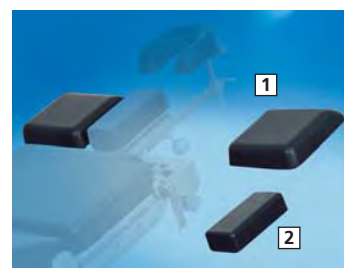

HEAD REST FOR SHOULDER OPERATION

Part No. 1296096

Translucent u-shaped helmet with soft pads and 2 fixation straps with velcro fasteners for safe positioning and lateral fixation of the head while the patient is anaesthetised; for the different head sizes from adults to children; pads removable




Extension

Lower leg extension



Upper leg extension in supine position

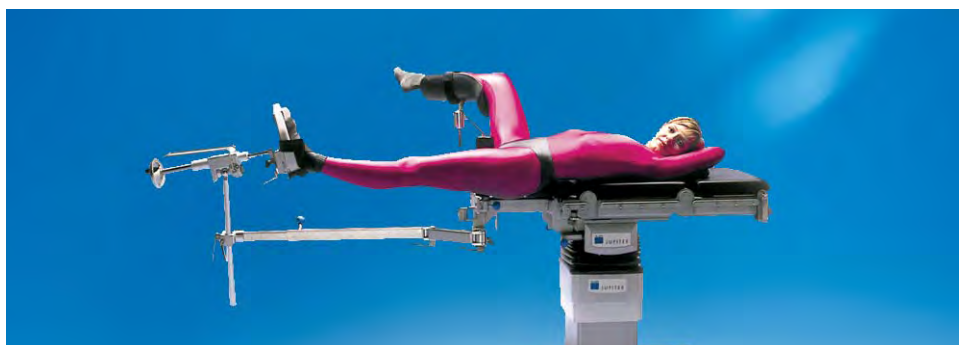


Upper leg extension in lateral position




COUNTER TRACTION POST FOR FEMUR

Part No. 4544448

For interventions in lateral position; counter traction post and perineal bow PUR integrally foamed and soft; height adjustable via cranc; stainless steel
Height adjustment: 125 mm

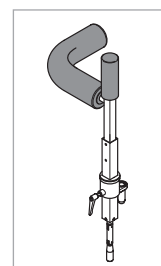

Orthopaedic extension, overview

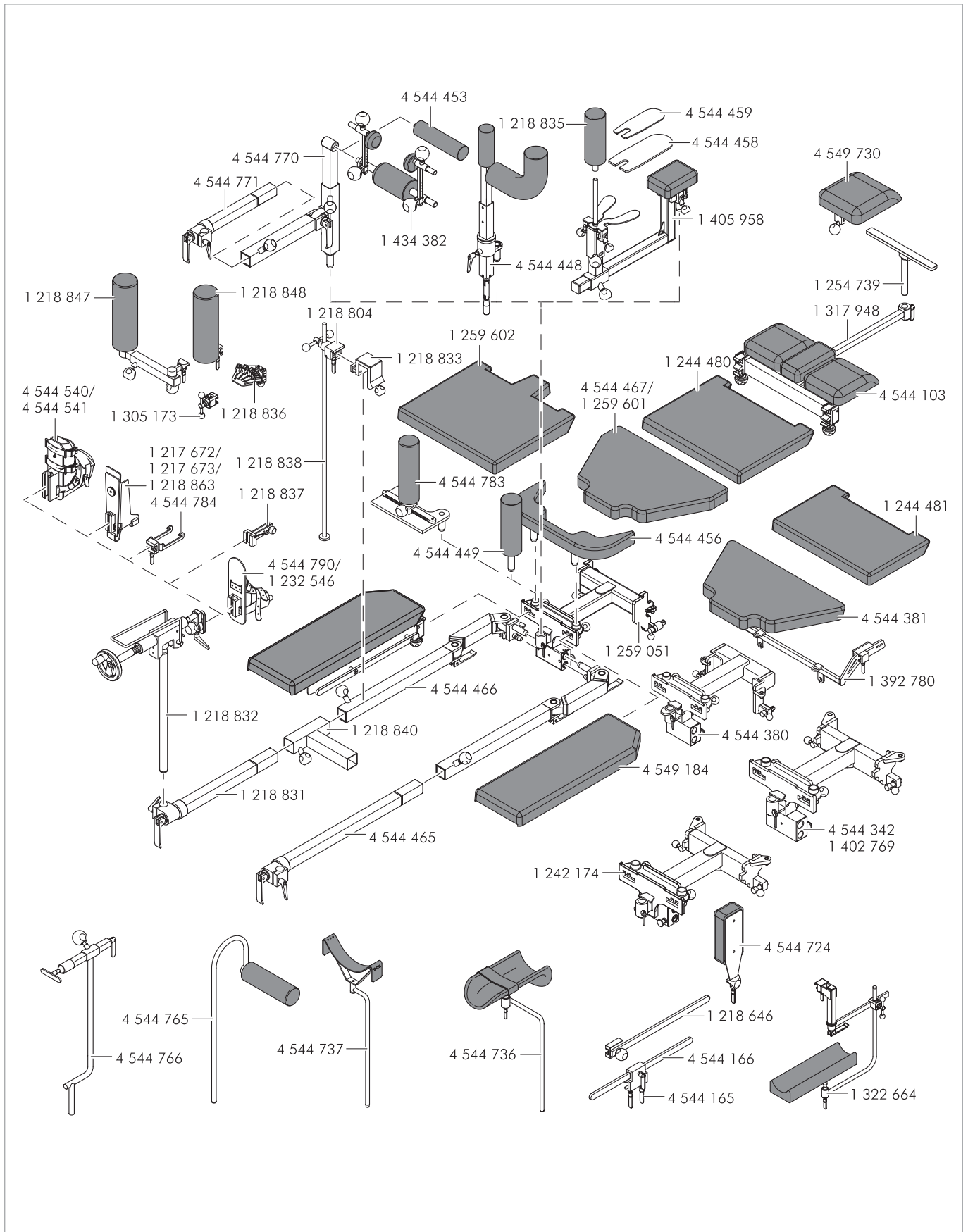

Adapter for operating table

1259051 OR-ADAPTER JUPITER/TITAN
 4544467 SUPPORT PAD JUPITER
 1259601 SUPPORT PAD TITAN
 1392780 OR HOLDER MARS
 4544380 OR-ADAPTER MARS
 4544381 SUPPORT PAD MARS / SATURN
 4544342 OR-ADAPTER SATURN SELECT
 1402769 OR-ADAPTER SATURN SELECT / MARS II
 1242174 OR-ADAPTER SATURN SYSTEM

Basic configuration

4544466 DOUBLE JOINT STRUT, PAIR
 4544783 COUNTER TRACTION POST ADJUSTABLE
 4544449 COUNTER TRACTION POST 80
 1218831 EXTENSION STRUT
 4544465 EXTENSION STRUT LONG
 1218832 SPINDLE TRACTION MECHANISM
 4544790 LEATHER EXTENSION SANDAL LEFT
 1232546 LEATHER EXTENSION SANDAL RIGHT

Accessories femur in supine position with one extension strut

4544165 SIDE RAIL ADAPTER
 4544166 SIDE RAIL
 4544736 LEG HOLDER GOEPEL
 1218804 CLAMP RADIAL SETTING

Accessories femur lateral position with two extension struts

4544448 COUNTER TRACTION POST FOR FEMUR
 4544456 ADDITIONAL PAD

Accessories femur lateral position with one extension strut

4544448 COUNTER TRACTION POST FOR FEMUR
 4544456 ADDITIONAL PAD
 1218840 T-ADAPTER
 1218833 RAIL CLAMP FOR EXTENSION STRUT
 1218837 ADAPTER FOR KIRSCHNER BOW
 4544784 ADAPTER FOR KIRSCHNER BOW BASIC
 1322664 LEG SUPPORTING DEVICE
 1218804 CLAMP RADIAL SETTING

Accessories for tibia and fibula nailing

4544770 COUNTER TRACTION POST FOR TIBIA
 4544771 EXTENSION STRUT SHORT
 4544453 COUNTER TRACTION POST DIAMETER 60
 4544165 SIDE RAIL ADAPTER
 4544166 SIDE RAIL
 4544736 LEG HOLDER GOEPEL
 1218804 CLAMP RADIAL SETTING

Accessories for pelvis plaster

1405958 PLASTER DEVICE CONSISTS OF:

- 1386130 PELVIS SUPPORT
- 1386131 PAD FOR SPINE COLUMN
- 1386129 ADAPTER FOR PLASTER BOARD
- 1385777 PLASTER BOARD MEDIUM, PAIR
- 1386629 COUNTER TRACTION POST PLASTER

4544459 PLASTER BOARD SMALL

4544458 PLASTER BOARD LARGE

NOT SHOWN:

1385776 PLASTER BOARD LARGE, PAIR

1386128 PLASTER BOARD SMALL, PAIR

1405077 COUNTER TRACTION POST PLASTER SHORT

Accessories for arm extension

4544724 LATERAL SUPPORT

1218847 ELBOW SUPPORT

1218848 AXILLA SUPPORT

1218836 WEINBERGER HAND TRACTION DEVICE

4544765 COUNTER TRACTION FOR HUMERUS

4544766 TRACTION DEVICE FOR HUMERUS

1218804 CLAMP RADIAL SETTING

General accessories for base support

1218833 RAIL CLAMP FOR EXTENSION STRUT

1218804 CLAMP RADIAL SETTING

1218838 BASE SUPPORT

Additional accessories

1305173 CLAMP BASIC

1218646 SIDE RAIL EXTENSION

4549184 LEG SECTION TWO PARTS

1244480 SUPPORTING PAD HIP J

1244481 SUPPORTING PAD HIP MS

1259602 SUPPORTING PAD HIP TITAN

4544103 SHOULDER POSITIONER

1317948 RETAINER FOR HEAD RESTS

1254739 CONNECTION HOLDER BASIC

4549730 HEAD REST BASIC

1218863 PLASTIC EXTENSION SHOES FOR KIDS, PAIR

1217672 PLASTIC EXTENSION SHOES LEFT

1217673 PLASTIC EXTENSION SHOES RIGHT

4544540 LEATHER EXTENSION SHOE RIGHT

4544541 LEATHER EXTENSION SHOE LEFT

4544737 LEG SUPPORT

1434382 CONDYLEN FIXATION DEVICE

Hip surgery

Hip surgery with 360° radiolucent carbon fibre section

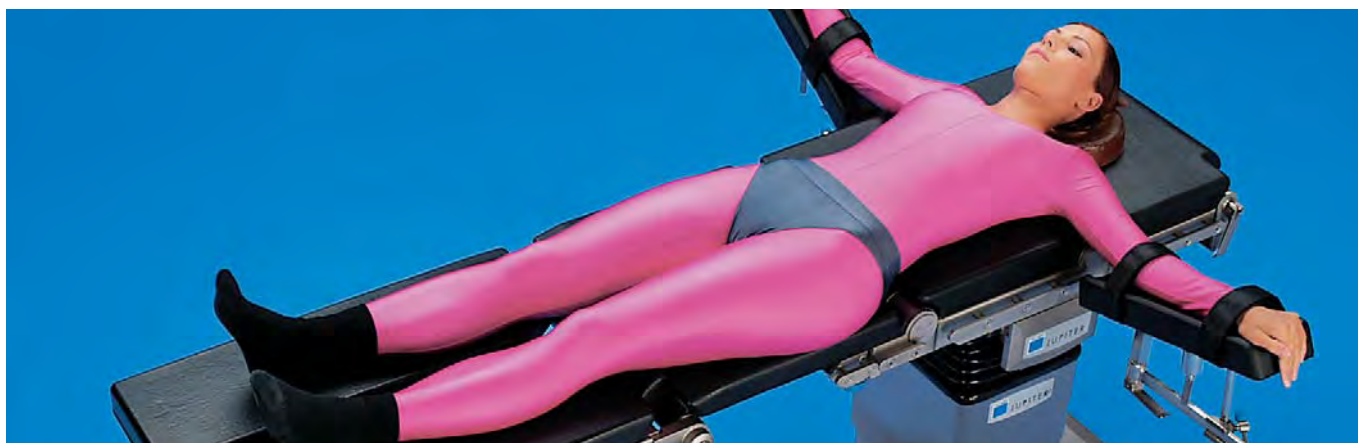


TABLE TOP SEGMENT CARBON 450

Part No. 1277921

To attach to TRUMPF coupling point system; for pelvis operation and low-artefact fluoroscopy; Comfort Plus pad integrally foamed, electrically conductive, soft and removable; carbon fibre; stainless steel coupling points
 Length x width: 485 mm x 530 mm


TABLE TOP SEGMENT CARBON 450 with Medifoam pad: 1424571

TABLE TOP SEGMENT CARBON 450 with FoamLine pad: 1424958





Thorax

Thorax plaster cast

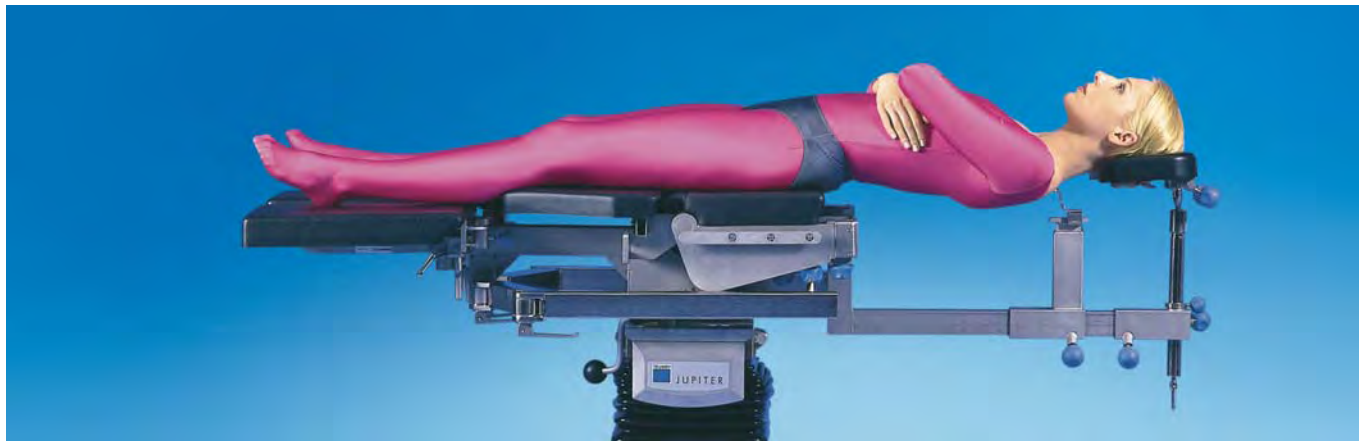

- 1 ADAPTER FOR ORTHOPAEDICS BENCH · Part No. 4544156
To attach to TRUMPF coupling point system; for mounting of accessories for pelvis and thorax plaster; stainless steel frame
- 2 ORTHOPAEDICS BENCH · Part No. 4544158
To attach to adapter for orthopaedics bench 4544156; stainless steel
- 3 ADAPTER FOR CONNECTION HOLDER · Part No. 4544157
To attach to orthopaedics bench 4544158; for mounting of connection holder with joint; stainless steel
- 4 CONNECTION HOLDER WITH JOINT · Part No. 4544729
To attach to central holder for head rest system 4544786, retainer for head rests Vario 1300079 and cross joint 4544787; stainless steel
Length of side rail: 185 mm
Adjustment ball joint: rotatable: 360° / tilt $\pm 30^\circ$
- 5 HEAD REST HORSESHOE-SHAPED TWO PARTS · Part No. 4549732
- 6 ADAPTER FOR THORAX PLASTER SUPPORT · Part No. 4544161
To attach to orthopaedics bench 4544158 and pelvis support 1386130; stainless steel
- 7 THORAX PLASTER SUPPORT · Part No. 4544163
To attach to adapter for thorax plaster support 4544161; stainless steel

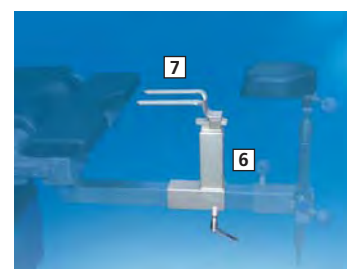
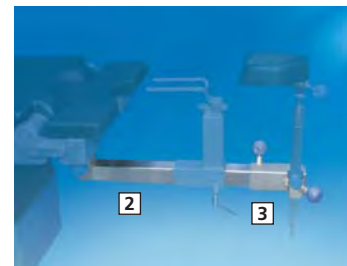


Pelvis

Putting a plaster cast on the pelvis



- 1 PELVIS SUPPORT · Part No. 1386130
To attach to the OR adapters of TRUMPF; it is used for the positioning of patients during plaster procedure; stainless steel
- 2 PAD FOR SPINE COLUMN · Part No. 1386131
To attach to pelvis support 1386130; for the additional support of the patient; pad Comfort Plus integrally foamed, electrically conductive, soft and detachable; stainless steel frame
Pad: 170 mm x 120 mm x 50 mm
- 3 COUNTER TRACTION POST · Part No. 1218835
To attach to adapter for plaster board 1386129, adapter for CALYPSO 1404892 and adapter OR; pad integrally foamed, electrically conductive, soft and detachable; with radiolucent rod
Diameter: 80 mm/length: 240 mm
- 4 COUNTER TRACTION POST PLASTER · Part No. 1386629
To attach to adapter for plaster board 1386129 and adapter for CALYPSO GT 1404892; stainless steel
Diameter: 16 mm/length: 200 mm
- 5 ADAPTER FOR PLASTER BOARD · Part No. 1386129
To attach to pelvis support 1386130; with side rails at both sides; stainless steel
Height adjustment:
Over pad level: 15 mm
Below pad level: -90 mm

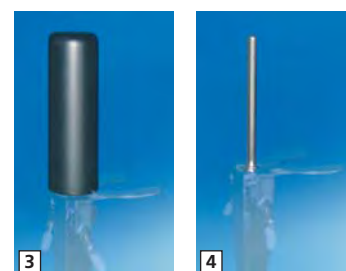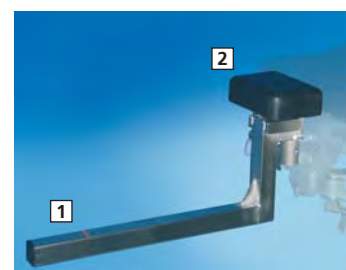

Plaster boards

1 Plaster boards for patients of various sizes

PLASTER BOARD LARGE, PAIR · Part No. 1385776

To attach to adapter for plaster board 1386129 and adapter for CALYPSO 1404892; for patient weights up to 135 kg; stainless steel
Dimensions: 280 mm x 175 mm

PLASTER BOARD MEDIUM, PAIR · Part No. 1385777

For patient weights up to 80 kg
Dimensions: 180 mm x 165 mm

PLASTER BOARD SMALL, PAIR · Part No. 1386128


For patient weights up to 30 kg
Dimensions: 160 mm x 135 mm

2 PLASTER BOARD LARGE · Part No. 4544458

To attach to pelvis support 1386130, adapter for plaster board 1386129 and adapter for CALYPSO 1404892
Dimensions: 200 mm x 80 mm

3 PLASTER BOARD SMALL · Part No. 4544459

To attach to pelvis support 1386130, adapter for plaster board 1386129 and adapter for CALYPSO 1404892
Dimensions: 160 mm x 60 mm

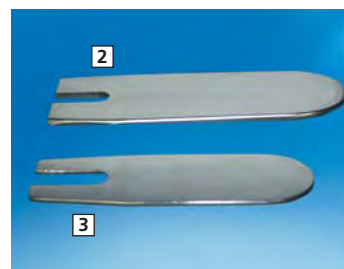

Storage systems

Extension trolley

EXTENSION TROLLEY · Part No. 1297815

For positioning and transport of the OR-Adapter including accessories during column or shuttle use; stainless steel

Length 1,158 mm x width 685 mm x height 765 mm

EXTENSION TROLLEY SATURN SYSTEM · Part No. 1307484

For positioning and transport of the OR-Adapter SATURN System 1242174 including accessories during column or shuttle use; stainless steel

Length 1,158 mm x width 685 mm x height 830 mm

EXTENSION TROLLEY Q · Part No. 1315543

For positioning and transport of the OR-Adapter Q 1309812 including accessories during column or shuttle use; stainless steel

Length 1,158 mm x width 685 x height 840 mm



TROLLEY FOR EXTENSION ACCESSORIES

Part No. 4545631

For storage of extension accessories; with basket; stainless steel chassis

Dimensions: Length 1.350 mm x width 670 mm x height 1.620 mm

Without accessory



TROLLEY FOR PLASTERING ACCESSORIES

Part No. 4545637

For plaster accessories; stainless steel chassis

Dimensions: 1.350 mm x 670 mm x 1.620 mm

Without accessory



TRUMPF

Positioning accessories



Positioning accessories

ASTOPAD DUO CONTROL UNIT

Part No. 1308289

To attach to infusionsstand; 2 heating elements can be controlled independently of one another; for heating blanket COV 150 1308290 and heating blanket COV 180 1308333

Dimensions: 180 mm x 260 mm x 210 mm

Output voltage: 24 V; operating voltage: 220 V–240 V



ASTOPAD COV 150 HEATED BLANKET

Part No. 1308290

Including one extension cable COV 50200 1308337; use as a table top blanket and an underblanket; X-ray-permeable, flexible, shape-adaptable, washable and disinfected;

Dimensions: 500 mm x 1.500 mm

COVER FOR COV 150 · Part No. 1308331 (not shown)

Cover for heated blanket COV 150 1308290; can be washed at 95°C and disinfected

DISPOSABLE COVER FOR COV 150 · Part No. 1308332 (not shown)

Disposable cover for heated blanket COV 150 1308290; pack of 50 pieces



ASTOPAD COV 180 HEATED BLANKET

Part No. 1308333

Including one extension cable COV 50200; use as a table top blanket and an underblanket; X-ray-permeable, flexible, shape-adaptable, can be washed down and disinfected; dimensions: 800 mm x 1.800 mm

COVER FOR COV 180 · Part No. 1308334 (not shown)

cover for heated blanket COV 180 1308333; can be washed at 95°C and disinfected

DISPOSABLE COVER FOR COV 180 · Part No. 1308335 (not shown)

disposable cover for heated blanket COV 180 1308333; pack of 50 pieces



CABLE BRACKET

Part No. 1308336

To attach to infusion stand; for 2 extension cables

Max. diameter: 25 mm

(not shown)

EXTENSION CABLE COV 50200

Part No. 1308337


Cable for connecting the ASTOPAD COV heated blanket to the ASTOPAD DUO control unit 1308289

(not shown)

HEATING PAD OPT 130

Part No. 1219018


For adults; electrically conductive; for Astopad Duo control unit 1308289
 Dimensions: 1.000 mm x 370 mm x 10 mm

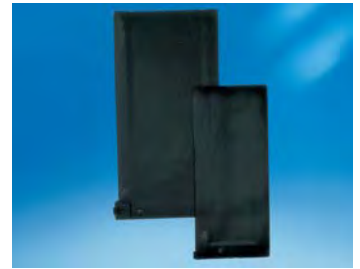

Anti-decubitus pads

TABLE GEL PAD LARGE · Part No. 1219020
 Suitable for heating pad OPT 130 1219018
 Dimensions: 1100 mm x 500 mm x 15 mm

TABLE GEL PAD SMALL · Part No. 1219021
 Dimensions: 860 mm x 370 mm x 15 mm

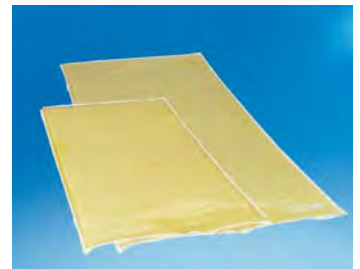

GEL PAD FOR HIP

Part No. 4140014

Dimensions: 500 mm x 500 mm x 13 mm



GEL PAD FOR ARM / FOOT

Part No. 4545670

Dimensions: 380 mm x 100 mm x 6 mm



GEL PAD FOR TABLE TOP

Part No. 4545667

Dimensions: 1.100 mm x 500 mm x 15 mm



GEL PAD FOR TABLE TOP WITH CUT-OUT

Part No. 4545671


Dimensions: 1.170 mm x 510 mm x 15 mm



GEL POSITIONING ROLL

Part No. 4545669

Dimensions: 350 mm x 110 mm x 50 mm

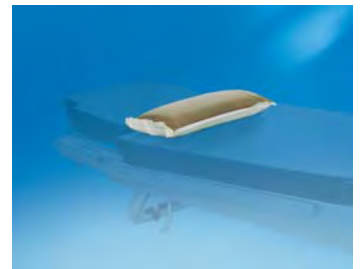

HEAD SUPPORT FOR OPHTHALMOLOGY

Part No. 4545673

Dimensions: 300 mm x 270 mm x 98 mm
(not shown)

HEAD RING 210

Part No. 4140016

Dimensions: 210 mm x 76 mm x 44 mm



GEL PAD HORSESHOE SHAPED

Part No. 4545672


Dimensions: 200 mm x 76 mm x 44 mm



PAD SEMI CIRCULAR SHAPED

Part No. 4145408

Pad integrally foamed, electrically conductive and soft
 Dimensions: 530 mm x 180 mm x 115 mm

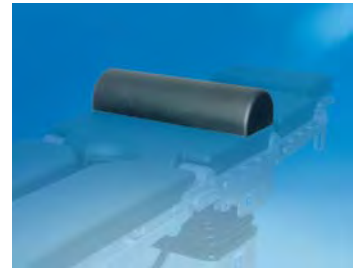

KNEE BENCH

Part No. 4145287

For knee positioning; pad integrally foamed, electrically conductive and soft
 Dimensions: 500 mm x 450 mm x 280 mm



ROLLBOARD FOLDABLE

Part No. 4545675

Dimensions: 1.800 mm x 500 mm





TRUMPF

